

Product instruction manual

multifunctional separate nursing bed

Seracare

1. Tips




1.1 This product will be shipped in 2 packages. They may not arrive at the same time. Please wait.

1.2 Thank you for your purchase! We are committed to providing you with high quality products. In the unlikely event that your product is damaged, missing parts or has problems with assembly, please contact our customer service team. We will offer you the best solution as soon as possible. In order to solve your problem quickly, please provide us with some pictures that clearly show the product problem. Thanks !

1.3 Please keep all small parts out of the reach of children when assembling. May be fatal if swallowed or inhaled. Children should not climb or play on the product to avoid serious injury from furniture tipping over. To avoid possible injury from movement, keep all parts and packaging (film, plastic bags, foam, etc.) away from infants and children.

1.4 Please match the corresponding parts and accessories to the part numbers in the manual. If parts and accessories do not match, check the information in this manual. When assembling, first align all screws loosely with the corresponding pre-drilled holes and then tighten the screws one by one. During product use, periodically check the tightness of screws and tighten as necessary to ensure structural integrity. Please keep sharp and corrosive objects away from the product to avoid damage or injury.

1.5 It is recommended to install it with two people.

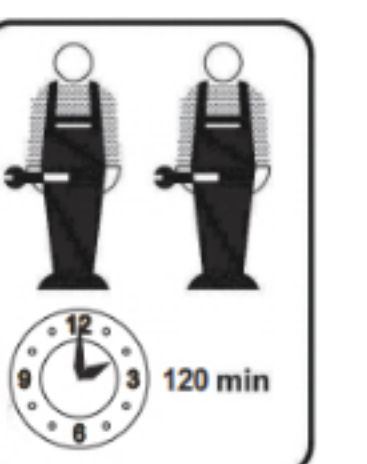

2. Product overview

2.1 Composition of product structure

The multifunctional separate nursing bed consists of a bed frame, foot bars, armrests, a motor with its driving system, and a handheld electric controller.

2.2 Product model and specification: KYG-F-D03.

2.3 Product functions

- 1) Assist with back lifting and lowering
- 2) Lower the legs to form a chair
- 3) Assist the user in getting out of bed

3. Product performance parameters

3.1 Dimension parameter table

Project	Size (in)
Bed length	84.25
Bed width	43.70
Bed height	37.09
Bed surface height	19.76

3.2 Electrical parameters

- 1) Input voltage: AC 1 00V~240V ; Operating voltage: DC 35V ;
- 2) Frequency: 50/ 60Hz ;

3.3 Performance parameters

- 3.3.1 Folding angle of electric bed backboard: 0°~ 75° (±5°) .
- 3.3.2 Folding angle of the bed's leg section: 0°~ 85° (±5°) .
- 3.3.3 The lifting assist angle of the bed frame: 0°~ 35° (±5°) .


3.4 Parameter specification

Appearance	Light Gray
Output Voltage	24V DC .
Battery Capacity	1.2 Ah
Battery Type	Lead-acid Battery
Ingress Protection (IP) Rating	IPX4
Operating Temperature	+10°C to +40°C
Certification Standard	IEC60601-1
Built-in Protection	Deep Discharge Protection

4. Product operating conditions



- 1) Ambient temperature: +5°C~+40°C;
- 2) Relative humidity: <90%;
- 3) Atmospheric pressure: 860hPa~1060hPa

5. Instructions for using the remote control

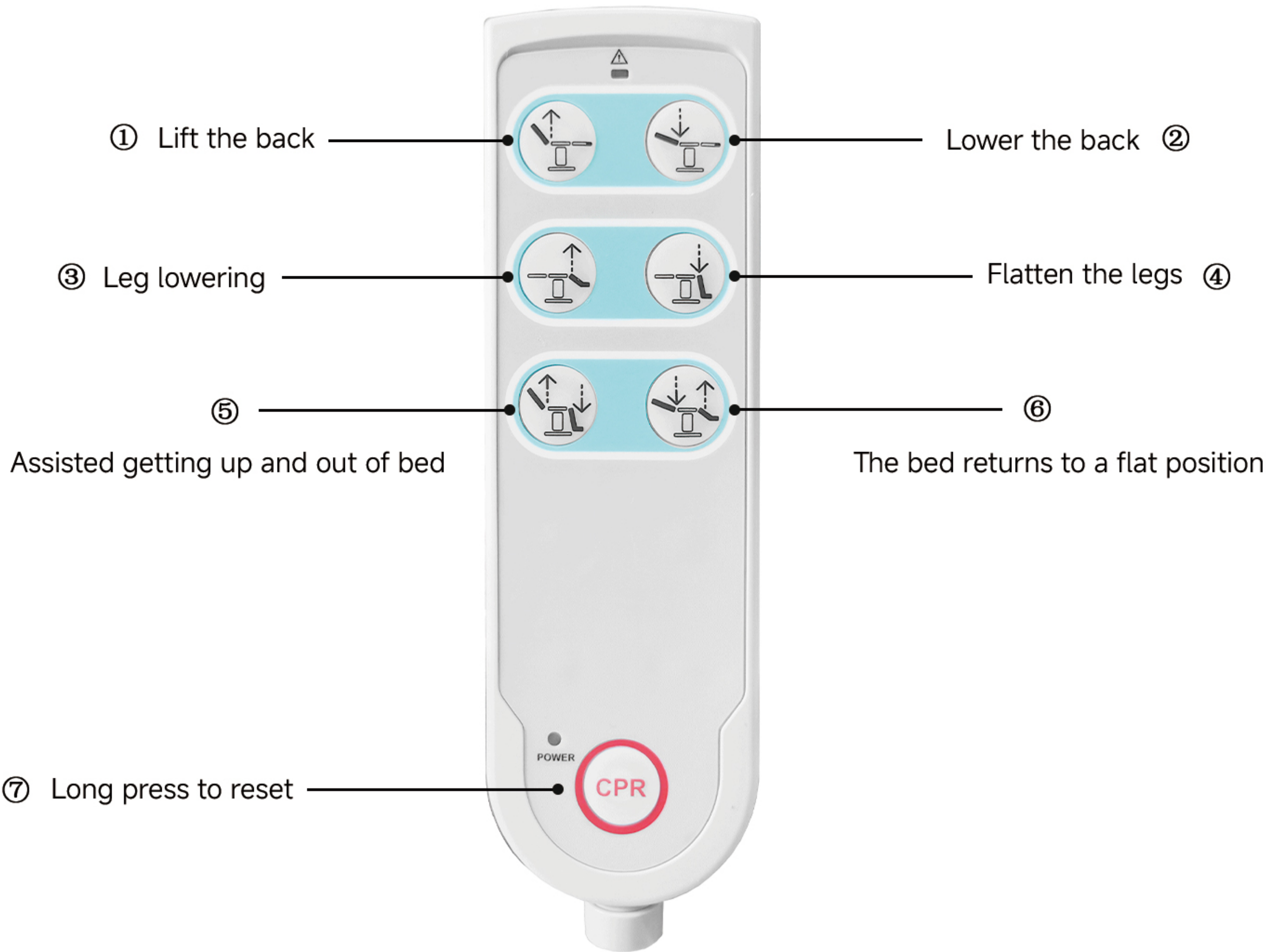

- ① Press and hold the button to raise the back, relieving back pressure.
back lifting angle: 0-75°(±5°).
- ② Press and hold the button to lay the back flat.
- ③ Press and hold the button to lower the legs.falling leg angle: 0-85°(±5°).
- ④ Press and hold the button to flatten the legs.
- ⑤ Press and hold the button to adjust the bed's position and assist in getting up.
- ⑥ Press and hold the button to lay the bed flat.
- ⑦ Press and hold the button to long press to reset.

6. List of accessories

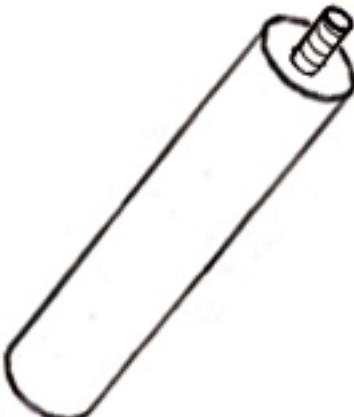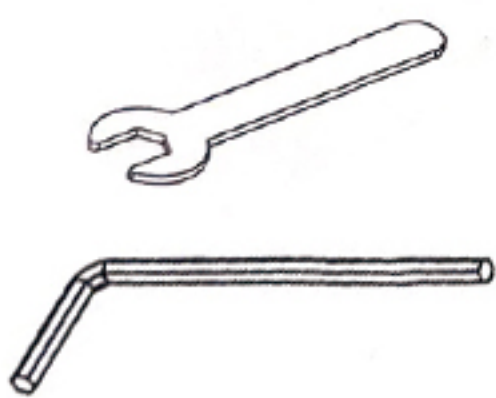
① x 8	② x 15	③ x 2	④ x 4
 Screw: M8*55mm	 Nut: M8		
⑤ x 8	⑥ x 8	⑦ x 4	
 Screw: M10*30mm	 M8 screw nut assembly	 Locking hook	
⑧ x 6	⑨ x 1	⑩ x 1	⑪ x 16
 Screw: M8*35mm	 Screw: M8*40mm	 Screw: M10	 Screw: M6*35mm
⑫ x 1			⑬ x 1
 Mattress			 Tool

7. Installation steps (A)

A1. Secure the left and right bed frames to the bed body with M8*55mm screws and M8 nuts.




A2. Manually install the brake buttons into the two round holes on the bed frame.

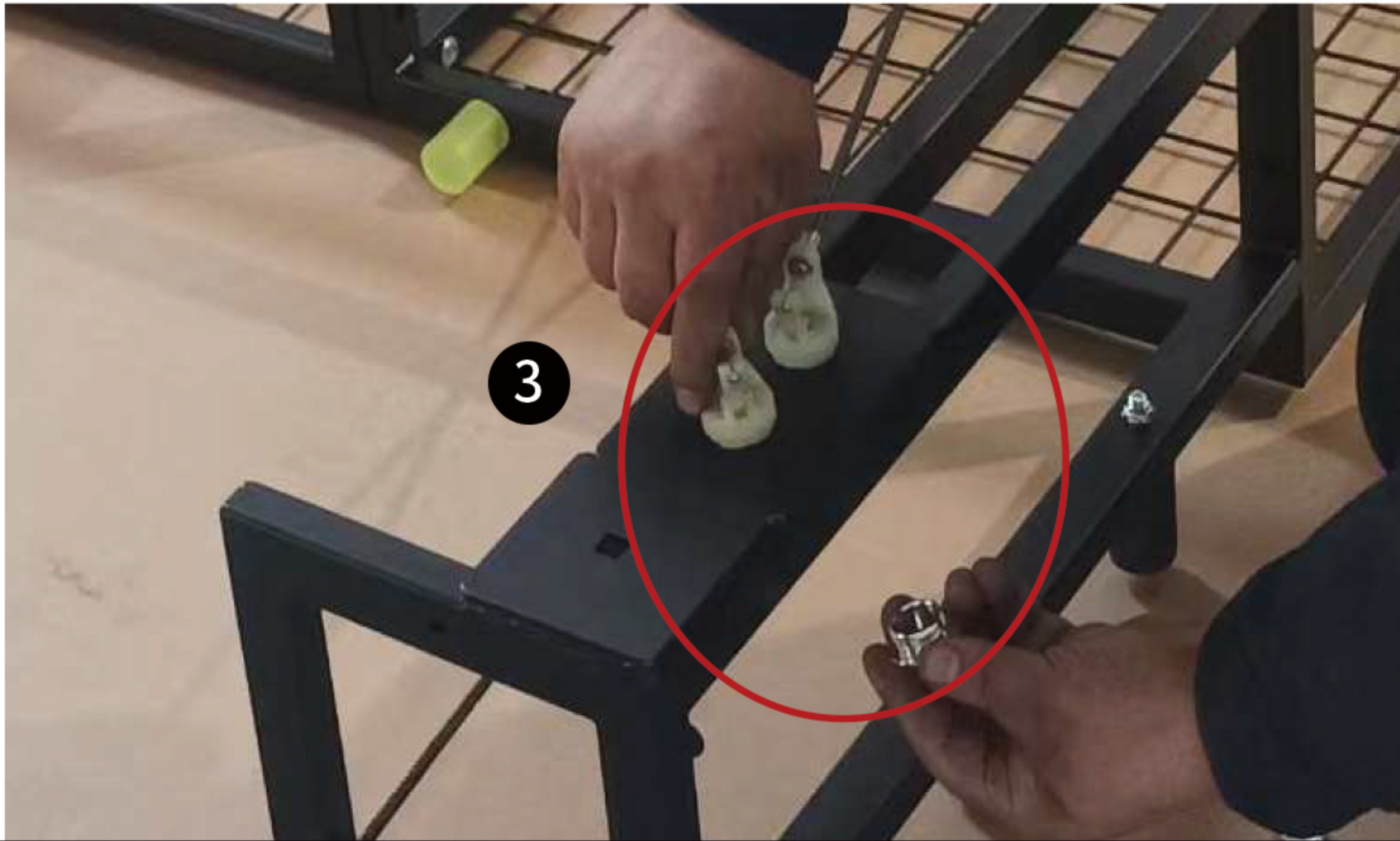


8. Installation steps (B)

B1. Fix the bed legs to the left and right bed frames with M10*30mm screws.

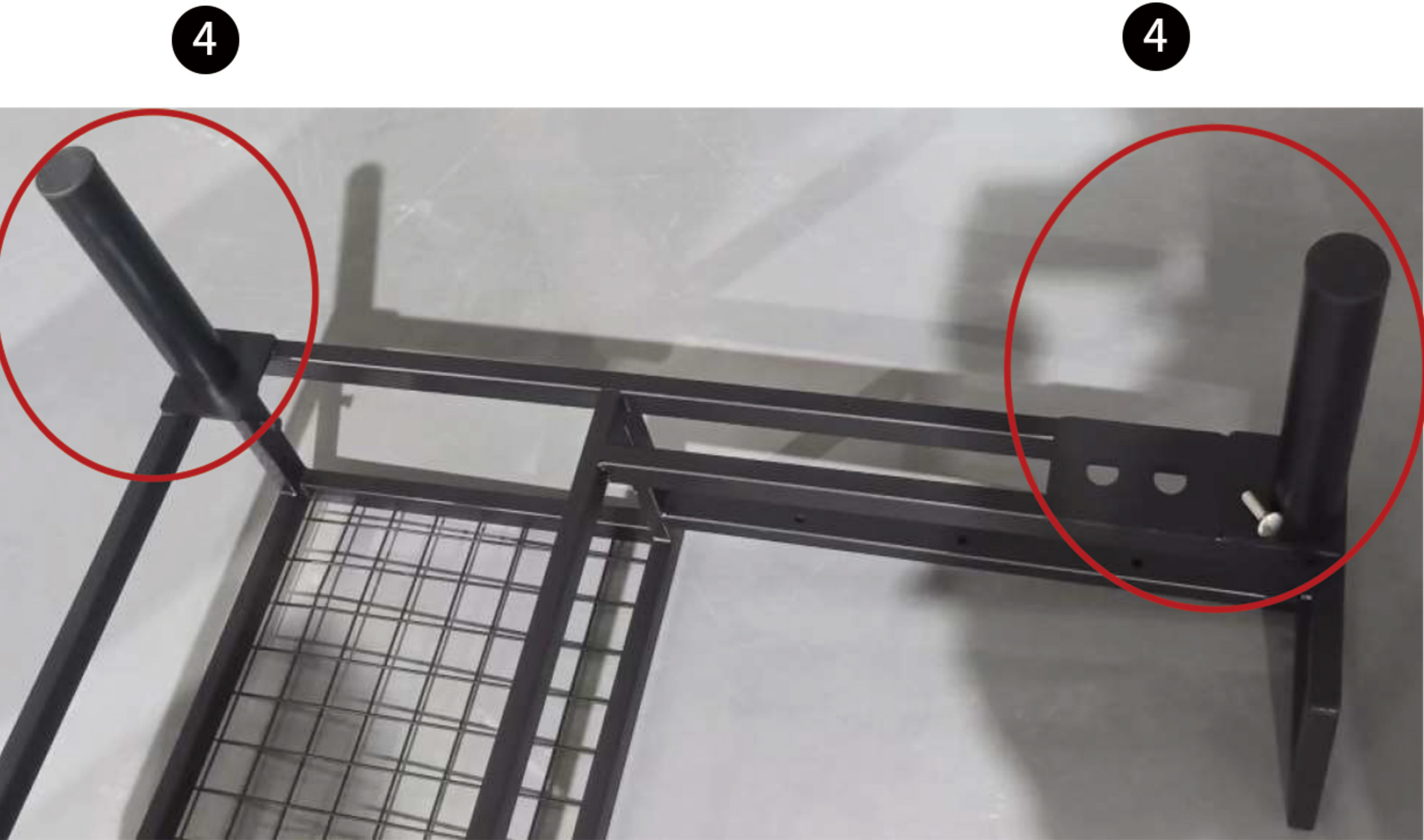


B2. Attach the bed side panels to the bed body with M6*35mm screws.

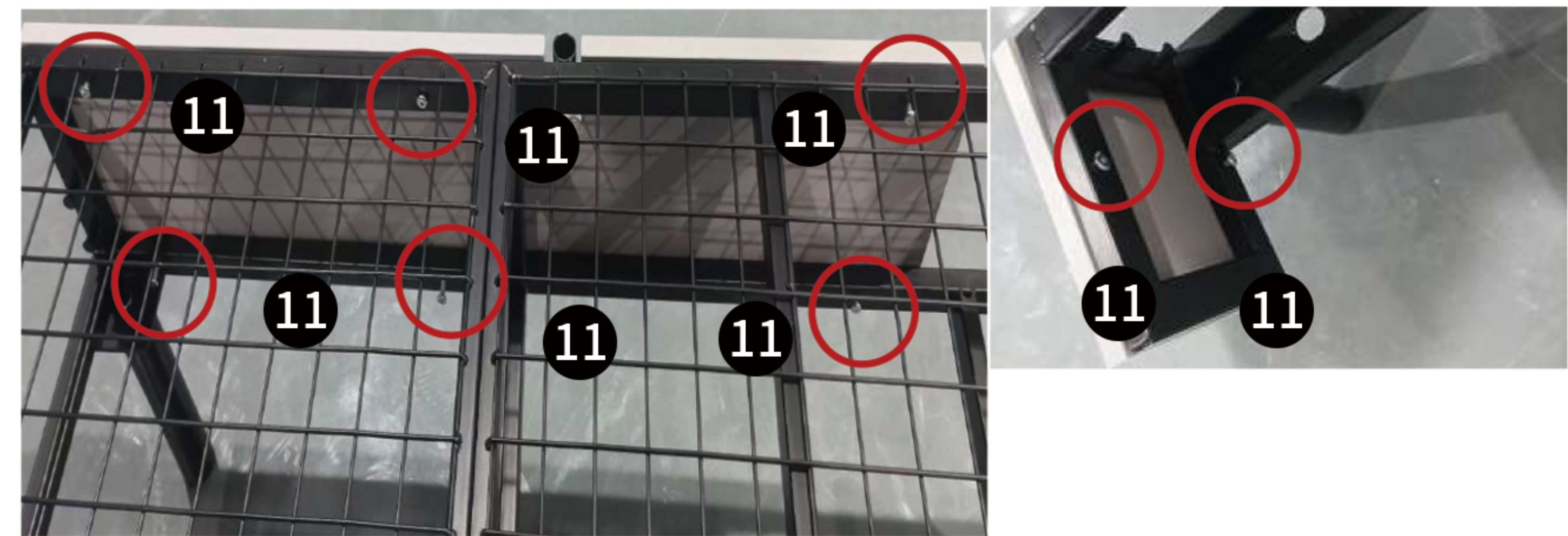

9. Installation steps (C)

C1. Fasten the buckle hooks to the headboard and footboard with M8 Chicago screws.

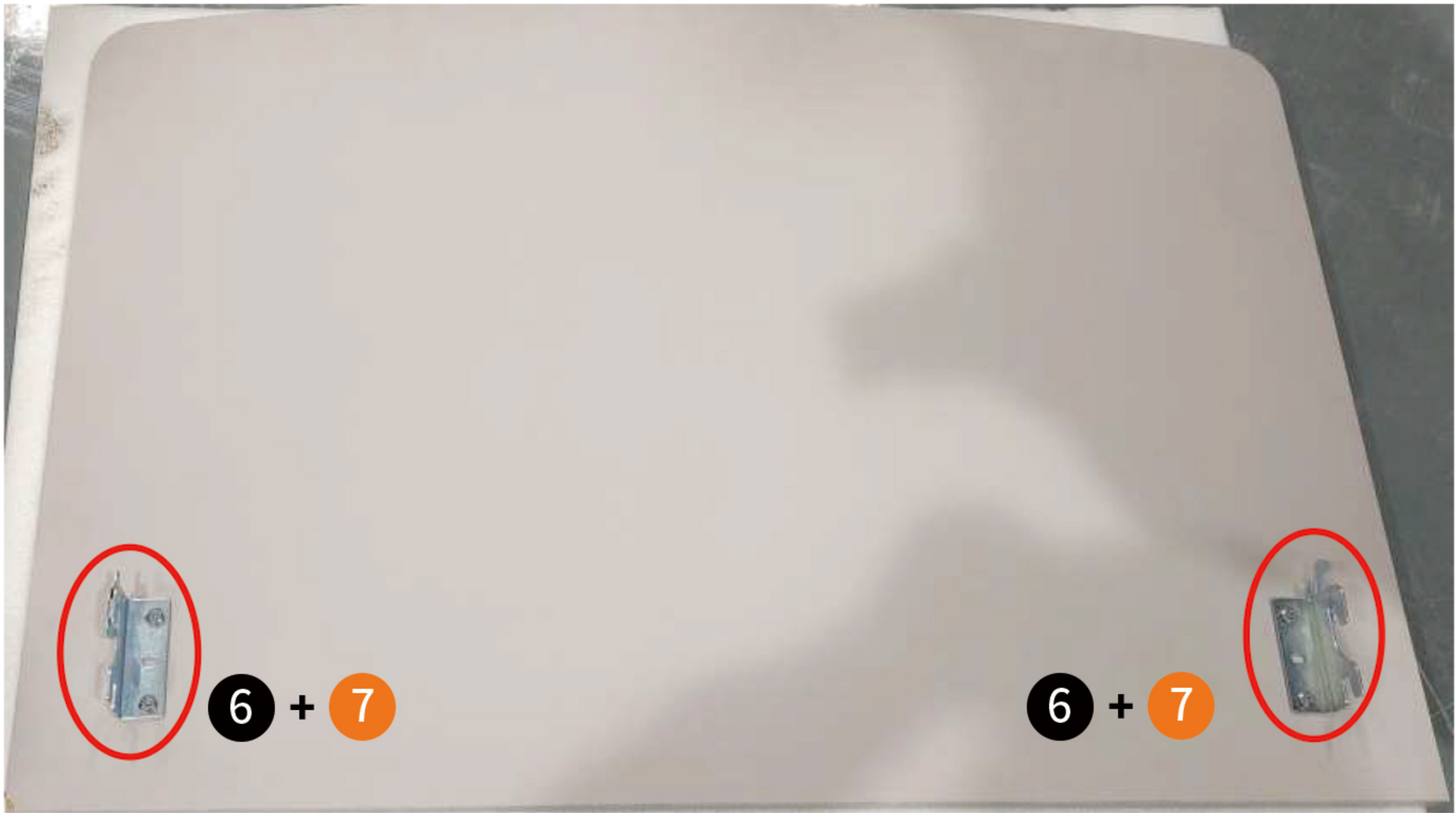

C2. Hang the headboard and footboard onto the front and rear of the bed body.



10. Installation steps (D)

D1. Insert the armrests into the corresponding holes on the bed body.




D2. Secure the left and right leg assemblies with M8*35mm screws and nuts; fix the middle part with M8*40mm screws.




11. Installation steps (E)

E1. Fasten the back assembly with M8*35mm screws and nuts.

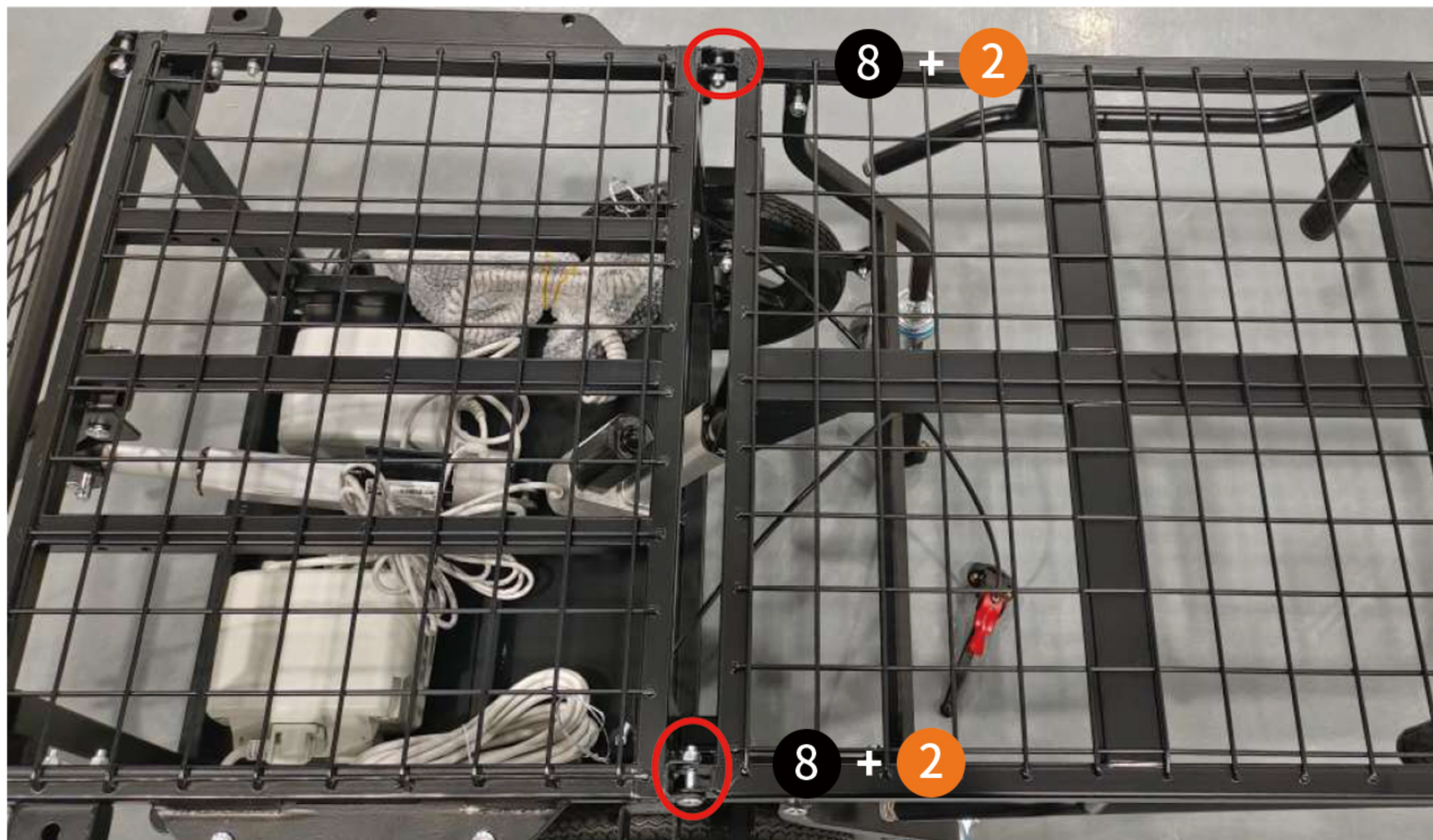

E2. Fix the small wheel bracket with M8*35mm screws and nuts.

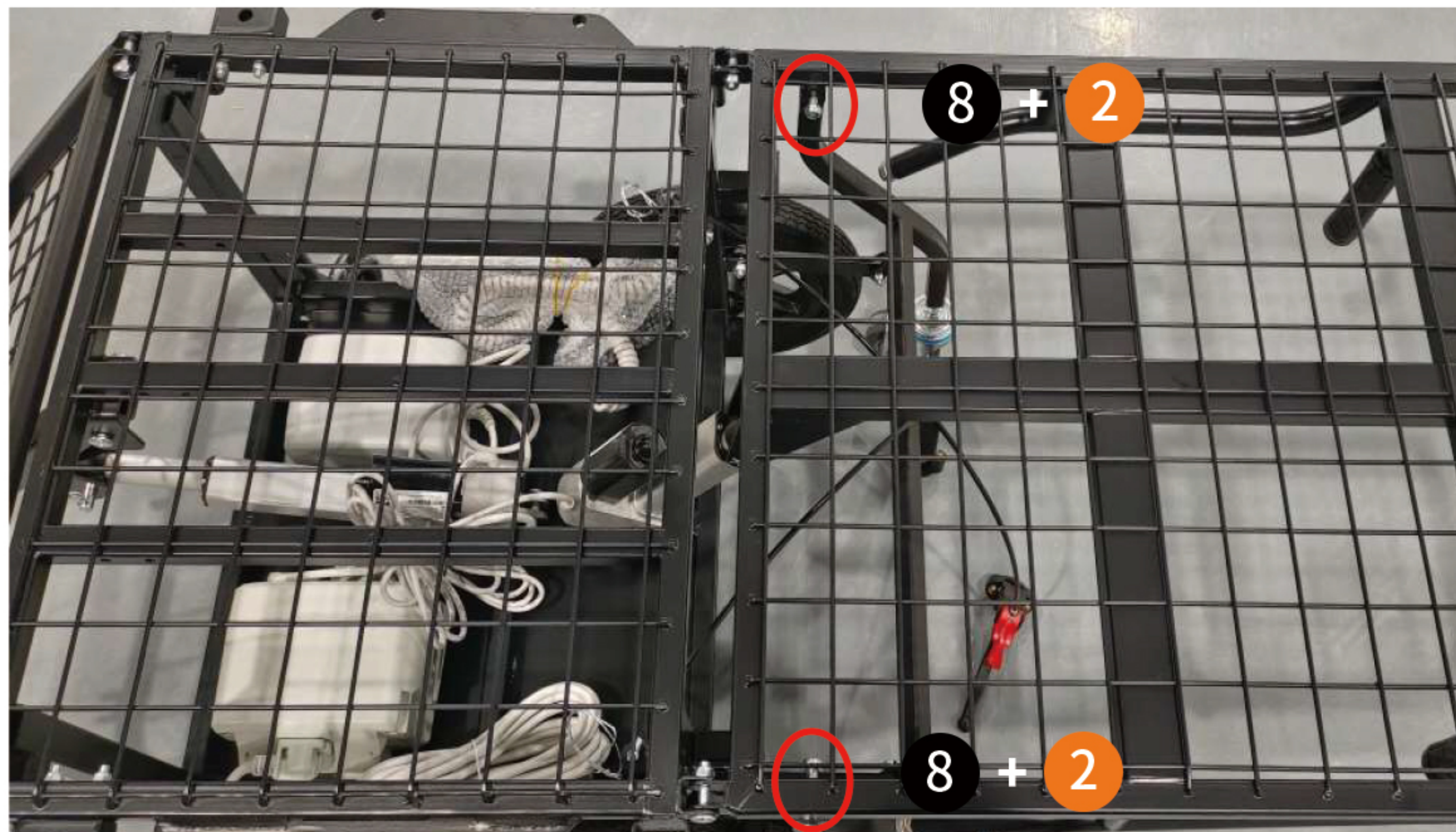


12. Installation steps (F)

F1. Attach the brake handle to the handle of the back assembly.



F2. After laying the wheelchair flat, adjust and tighten the M10 screws against the small wheel bracket.

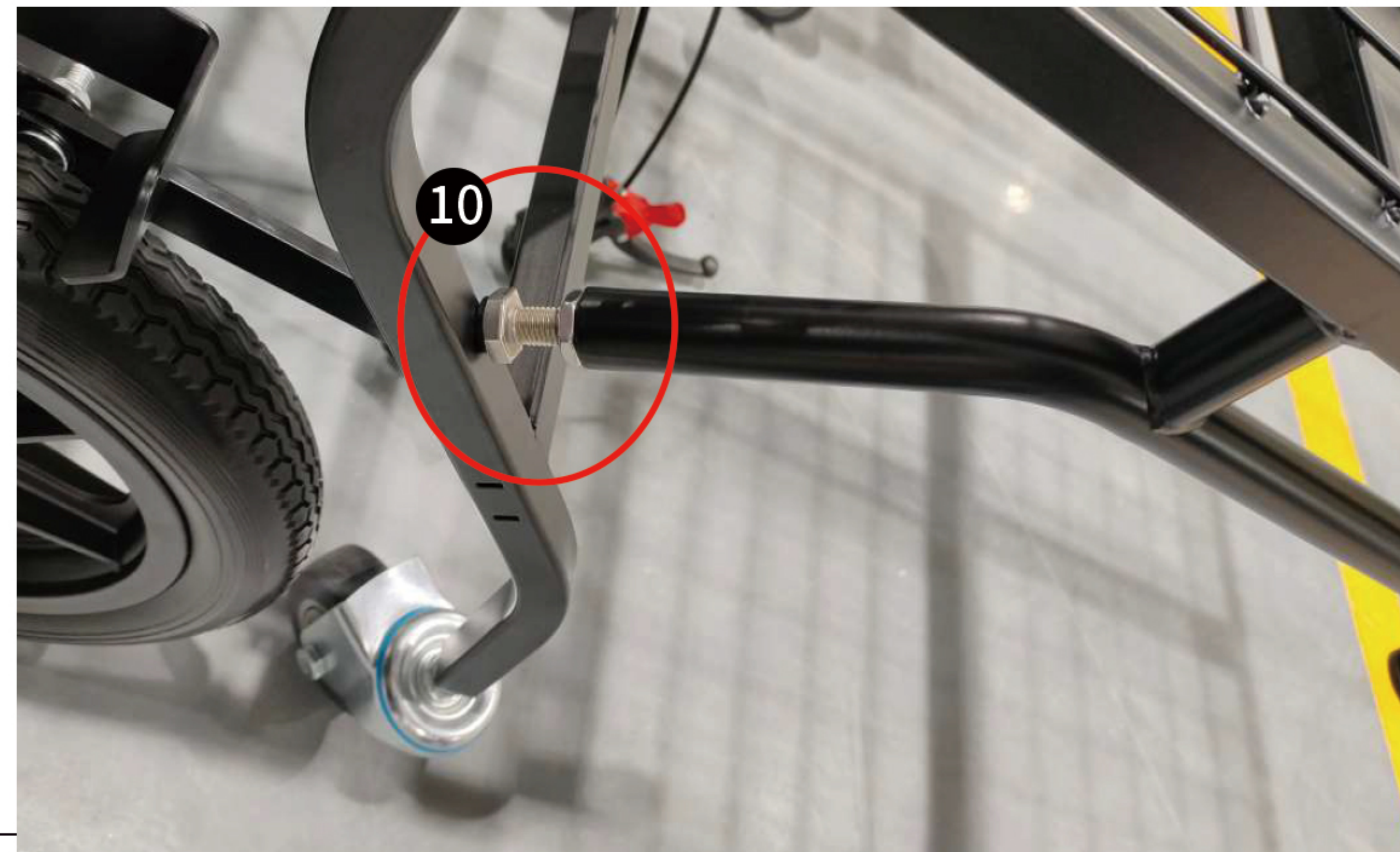

13. Installation steps (G)

G1. Lay the wheelchair flat, connect it with the bed frame, and lock them together.

