

Preface

Dear User:

- Thank you for purchasing our shower chair.
- Before using the product, please carefully read this instruction manual to ensure proper usage and full understanding.
- After reading, please keep this manual in a safe place for future reference.

Product Specifications/Structure

Manual Measurement Error 1-5 cm



Product Configuration: YC746

Product Dimensions: 95x55x(113-118)cm

Product Load-bearing Capacity: 100 kg

Attention



1. Before the user sits down, ensure both front and rear brakes are locked and the footrests are retracted.
2. After the user is seated, lower the footrests and have the user place their feet lightly on them. The caregiver should release the brakes when pushing is needed.
3. Never step directly onto the footrests when getting out of the wheelchair, as this may cause forward tipping! The caregiver should first press the brake to ensure it is locked, then retract the footrest, and finally assist the user in getting out of the wheelchair.
4. The wheels of this product are relatively small, and it is prohibited to push it over bumps or on uneven roads when carrying people.
5. Before bathing, shampooing, or using the commode, you should also lock the wheels.

Product Installation Steps



1. Open the packaging and take out the product and accessories.



2. Install the wheel onto the main frame, aligning it with the holes and inserting the C-clamp. Repeat the same installation process for the remaining wheels.



3. Attach the backrest to the main frame (Check if nuts are present at the holes).



4. Install the air spring rod onto the main frame, insert the short screw through the combined hole, then screw in the nut, and tighten using an L-shaped wrench and an open-end wrench.



5. Install the long screws by threading them through the washers and inserting them into the armrest holes.



6. Please note that armrests are designated as left or right—L for left, R for right.



7. Install armrests onto the main frame. Insert armrests through the curved plastic spacers, align with holes, and tighten using an L-shaped wrench.



8. Install the footrests onto the main frame (align with holes).



9. Place the commode into the shower chair to complete assembly.

Adjustment Methods



Press the backrest adjustment handle, then pull the back panel downward to lower the backrest.



Adjusting the Headrest: Loosen the headrest screw bolt to adjust the headrest angle, then tighten the screw bolt.
Screw Bolt Adjustment: Push the screw bolt outward to adjust its angle.



Feet Tube Height Adjustment: Remove the wheel's locking clip to adjust the hole position height according to usage requirements, then secure with the clip.



Footrest Height Adjustment: Unscrew the footrest's nut and screw to adjust the hole position height as needed, then tighten the screw.