



**Shower and Toilet Stool Atlantic**

User Instructions



712-022



Fully read the user instructions carefully before using the Shower and Toilet Stool Atlantic for the first time. Keep the user instructions in a safe place so that they are available for later use.

**Intended Purpose**

The shower and toilet stool is suitable for domestic use by patients who, for example, due to paralysis, loss of limbs, frailty, cardiovascular illness or through injury require a stable support when showering or when using the toilet and are not able to visit the bathroom / WC independently.

**Indications**

The shower and toilet stool can be used in the event of restricted mobility of the legs and / or knees as well as the upper limbs, loss of strength, balance disorders and slight visual impairment.

**Instructions / Assembly: Have this carried out by your specialist dealer / care assistant**

1. Remove all the individual components from the cardboard box and check the contents and the packaging for damage and completeness.

2. Place the frame on the rubber feet on the floor and secure the upholstered toilet seat with the aid of the fixing brackets.




3. Secure the four wheels with the clips – ensure that these are all set to the same height! From the back, push the toilet pan with the lid into the holder and place the cover on to it.




4. Press down the red brake pedals so that the frame is standing securely on the floor and then insert each of the left and right-hand tubes with the footrests into the openings intended for this.

Ensure that the black bracket runs on the tube. Turn the footrests to the front until the pin engages in the bracket. The rests for the feet point inwards. Fold the footrests down.

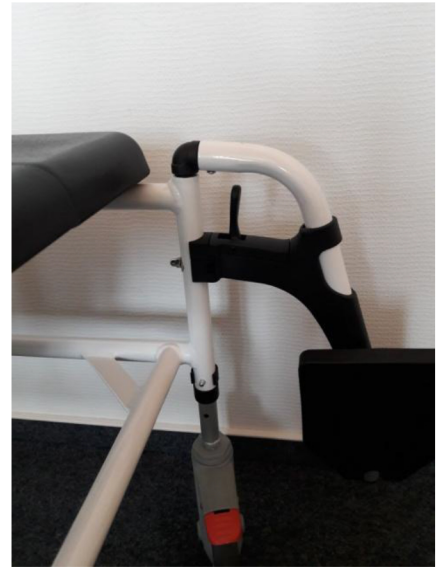

5. Insert the lower tube ends of the backrest in the openings at the back of the frame and secure them using the two knob screws. Fold the armrests down.

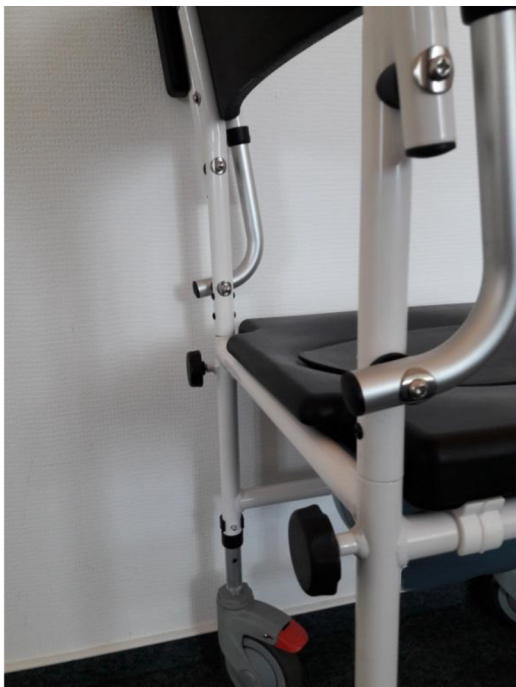

6. To release the brakes, press the black locking button.



7. To adjust the height of the footrests, loosen the wing nuts, remove the screws and set the tubes to the desired position. Then replace the screws and firmly tighten the wing nuts.



### Safety Instructions



1. Ensure that the brakes have been applied before a person sits on the Shower and Toilet Stool Atlantic.
2. Before **every use**, check that the backrest and the footrests are secure.
3. Ensure that the wheels and the footrests have been set to the same height!
4. Before use, check that the toilet pan is seated firmly in its mount and if needed that the cover closes with the edge of the toilet seat.
5. During use ensure that the armrests are folded down. **Risk of being trapped!**
6. Ensure that the footrests are first folded down when the patient is sitting on the Shower and Toilet Stool Atlantic. Otherwise there is the risk of **tripping and injury!**
7. The maximum permissible load on the shower and toilet stool is 136 kg!
8. Do not use a defective shower and toilet stool and contact your specialist dealer if you notice damage to the material!
9. Do not stand on the footrests and do not lean one-sided on the armrests! **Risk of tipping over! Risk of falling!**
10. Do not stand on the shower and toilet stool and do not allow unattended children in its vicinity! **Risk of injury!**

### Cleaning

Clean the shower and toilet stool regularly with a mild soap solution and clean water. Use a soft cloth for drying. Do not use any abrasive or corrosive cleaning agents. The removable seat can be washed off under running water and can then be disinfected. The toilet pan should be treated in exactly the

same way after every use. Use a commercially available disinfectant to disinfect the shower and toilet stool. Do not use high-pressure or steam cleaners for cleaning!

If they are dirty, clean the wheels using a soft brush or a cloth and a mild soap solution and afterwards dry them using a cloth.

### **Multiple Use**

The Shower and Toilet Stool Atlantic is intended for daily use. It might be used by several people. Before the shower and toilet stool is used by someone else or ownership is transferred to another person, it must be thoroughly cleaned and disinfected in order to prevent the transmission of germs (contamination). Here the procedure described under the point *Cleaning* and a commercially available disinfectant must be used.

### **Storage**

Good ventilation must be ensured during use in the bathroom in order to avoid the building up of deposits. The shower and toilet stool must be dried after use. Before storing the shower and toilet stool it must be completely dry. Subsequently store it in a dry location protected against light in the polybag and cardboard box supplied.

### **Contraindications**

The use of the shower and toilet stool is not suitable in the event of joint contractures, severe occurrences of paralysis, severe balance or visual disorders or severe pains in the trunk or in the upper and lower limbs. With significant loss of strength when sitting, the use of the shower and toilet stool must be refrained from as safe seating cannot be guaranteed.

### **Maintenance**

The Shower and Toilet Stool Atlantic is maintenance-free.

Functional Faults: 

If you notice the weakening of the brakes, stiffness of the wheels or a defect in the material, please contact your specialist dealer!

Do not use a defective or damaged Shower and Toilet Stool Atlantic!

### **Scope of Supply**

Shower and toilet stool frame, 4 wheels including insertion tubes and clamps, 2 footrests including insertion tubes and fixing screws and nuts, backrest including insertion tubes and 2 locking screws, seat with lid, toilet pan with lid, user instructions, polybag, cardboard box.

### **Technical Data / Dimensions and Weight**

Material: frame - aluminium, powder-coated, backrest, armrests and seating surface / toilet seat - PU, footrests – thermoplastic elastomer

Weight: 13 kg

Height: 92-100 cm

Seat height: 48.5 – 56 cm

Armrest height: 71 – 79 cm

Backrest height: 18 cm

Width of the backrest: 44 cm

Seat width: 43 cm

Seat depth: 39 cm

Overall width: 45 cm

Overall depth: 82.5 cm

Footrests: approx. 15 x 14 cm

Distance between the seat surface and the lower end of the backrest: approx. 22 cm

Maximum load-bearing capacity: 136 kg

### Disposal

A worn-out shower and toilet stool is a source of valuable materials which can be reintroduced into the industrial cycle. Therefore do not dispose of your shower and toilet stool in the household waste, instead have it disposed of professionally by an officially approved waste disposal centre. Your local waste collection department will be pleased to provide you with the relevant information.

### Guarantee

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects.



001-877-644-9366  
customerservice@aosom.com