

# Instruction Manual

## Folding Commode

Thanks for using our product, before using, please read the manual carefully.

Item No: CST-2069



1. Backrest combination
2. Amrest combination
3. Seat combination
4. Bucket
5. Leg
6. Foot rest


Seat Size: 400\*420mm

Seat Height (seat to floor height): 480~530 ± 5mm

Net weight: 7.2kg

Weight Capacity: 200kg

## Installation methods and procedures

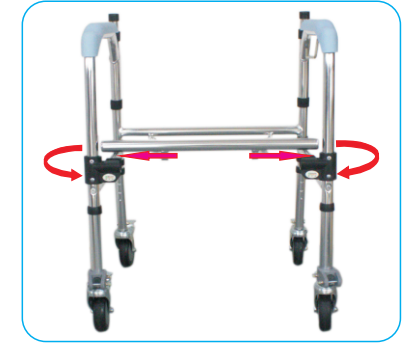

1. Take all the items out of package.
2. Install front seat tube and armrest combination. Open the left/right armrest. To align the armrest, the front seat tube insertion is pushed to a marble pop.



3. Install backrest and bucket. put the backrest combination in the direction of the arrow, the plastic snap joint for seat on the back plug-in mounting, and then put the bucket on the seat tube that tube, then press down hard.
4. Install the seat.



5. Install foot rest. Place the foot rest on the reserved installation, use a wrench to lock and hold.
6. Setup Complete.