Lift Chair ASSEMBLY INSTRUCTION




ADULT ASSEMBLY REQUIRED

Read all instructions before assembly and use.


Due to the presence of small parts during assembly, keep out of reach of children untill assembly is completed.

THIS LIFT CHAIR ELECTRIC RECLINER HOLDS
A MAXIMUM WEIGHT OF 350 LBS
DO NOT EXCEED

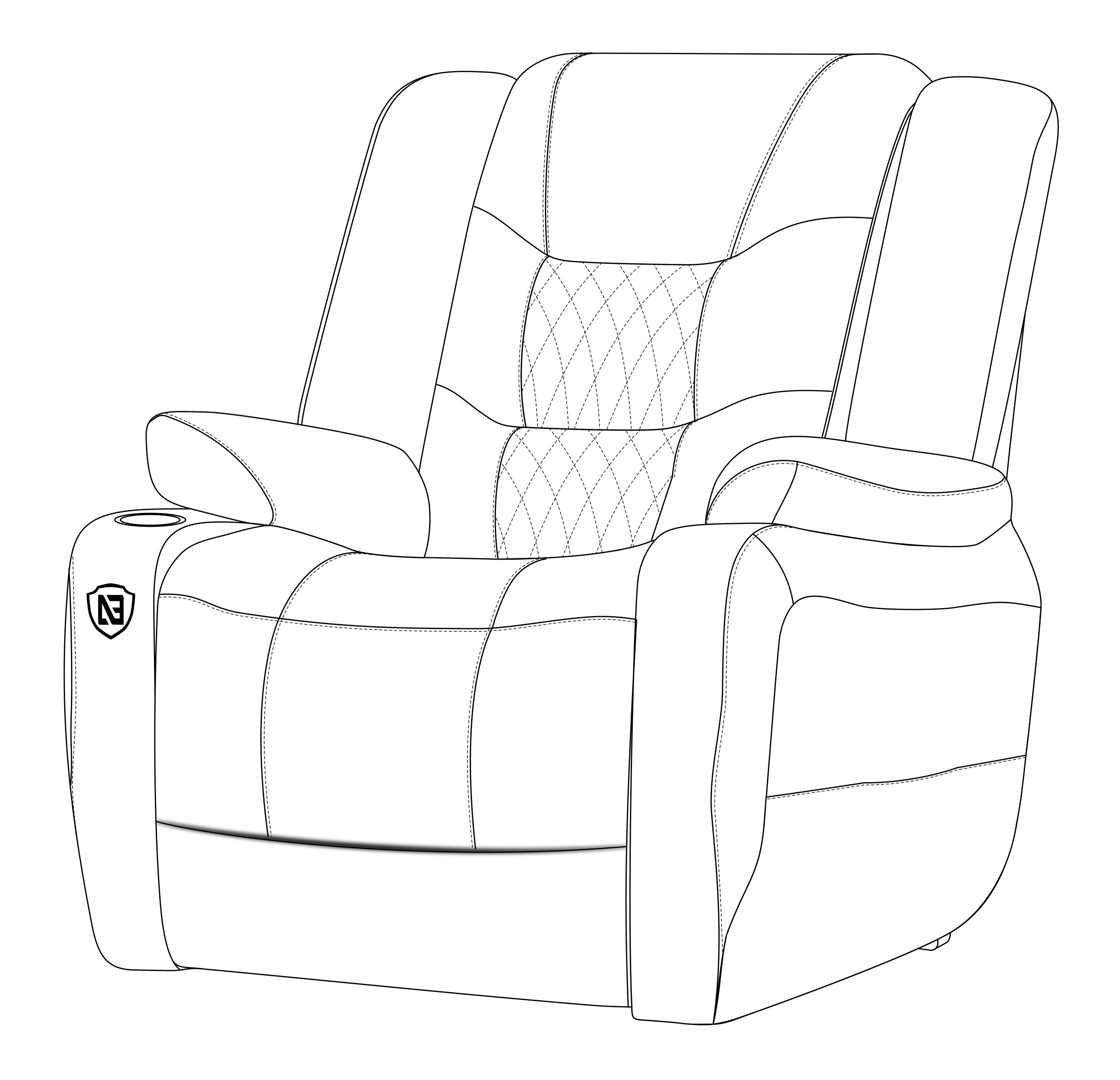



Assembly Instruction

STEP 3

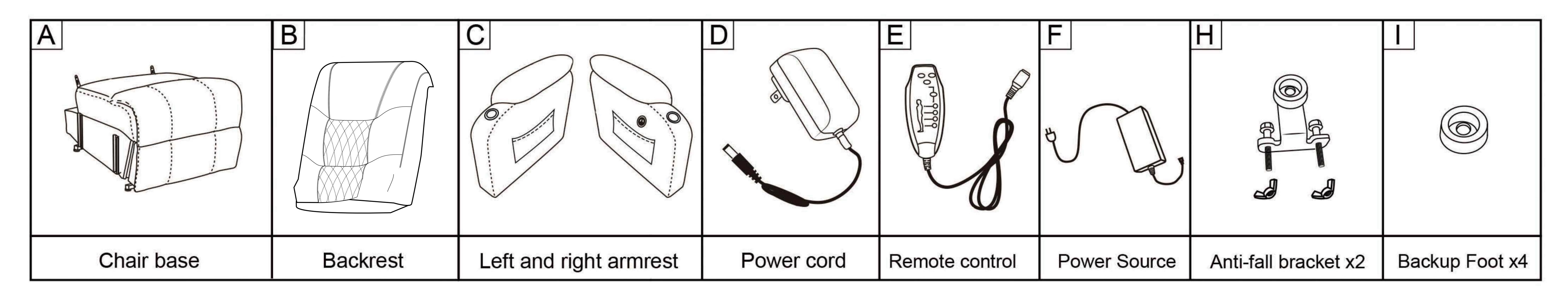



You will receive totally 2 boxes, please make sure you get 1x (A) box and 1x (B) box before assembly. Just feel free to chat us if you have any problem, we will try our best to help you solve the issue.

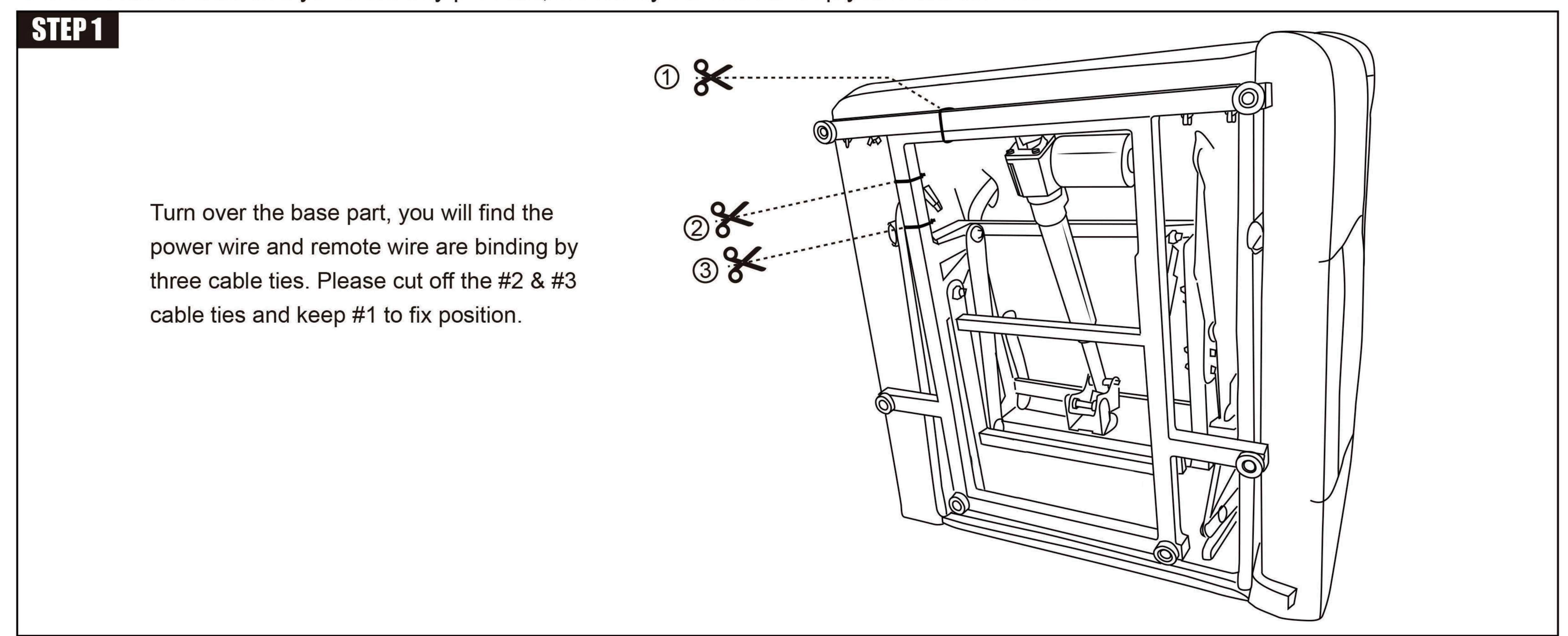

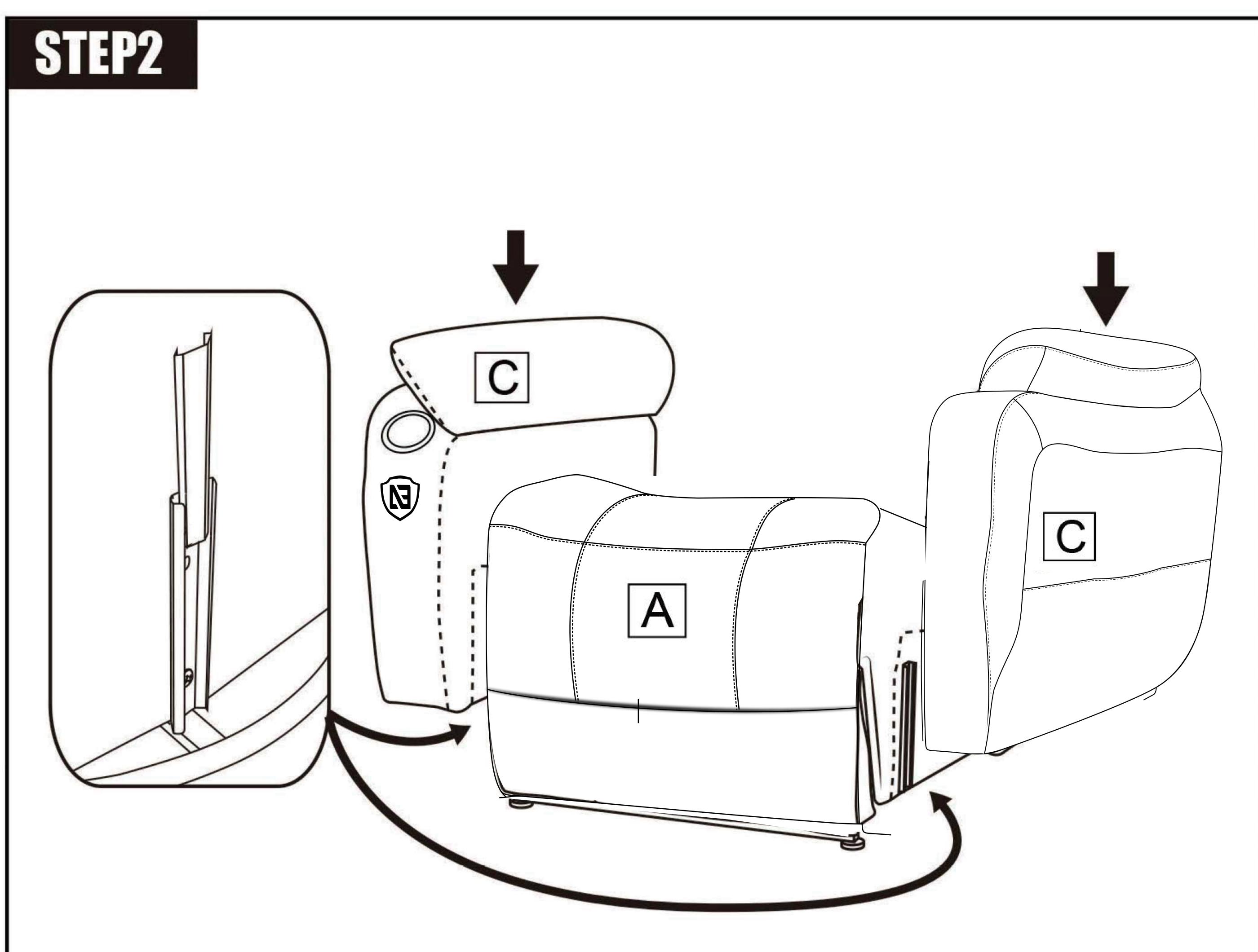



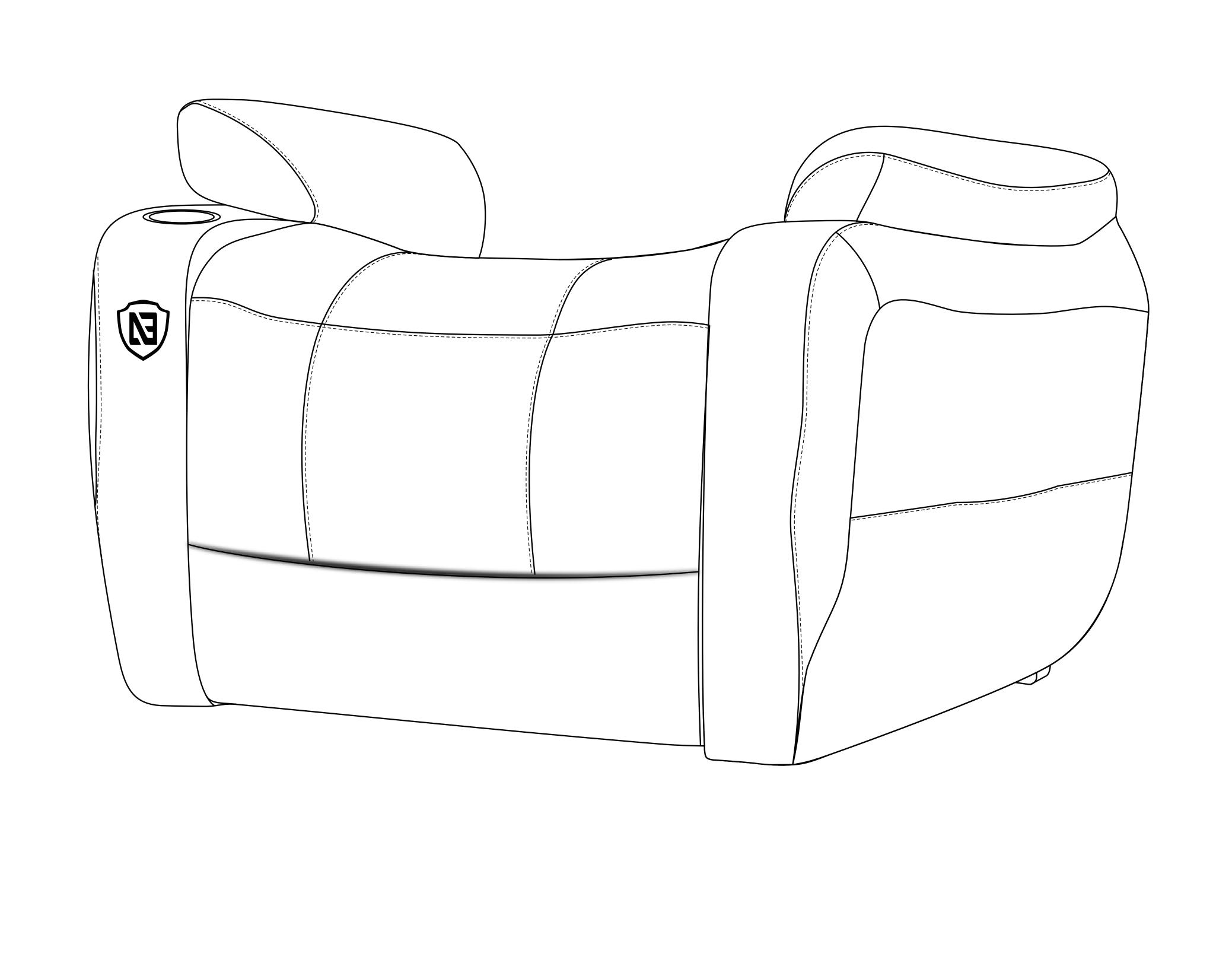

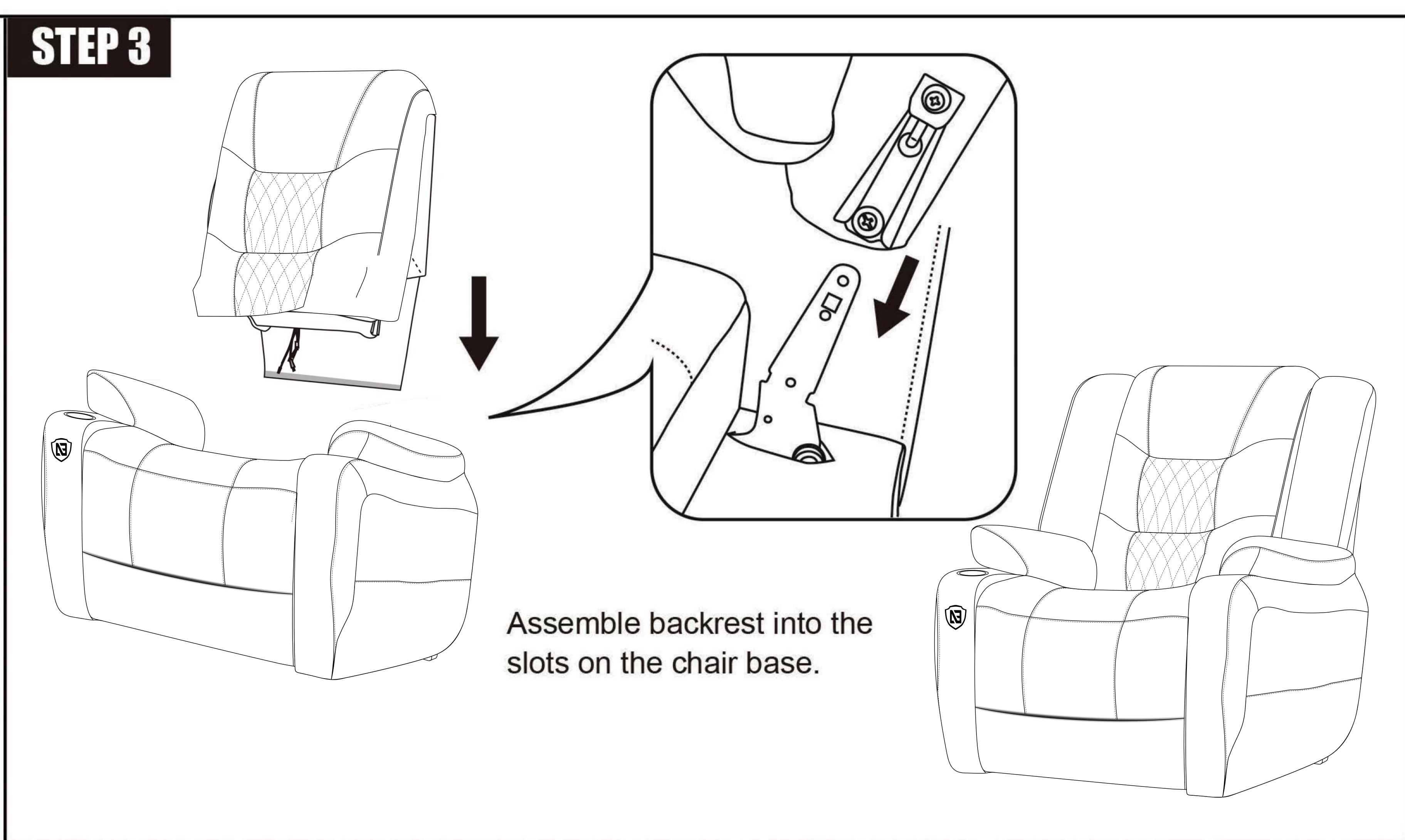



STEP 4


- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power cord(D).
- No.4 wire connects with backrest and remote control(E).
- No.5 wire connect the motor and armrest with control panel.
- No.6 wire connect the motor and power source (F).

NOTE

- 1: Two same color wires connect.
- 2: Plug power cord and power source into outlet, then you can use all function.

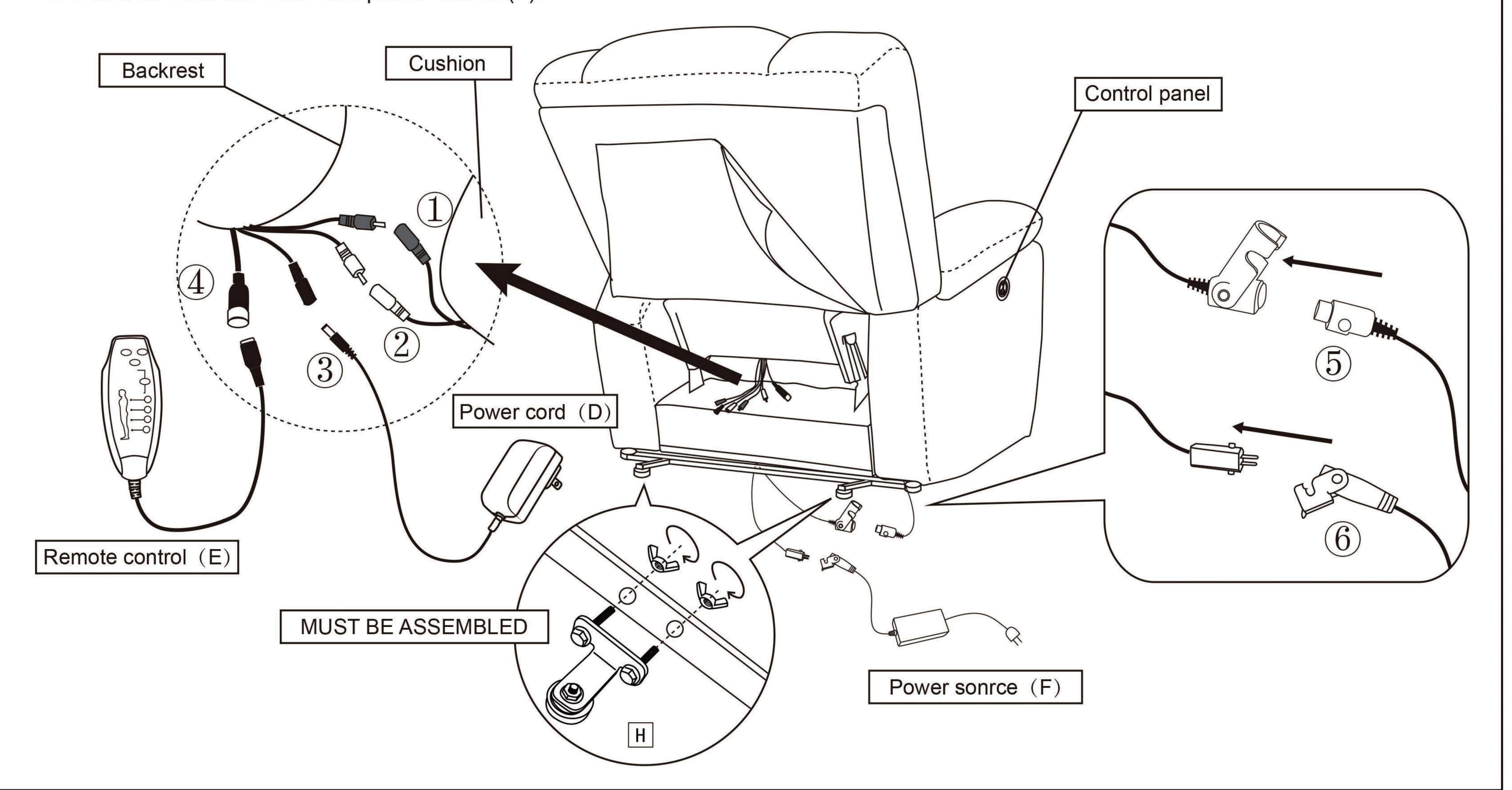

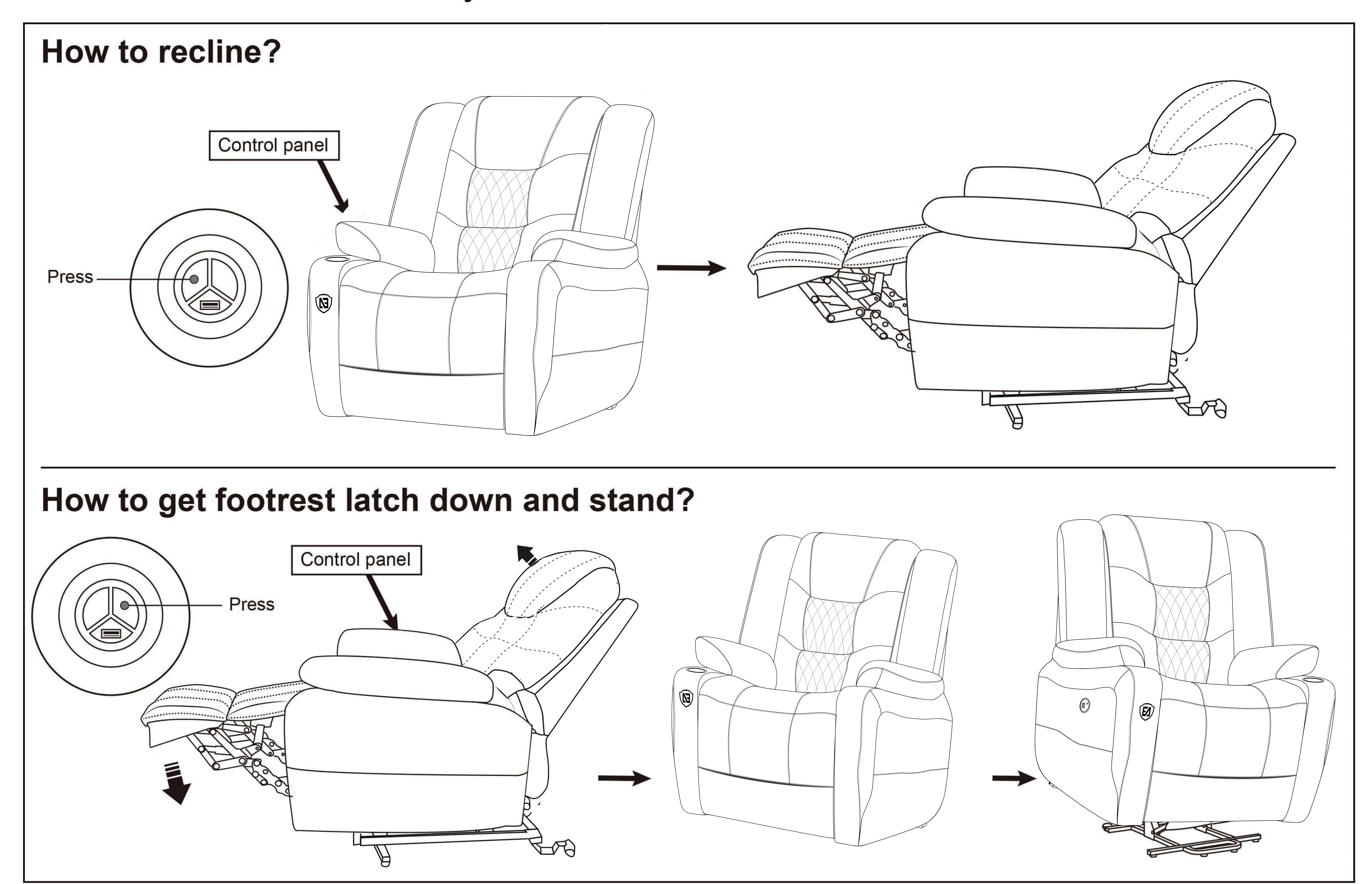

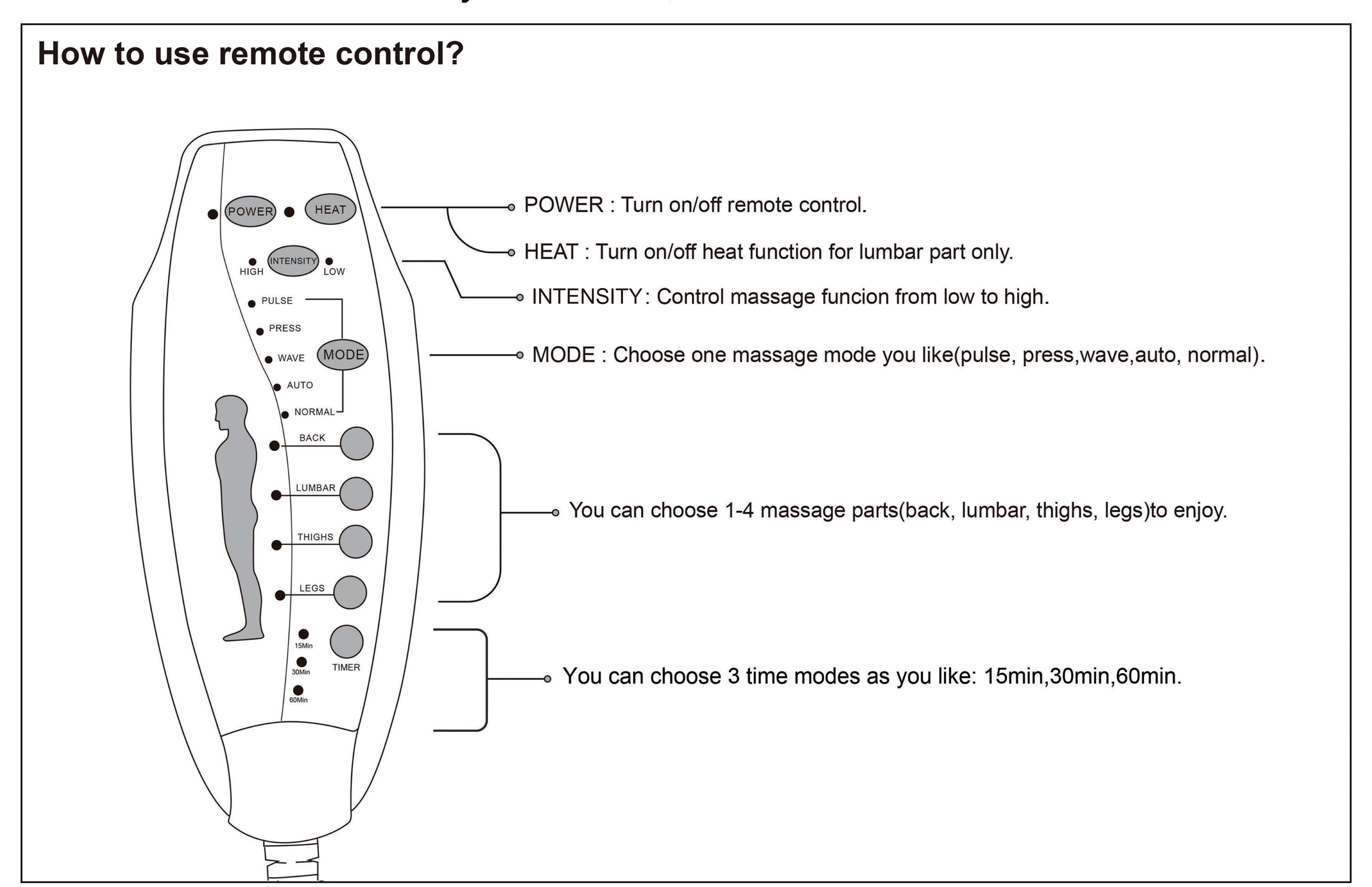

FAQ

1.Q: Missing parts

A: Please check if you have already received two boxes. If so, kindly send messages to let us know missing part name regarding this instruction, we will send replacement for you asap.

2.Q: Some parts damaged when arrived

A: Please send damaged parts photos to us as message attachment, we will check and offer good solution soon.

3.Q: The lift and reclining function does not work, e.g. The chair is stuck on lifting or reclining position, and there is no response when you press the lifting or reclining button.

A: Please help to unplug and plug all the wires as per Step 4. When all the wires are connected well, first, please check if the indicator light on power source(F) is ON or NOT, if the light on power source does not turn on, the power source is defective. If the light on power source is ON, but the control panel at left arm has no blue light, that's the problem with the control panel.

When the power source and control panel both have light, that's the problem of the lift motor.

When you have this problem, please don't worry, you just need to contact us thru Amazon message and describe the problem, then we will send the needed replacement to you immediately.

4. Q: The vibration massage functions is not working.

A: Please check first if all wires at the back of recliner chair are connected correctly. The power cord also been plug into the wall outlet. Then please check the lights on when you press "POWER" button on the remote. If no, the remote is defective and you can contact us for new one to replace.

5. Q: The Heat function not work or can't feel.

A: Please check all wires connection as same as question 5 first. If the remote also works well, you will need wait for 5-10 mins after you press the "HEAT" button. The heating is only works at lumbar position and the heat temperature isn't very hot.