

Scope of application and instructions

1. This shower chair is an assistant facility for disabled people to take a bath more conveniently.
2. Before using the shower chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole.
3. Check seat board, the backrest are securely installed, please sit in the middle position of the Shower chair when using it.

Storage Method

Please put the shower chair in a dry and ventilated place. Do not exposure it under the sun for a long time. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Precautions

To prevent the injury caused by shower chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water for a long time and keep the board dry.

⚠️ WARNINGS:

- ⚠️ Prior to each use check to assure that all screws, joints and fittings on the bath bench are tight. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times. Be sure that all gold locking buttons fully protrude thru holes of each Adjustable Leg.
- ⚠️ DO NOT place bath bench on uneven surfaces.
- ⚠️ DO NOT use if wobbly, unstable or not level.
- ⚠️ DO NOT exceed user static pressure bearing weight of 205kg(450LBS).
- ⚠️ DO NOT stand on the bath bench.
- ⚠️ DO NOT use the armrest to fully support body weight when using the unit.
- ⚠️ Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions
- ⚠️ This shower chair cannot be used in the bathtub.
- ✅ Excessive twisting or turning while seated on the bench may cause a loss of balance and damage to the product.
- ✅ Check rubber tips for rips, cracks or wear. If these conditions exist, DO NOT use the bath bench and replace the tips immediately.
- ✅ Check the bench for cracks or ruptures in the plastic.
- ✅ Check that all push buttons are fully protruded from the mainframe through the leg extensions and are properly engaged.
- ✅ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use.



SHOWER CHAIR CST-3012

**CAUTION
UP TO 450LBS**



To ensure the safe installation, adjustment, and operation of the shower chair, these instructions MUST be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER AND/OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions, and instructions, contact a professional or dealer before attempting to use this equipment. In particular, we are not to be held liable for any indirect or direct damage or consequential damage, loss or expenses in connection with the use or inability to use the device for a specific purpose.

SHOWER CHAIR CST-3012

Thanks for using our product.
Before using, please read the manual carefully.



Chair back: 410*170mm/16.1*6.7 "
Seat: 510*310mm / 20*12.2 "
Seat height (seat to floor height): 355-510mm/14-20 "
Chair back Height: 710-870mm/28-34.3 "
Net weight: 2.72kg / 6LBS
Static pressure bearing force: 205kg / 450LBS


Elements of the chair and manual for installation:



1. Chair back
2. Tube for chair back
3. Seat
4. Chair leg
5. Anti-slip pad for chair



1. Take all the items out of package.

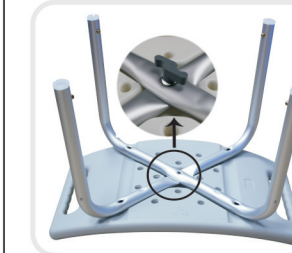


2. Put the seat on the seat frame and fix with short screws. (please make clearly which side is the back/front side of the seat)



3. Put the two straight tubes in the corresponding place and fix with long screws. (remember to use the plastic ring with the screw)


The dimension with 3 holes should face outside because it's used to connect to the tube of the chair back.

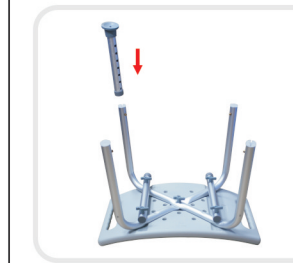

4. Install the foot tube. First press the pinball on the seat tube, then insert the foot tube into alignment, push to fit the hole position, the ball can jump out. (Note: 4pcs chair legs must be in the same level of hole)



5. Fix the chair back and the tube with short screw. (with plastic ring)

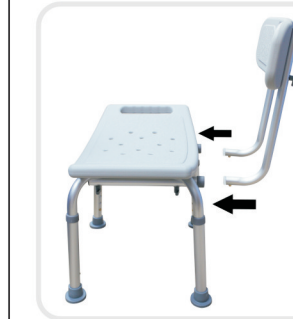

6. The backrest tube is inserted into the seat plate under the straight hole, out of the pinball.



7. Setup Complete;

Thanks for your purchase. Any concerns please don't hesitate to contact us, we will resolve all the issues timely. Customers are always the best teachers for us to keep growing!