


Shower chair

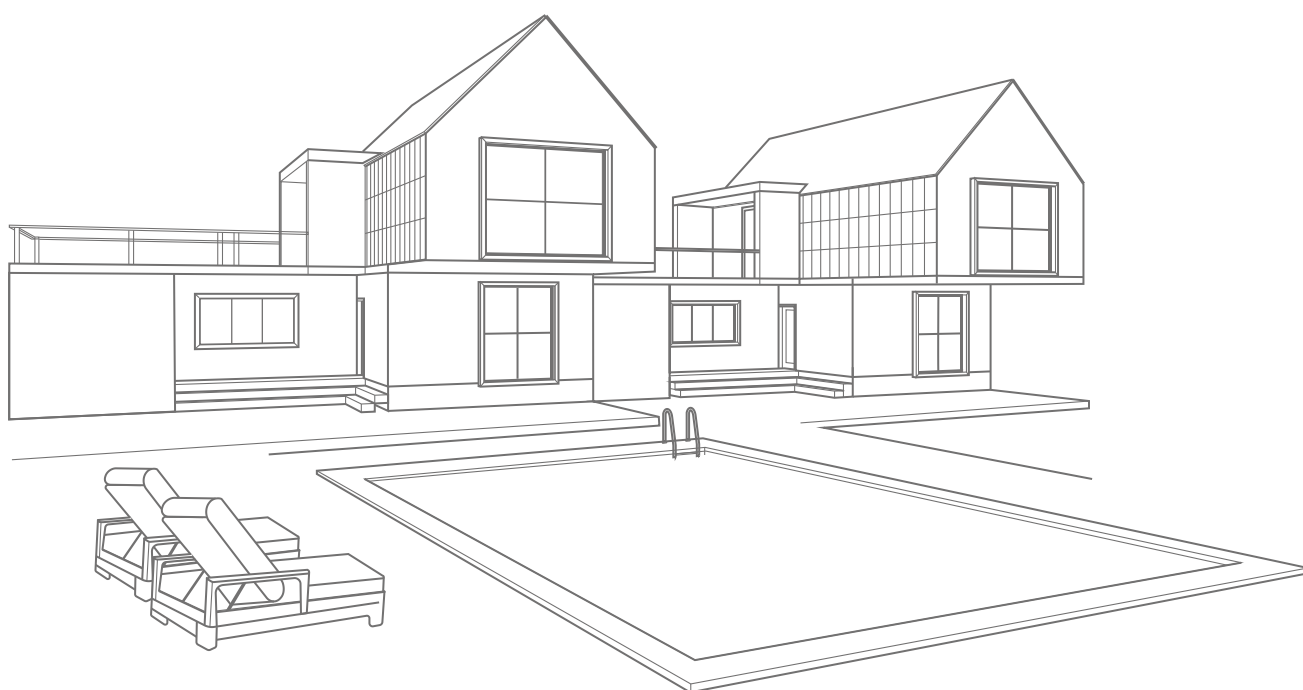

01 TROUBLE SHOOTING :

Missing / Wrong / Broken parts and installation issues



Send email with your question, pic and order NO.



Respond within 24 hours




Issue Resolved







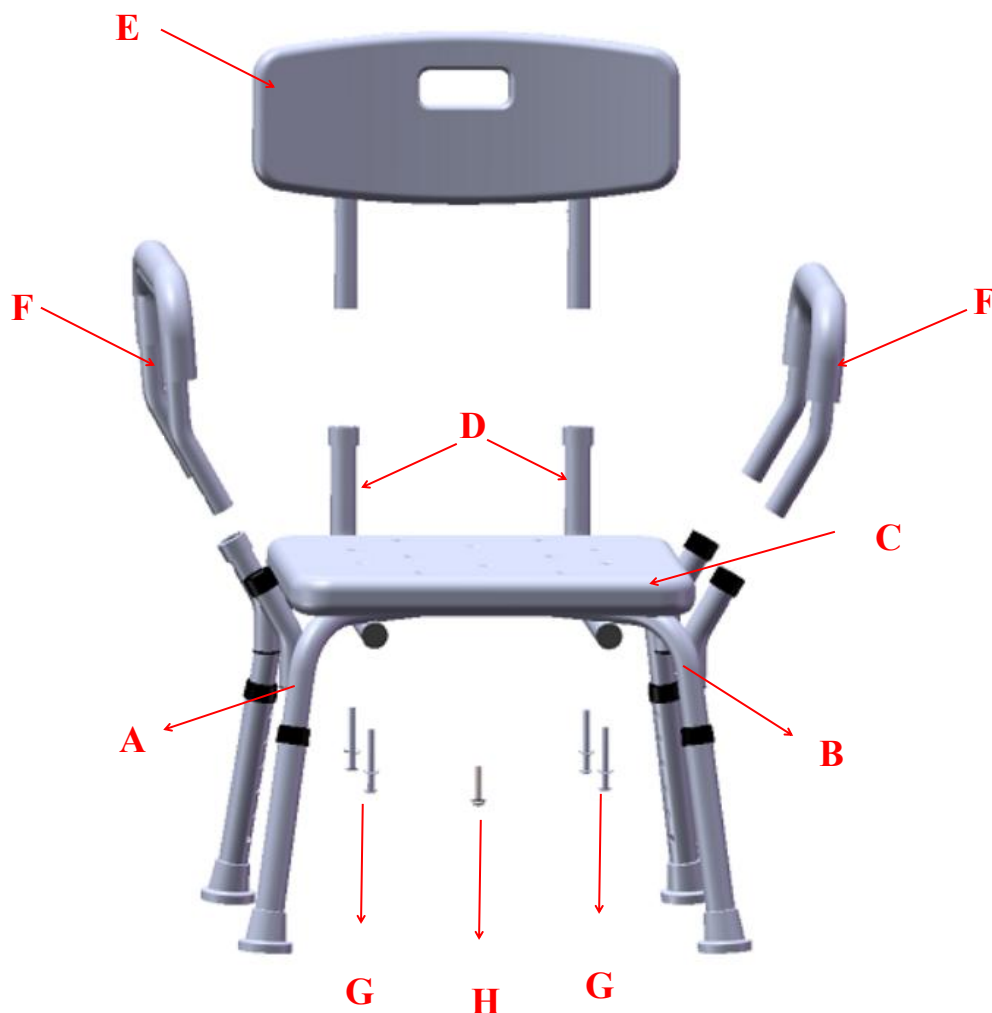


02 PRODUCT BRIEF :

The shower chair is made of aluminum alloy, the seat is made of PE hard material. All of the legs with 6 adjustable holes, non-slip rubber foot pads. Screw gaskets are made of SS 304.

This product is suitable for the elderly, pregnant women, disabled people and other people with limited mobility.




03 PART LIST

<p>A Upper tripod part</p>  <p>X1</p>	<p>B Down tripod part</p>  <p>X1</p>	<p>C Cushion</p>  <p>X1</p>
<p>D Seat support tube</p>  <p>X2</p>	<p>E Backrest</p>  <p>X1</p>	<p>F Armrest</p>  <p>X2</p>
<p>G Long screw with hardware washer</p>  <p>X4</p>	<p>H Short screw with hardware washer</p>  <p>X1</p>	<p>I Driver stool</p>  <p>X1</p>

04 HOW TO INSTALL?

Step1: Put A on C, then put B on A, and then put D on A and B. Lock the screws with hardware washer (G and H) in the five holes with driver stool (I).

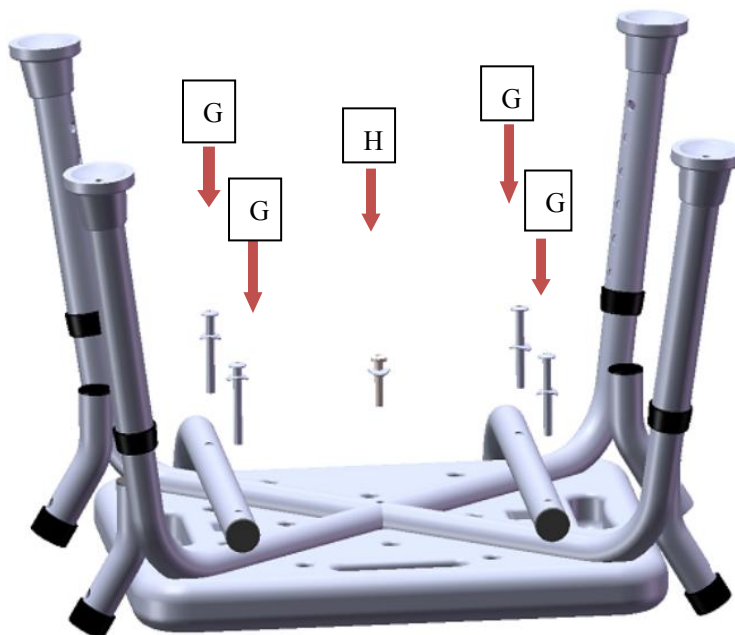

Step2: Install the backrest into the seat support tube by pressing the button.



Step3: Install the armrest into the corresponding pinball holes.



Inspection after assembling

1. Whether the bath chair is installed correctly, you can refer to the diagram in the instruction manual for comparison.
2. Check whether all accessories of the bath chair are fully assembled, and check whether there are any redundant accessories.
3. Check whether there is any looseness in the installation of the bathing chair. If there is any looseness, tighten the screws and nuts.
4. Check whether all components of the bath chair are skewed, and make appropriate adjustments

Product maintenance and maintenance methods:

1. Do not let hard objects scratch or scratch the surface of the product to prevent damage to the material.
2. For daily cleaning and maintenance, use a neutral synthetic detergent, wipe with a damp cloth, and then wipe clean with a dry cloth.
3. Avoid exposure to strong acid and strong alkali to prevent corrosion.

Storage conditions and methods

After packaging, the bath chair should be stored at a temperature not exceeding $-10\text{ }^{\circ}\text{C} \sim 40\text{ }^{\circ}\text{C}$, and a relative humidity not exceeding 80%. No corrosive gas and well-ventilated, clean room. During transportation, it should be protected from moisture, light, and heat sources.