

**Power Lift Massage Recliner**


# **Assemble Manual**

**Note:** This chair will come with 2 separate boxes, we ship them on the same day but the carrier may deliver them to your door on different days, please kindly wait for more 1-3 business days if you only receive one box.






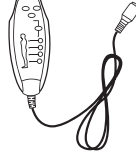




Please make sure you get 1 x Box(A) and 1 x Box (B) before assembly.

For any query, feel free to contact us, we'll offer you an ideal resolution within 24 hours.

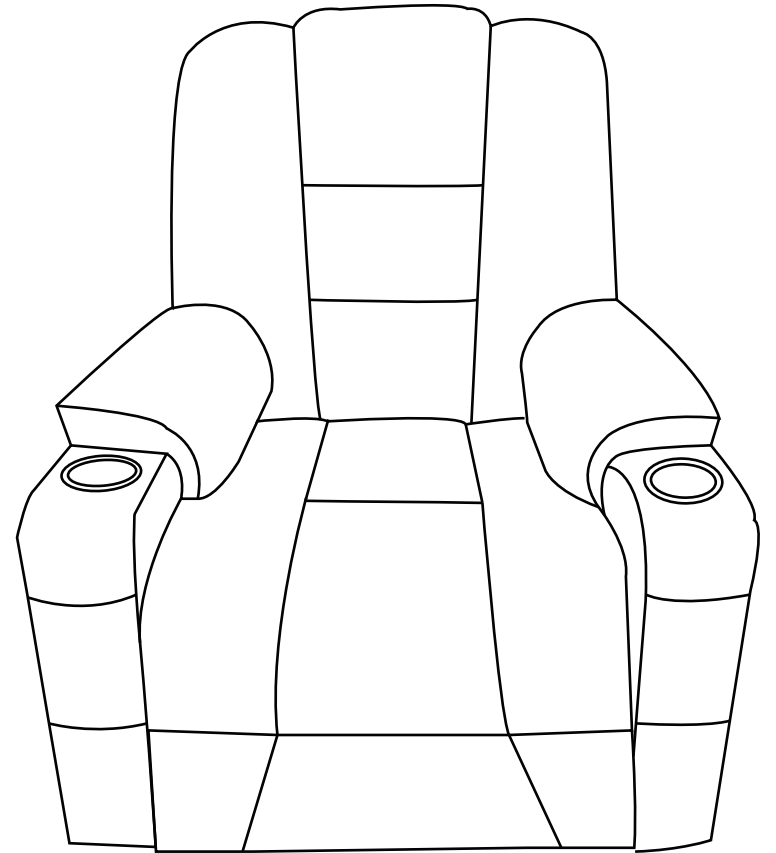**Direct email address:**  
**[aftersale2020@outlook.com](mailto:aftersale2020@outlook.com)**

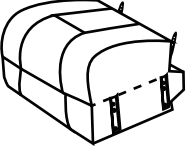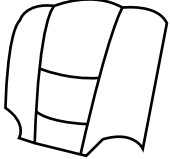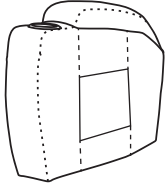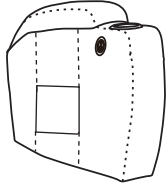


# Lift Chair Assembly Instruction-CH1005
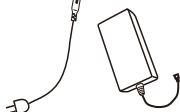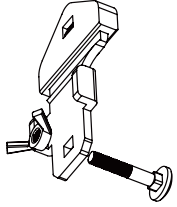
<b>A</b> 	<b>B</b> 	<b>C</b> 	<b>C1</b> 	<b>D</b> 	<b>E</b> 	<b>F</b> 	<b>G</b> 	<b>H</b> 	<b>I</b> 
Chair base	Backrest	Left armrest	Right armrest	Power cord	Massage Remote	Lift Remote	2 IN 1 Line	Power Source	Anti-fall bracket x2

**STEP 1**


With the USB Charging Port

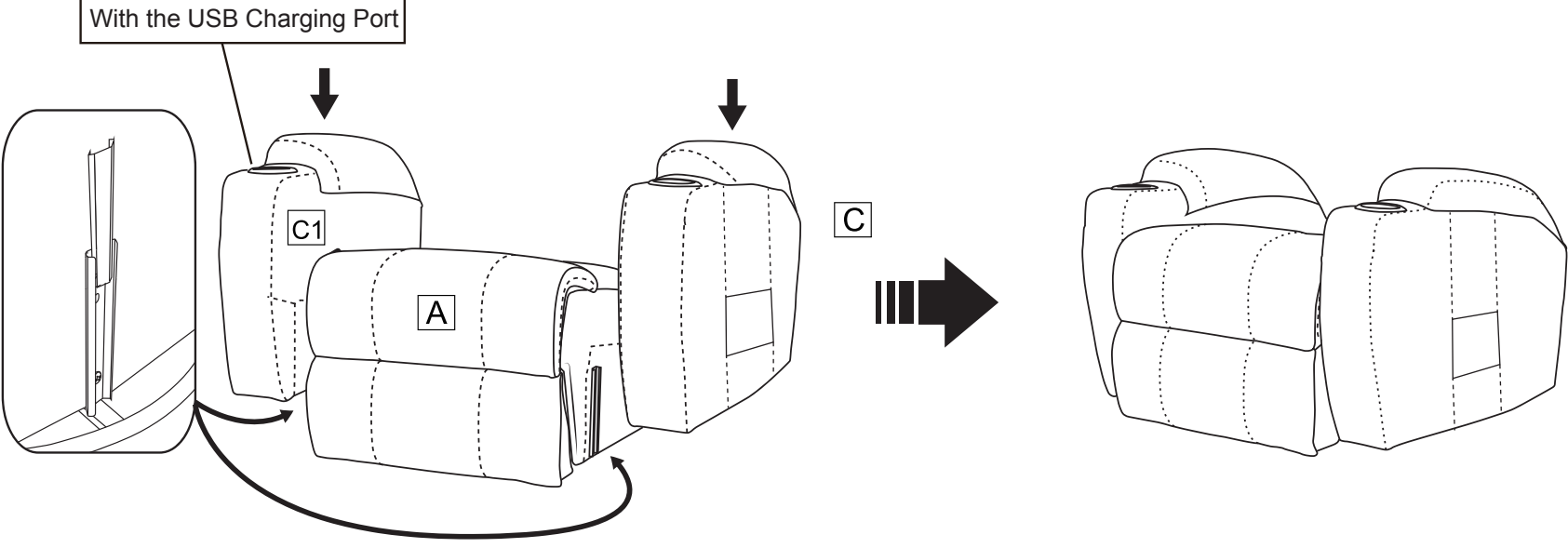

Insert two armrests to the chair base, make sure the slots on the armrests fully assembled to the bottom part.


# Lift Chair Assembly Instruction-CH1005

## STEP 2

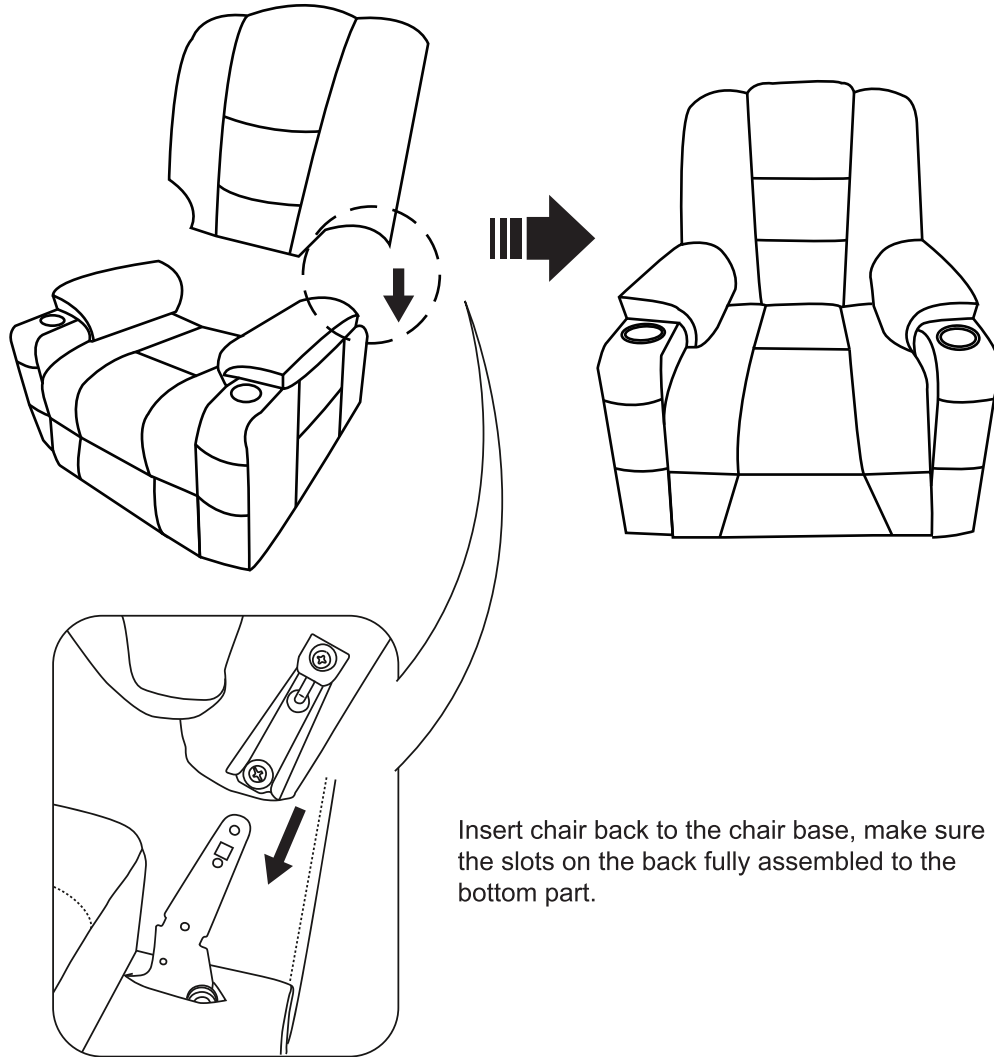

## STEP 3

Butterfly nut up, screw bolt down

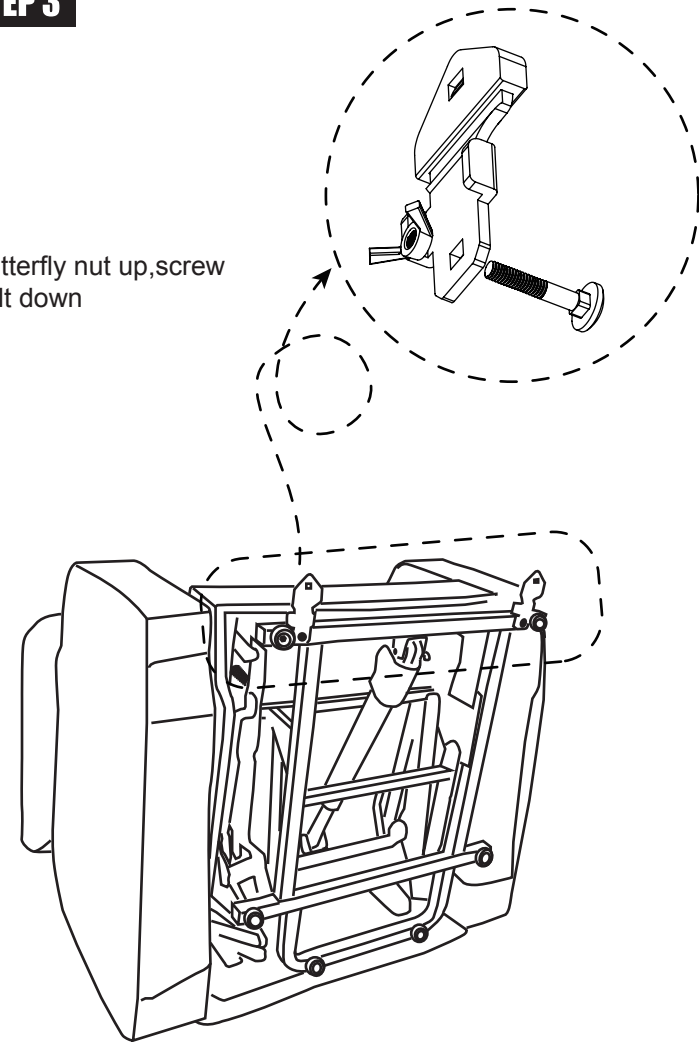


# Lift Chair Assembly Instruction-CH1005

## STEP 4


### NOTE :

- Wire 1: RED wire from the backrest connect to the RED wire from the base.
- Wire 2: WHITE wire from the backrest connect to the WHITE wire from the base.
- Wire 3: BLACK smaller wire from the backrest connect to the 2 IN 1 Line(G).
- Wire 4: BLACK bigger wire from the backrest connect to the Massage Remote(E).
- Wire 5: The other end of this 2 IN 1 line connect to the wire from the right armrest.

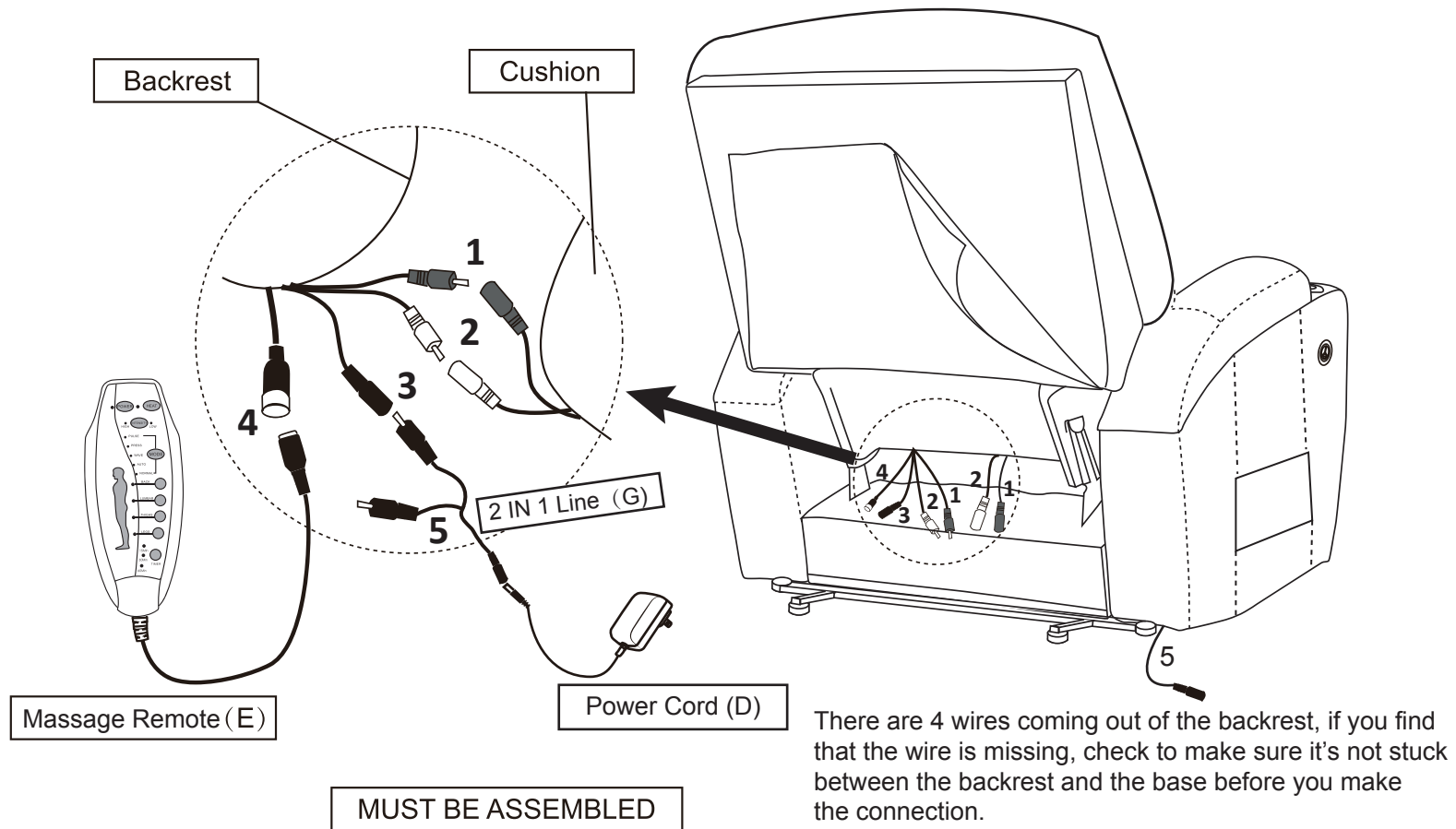

# Lift Chair Assembly Instruction-CH1005

## STEP 5

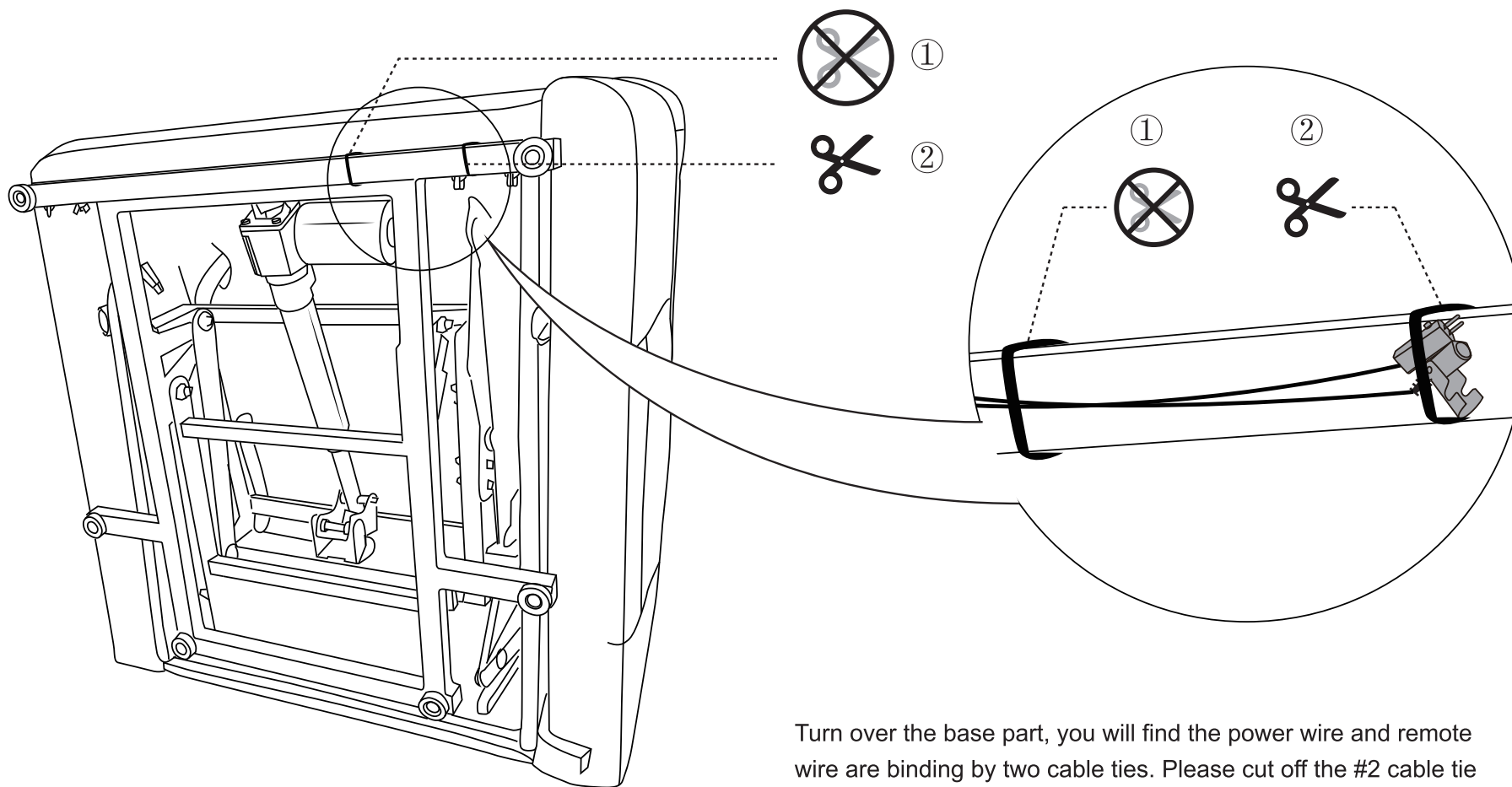

Turn over the base part, you will find the power wire and remote wire are binding by two cable ties. Please cut off the #2 cable tie and keep #1 to fix position.


# Lift Chair Assembly Instruction-CH1005

## STEP 6

**NOTE : Plug power cord and power source into outlet, then you can use all function.**




Wire 6: The wire from lift motor connect to the Lift Remote(F).

Wire 7: The wire from lift motor with 2 pins connect to the Power Source(H).




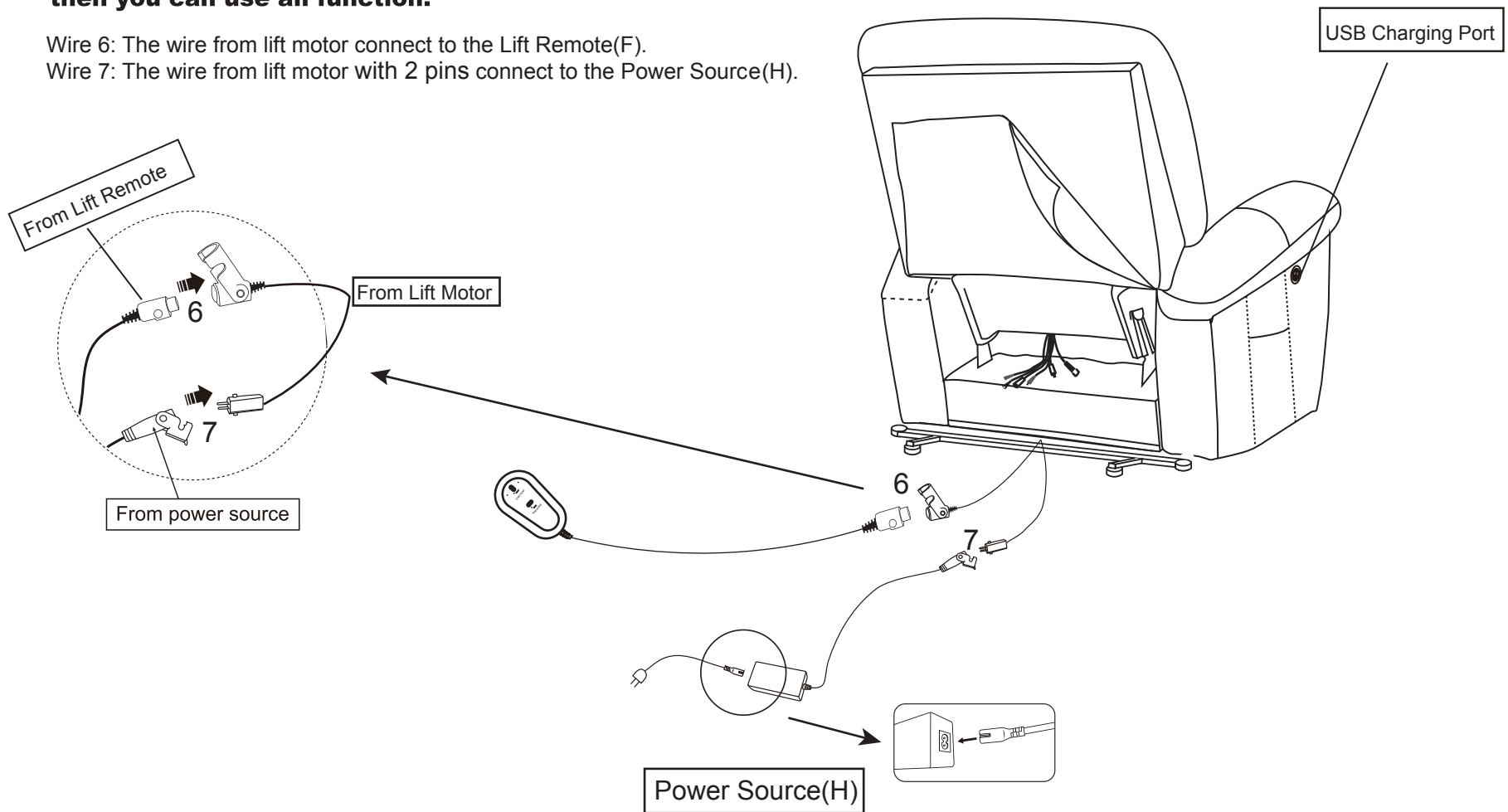


# Lift Chair Assembly Instruction-CH1005

## How to recline?

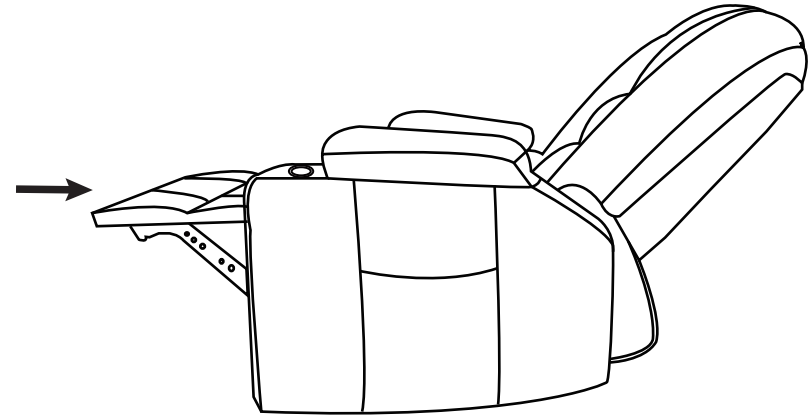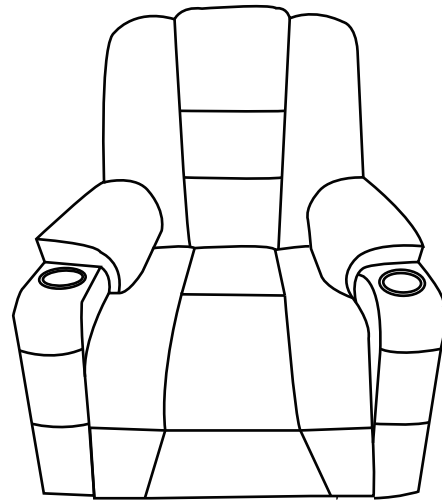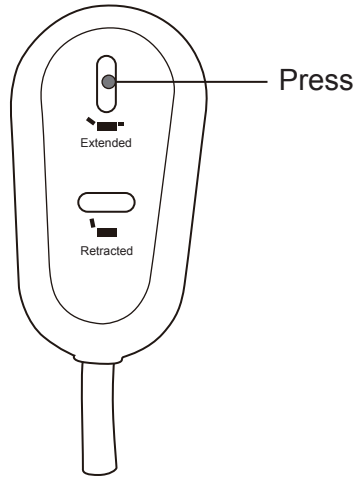

## How to get footrest latch down and stand?

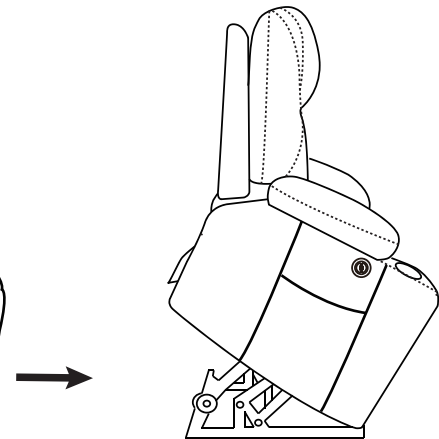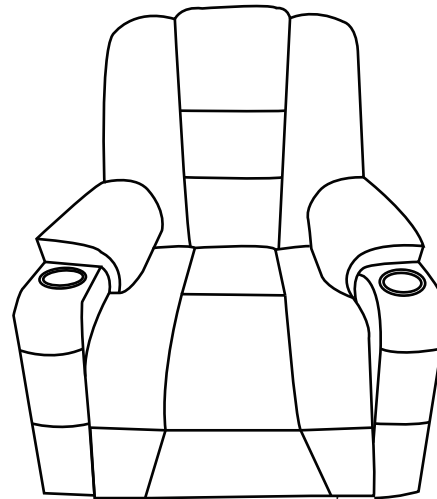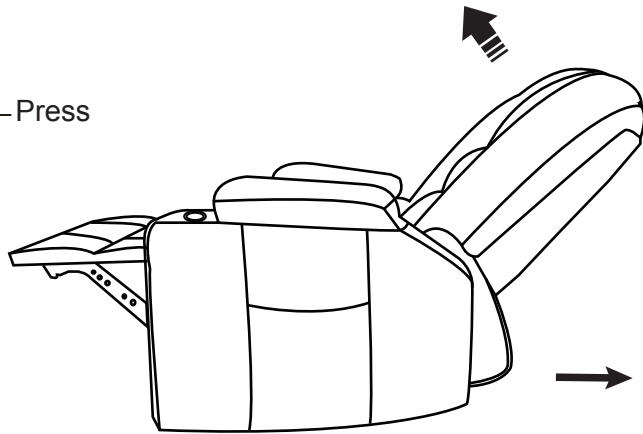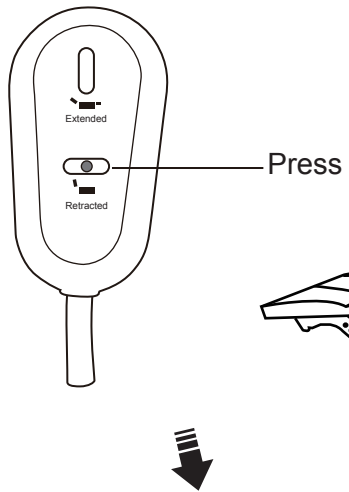


# Lift Chair Assembly Instruction-CH1005

How to disassemble the chair?


**Please note that you need to take apart the backrest from base before removing the arms.**

1. Pull up the buckles firstly while take the backrest out from the base.

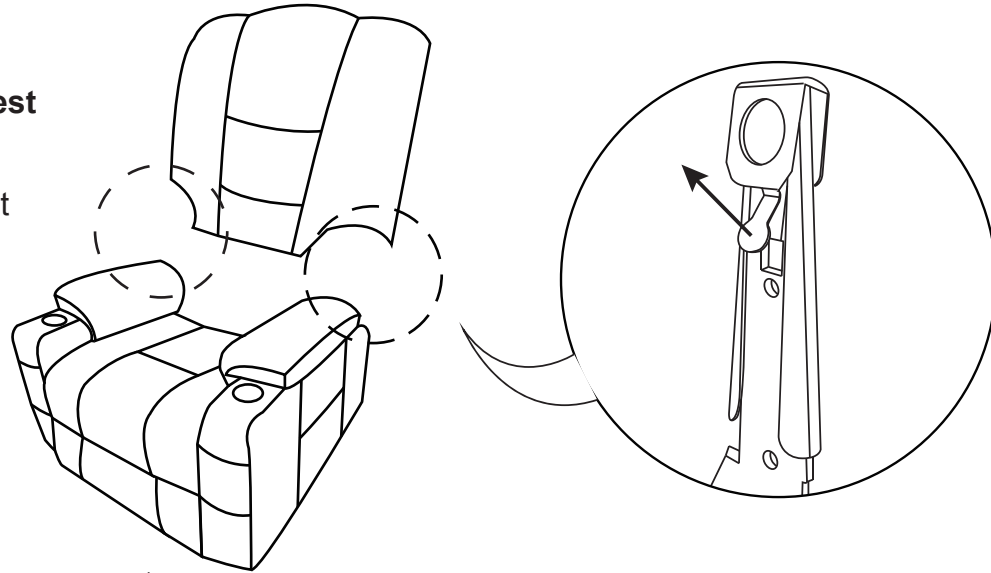


2. Then pull out the arms from the base.

**Note: Please grab both ends of the armrest with hands and then pull the armrest up vertically at the same time.**

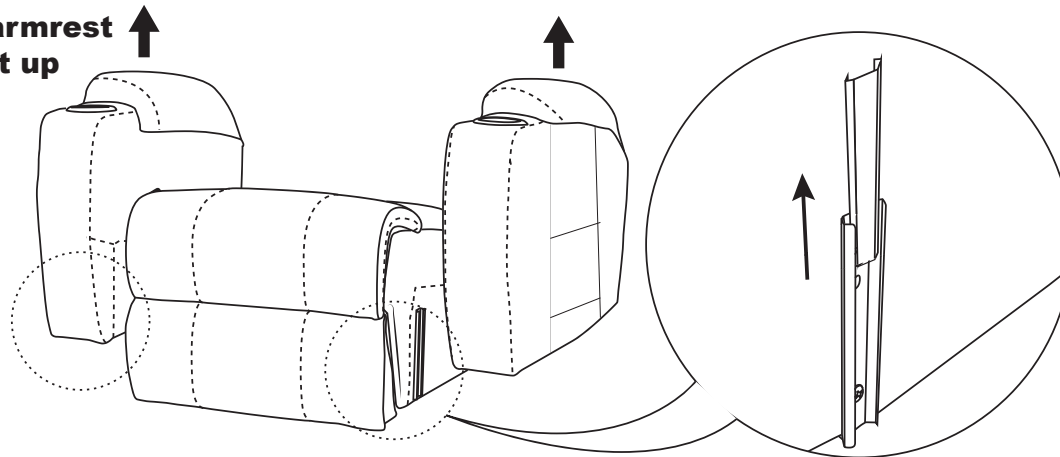

# Lift Chair Assembly Instruction-CH1005

## How to use remote control?

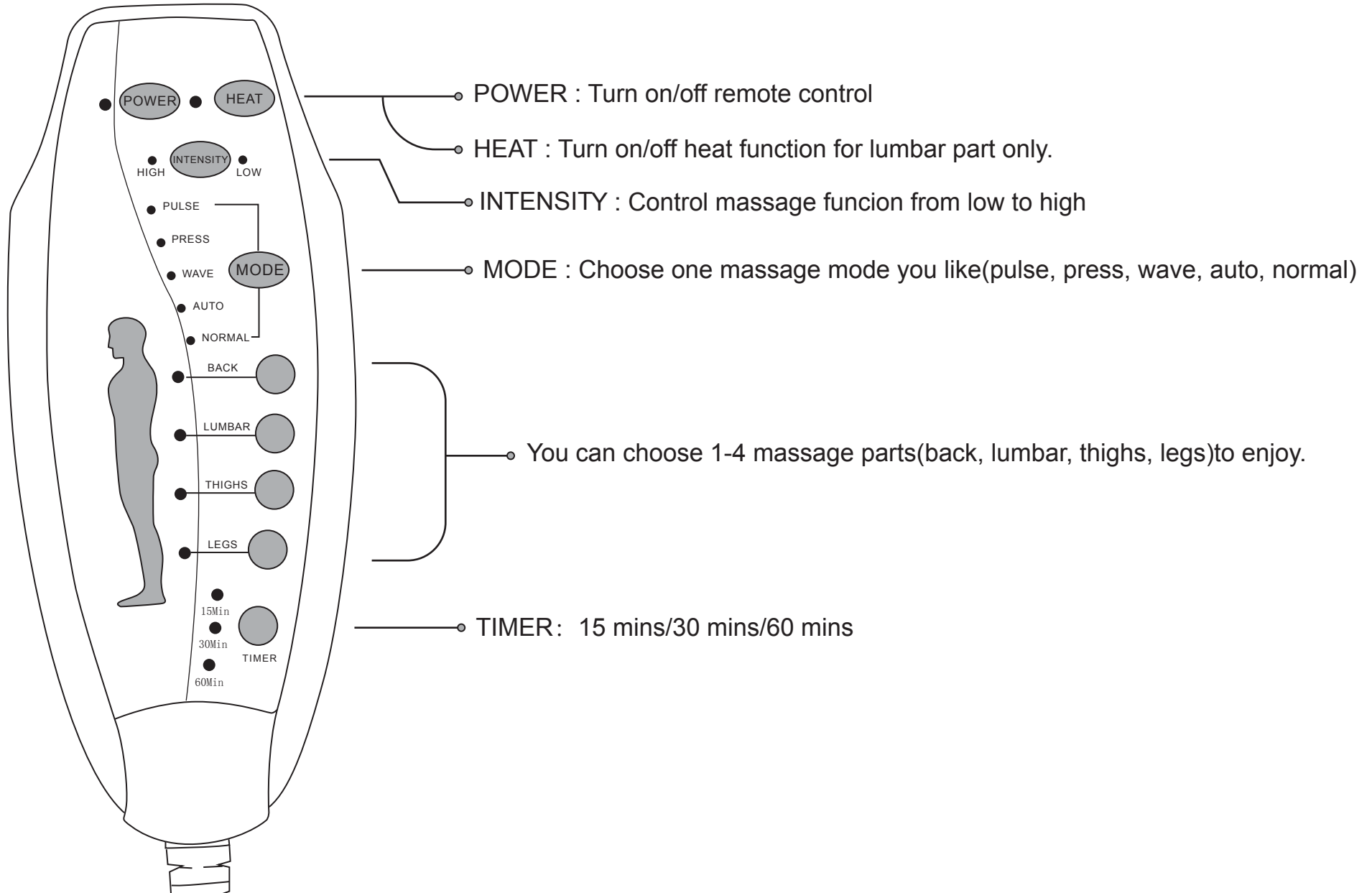

# Lift Chair Assembly Instruction-CH1005

## FAQ

1.Q: Missing parts when received.

A: Please check if you already received two boxes we shipped to you. Sometimes the two boxes will not arrive in same date, hope you can wait for 2-3 more days or contact us for more help.

If you already have two boxes, please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

2.Q: Some parts damaged when arrived.

A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

3.Q: The massage function does not work.

A: Please check if all the wires were connected correctly. If you have connected all wires correctly, please try to re-plug all the wires to avoid poor wiring contact problems.

If the massage still does not work after you have re-plugged all the wires, then check these 2 parts:

Part 1: The indicator lights on massage remote control are working ? YES or NO

Part 2: The USB ports are charging properly ? YES or NO

After confirming the answers to these 2 questions, contact us by email and let us know the answers, we will offer you a solution for this issue.

4. Q: The Heat function does not work or can't feel.

A: Please check all wires connection the same as question 3 first. If the remote also works well, you will need wait for 5-10 mins after you press the "HEAT" button. The heating function only works at lumbar position and the heat temperature isn't very hot.

5.Q: The chair can not lift and recline.

A: Please re-plug all wiring connected to the motor to ensure they were properly connected.

If the chair still does not work after you have re-plugged all the wires, then please check if the indicator lights on the Lift remote control and power source (Part G) are on or not, and contact us by email and let us know the answer, we will offer you a solution for this issue.