


Scan the QR code to watch the installation video.




Adjusting height: Press the ball bearing on the leg tube to extend or retract it. The tube will pop out from the corresponding hole.

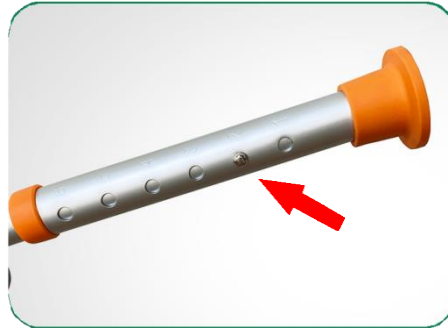

Anti-slip footpads: Provide stability and grip on most surfaces.

Vacuum suction cups: Prevent tipping on smooth, flat, seamless surfaces.

Instruction

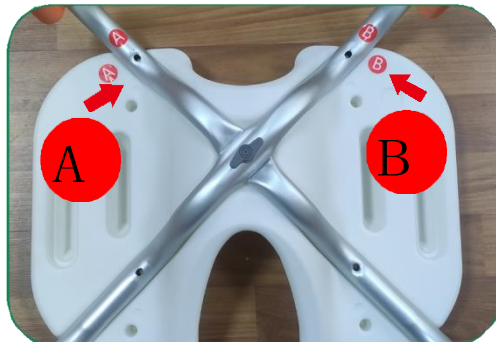

1) As shown: Position tubes A and B into their corresponding slots. Tube A goes below, tube B above. Match identical letters. (Secure tubes A and B at their midpoints using orange medium screws.)

2) As shown: Attach the two L-shaped backrest tubes to the outer holes of tubes A and B. Then secure the backrest tubes with long screws. (Do not use long screws before installing the backrest tubes.)



3) As shown: Secure the backrest tubes with 4 short screws. First, loosely secure them with the short screws. After all 4 screws are in place, tighten them individually.



As shown: Take one armrest. Press the two side balls on the armrest tube and insert the armrest tube.

Completed