Installation Instruction

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Installation
Diagram:

1. Fully folded state
2. Pull the bottom frame apart front and back
3. Insert the backrest and armrests into the tubes, and then lock the clips
4. Installation complete

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Folding
Diagram:

1. Lift the product and fold it in half front to back
2. Folding complete

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Important Details:

1. Height adjustment is achieved by manipulating the four sleeves as shown below. Press the ball bearing and slide the sleeve into different holes to select various height settings.
2. The armrests feature a flip-up design for easy cleaning of the sides.
3. For the cold-weather pad, peel off the protective film on the back and attach it directly to the seat board instead of placing it on top. Im on the back and attach it directly to the seat board instead of placing it on top.

Safety Precautions:

1. Thoroughly read and understand this manual before installing the bath chair.
2. Ensure the floor is level and all four legs of the bath chair frame make stable contact with the ground before use.
3. Verify that the height adjustment ball bearings are fully engaged and aligned in the same position before use.
4. Inspect the frame and all components for shipping damage before installation. Cease assembly if any damage is detected.
5. Carefully inspect the non-slip footpads for tears or other damage. If any is found, contact us immediately for replacement parts.
6. Users with physical disabilities should install and use this product with assistance from family members.