**SuperTrapeze™**

STP-S  
STP-AK  
(DC222 - RevA)

[healthcraftproducts.com](http://healthcraftproducts.com)

## ⚠️ WARNING FOR BED SAFETY

### ⚠️ WARNING

#### ENTRAPMENT HAZARD


There is a risk of entrapment associated with all bedside mobility aids. Entrapment occurs when an individual becomes wedged between a bed rail and another object (example: mattress). Entrapment can result in serious injury and death. Read and understand the enclosed Entrapment Prevention Guidelines document. See the additional Entrapment Prevention Guidelines:

- [www.fda.gov/bedsafety](http://www.fda.gov/bedsafety)
- <https://www.cpsc.gov/safety-education/safety-guides/furniture-furnishings-and-decorations/adult-bed-rails>


PRIOR TO INSTALLATION, review all of the product specifications and instructions to ensure that it is safe to use with your bed and mattress. **Stop using immediately if product or bed shifts from its original position, if product is damaged, broken or missing parts.**

PRIOR TO EVERY USE, ensure the pole is installed correctly in one of the specified locations:

(a) No gap between the pole and the side of the mattress.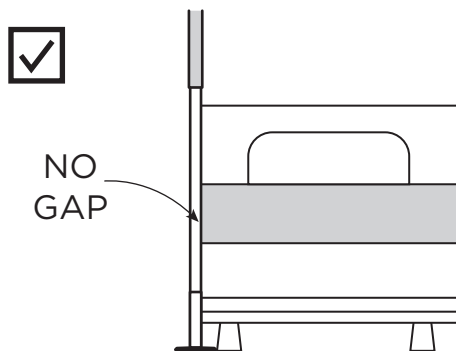

(b) A gap of at least 12.5"/320mm between the pole and the side of the mattress.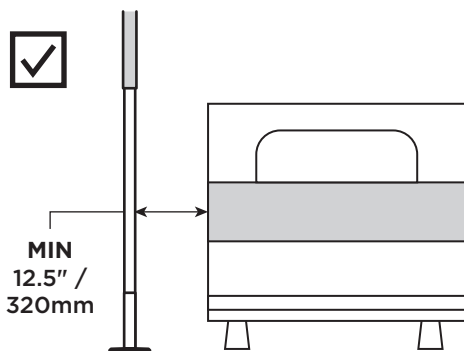

### ⚠️ WARNING


#### ENTRAPMENT HAZARD

NEVER use product without properly securing beside the bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

### ⚠️ WARNING

#### ENTRAPMENT HAZARD

(a) There is an increased risk of entrapment when using a safety product beside a bed with a soft mattress. Test mattress by applying 12lbs (5.4kg) of weight on the edge of the mattress. If the mattress can compress by 2" (51cm), do not use this product.

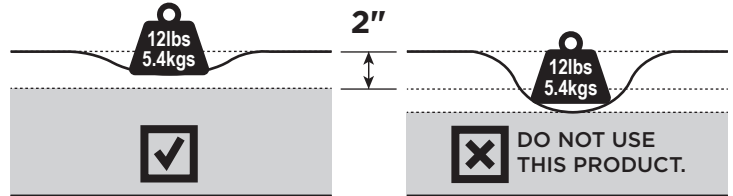(b) If product is used in a care facility, follow the installations policies of the institutions. If this product does not comply with those policies, do not install this product.

(c) When changing bed linens, always confirm that product is installed correctly as per installation instructions. Re-position to eliminate a gap between product and side of mattress.

For questions about product safety or installation, or in case of a serious event with this product, contact a the HealthCraft at 1-888-619-9992 immediately. In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at this website: <https://www.saferproducts.gov/> or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

### ⚠️ WARNING

#### SUFFOCATION, STRANGULATION, ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position gaps can occur which can entrap and cause fatality. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- NEVER use unless product is tight against mattress, without gaps, and at least 12-1/2 in. from headboard and footboard.
- NEVER use with children.
- NEVER use on toddler, bunk, water, electrical or inflatable beds, or on beds with mattress toppers or soft compressible pads.

This product is under the jurisdiction of the FDA; to report problems or injuries, call 1-800-332-1088.

### ⚠️ WARNING

#### ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured beside the bed. If product can move away from bed or mattress, it can lead to entrapment and death.

# English

## OVERVIEW

Thank you for choosing the SuperPole System from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your SuperPole System is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your SuperPole System installed by a qualified contractor.

The SuperPole System is intended for moderate vertical load bearing to provide sitting and standing support for people with disability. The floor to ceiling range and maximum user weight dependent on model (see Figure E for details and product / accessory compatibility chart). Product is not intended to support full body weight. The SuperPole System is not to be used in any other way than described above.

The SuperPole is a high quality, floor to ceiling safety pole. The SuperPole must be installed between a structural floor and ceiling by turning a jackscrew at the bottom of the pole. The pressure created by the jackscrew is strong enough for the SuperPole to support the user, yet it does not require drilling holes into the floor or ceiling. This is ideal for relocating or removing the pole. NOTE: The top plate has two holes that may be used to fix the plate directly to the ceiling joist if necessary. See Figure C for installation warnings.

The SuperTrapeze is an add-on to the SuperPole that increases its functionality for in-bed support. The SuperTrapeze features ergonomic offset grips that allow the user to naturally transfer from a lying position to a seated position in bed. It is important to set proper height of the SuperTrapeze handle to optimize the user's range of motion. Additionally, it can easily be stored out of the way when not being used.

## LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.

Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. 1-888-619-9992. Patent Pending. © 2026 HealthCraft Products Inc.

## DELIVERY CONTENTS - SUPERPOLE

### FIGURE A.

- |                                      |                  |
|--------------------------------------|------------------|
| 1. Pole                              | 8. Top Plate     |
| 2. Pole Grip, 20" length             | 9. Base Plate    |
| 3. Cover Sleeve (with warning label) | 10. Nylon Washer |
| 4. Threaded Collar                   | 11. Cotter Pin   |
| 5. Jam Nut                           | 12. Clevis Pin   |
| 6. Extension Screw                   | 13. C-Clip       |
| 7. Plastic Washer                    |                  |

## DELIVERY CONTENTS - SUPERTRAPEZE

### FIGURE A.

- |                                      |                            |
|--------------------------------------|----------------------------|
| 15. Spring Ring (with warning label) | 17. 5/16-24 Set Screw (x4) |
| 24. Support Arm                      | 21. 5/32" Hex Key          |
| 25. Rectangular Plug                 |                            |
| 26. Plastic Buckle                   |                            |
| 27. Safety Strap                     |                            |
| 28. Trapeze Handle                   |                            |
| 29. Top Grip, 6" length              |                            |
| 30. Bottom Grip, 10" length          |                            |

## TECHNICAL DATA

### FIGURE B.

## INSTALLATION WARNINGS

### FIGURE C.

1. Use caution while maneuvering pole during installation.
2. Standard pole can be installed directly under 1 ceiling joist.
3. Standard pole can be installed directly under 2 ceiling joists.
4. Heavy Duty (HD) pole must be secured to ceiling joist (s).
5. If subject to heavy usage, pole must be secured to ceiling joist (s).
6. CAUTION - Damage may occur if extension screw is overtightened.
7. CAUTION - Do not install if the structure above ceiling is unknown.
8. CAUTION - Do not install on loose floor coverings.
9. CAUTION - Do not install in rooms with radiant ceiling heat.
10. CAUTION - Do not install on angled ceilings.
11. CAUTION - Do not install pole where it is directly exposed to water.

## SUGGESTED PLACEMENT

### FIGURE D.

1. BEDROOM - Locate pole at least 12.5"/320mm from headboard or footboard, adjacent to waistline and either (a) against the mattress - no gaps, or (b) 12.5"/320mm away from mattress.
2. BATHTUB ONLY - Locate pole base against tub, approximately half way along its length.
3. SEATING (TOILET or CHAIR) - To provide clearance when standing, locate pole 3-10" / 76mm-254mm forward of the knees, and 2-6" / 51mm-154mm to the side of knees when sitting.

## COMPATIBILITY CHART

### FIGURE E.

## INSTALLATION INSTRUCTIONS

### FIGURE F.

1. Slide support arm onto top of pole. Ensure correct orientation; safety strap should extend towards pole grip. Rest support arm on top of pole grip.
2. Connect top plate to pole and align holes.
3. Insert clevis pin and nylon washer (on opposite side).
4. Install cotter pin into clevis pin to secure the connection.
5. Place base plate on floor at intended location, and tilt pole upright.
6. When upright, insert the pole into the base plate hole.
7. a) Lift cover sleeve and tape to pole, then  
b) Insert shaft of screwdriver through hole of extension screw.
8. Hold pole and turn screwdriver clockwise to raise the pole.
9. Ensure top plate contacts the ceiling squarely.
10. Continue turning screwdriver approx. 2-5 turns until pole is secure.
11. Turn jam nut counter-clockwise until it's tight against bottom of pole.
12. Crank screwdriver and jam nut counter-clockwise at the same time until the jam nut is locked.
13. Cover exposed hole on the extension screw with c-clip.
14. a) Lower cover sleeve to conceal screw.  
b) Inspect pole monthly - tighten extension screw if needed.
15. a) Rotate support arm over bed.  
b) Hold the trapeze handle under the support arm, such that the top grip of the handle is closest to the head of the bed.
16. Rotate the end of the safety strap 180° and slide onto the top grip of the trapeze handle until it reaches the top bend of the trapeze handle.
17. Position support arm such that the trapeze handle hangs directly over the user's hands (on thighs) while they are laying in bed.
18. Raise support arm such that the distance between the floor and the top of the support arm is approx 84"/2134mm.
19. Slide spring ring upwards on support arm to expose threaded holes.
20. Install support arm to pole by tightening 3 set-screws with hex key. The set-screws MUST be tightened until the back of the screw is flush with the collar or injury may result.
21. Slide spring ring downwards on the castellated collar to retain set screws.
22. Adjust length of safety strap such that the user can comfortably grasp the bottom rail of the trapeze handle when laying down.

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