


LiftPlus™ Startup Instructions

NOTE: Connect the battery and charge the LiftPlus™ for six hours before using it the first time. If the LiftPlus battery completely discharges, recharging will take 72 hours.

When not in use, it is recommended to keep the LiftPlus plugged in to maintain a maximum charge. For more information on charging, please refer to the Owners Manual.

Step 1 - Folding the Unit

1. Set and lock the brakes.
2. Grasp the folding latch handle and rotate it slightly forward to remove some tension on the red latch. Press the red latch release lever up with your thumb.
3. Allow the folding latch handle to spring upwards.
4. Pull up on the folding latch handle with one hand and grasp the column handle with the other hand to guide the column down. The column will slowly fall into the folded position.
5. Push the column as far down as possible.
6. Push down on the folding latch handle until the folding latch reaches the bottom of its travel.
7. Rotate the folding latch handle downward into its locked position. There will be a loud click as the latch release lever engages.
8. Tug the folding latch handle upwards to be sure that it is properly locked.

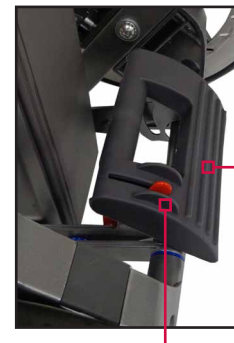


Folding
Latch Handle

Latch Release Lever




Step 2 - Connecting Battery Terminals

1. Slide the battery cover down and connect the battery. Connect the red wire to the red tab and the black wire to the black tab.
2. Replace battery cover.

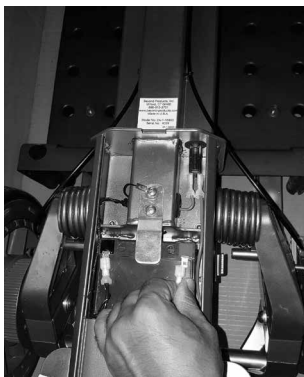


Step 3 - Unfolding the Unit

1. Set and lock the brakes.
2. Grasp the folding latch handle and rotate it slightly forward to remove some tension on the red latch. Press the red latch release lever up with your thumb.
3. Allow the folding latch handle to rotate toward you and then pull it slightly upward.
4. Squat down beside the unit and grasp the column handle.
5. Pull up on the column handle and rotate the column as far back as possible.
6. Push down on the folding latch handle until the folding latch reaches the bottom of its travel.
7. Rotate the folding latch handle downward into its locked position. There will be a loud click as the latch release lever engages.
8. To ensure unit is locked, pull upward on the folding latch handle (it should not move). Push forward on the column handle. The column should not rotate forward at all (see pictures below).
9. Charge for six hours before using the LiftPlus for the first time.

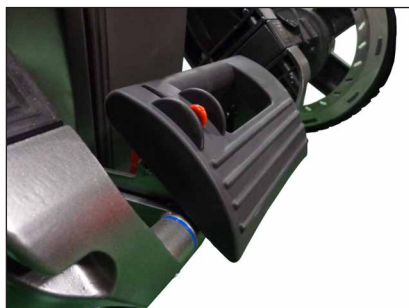

Locked Properly

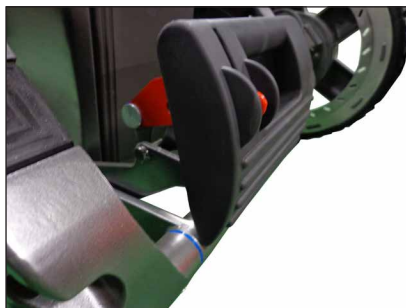

Not Locked Properly



Not Locked Properly

www.magliner.com

Magline, Inc., 1205 West Cedar Street, Standish, MI 48658 • (800) 624-5463 • (989) 512-1000