









# ASSEMBLY INSTRUCTIONS 13012

## —— Electric Recliner Chair ——

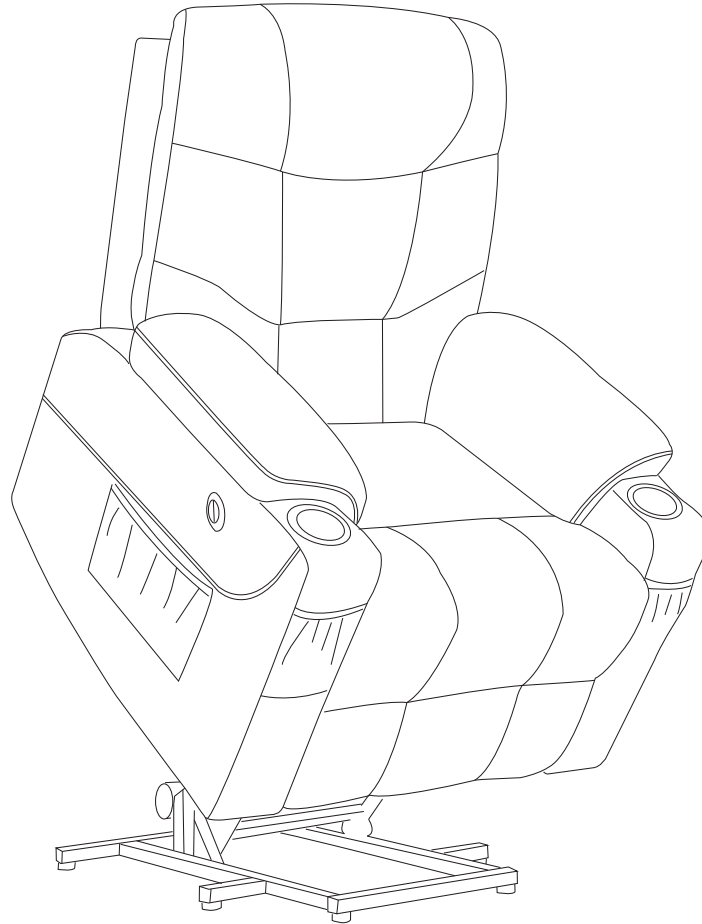


Thank you for choosing our product. There will be two boxes for this product, sometimes the two boxes will not arrive in same date, kindly wait and you will get it soon.

# Power Lift Chair Assembly Instruction\_13012
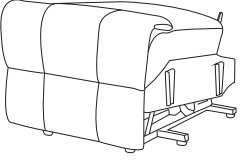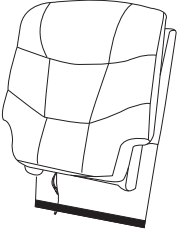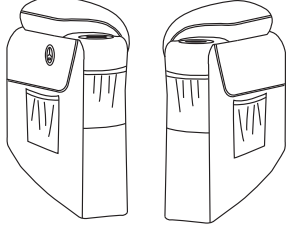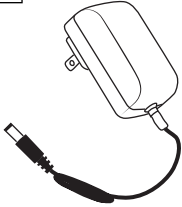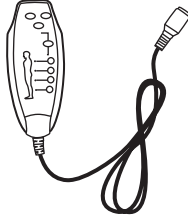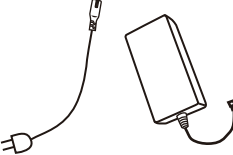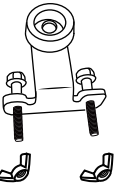
|                                                                                              |                                                                                               |                                                                                                |                                                                                                 |                                                                                                 |                                                                                                 |                                                                                                 |
|----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|
| <b>A</b><br> | <b>B</b><br> | <b>C</b><br> | <b>D</b><br> | <b>E</b><br> | <b>F</b><br> | <b>G</b><br> |
| Chair base                                                                                   | Backrest                                                                                      | Left and right armrest                                                                         | Power cord                                                                                      | Remote control                                                                                  | Power Source                                                                                    | Anti-fall bracket x2                                                                            |

You will received totally 2 boxes, please make sure you get 1x (A) box and 1x (B) box before assembly.  
Just feel free to chat us if you have any problem, we will try our best to help you solve the issue.


## STEP 1

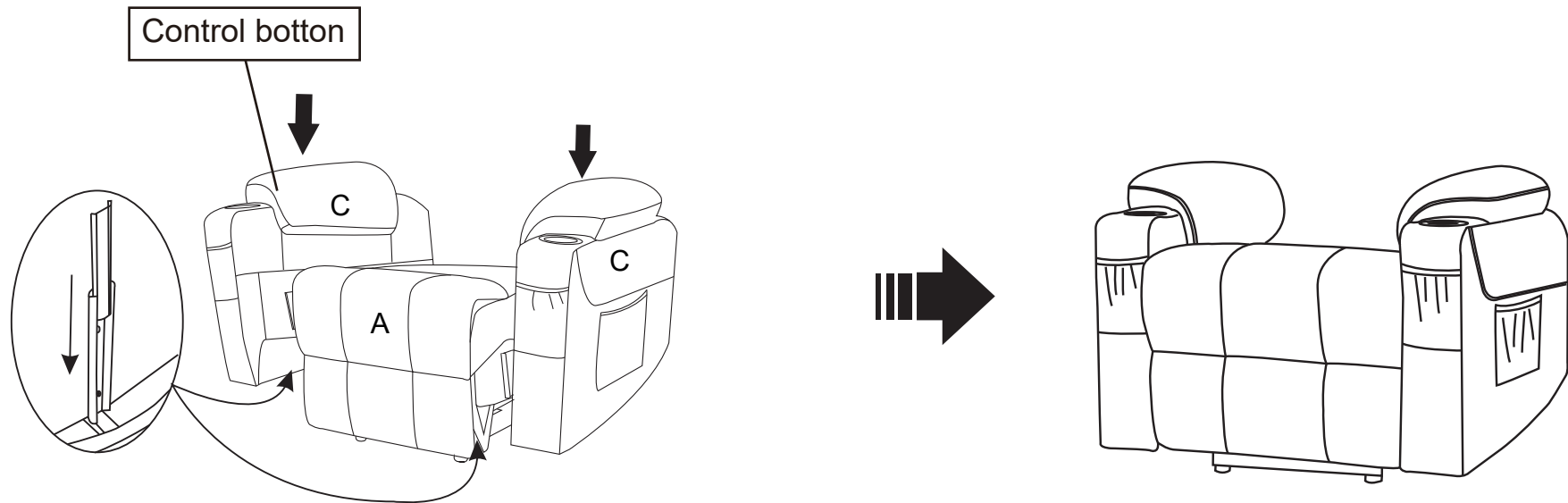


Insert two armrests to the chair base, make sure the slots on the armrests fully assembled to the bottom part.

# Power Lift Chair Assembly Instruction\_13012

## STEP 2

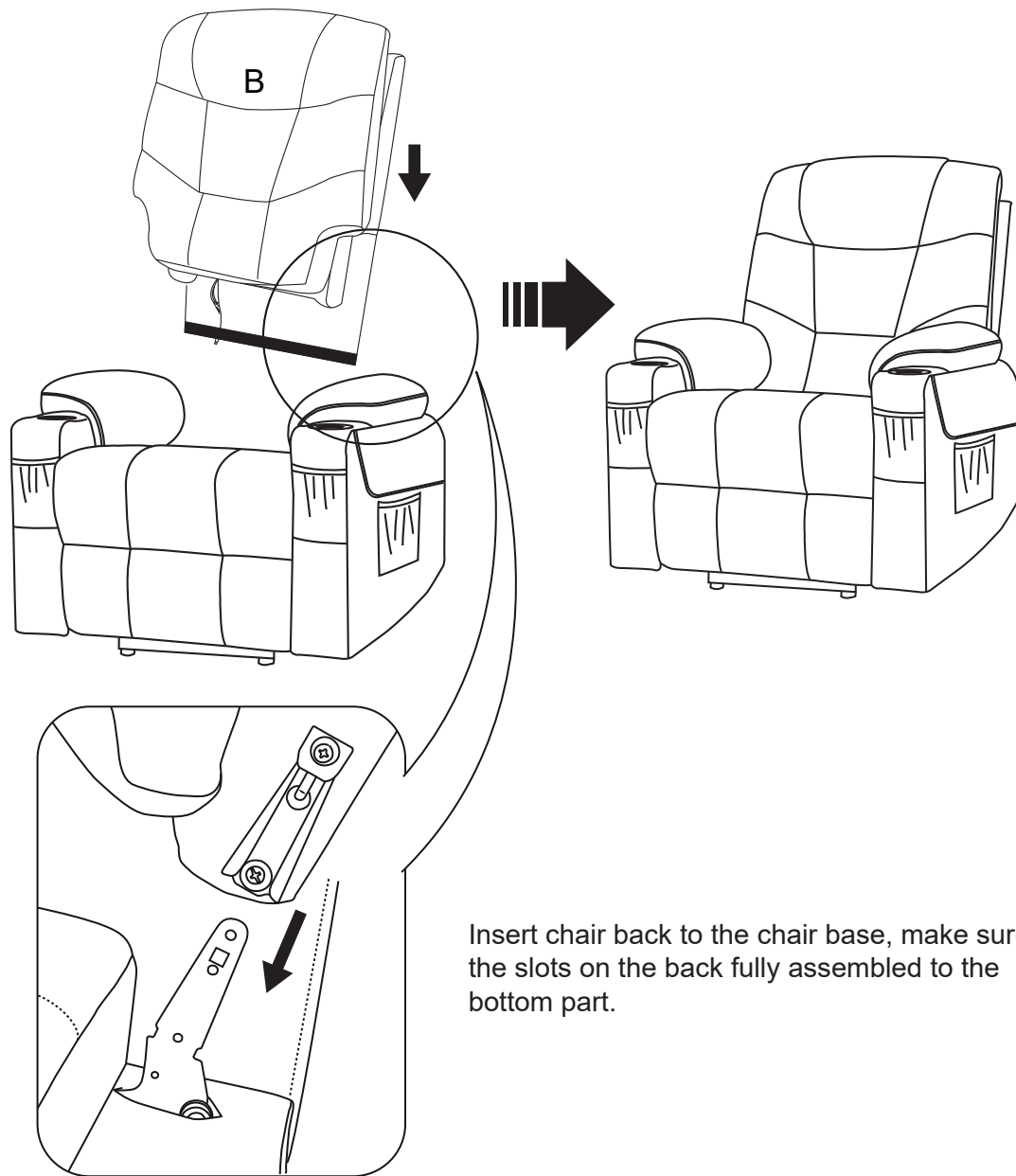

## STEP 3

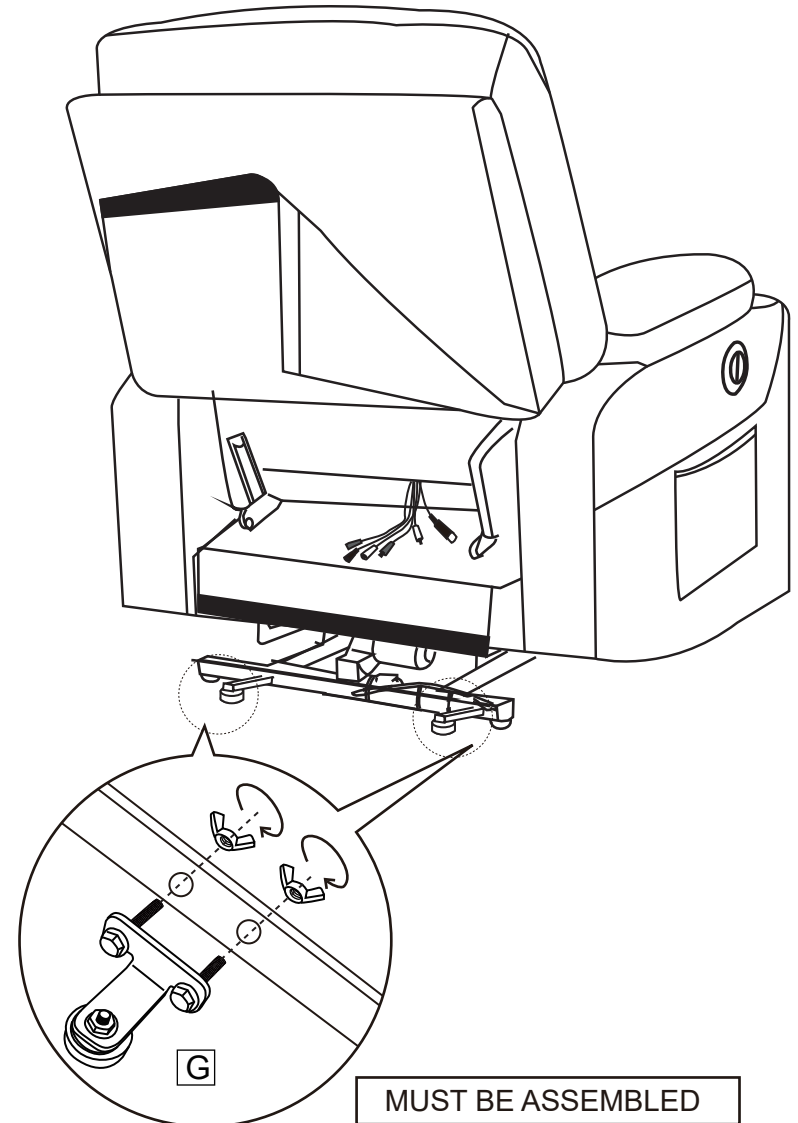

# Power Lift Chair Assembly Instruction\_13012


## STEP 4

**NOTE : Two same color wires connect.**

No.1 and No.2 wire connect with the backrest and chair base.


No.3 wire connects with backrest and power cord(D).

No.4 wire connects with backrest and remote control(E).

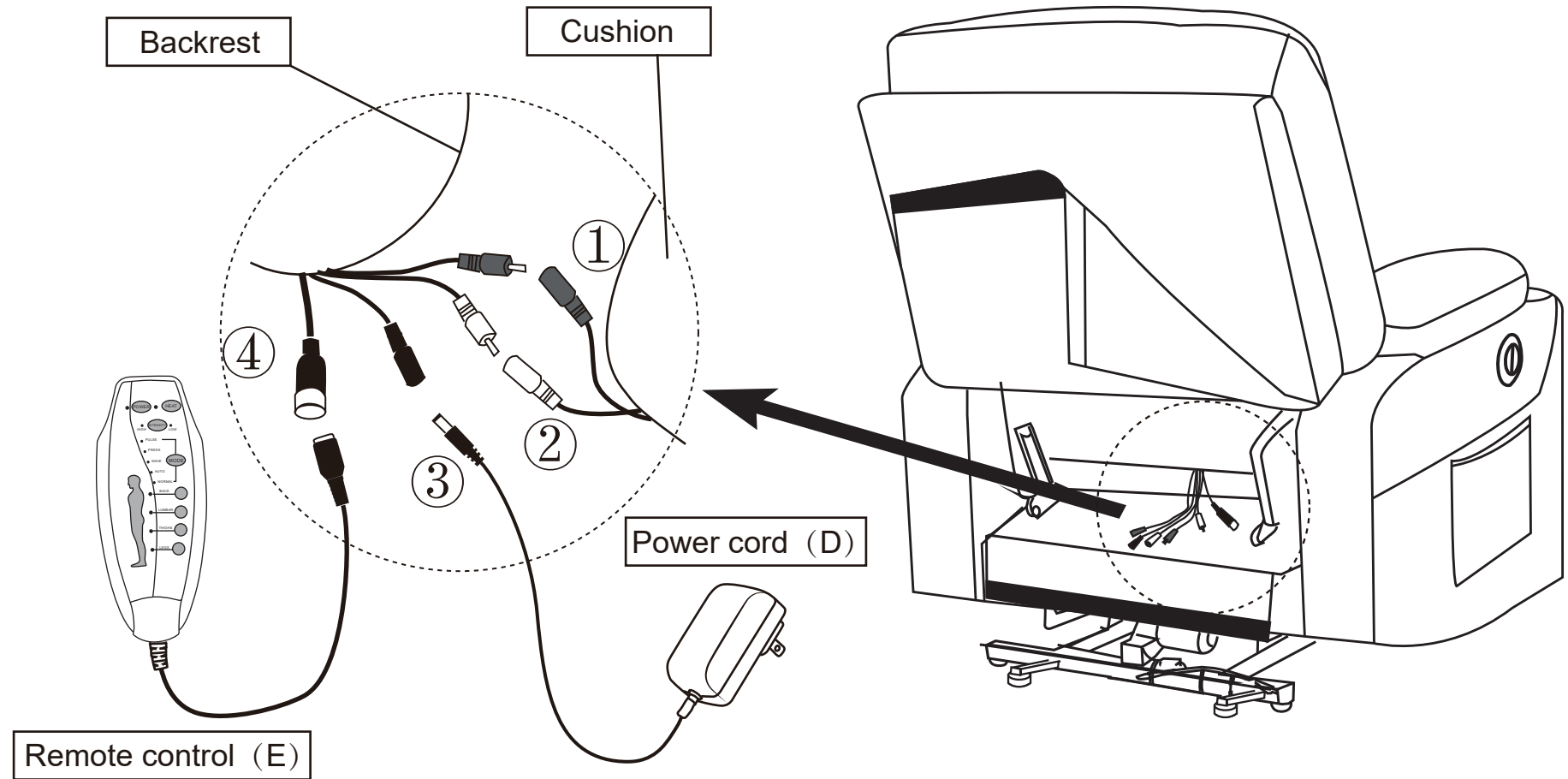

# Power Lift Chair Assembly Instruction\_13012

## STEP 5

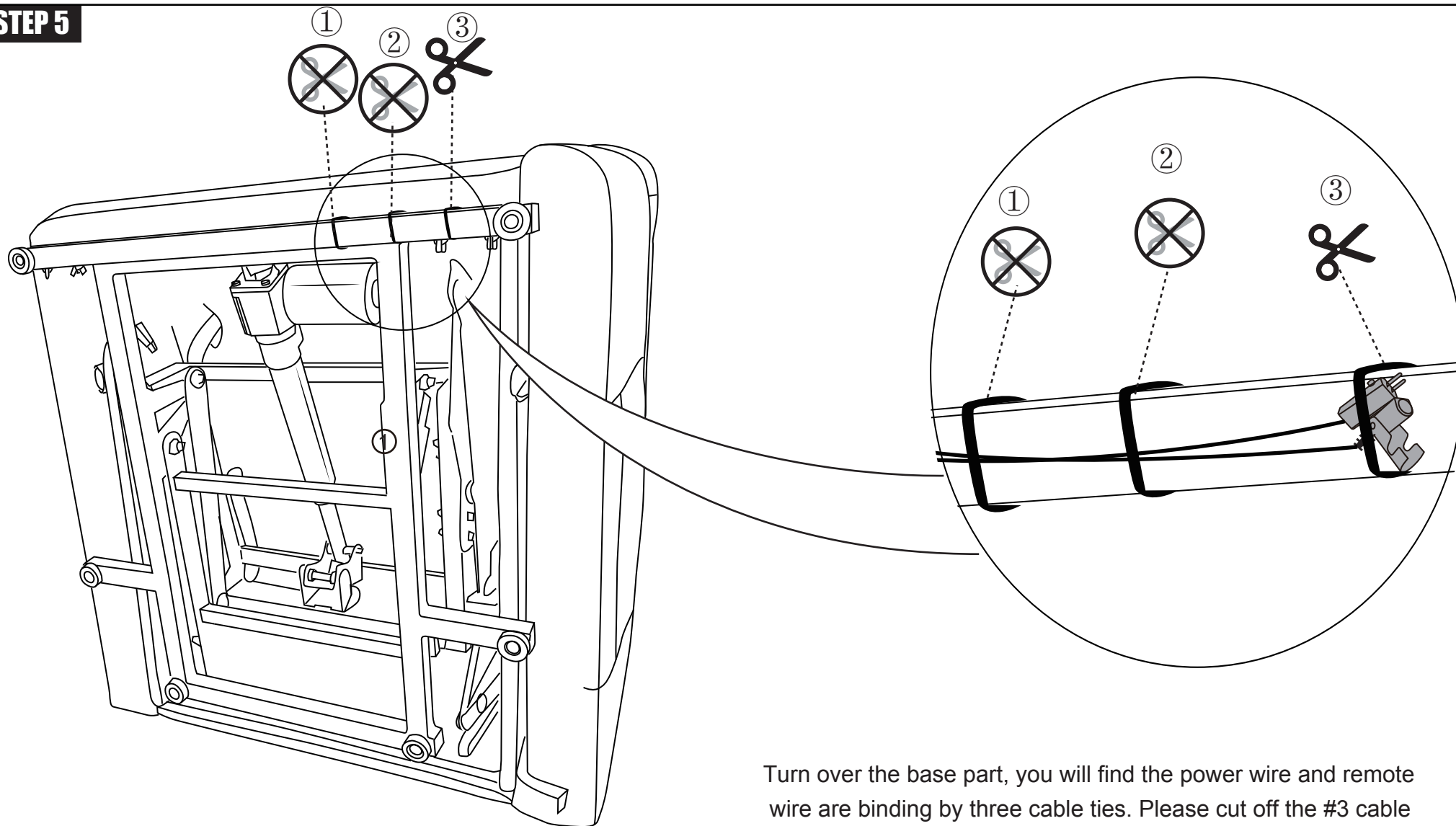

Turn over the base part, you will find the power wire and remote wire are binding by three cable ties. Please cut off the #3 cable tie and keep #1 and #2 to fix position.


# Power Lift Chair Assembly Instruction\_13012

## STEP 6

**NOTE : Plug power cord and power source into outlet, then you can use all function.**


No.5 wire connect the motor and armrest with control panel.

No.6 wire connect the motor and power source (F).

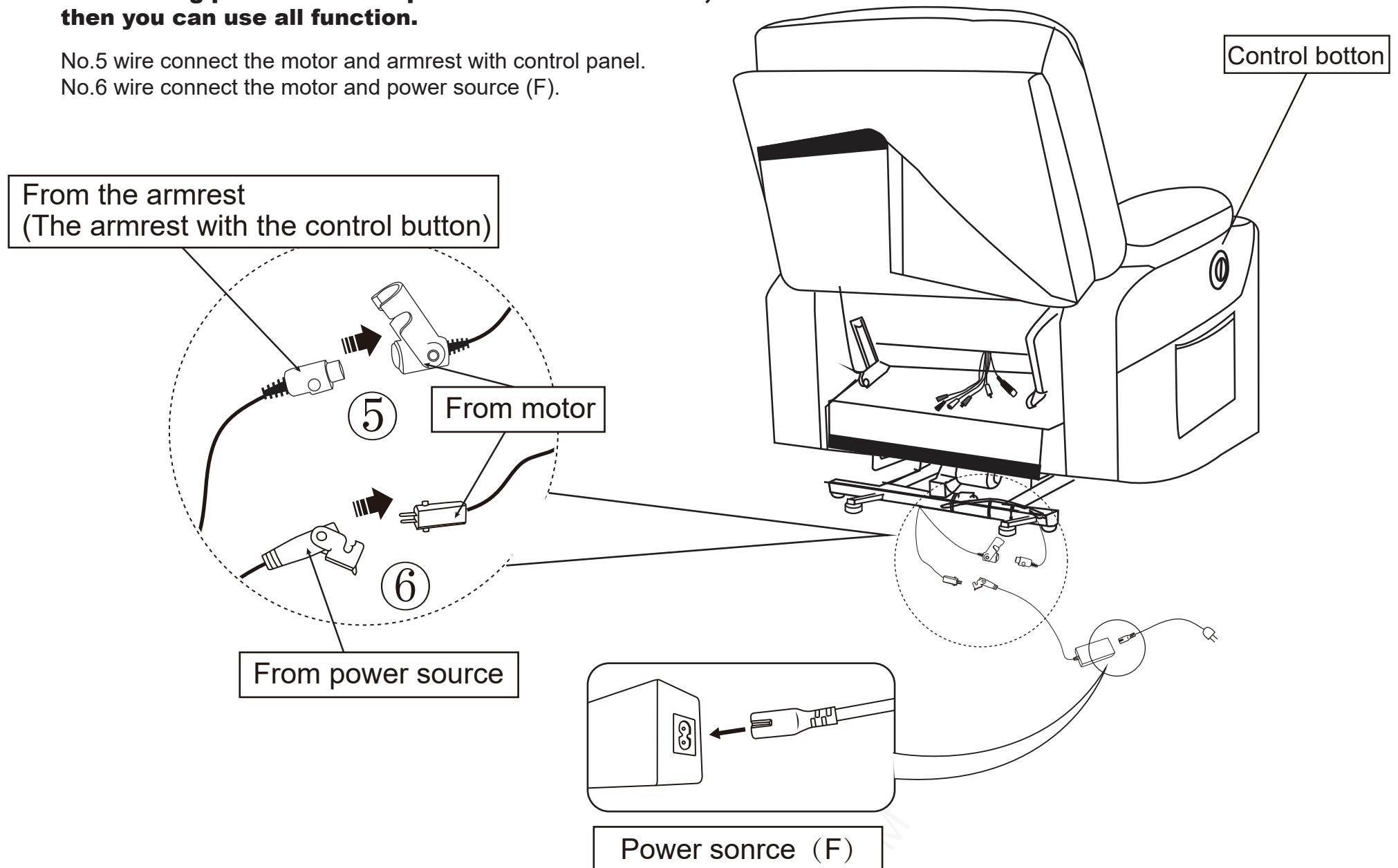


# Power Lift Chair Assembly Instruction\_13012

## How to recline?

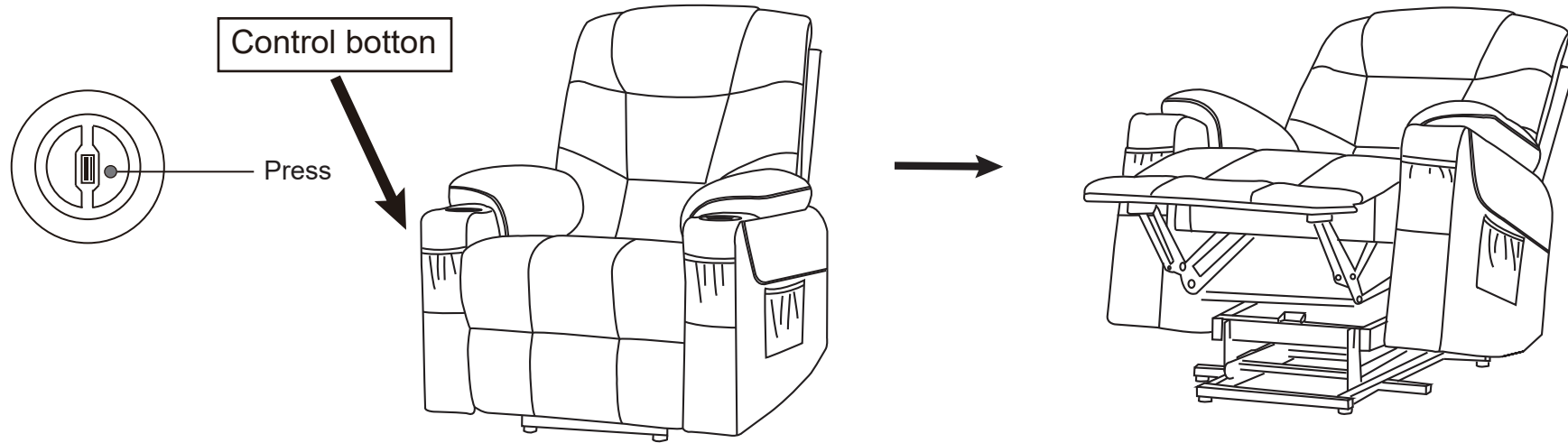

## How to get footrest latch down and stand?

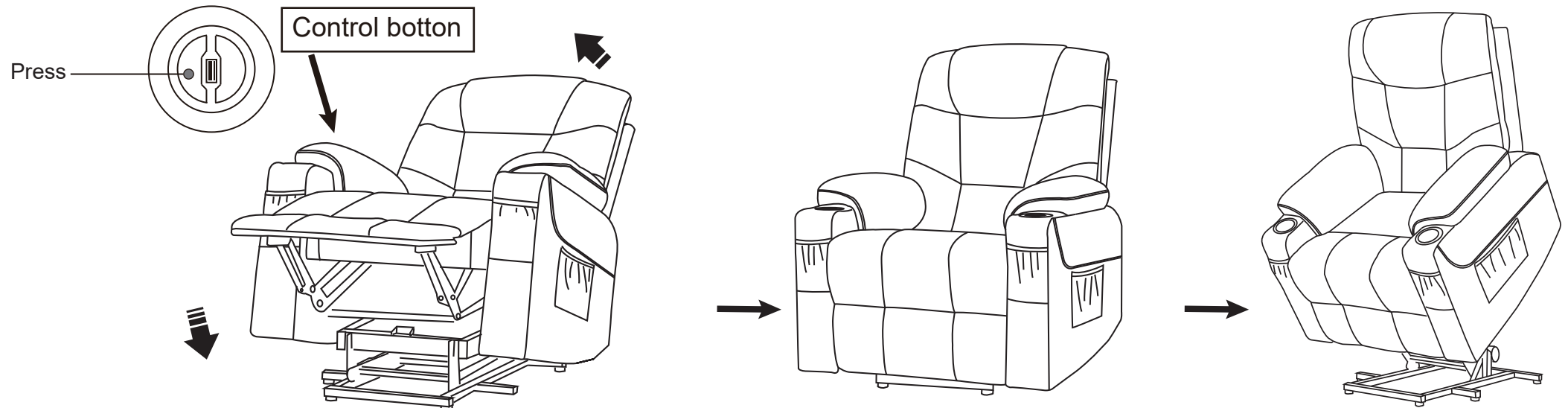


# Power Lift Chair Assembly Instruction\_13012

## How to remove the chair ?


**Please note that you need to take apart the backrest before removing the arms.**

1. Force apart the buckles while take the backrest out from the arms.

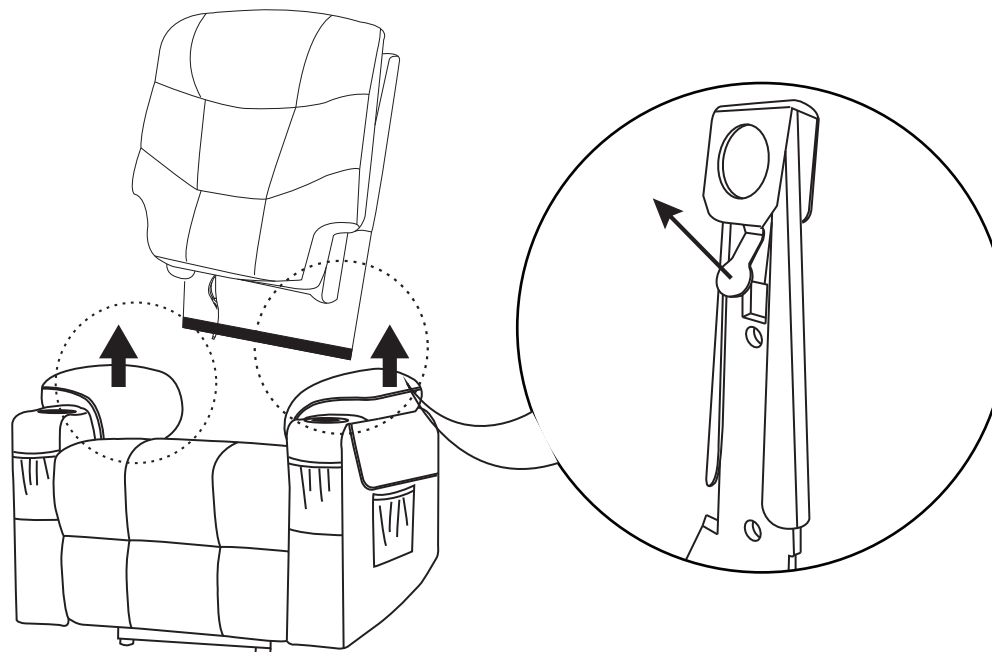


- 
2. Then pull out the arms from the base.

**Note: Please pull out the armrests from the middle of the armrests.**

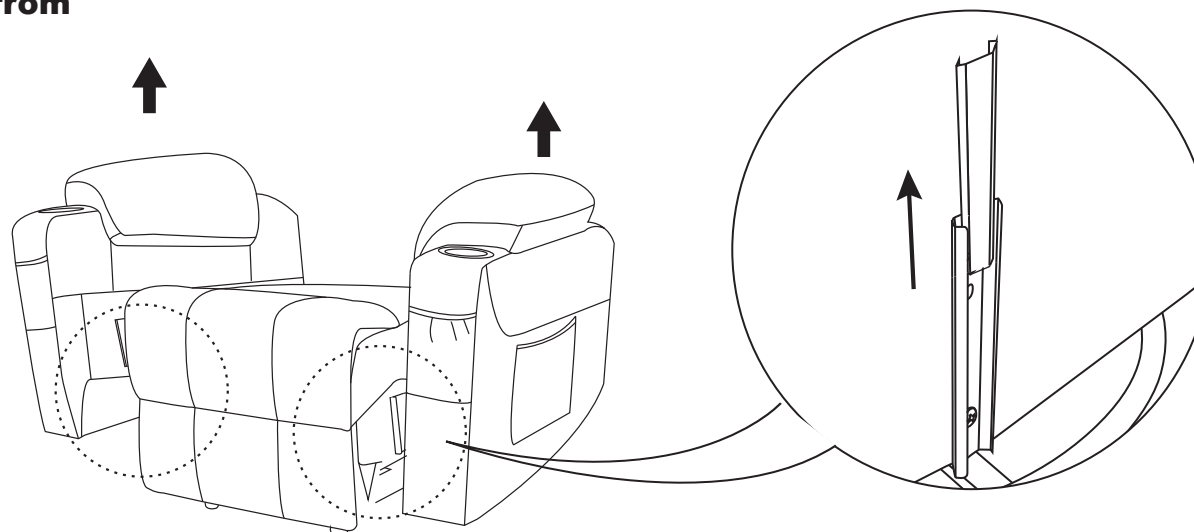

# Power Lift Chair Assembly Instruction\_13012


## How to use remote control?

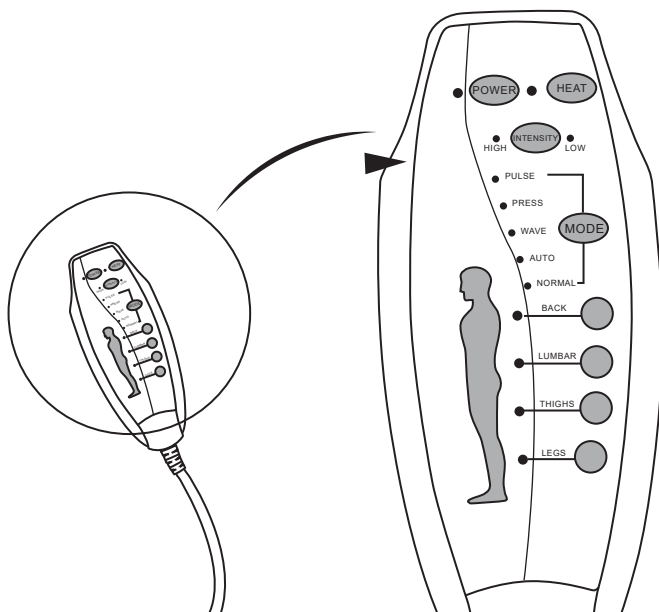

- POWER : Turn on/off remote control.
  - HEAT : Turn on/off heat function for lumbar part only.
  - INTENSITY : Control massage function from low to high.
  - MODE : Choose one massage mode you like(pulse, press,wave,auto, normal)
- You can choose 1-4 massage parts(back, lumbar, thighs, legs)to enjoy.

## How to use control panel that outside of right armrest?

Control panel is for chair lift and reclining function, also has dual USB outlets (5V 2A) that keep your devices charging, such as nearly all smartphone(like iPhone, Samsung..), ipad, headphones, Kindle...

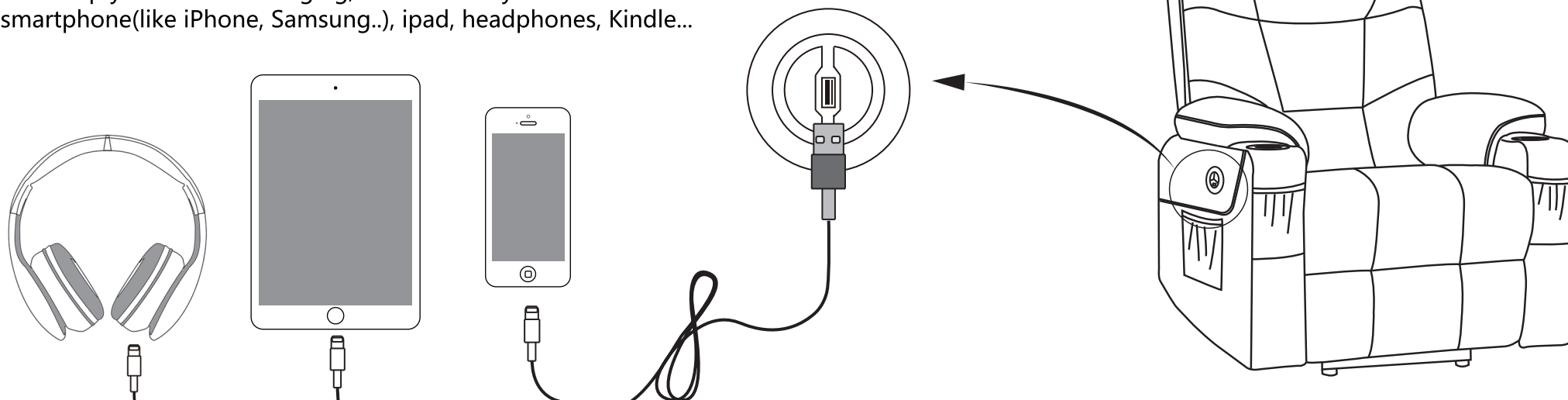

# Power Lift Chair Assembly Instruction\_13012

## FAQ

1.Q: Missing parts when received.

A: Please check if you already received two boxes we shipped to you. Sometimes the two boxes will not arrive in same date, hope you can wait for 2-3 more days or contact us for more help.

If you already have two boxes, please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

2.Q: Some parts damaged when arrived.

A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

3.Q: The massage function does not work.

A: Please check if all wires connect correctly, the power cord (D) may be defective if indicator does not light on, or if the lights doesn't turn on when you press "POWER" button on the remote, that means the remote is defective.

Please kindly send messages with clear photos to describe the problem, then we will send necessary replacement to you.

4. Q: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.

A: Please kindly check if the power source (F) and control panel are connected well with lift motor(Wire #5 & #6). Sometimes the power source (F) and control panel may fall off from the motor. If they connect well, then check the lift motor if it has some anomaly, like some noise comes out when you press the lift and recline button. If you can make sure the motor is defective, please contact us for replacement.

5. Q: The Heat function does not work or can't feel.

A: Please check all wires connection the same as question 3 first. If the remote also works well, you will need wait for 5-10 mins after you press the "HEAT" button. The heating function only works at lumbar position and the heat temperature isn't very hot.