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
CP2972-XXX-02

REQUIRED TOOL:

Standard size Phillips head screwdriver

INSTRUCTIONS:

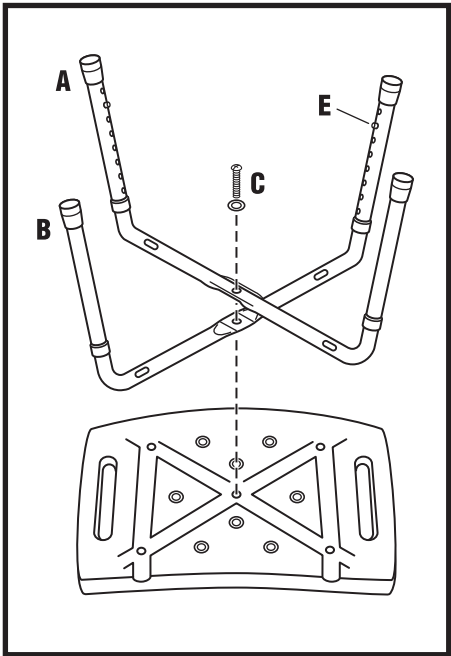- Place flattened section of leg **A** over flattened section of leg **B** in perpendicular position.
Insert 1-3/8 in. screw **C** through washer and center holes of legs into threaded insert at the center of either seat. Both legs should be seated in the grooves at the bottom of the seat.
- Assemble (L-shaped) backrest supports over legs by aligning all screw holes using 2-1/4 in. screws **D**. Make sure that the shorter end (labeled **S**) of the L-shaped supports are fastened underneath the plastic seat bottom. Securely tighten screws **D** through washers and into the four threaded inserts.
- Assemble backrest by fastening 1-3/8 in. screws **C** through holes in both backrest supports into threaded inserts in the back of the backrest.
- To adjust height of seat, push in button **E** on legs and slide tubular sleeve so that button **E** locks (pops out) securely into desired height position. Check to see that all legs are in the same height position before using this seat.

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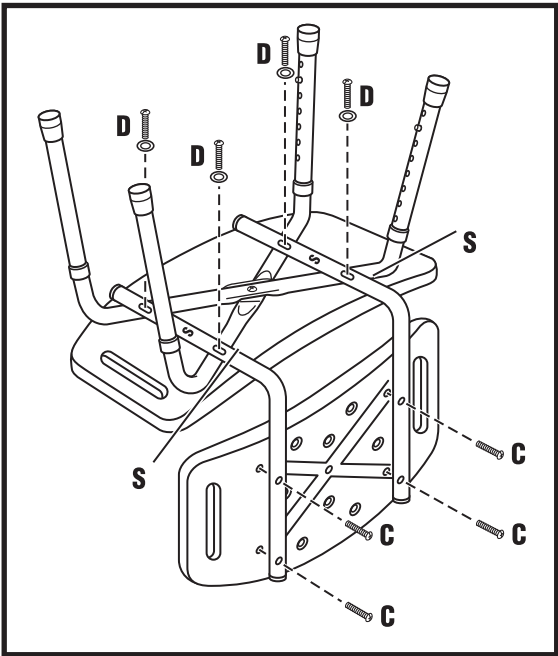
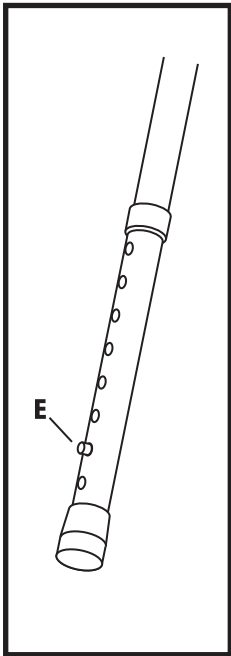
IMPORTANT:

Wipe seat dry after every use. Periodically check that all assembly screws are securely fastened.

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WARNING / ATTENTION / ADVERTENCIA

WARNING:

Do not jump or stand on the chair. This product has a maximum user weight limitation of **250 pounds**. If components are damaged or missing, contact your dealer immediately. **NEVER MAKE SUBSTITUTIONS OR MODIFY THIS PRODUCT IN ANY MANNER.** Before use of chair, check that all components are fastened securely and that buttons on adjustable legs are popped out completely and fully engaged in selected holes. Liberty Hardware Manufacturing Corporation assumes no liability for any damage or injury caused by improper installation, assembly or use of this product.

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CLEANING INSTRUCTIONS:

To clean, wipe with soft damp cloth. Do not use detergents or cleansers that may be harmful to the finish.

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