




# Power Lift Chair Assembly Instruction

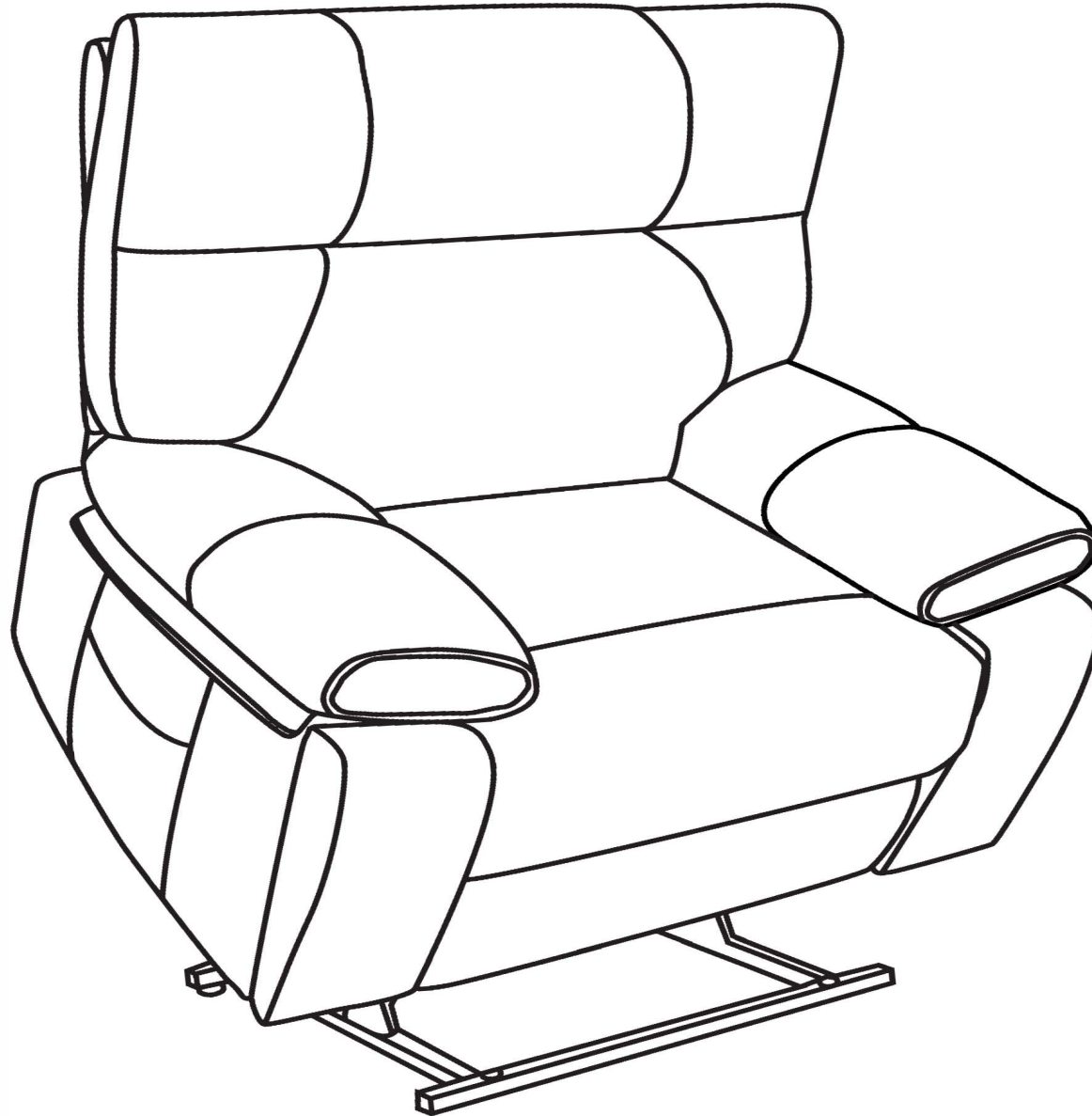


Please make sure you've got everything lists as following before assembly.

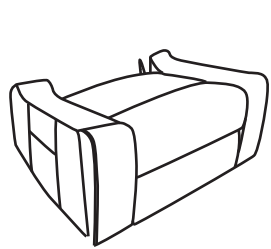


**A** Chair base

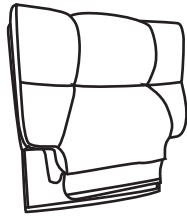

**B** Backrest

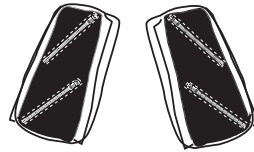


**C** Left and right armrest

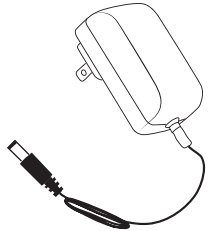


**D** Power cord



**E** Remote control

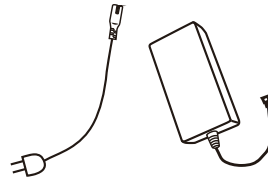

**F** Power Source

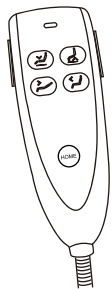


**G** Lift control



**H** Supporting Feet x4


### STEP 1

Find a small bag in the accessories box. Take out Supporting Feet from the small bag. Assemble the four Supporting Feet as shown on the bottom of mechanism.

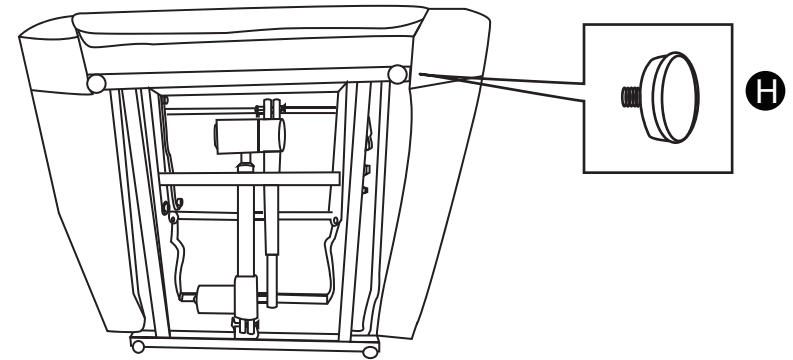


### STEP 2


Zip the zipper to make sure both armrest cushions(C) are well attached to the right direct of arm.

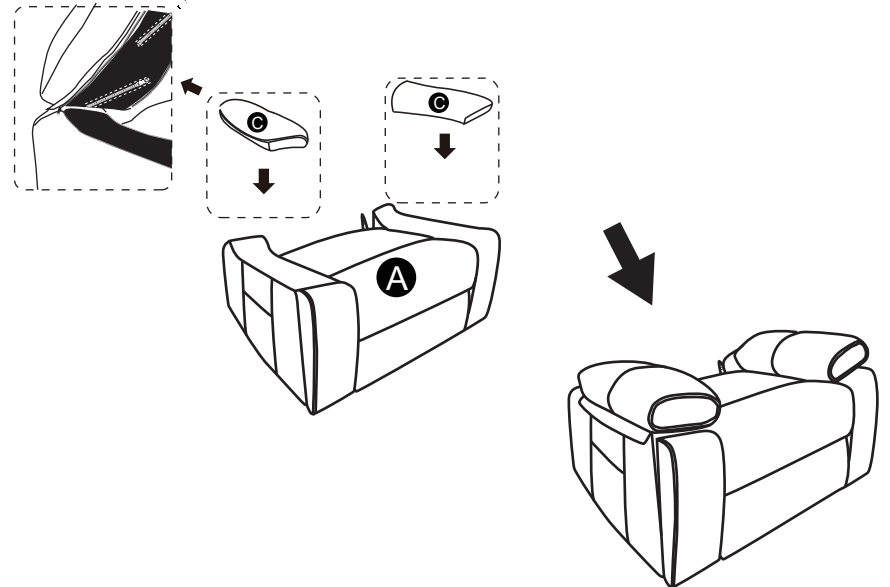

**STEP 3**

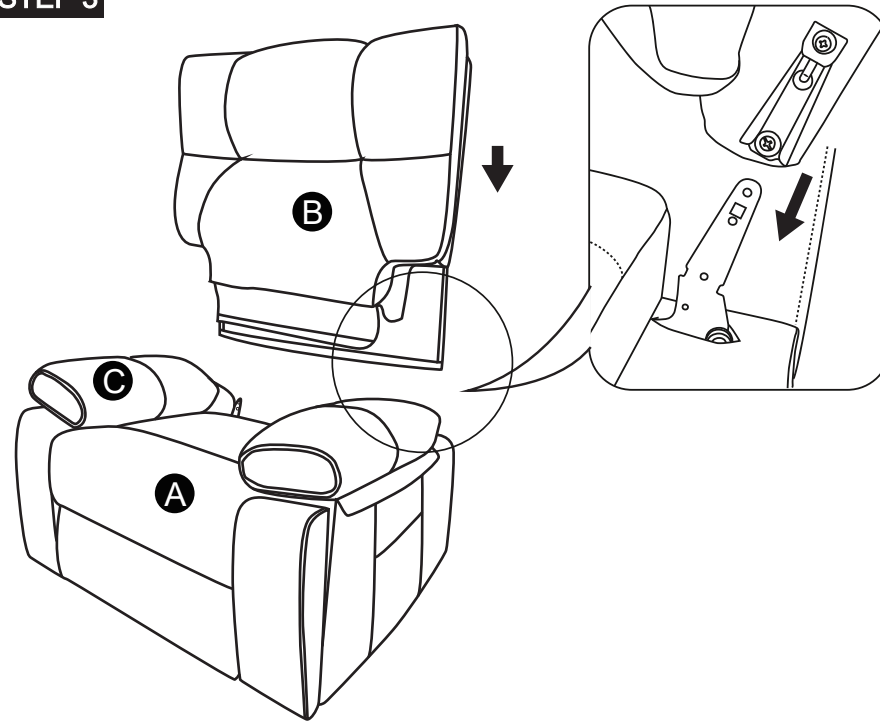

Insert chair back to the chair base, make sure the on the back fully assembled to the bottom part.


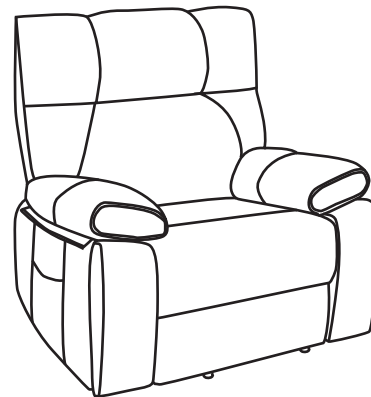


## STEP 4


- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power cord(D).
- No.4 wire connects with backrest and remote control(E).
- No.5 wire connect the motor and the wire from lift control(G).
- No.6 wire connect the motor and power source (F).

## NOTE

- 1: Two same color wires connect.
- 2: Please connect all wires BEFORE plug power cord and power source into outlet.

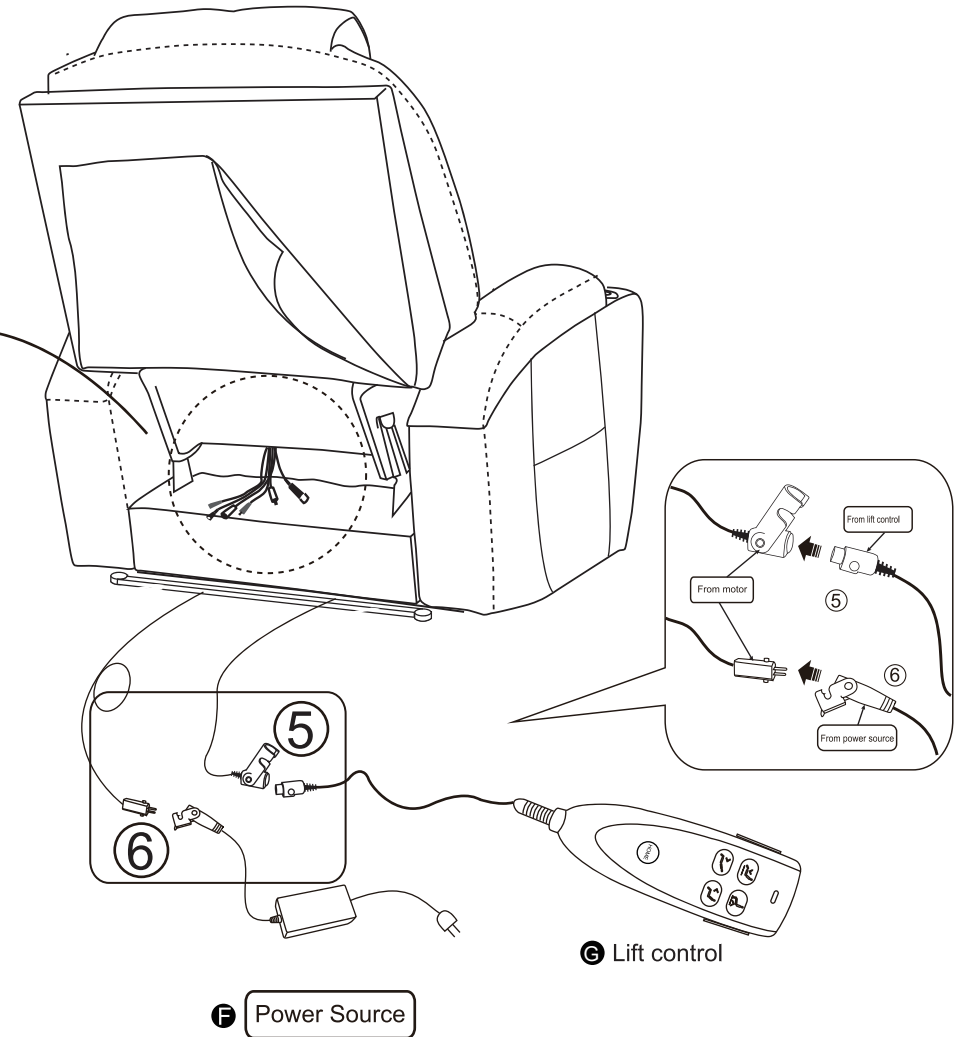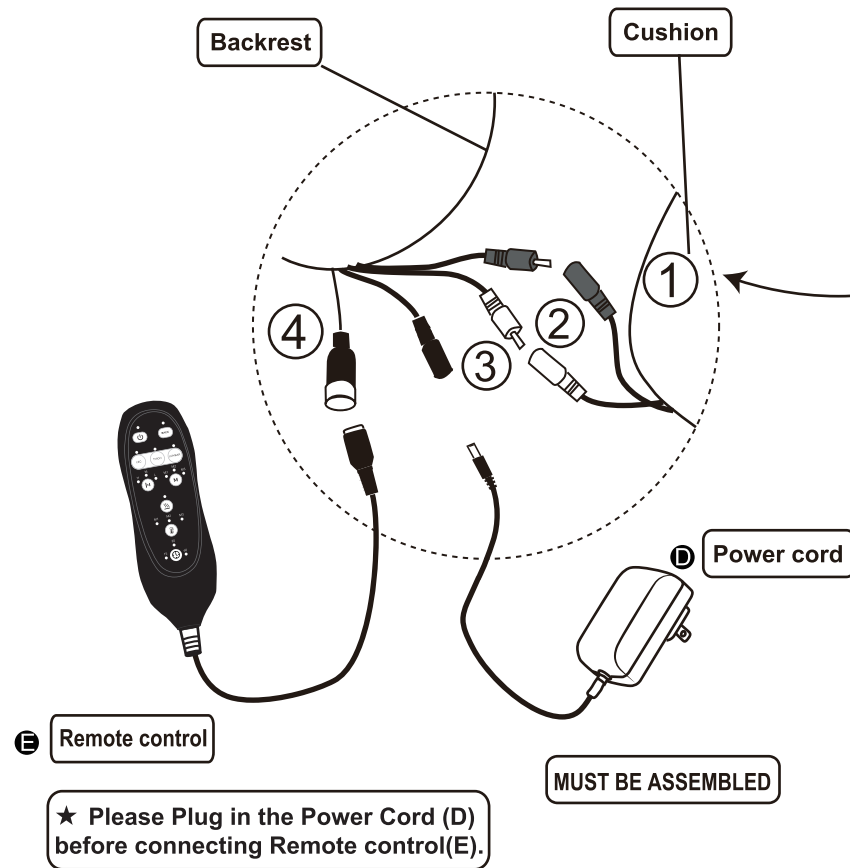


## How to recline?

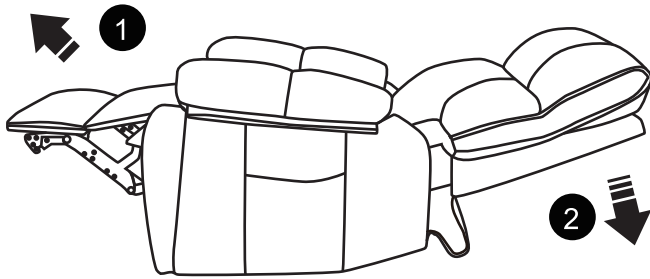



Press the button to descend the seat and unfold the footrest.

Press the button to recline the backrest.

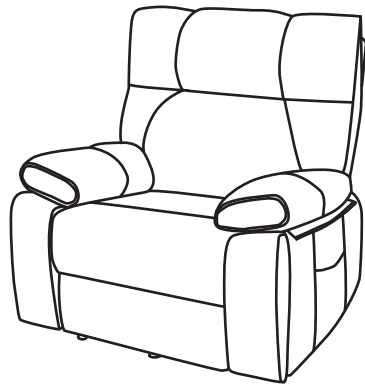

seat position


## How to get footrest latch down and stand?

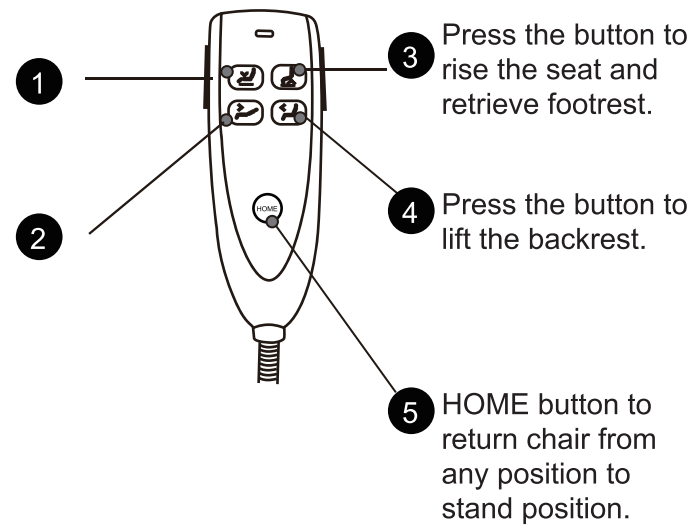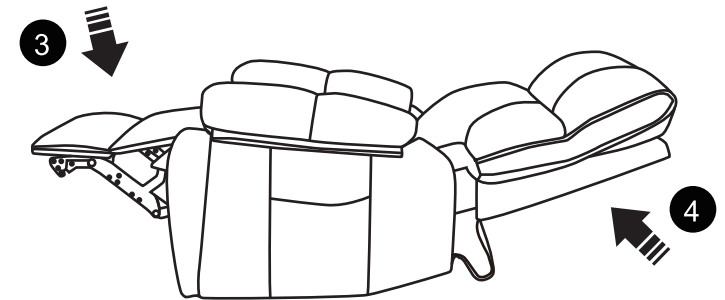

3 Press the button to rise the seat and retrieve footrest.


4 Press the button to lift the backrest.

5 HOME button to return chair from any position to stand position.

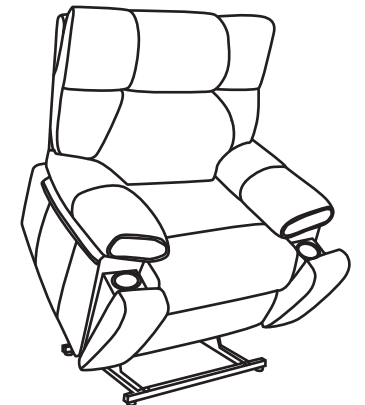

stand position

## How to use remote control?

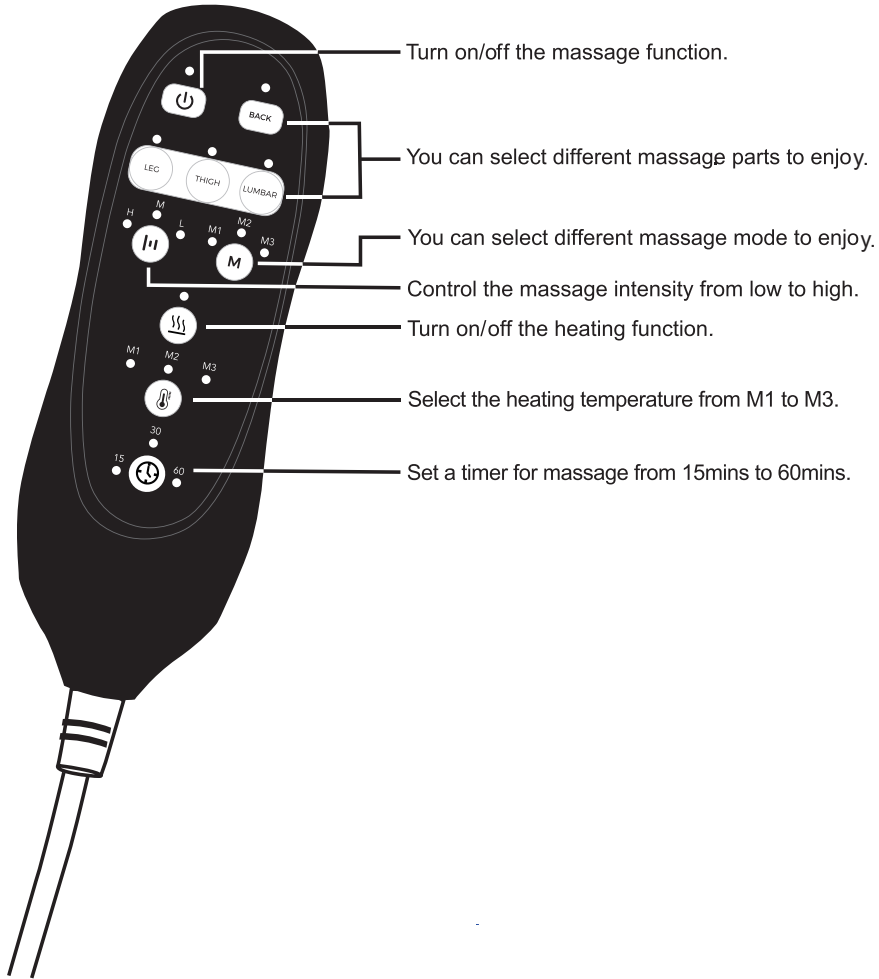

## FAQ

### Q1: Missing parts when received.

A: If you find some parts are missing from the box, please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

### Q2: Some parts damaged when arrived.

A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

### Q3: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.

A: Please kindly check if the power source(F) are connected well with lift motor. Sometimes the power source (F) may fall off from the motor.

If the connection is good, then check the lift motor if it has some anomaly, like some noise comes out when you press the lift button. If you can make sure the motor is defective, please contact us for replacement.

### How to remove the chair ?

Please note that you need to take apart the backrest before.

1. Force apart the buckles while take the backrest out from the arms.

