



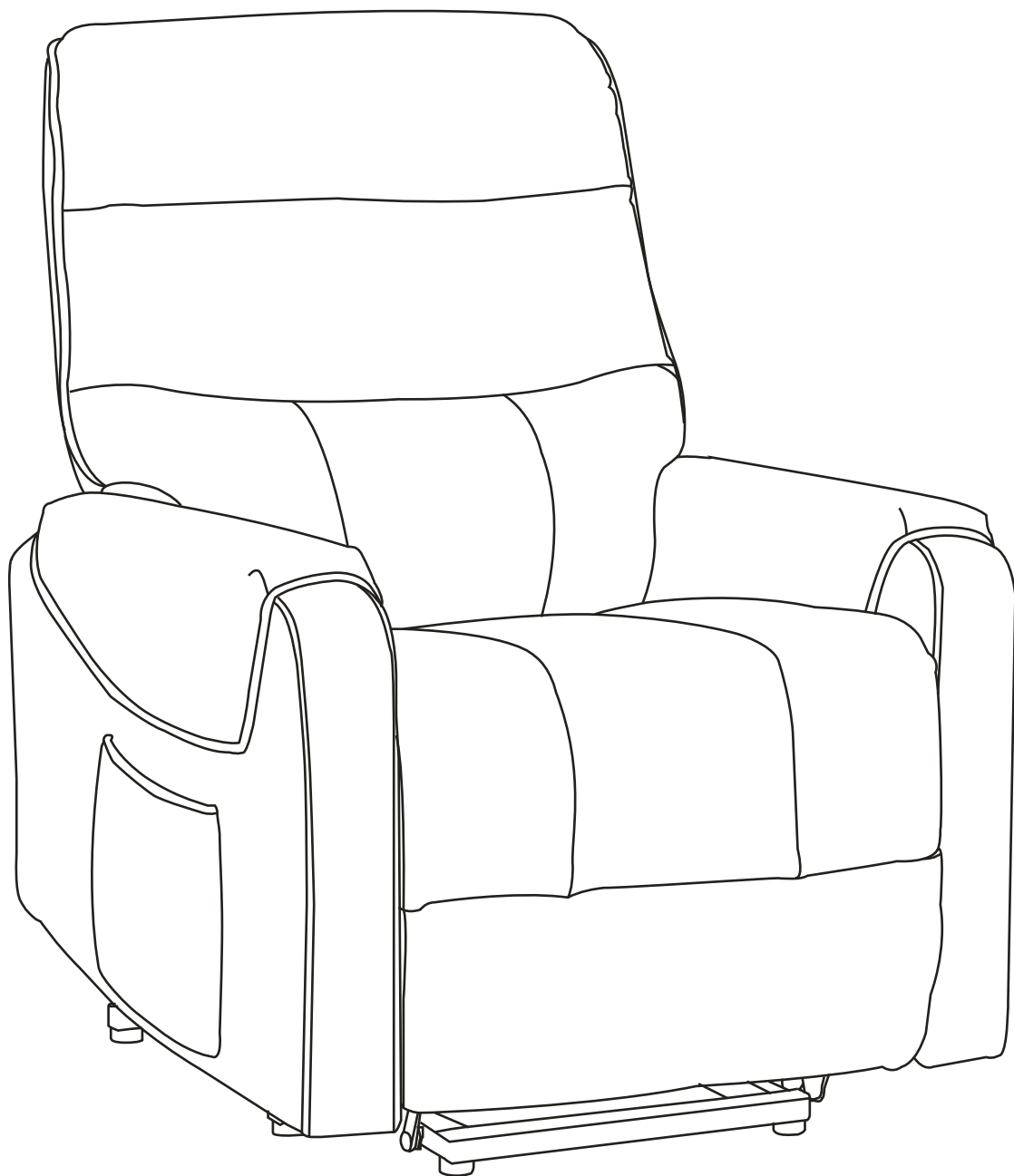

INSTRUCTION MANUAL

ELECTRIC LIFTING & RECLINING

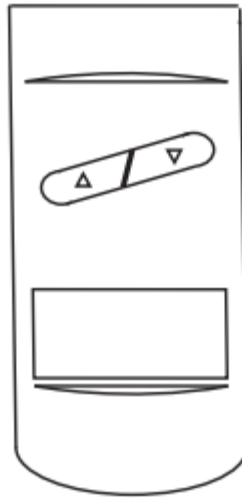

Operation Instructions

- Plug the direct current end of the cord into the AC Adapter outlet, then insert the AC adapter plug into a wall outlet for power.
- Press the button to ▽ recline the chair back and foot-rest.
- Press the button ▲ to return the chair back and foot-rest to their original position. Push the button a ▲ second time to lift the entire chair.
- Press the button to ▲ lower the chair and return it to the original position.

PRODUCTFEATURES

- Power Reclining
- Power Lifting

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including:

- Always unplug the lift chair from the power source after use, and before cleaning.
- Do not touch lift chair if large amounts of liquid have spilled on it. Unplug immediately.
- The chair should never be left unattended when plugged in. Unplug from the outlet when not in use or before putting on or taking off parts or attachments.
- Do not use outdoors.
- Do not carry this appliance by the attached power cord;do not use it as a handle.
- Keep the cord away from heated surfaces.
- Close supervision is necessary when this appliance is used by or near children, invalids or disabled persons.
- This unit is not a toy. Children should not use or play with it.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not use near eyes or other highly sensitive areas.
- To disconnect, press the OFF button, make sure all functions are stopped, then remove the plug form the outlet.
- Use this appliance only for its intended use as described in this manual. Do not use attachments that haven't been provided with the unit. This appliance is not intended use for persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety, children should be supervised to ensure that they do not play with the appliance.

HEALTH CAUTION

- If you have any concerns regarding to your health, consult your doctor before using this product.
- Individuals with pacemakers should not use this product.
- If you have pain in a muscle or joint after using for a prolonged period of time, discontinue using and consult your doctor. Persist pain could be a symptom of a more serious condition.

MAINTAINANCE INSTRUCTIONS


TO STORE:

Place unit in its box or safety dry and cool place. Avoid putting chair in contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap attached power cord around the unit. DO NOT pull the unit by power cord.


TO CLEAN:

- Ensure the unit is unplugged from the power source, and allow the unit to cool down before cleaning.
- Do not use any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat with clean, dry cloth.
- Do not use any abrasive cleaners, brushes, gasoline, kerosene, glass furniture polish, paint thinner or water hotter than 120 °F to clean the unit.

OPERATING INSTRUCTION

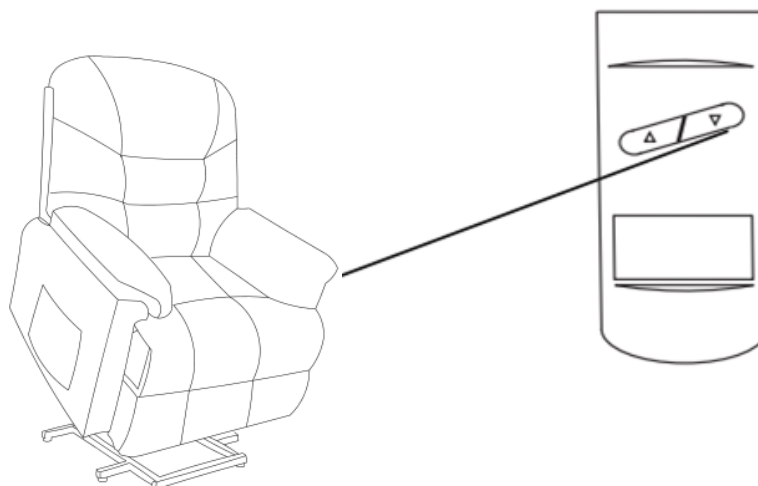

Press the $\triangle$ button to make the sofa lifted.

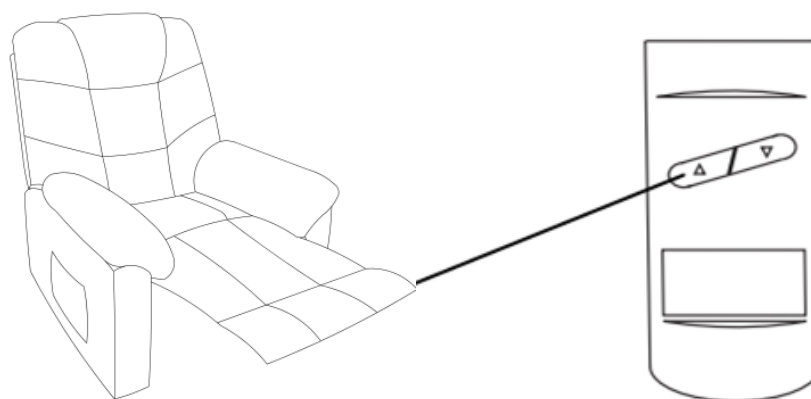

Press the ∇ button to recline the sofa.

TECHNICAL SPECIFICATIONS

Rated Voltage :100V-240V~

Rated Frequency :50Hz/60Hz

ASSEMBLY GUIDE

.01

Congratulations, you now have a new function sofa with massage.

Before enjoy the comfort chair, you have to put it together. This Assembly Guide will take you through the basic assembly process. To make sure your assembly goes smoothly, please follow the assembly instructions, **and especially the important safety information.**


Also, while it is possible to assemble the chair on your own, **having a second person to help you makes it even easier.**

Before you begin, please check if there are any missing parts from the package. Should there be any issues, please don't hesitate to drop us an email and we'll help you out ASAP.


PACKAGE LIST

.02


Ⓐ

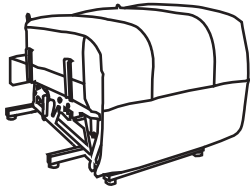


Ⓑ

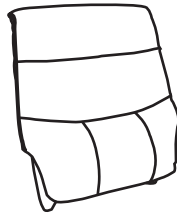

Ⓒ

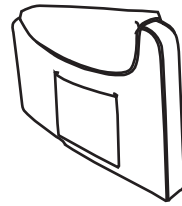


Ⓓ

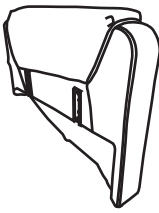


Ⓔ



Ⓕ

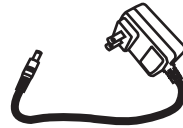


Ⓖ

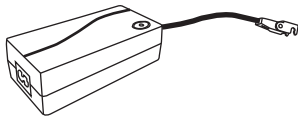


Ⓖ



Ⓖ

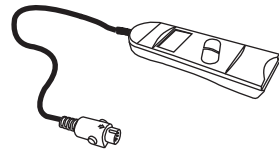

Ⓙ

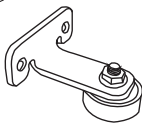


x2

Ⓚ



x4


Ⓛ

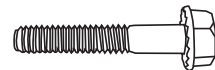


x4

ASSEMBLY IN STRUCTION.03

STEP 1:

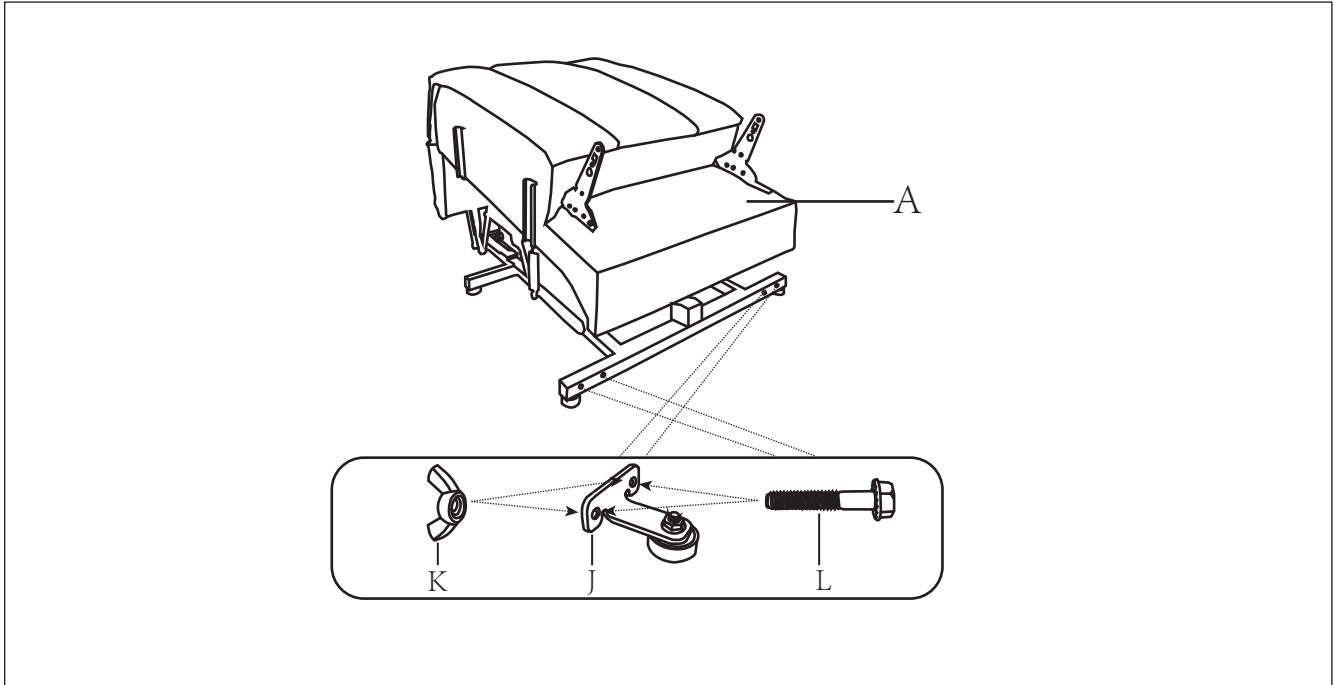


STEP 2:

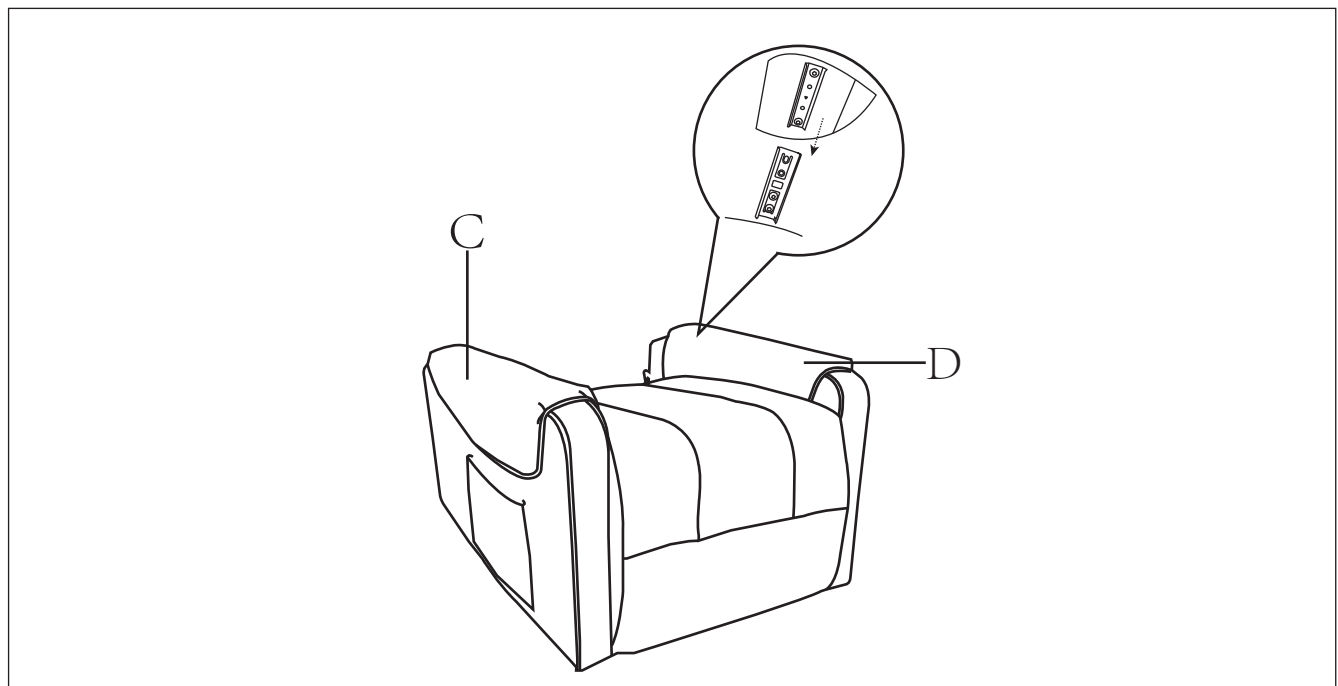


ASSEMBLY IN STRUCTION .04


STEP 3:

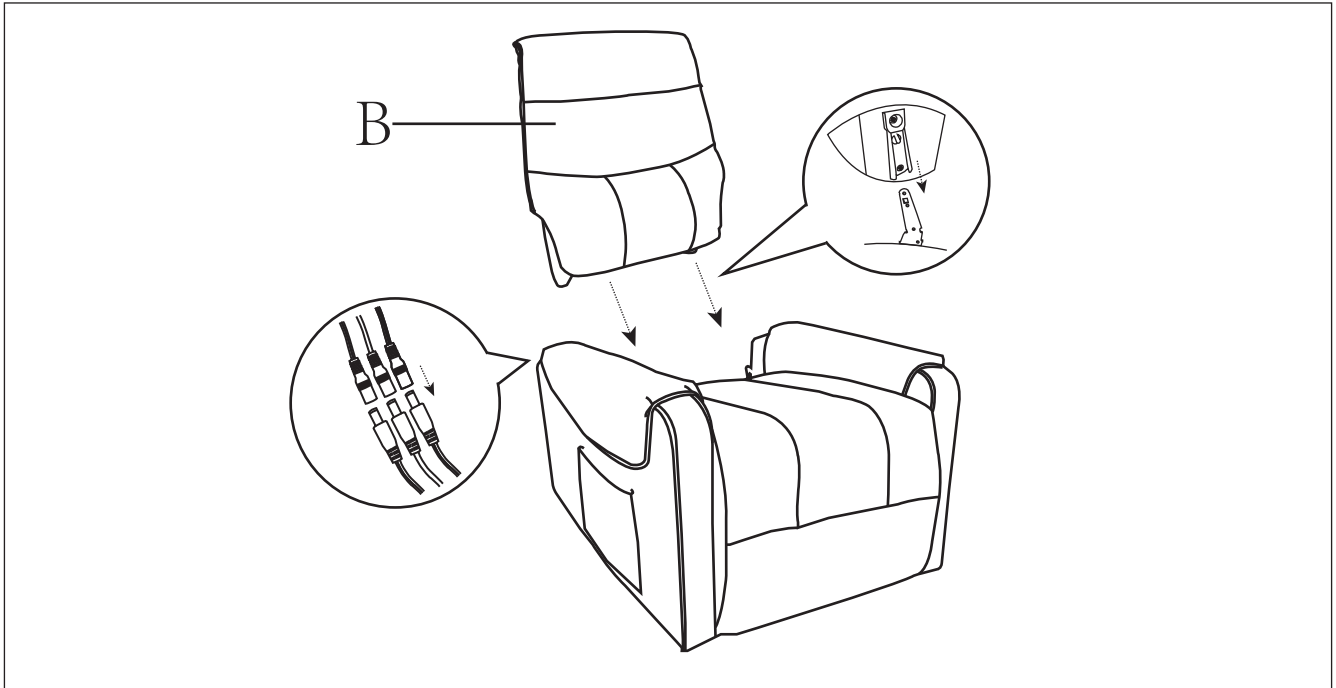

STEP 4:

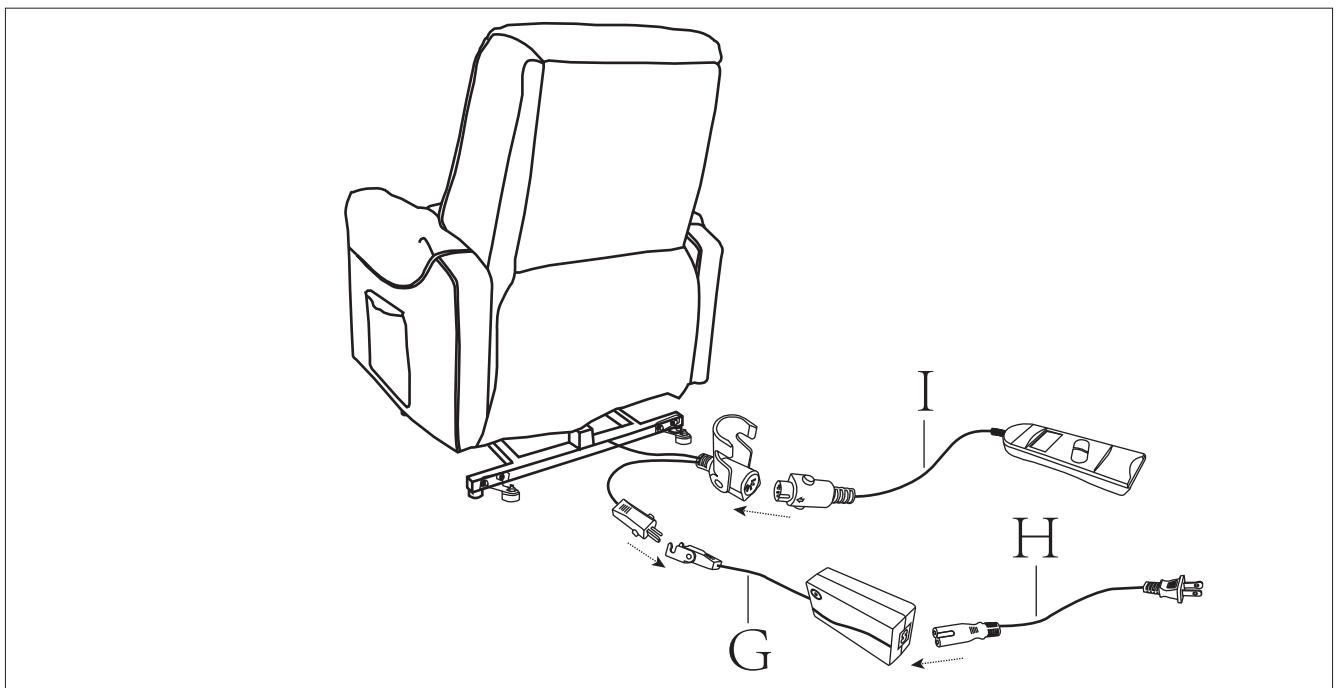

**Congratulations! You've assembled
your chair.**

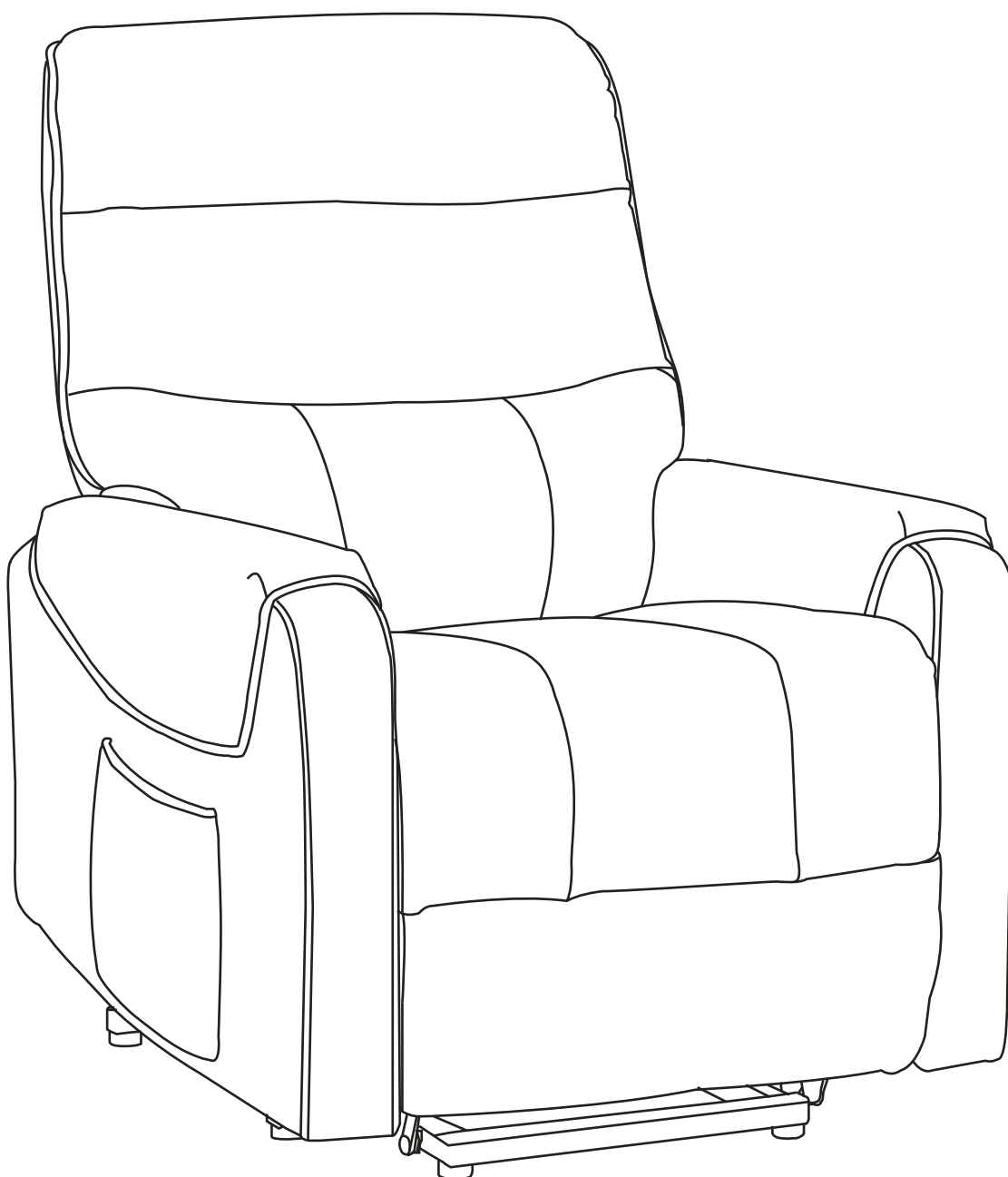

Warnings


- Sit down in the chair gently. To avoid tipping over, do not throw yourself into the chair. Move the chair into the reclining position slowly, making sure that no children or pets are in close proximity. Do not allow children to play on or around the chair, particularly when it is in an open position.
- Close the recliner completely before getting out of it.
- Never sit on the footrest when it is open, even if it is a child.
- To eliminate the risk of tipping over, do not sit on the arms or footrest of the chair.
- This chair is designed to be used by 1 adult at a time.
- Maximum weight capacity is 330LBS, Do not allow any person who over 330LBS to sit on the recliner.
- Please forbid people with the following ailments from using this sofa: Those people with severe symptoms of heart disease, high blood pressure, cerebral thrombosis and others who are sensitive to heat or completely insensitive to heat, those with brittle bones or have issues with waist support pregnant women, children and others where it has been determined they are not suitable to use this product under the guidance of doctors.
- To fully recline the sofa, ensure the sofa well, you need to put the sofa at least 100cm away from the wall or any other obstacle.
- Product cannot touch water, fire, etc. Product can also not come in contact with sharp objects (knives, nails, etc.)

Maintenance Manual Instructions

- To avoid dropping, you should carry the sofa carefully.
- If you find dirt and or stains on the chair, spray the sofa with a sofa specific cleaner then use a sponge or cloth to wipe the dirt away. Do not use water, soapy water or cleaner with waxed wooden areas. Take care to wipe the sofa carefully and gently.
- Do not put the sofa in an area with direct sunlight and do not put the sofa directly in front of an air conditioner, this will fade the upholstery and make it brittle.


How to Disassemble

Please find a strong tool to prod the small piece of metal grooves, then the backrest will be pulled out. The tool is not included.

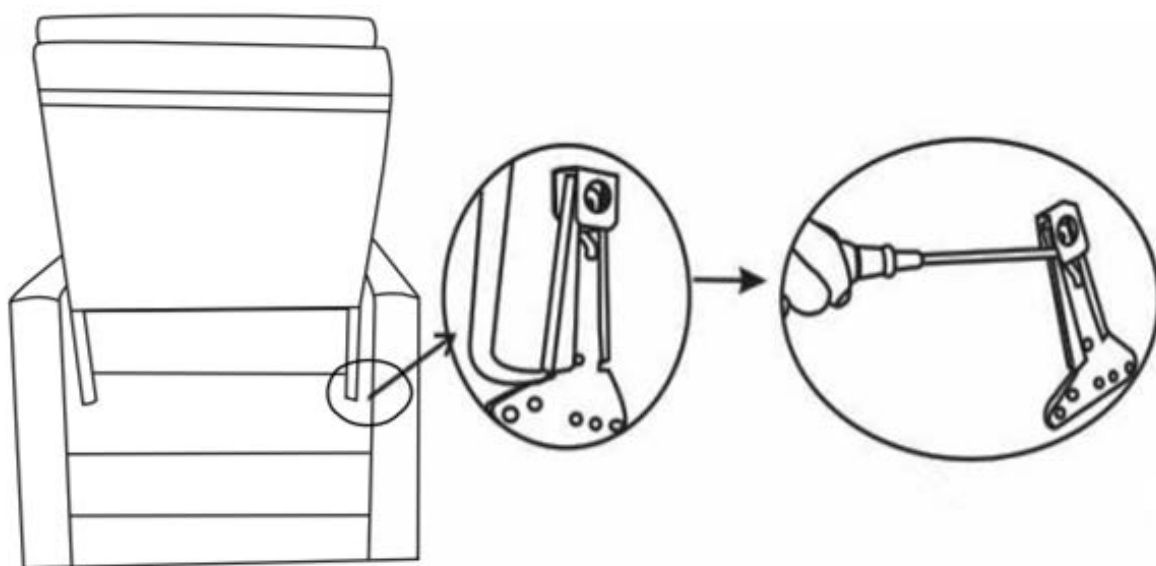

CONTEOL 2.4GHz MOTOR AND HEATING

READ ALL INSTRUCTIONS
BEFORE USING THE ITEM

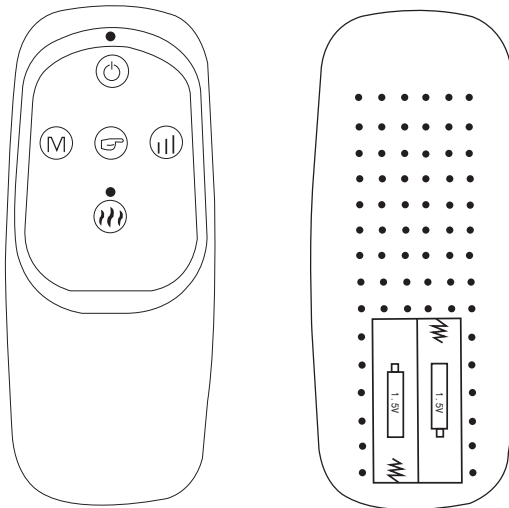

Adopt America and European 2.4GHz solution, with strong anti-interfere feature


PRODUCT FEATURE

- 8 Vibration Motors
- Heating Massage
- Three Massaging modes
- Three Adjustable intensity
- Four massage zones, each zone can be operated individually
- The default timing is 15 min
- The product is 4 position motor massage core. Pull through it, it is in standby state. Users can realize the function control through the hand controller

CONTEOL 2.4GHz MOTOR AND HEATING

HOW TO USE

- Insert adapter jack into the socket found on back of the massage chair, then plug the adapter into a wall outlet. The specification of power adapter: input, AC110-240V, 50Hz; output, DC 12V, 1000mA.

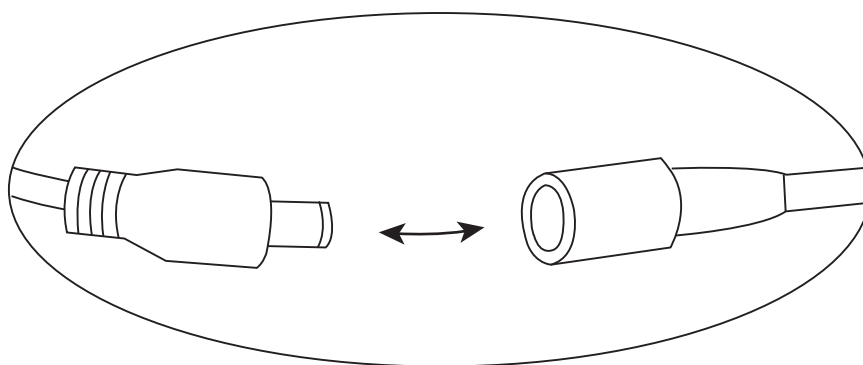

- Ⓢ **POWER:** Turn the unit ON by pressing the ON/OFF button once, Turn on the massage in each zone by pressing the Massage zone. To turn off a zone, press the button again.
- Ⓜ **MODE:** To choose a desired massage mode, press the MODE button. To change modes, press MODE button until desired mode Massage.
- Ⓜ **INT:** To change the massage intensity press the INT button.
- Ⓜ **Heating :** if users want the product to have the spontaneous heat, they can press heating switch key for heating. The maximum heating value is 65 °C . There is a built-in protective chip in the product, and the continuous heating will not make the heat value exceed the maximum peak value.
- Ⓜ **Position:** Turn on the power and massage four position at the same time. You can operate this button according to your needs. The parts are waist-thigh-leg-back---

CONTEOL 2.4GHz MOTOR AND HEATING

HOW TO USE

- Time: the default preset time of the product is 15 minutes. After it is used for 15 minutes, the product will shut down automatically for protection.
- NOTE: Discontinue use immediately if you feel any discomfort or pain.
- NOTE: for your benefit, the massage chair is equipped with an auto shut-off feature that will turn the chair unit off after 15/30 minutes of continuous use.
- DO NOT use for more than 30minutes of continuous use.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

- This product is for use on a 12V circuit Make sure that the product
- Only use this product with original power supply type nr :IVP1200-1000
- Always unplug the massager from the power source after use, and before cleaning.
- Do not reach for massager if it has fallen into water. Unplug immediately.
- An appliance should never be left unattended when plugged in.
Unplug from outlet when not in use, before putting on or taking off parts or attachments.
- Do not use outdoors.
- Do not carry this appliance by supplied cord as a handle.
- Keep the cord away from heated surfaces.
- Close supervision is necessary when this appliance is used by on or near children, invalids or disabled persons.

CONTEOL 2.4GHz MOTOR AND HEATING

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

- This unit is not a toy. Children should not use it or play with it.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not use near eyes or other highly sensitive areas.
- To disconnect, push "POWER" key to "OFF" place, make sure all functions stopped, then remove plug from outlet.
- Keep cord away from heated surfaces.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not provided with the unit.

HEALTH CAUTION FOR MASSAGE CHAIRS

- If you have any concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should not use this product.
- If you experience pain in a muscle or joint for a prolonged period of time. Discontinue use and consult your doctor. Persistent pain could be a symptom of a more serious condition.

MAINTENANCE INSTRUCTIONS

TO STORE:

Place unit in its box or safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap power cord around the unit. Do not hang the unit by power cord.

CONTEOL 2.4GHz MOTOR AND HEATING

MAINTENANCE INSTRUCTIONS

TO CLEAN:

- Insure the unit is unplugged from the power source, and allow to cool before cleaning.
- Do not place in any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat dry with clean, dry cloth.
- Do not place in any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat dry with clean, dry cloth.
- To clean the unit, make sure the massager is unplugged. Allow the unit to cool and the wipe down with a damp clean cloth. Do not use any strong chemical cleaners.