

Installation Instructions



Attentions:

- 1) This product cannot be used as a wheelchair to push people, in order to avoid overturning accidents.
- 2) Please check all parts and wheels for integrity before use.
- 3) When sitting, be sure to place the side brake and secure the car to prevent accidents caused by sliding.
- 4) The weight of the passenger must be less than the maximum carrying capacity of the vehicle --100kg.
- 5) If the passenger is overweight, you can use it without installing side handrails.


HOW TO ASSEMBLE



- 1) Pull the bending rod here and open the frame, then check all components, and make sure screws are in the backrest bag.




- 2) Firstly, install the wheels with the red buckle facing upwards, align with the slot and insert the wheels. Then, install the backrest and insert it into the U-shaped frame.

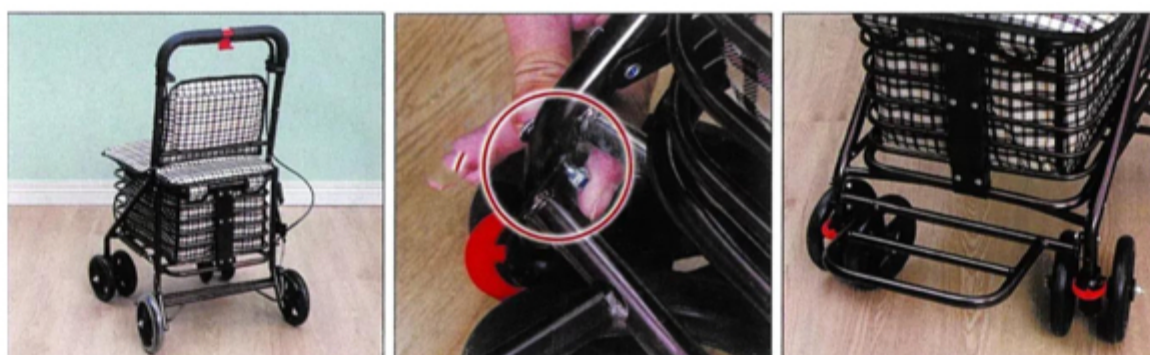



3) Next, install the foot pedal with short screws, paying attention to the "metal pieces" on both sides of the pedal facing downwards.

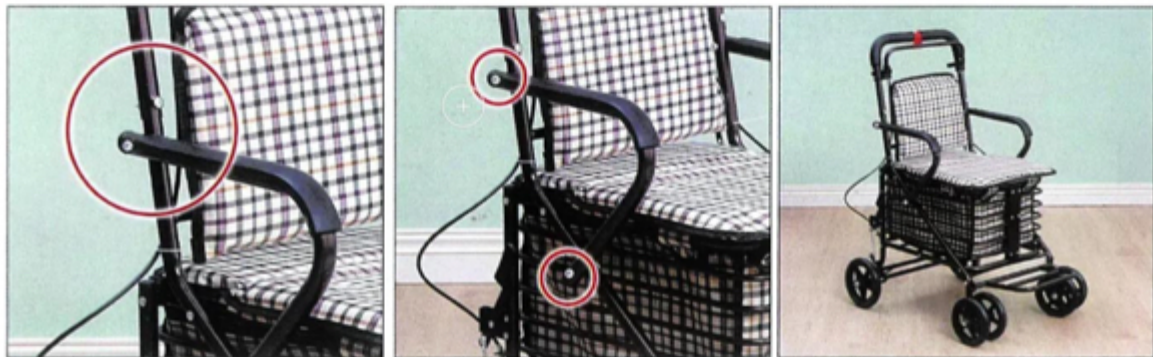

4) Finally, use long screws to install the armrest, ensuring that both the upper and lower positions are connected and tightened. Now, you have finished it.

!!!Congratulations!!!