

How to readjust the height of the shifter:

- ▶ Adjust the height of the seat by rocking the handle of the control assembly up and down as required.

7. Precautions:

- ▶ Before use, please check whether the components are complete and stable to ensure safe use;
- ▶ Do not stand on the pedals each time you sit in or get up from the patient shifter. Step on the ground.
- ▶ Adjust the patient shifter to the proper height before use:
- ▶ Make sure that both sides of the base plate are kept at the same height before use.
- ▶ Do not adjust the height when the user is sitting on the patient shifter.
- ▶ Ensure that the wheels are locked when the user sits on and off the patient shifter.
- ▶ Always keep the four wheels locked when staying on a sloped ground.

8. Maintenance:

- ▶ In order to protect users from accidental injury, please check and maintain the unit frequently.
- ▶ Please use neutral detergent to clean the stains;
- ▶ Regularly check whether nuts are loose, parts are damaged or broken. If necessary, replace damaged parts.
- ▶ When disassembling and disassembling, carefully follow the instructions to avoid unnecessary damage.

9. Packaging, Transportation and Storage:

- ▶ Packaging: This product is packed in cartons.
- ▶ Transportation: When transporting, should handle with care, prohibit the handstand, prevent rain.
- ▶ Storage: Stored in a relative humidity of not more than 80%, free of corrosive gases and well ventilated.

10. Service life: 5 years

11. Instruction preparation time: August 01, 2023

Warehouse No. 4, 599 Yongxing East Road, Jizhou District, Hengshui City

For support, please attach your order ID and relevant images and contact us. We are committed to delivering a timely and satisfying solution.

✉ YBSY918@outlook.com

Patient lift Installation instruction



Enjoy delicious food anytime, anywhere
Widening and thickening
Double decker dining table

Medical device production record certificate number: Jiheng Drug
Administration machine production No. 20230035
Medical device product record number: Jiheng machine 20230213
Medical device product technical requirement No: Jiheng Equipment
20230213

Please read the product instructions carefully or purchase and use under the guidance of medical personnel. Installation video Scan code

1. Product Name: Patient Transfer Machine

2. Product model: KYD-YWJ-01 Production batch number and service life see product certificate

3. Product performance and structural composition: composed of brackets, casters, base legs, suspension rods, controller components, and handrails.

4. Intended use: Used for medical institutions to transport and move patients.

unit: cm error: ± 10 cm

Specification and model	length	Castor width	Castor height	Seat height	Overall height
KYD-YWJ-01	68	58	13	41-65	85-109

5. Assembly method:

Remove the mute wheel and wrench. Screw the mute wheel onto the base.



Install the hydraulic pump to the base.



Use an Allen wrench to tighten the bolts at the front end of the hydraulic drum. Unscrew the decorative cap on the cover and screw back the bolt.



Install the foot lever and tighten the hex socket screw. Tighten the waterproof cap on the outside.



Insert the steel shaft into the holes on both sides of the base. And fix it with white bolts at the rear.



Apply lubricating oil evenly on both sides. Above the steel shaft for smooth use.



Connect the white connecting pipe to the hydraulic pump hole.



Press the control lever to the bottom and do not loosen it. The rotatable white connecting pipe is connected to the upper part. Armrest hole alignment.



Connect the armrest to the steel shafts on both sides. And intermediate connecting pipe. If the connecting holes are not correct, refer to the previous step (height adjustable).



Connect the seat and armrest by stringing them into a thin steel shaft.



Buckle the waterproof cap of the entire machine and complete the installation!



How to readjust the height of the patient transfer machine:

- ▶ Loosen the locking lever and adjust the height of the seat according to the control handle of the height control component as needed.
- ▶ After adjusting the height appropriately, tighten the locking rod.

6. Usage:

- ▶ Ride using pan:
 - ▶ Have the user sit upright, open the transfer machine seat and place it in front of the user; Lift the user's foot and place it on the footrest; Lock the brake lock before the user sits on the transfer machine;
 - ▶ Allow the user to tilt their body slightly to the left and right in sequence, while rotating the corresponding seat on one side, allowing the user to sit in the left and right seats of the transfer machine in sequence; When the user is fully seated, fold the back of the transfer machine and lock the automatic buckle; Lock the safety buckle.
- ▶ When the user is fully seated on the transfer machine and both the back buckles are locked, open the brake latch; at this point, the front handle can be used to move the user.
- ▶ When the user moves to the target position, open the double lock on the back, rotate and open the seat of the transfer machine, and the user can smoothly move to the target position.