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
Shower Chair Installation Instructions

Two views of a metal rod with orange caps and a red arrow pointing to a hole. The top view shows the rod with a red arrow pointing to a hole. The bottom view shows the rod without the arrow.

2. ☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐☐ Locate the leg tube, at the center of the marble, and press down on the marble to pull out the leg tube.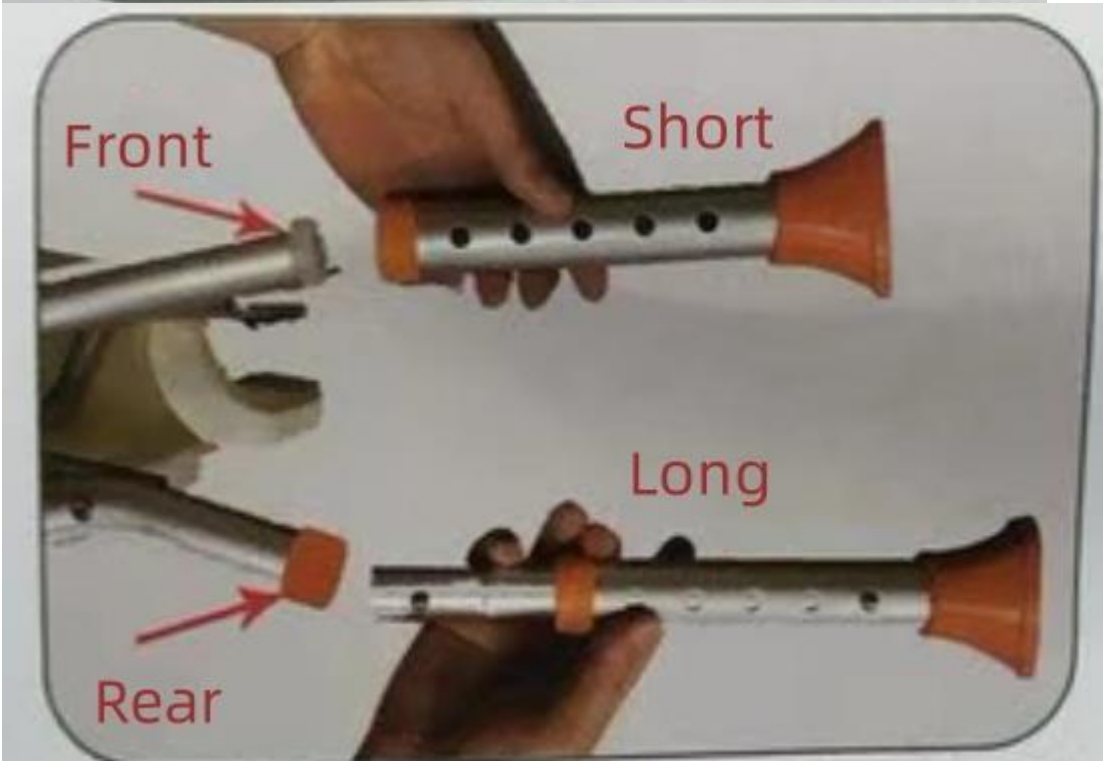

3. □□□2□□□□□□□□□□□□□□□□2□□□□□□, □□□□□□□□□□□□


Attach the 2 short leg tubes to the front legs of the chair with 2 white plugs, insert them and adjust them to the lowest position.


□ □ □

A diagram of a graduated cylinder with a blue stopper on the left and a blue pouring spout on the right. The cylinder has a scale with 10 major markings. A red arrow points to the 4th major marking from the left, which is labeled '4'.

1. To adjust the height of the chair, you need to press the button on the leg tube, and then extend and retract the leg tube up and down to adjust the height of the chair.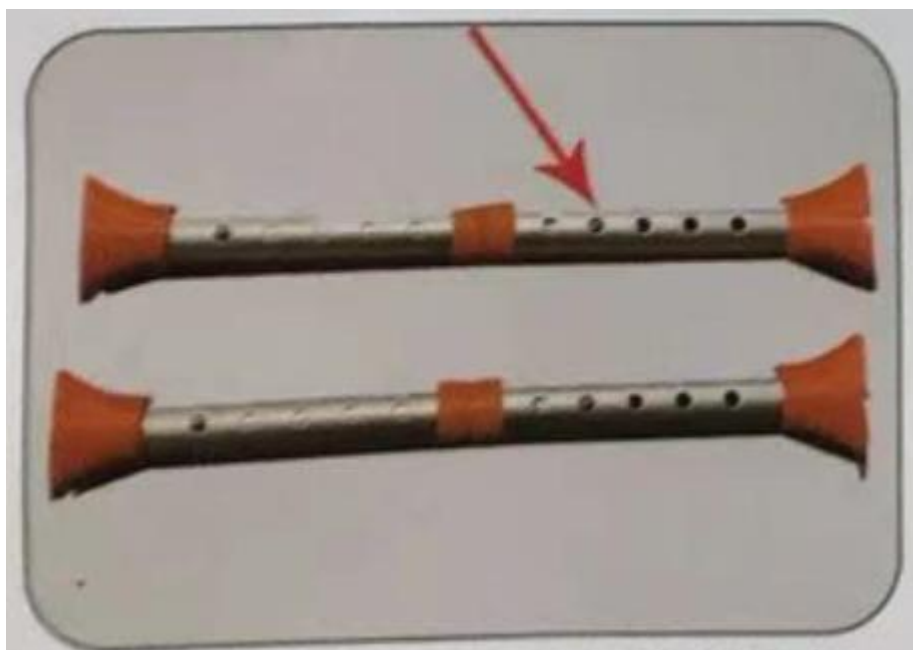

2. Locate the leg tube, at the center of the marble, and press down on the marble to pull out the leg tube.

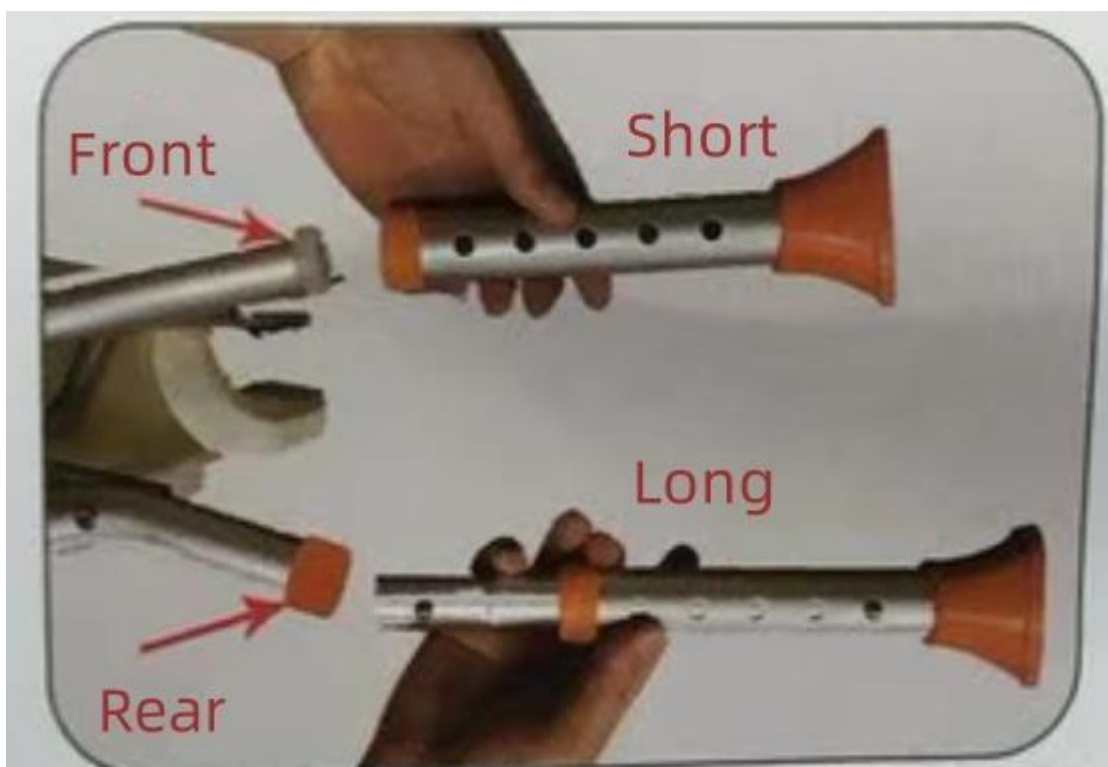3. Attach the 2 short leg tubes to the front legs of the chair with 2 white plugs, insert them and adjust them to the lowest position.

4. Install the 2 long leg tubes on the back legs of the chair with 2 hollow sleeves and mount them with a firm push-in until the marble pops out.

5. With the leg tubes installed, open the chair. There are switches on the left and right sides of the armrest for folding the tubes, which will lock when they pop out.

6. To fold the chair, press the folding switch buttons on both sides of the armrests at the same time. Stand behind the chair and lift the front of the chair upwards to fold it.