

## Scope of application and instructions

Remove Demo Watermark from <http://www.wancainfo.com/>

1. This shower chair is an assistant facility for disabled people to take a bath more conveniently.
2. Before using the shower chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole.
3. Check seat board, the backrest are securely installed, please sit in the middle position of the Shower chair when using it.
4. To release suction, use the pull tabs on the suction cups. Failure to do so will cause damage to the product.

## Storage Method

Please put the shower chair in a dry and ventilated place. Do not exposure it under the sun for a long time. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

## Precautions

To prevent the injury caused by shower chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water for a long time and keep the board dry.

## WARNINGS:

- ⚠️ Prior to each use check to assure that all screws, joints and fittings on the bath bench are tight. All four leg extensions with rubber tips **MUST** touch the floor simultaneously at all times. Be sure that all gold locking buttons fully protrude thru holes of each Adjustable Leg.
- ⚠️ DO NOT place bath bench on uneven surfaces.
- ⚠️ DO NOT use if wobbly, unstable or not level.
- ⚠️ DO NOT exceed user static pressure bearing weight of 150kg(330lbs)
- ⚠️ DO NOT stand on the bath bench.
- ⚠️ DO NOT use the armrest to fully support body weight when using the unit.
- ⚠️ Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions
- ✅ Excessive twisting or turning while seated on the bench may cause a loss of balance and damage to the product.
- ✅ Check rubber tips for rips, cracks or wear. If these conditions exist, DO NOT use the bath bench and replace the tips immediately.
- ✅ Check the bench for cracks or ruptures in the plastic.
- ✅ Check that all push buttons are fully protruded from the mainframe through the leg extensions and are properly engaged.
- ✅ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use.



## SHOWER BENCH CST-3062

**CAUTION**  
UP TO 330LBS



To ensure the safe installation, adjustment, and operation of the shower chair, these instructions **MUST** be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER AND/OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions, and instructions, contact a professional or dealer before attempting to use this equipment. In particular, we are not to be held liable for any indirect or direct damage or consequential damage, loss or expenses in connection with the use or inability to use the device for a specific purpose.

## SHOWER BENCH CST-3062

Thanks for using our product.  
Before using, please read the manual carefully.



1. Remove contents from carton.
2. Lay transfer bench upside down on a flat surface.
3. As shown, insert the tube into the other tube until the marble pops out. (Figure 1)

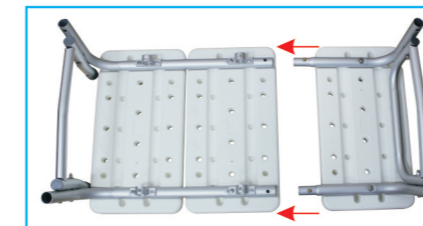


Figure 1

4. Attach leg with the large suction cup (marked "1") by depressing the brass push pin and inserting into the receptacle with the matching "1", Repeat for second leg (Figure 2).



Figure 2

5. Attach leg with small rubber tip (marked "2") by depressing brass push pin and inserting into the receptacle with matching "2", Repeat for second leg (Figure 3).



Figure 3

6. Turn transfer bench right-side-up, Attach armrest by depressing brass push pins and sliding into the arm receptacle on the side of the bench, Make sure the arm arcs outward, not inward (Figure 4).



Figure 4

7. To assemble backrest, insert pole into backrest support by depressing push pin and sliding into receptacle, Repeat for second pole (Figure 5).

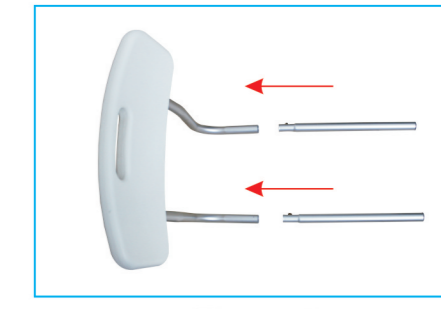


Figure 5

8. Insert backrest into base by depressing push pins and sliding supports through both receptacles. Note: backrest must go through both sets of receptacles to become secure (Figure 6).



Figure 6

**NOTE:** back support is reversible to accommodate left or right tub entry, You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

9. Place legs with suction cups into bath tub (Figure 7).



Figure 7