




Massage Chair

**R8328
Manual**

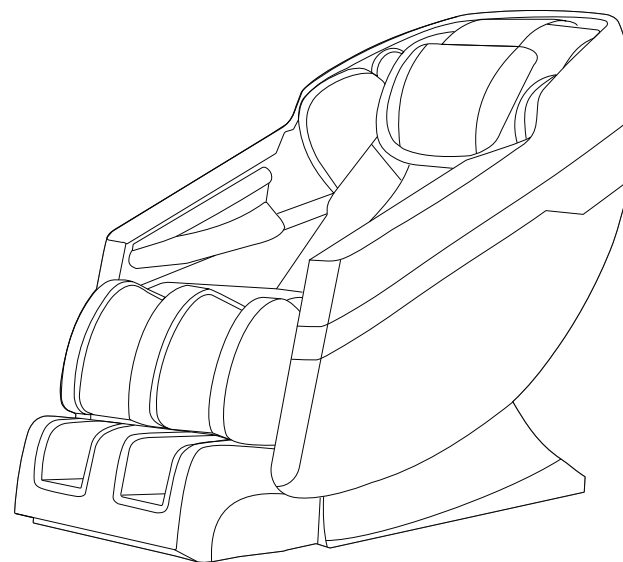

Questions, problems, missing parts? Email our technical support department for fastest response at support@scsources.com or call 657-341-0362, 9:00 a.m. - 5 p.m., CST, Monday - Friday
Please have your model and serial number ready.
(Located on the back of your unit)



M_R8328_2026
Printed in China

Thank you for choosing our product.

Please read this manual carefully before use to ensure proper operation. Keep this manual for future reference.

We reserve the right to update product design and specifications without prior notice as part of ongoing improvements.

Contents

Safety	1-6
Preparation Before Use	7-9
Move the Chair	7
Place the Chair	7
Installation	8-9
Components	10-13
Outer Structure	10
Controller	11
Function of Controller	12-13
Cleaning and Maintenance	14
Troubleshoot	15
Specifications	16

IMPORTANT SAFETY INSTRUCTIONS

This massage chair is intended for household only. When using an electrical appliance, basic precautions should always be followed, including the following:

Read all instructions before using this massage chair.

DANGER

-To reduce the risk of electric shock:

- 1) Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- 2) Never use pins or other metallic fasteners with this appliance.
- 3) Carefully examine the covering before each use. Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.
- 4) Keep Dry - Do not operate in a wet or moist condition.

WARNING

-To reduce the risk of burns, fire, electric shock, or injury to persons:

- 1) An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- 2) Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- 3) Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons. Keep children away from extended foot support.
- 4) Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- 5) Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
Return the appliance to a service center for examination and repair.
- 6) Do not carry this appliance by supply cord or use cord as a handle.
- 7) Keep the cord away from heated surfaces.
- 8) Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- 9) Never drop or insert any object into any opening.

- 10) Do not use outdoors.
- 11) Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- 12) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 13) Temperatures sufficiently high to cause burns may occur regardless of the control setting. Do not use on an infant or invalid or on a sleeping or unconscious person. Do not use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
- 14) Do Not Crush - Avoid sharp folds.
- 15) Do not use massager in close proximity to loose clothing or jewelry.
- 16) Keep long hair away from massager while in use.
- 17) Connect this appliance to a properly grounded outlet only. See Grounding Instructions.
- 18) The apparatus shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
- 19) An apparatus with CLASS I construction shall be connected to a MAINS socket outlet with a protective earthing connection
- 20) The appliance coupler is used as the disconnect device, the disconnect device shall remain readily operable.
- 21) This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- 22) If it is not in use for a long time or after use, please turn off the power and pull out the power plug.
- 23) Do not damage, break, over-bend, compress, tighten, wind, rotate or knot the power cord.
- 24) This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

SAVE THESE INSTRUCTIONS

This massage chair is intended for household use only.

● Usage Environment

- Do not use the chair in excessively humid or dusty environments. This may cause malfunction or electric shock.
- Do not use the chair in rooms with temperatures above 104°F (40°C).
- Keep the chair away from heat sources such as heaters, stoves, and direct sunlight.
- Place and use the chair on a stable, level, non-slip surface.
- This product is for home use only and is not intended for commercial use.

Caution

- For indoor use only.
- If using the chair in a cold room, do not raise the room temperature suddenly. Increase the temperature gradually to normal levels.
- If the chair has been stored in a cold environment, allow it to sit at room temperature for at least 1 hour before use.
Condensation may form on internal components, which can affect performance or cause malfunction.

IMPORTANT SAFETY INSTRUCTIONS

● Safety Precautions

- Read this manual carefully before using the massage chair to ensure safe and proper operation.
- Do not use the chair if the upholstery or seat is removed. This may cause injury or damage.
- Do not use the chair together with other therapeutic devices (such as electric blankets). This may cause injury or reduce effectiveness.
- Do not use the chair within one hour before or after eating. This may cause discomfort or adverse effects.
- Do not allow children or pets near the chair during operation, especially behind the backrest or under the seat or leg rest. This may result in injury.
- Do not place heavy objects on the armrests, leg rest, or backrest. This may cause damage or injury.
- Do not use the massage chair if your body is wet.
- Do not operate the chair with wet hands.

- Limit each session to 30 minutes or less.
- Do not massage the same area for more than 5 minutes at a time to avoid overstimulation.
- Do not place your hands or feet near the massage rollers during operation. This may cause injury.
- Stop using the product immediately if you feel discomfort.
- This product is not intended to treat medical conditions. Consult a qualified healthcare provider if needed.
- If you have a medical condition or are taking medication, consult your doctor before use.
- Do not use the product if you are feeling unwell.
- Remove scarves, neckties, necklaces, and other loose items before use.

● People with Medical Conditions

Consult your doctor before using this product if you:

- Are under medical rest as prescribed by a doctor
- Have a spinal condition, spinal injury, or abnormal spine
- Have back problems
- Have diabetes, osteoporosis, or reduced sensation
- Have joint problems
- Use a pacemaker or other electronic medical device
- Are pregnant or may be pregnant
- Have phlebitis or thrombosis
- Are at increased risk of blood clots
- Have had recent surgery
- Have surgical implants such as pins, screws, or other metal devices.

● Instances When the massage chair Should Not Be Used

If any of the following occur, turn off the power immediately and unplug the chair:

- Liquid is spilled on the chair. This may cause electric shock or malfunction.
- The backrest fabric is torn and internal components are exposed.
- You feel pain or discomfort during use. Stop immediately and consult your doctor.
- The chair is not working properly or shows any unusual behavior.
- There is a power outage. The chair may restart unexpectedly when power is restored.
- There is lightning or a storm.

Safety

● Assembly and Repair of the massage chair

- Do not disassemble the backrest cover of the massage chair. Touching the internal components may result in malfunction or electric shock.
- The massage chair must only be professionally serviced or repaired. Do not attempt to disassemble or repair the massage chair yourself.

● Things to Note About the Power Plug and Cord

- Check that the Alternating Current (AC) voltage corresponds with the specifications indicated for the massage chair.
- Do not connect or disconnect the power plug from the electrical outlet with wet hands. This may result in malfunction or electric shock.
- When disconnecting the power plug, pull it by holding the plug, not the cord.
- Although operation will be stopped automatically by the Auto-Timer function, always remember to switch off the main power switch after use.
- Do not place the power cord under the massage chair or any other heavy objects.
- Do not wind the power cord around the massage chair as this may damage the cord and result in a fire or electric shock.
- Do not operate the massage chair with a damaged cord or extension cord.
- Do not use the massage chair if the electrical outlet is loose.
- If the power cord is damaged, it should be replaced by a qualified technician.


● Grounding Instructions

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER - Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product - if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

Safety


This product is for use on a nominal 120V~ circuit and has a grounding plug that looks like the plug illustrated in sketch A as following illustration. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

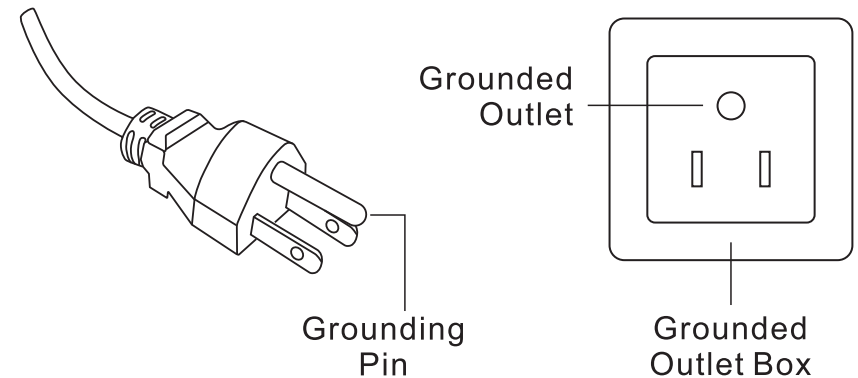

Preparation Before Use

Moving the Chair


1. Turn off the power and unplug the chair from the electrical outlet.
2. Tilt the chair backward as shown and roll it to the desired location.
3. Place a mat or protective covering on the floor to prevent damage during movement.

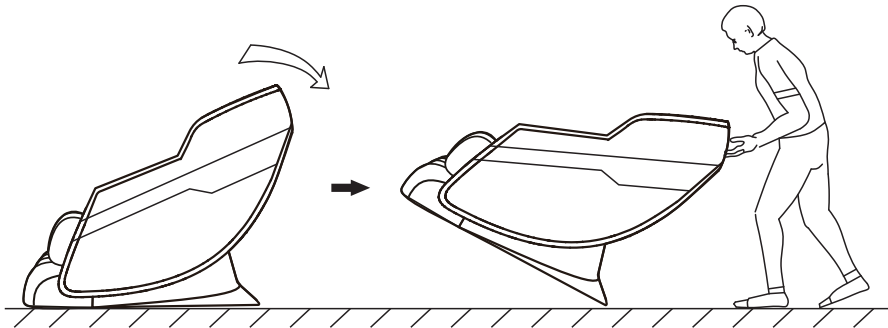

Placing the Chair

Ensure there is at least 20 in (50 cm) of clearance in front of and behind the chair.

This allows the backrest to recline fully without obstruction.

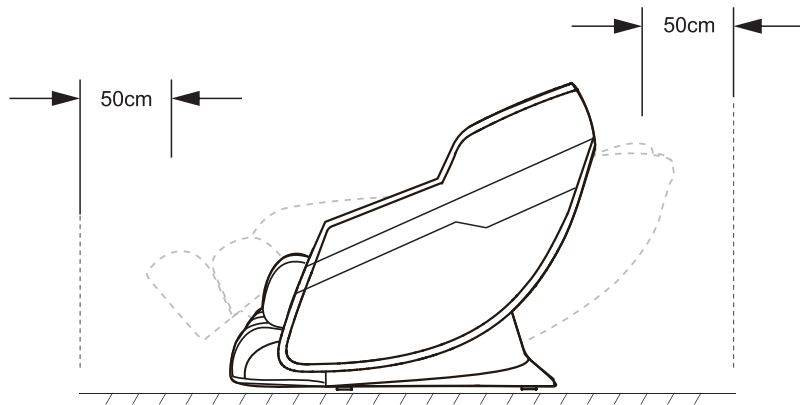

Preparation Before Use


Installation

1. Unpack


- ▶ Remove all packaging and protective wrapping. Confirm all parts are included.

2. Installing the Leg Rest

- ▶ Locate, nuts and bolts from accessory bag.
- ▶ Attach leg rest onto anchor points A and B. Slide cotter pin (E) onto post (B) to secure.

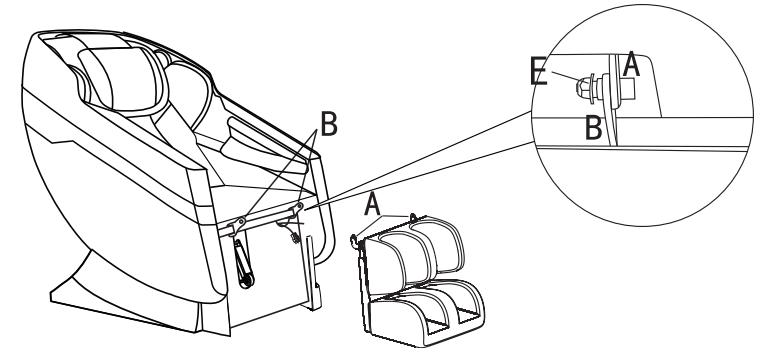


- ▶ Lift the leg rest and connect the air tubes to the ports on the leg rest.

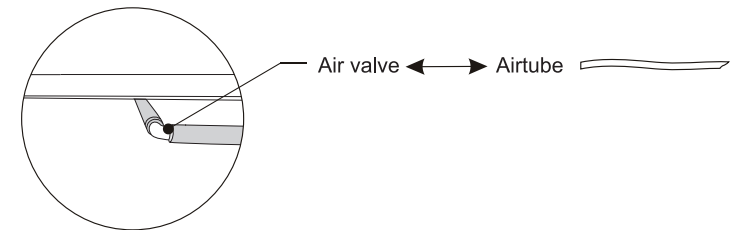


3. Install Cushion

- ▶ Place cushion with air hose facing inward.
- ▶ Connect air hose to seat frame.

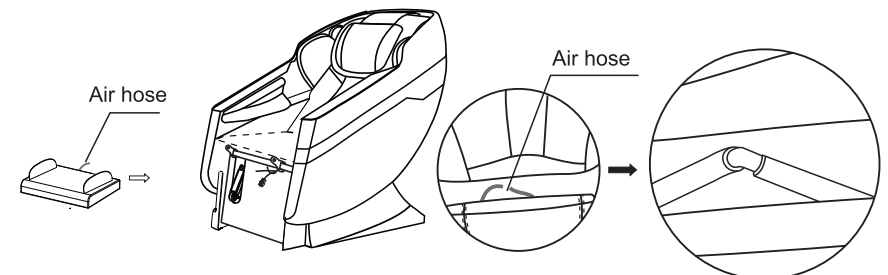


Preparation Before Use

- ▶ Connect the heater cable from the backrest to the seat (Fig. 2).
- ▶ Zip the backrest and seat together (Fig. 3).

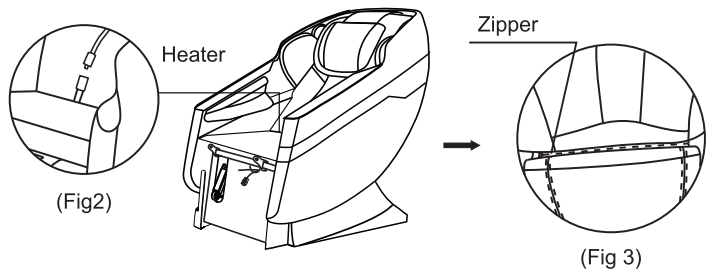


4. Install Backrest Pad and Pillow

- ▶ Zip the backrest pad onto the chair.
- ▶ Attach the pillow using the Velcro.

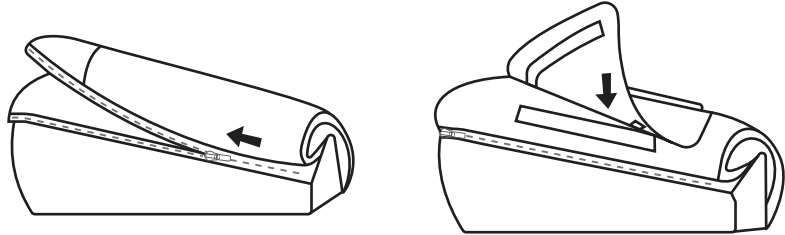



5. Power Supply

- ▶ Plug the power cord into the socket and turn on the switch.
- ▶ Press the power button on the remote to start.

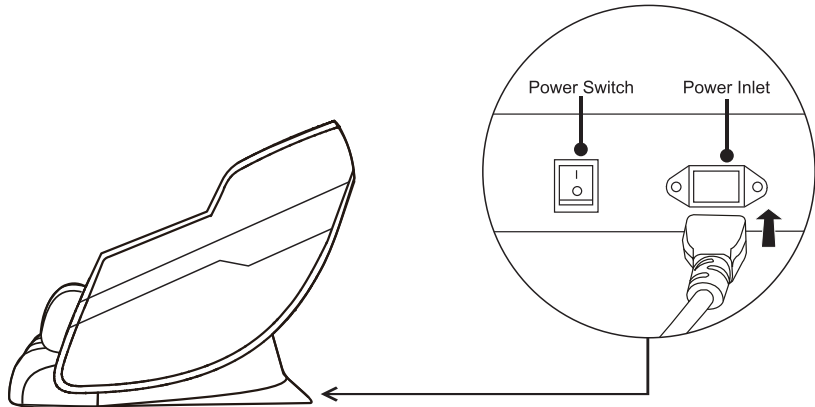

Components

Outer Structure

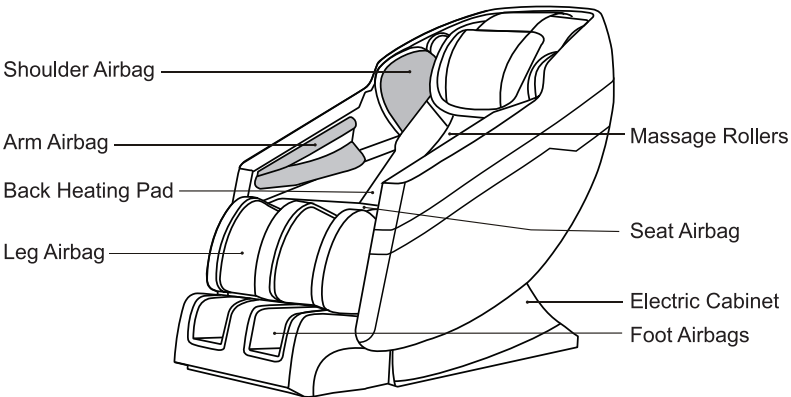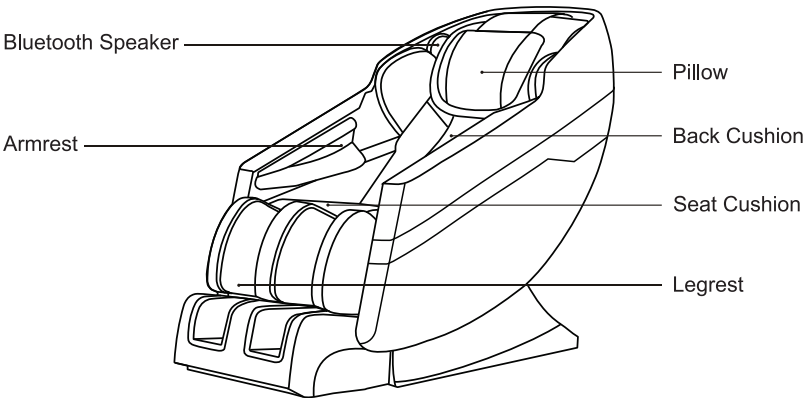

Components

Controller

(The specific display is subject to the actual product.)



Caution

- Do not sit, step or place heavy objects on the controller.
- Do not drop the controller. It is recommended that the controller unit be placed in the controller side pocket when not in use.
- Do not exert excessive force on the controller.
- Do not handle the controller with wet hands.
- Do not operate the controller by using sharp-edged or hard objects. It may result in damage.

Components

Function of controller

FUNCTION	ICON	DESCRIPTION
Power		Press to turn on/off.
Zero-G		Press to toggle Zero-G mode on/off.
Pause		Press to pause massage function. Press again to resume.
Bluetooth		Press to turn on/off Bluetooth.
Heat		Press to turn on/off heating.
Roller Speed+ / Roller Speed-		Press + to increase roller speed in manual mode.
		Press - to decrease roller speed in manual mode.
Backrest Up / Down		Press and hold to adjust backrest position up. Release to lock.
		Press and hold to adjust backrest position down. Release to lock.
Neck & Shoulder		Press to turn on/off massage rollers.
		Press again to change roller direction.
Timing		Press to cycle off-timer function: 15 / 20 / 25 / 30 min
Waist & Back		Press to turn on/off.
		Press again to change roller direction.
Airbag		Press to turn airbag massage on/off (shoulders, arms, hips, calves, feet).
Airbag Intensity		Press this button to increase the airbag intensity.
		Press this button to decrease the airbag intensity.


Components

Auto Mode

FUNCTION	DESCRIPTION
Refresh	Full body massage, strong roller and airbag intensity.
Relax	Full body massage, medium roller and airbag intensity.
Yoga	Full body stretch-style massage.
Neck & Shoulder	Focused massage for neck and shoulders.
Waist & Back	Focused massage for lower back and waist.
Dream	Full body massage, light roller intensity.

Bluetooth Pairing

On your phone Bluetooth list, choose the model “R8328 ” to do the pairing.

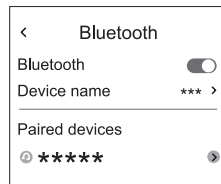

Cleaning and Maintenance

1. Storage

- Wipe with a soft, dry cloth after use.
- Keep clean and away from humidity.
- For long-term storage: cover and unplug.

Caution

- Avoid direct sunlight and high temperatures.
- Do not place against vinyl or wallpaper for long periods (may cause discoloration).

2. Cleaning

- Turn off power and unplug before cleaning.
- Wipe with a soft, dry cloth after use.
- Use a slightly damp cloth for deeper cleaning.
- Do not iron or machine wash upholstery.

Caution

- Do not use harsh chemicals (e.g., benzene, thinner).

3. Controller

- Clean with a dry cloth only.

Caution

- Do not use a wet cloth as it may cause malfunction.

Troubleshoot

Common issues can be resolved following the below steps.

Situations	Solutions
No Power	•Check the power plug is securely connected. •Ensure the main power switch is ON (I).
Controller Works, Chair Does Not	•The chair may be overheated. •Turn off power and allow to cool for 30–50 minutes.
Controller and Chair Not Working	•Check if the timer has ended. Restart the program. •Turn power OFF, then ON again.

Specifications

Model	R8328
Rated Voltage	120V~
Rated Frequency	60Hz
Rated Power	100W
Upright position-Approximately	53.54×29.33×41.18in
Reclined position-Appr	65.19×29.33×25.19in

