
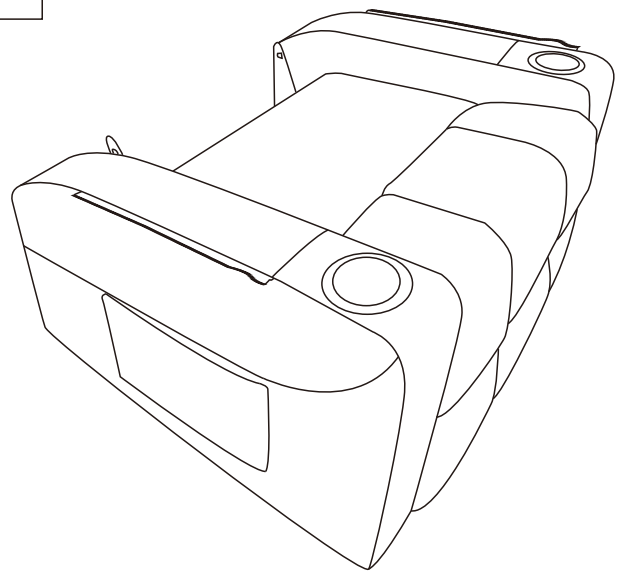
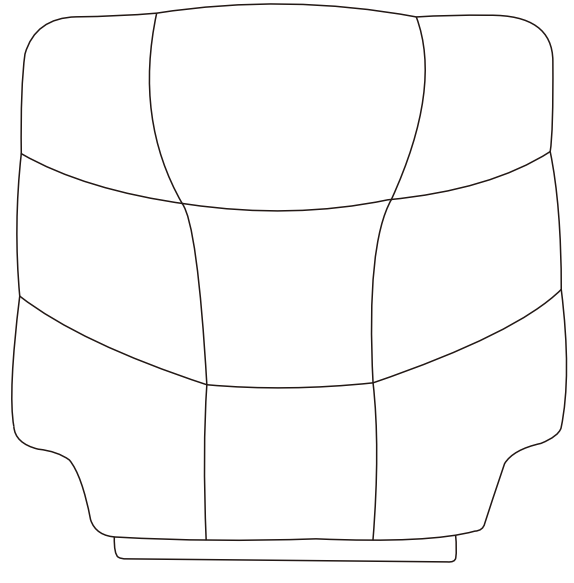
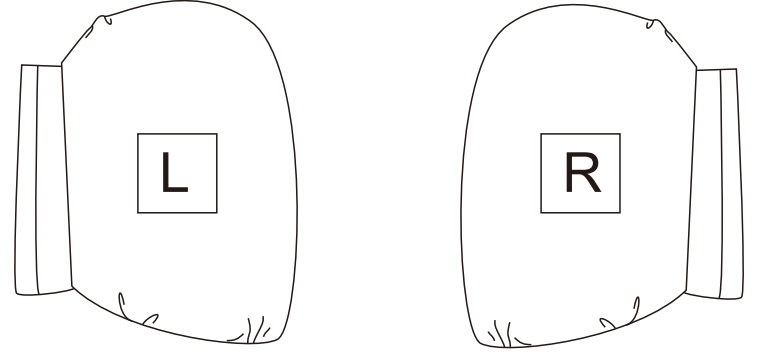
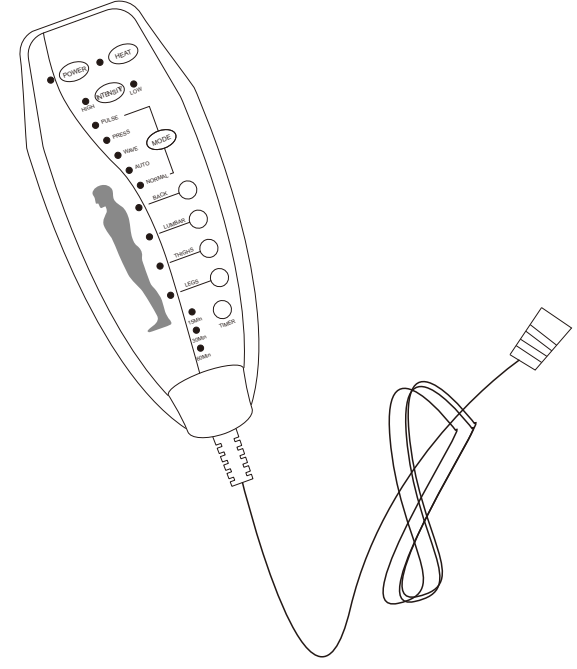
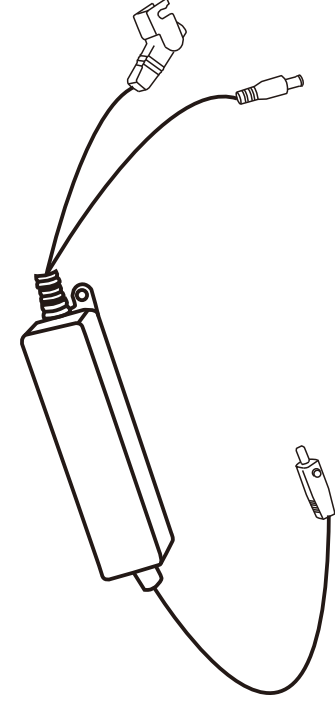
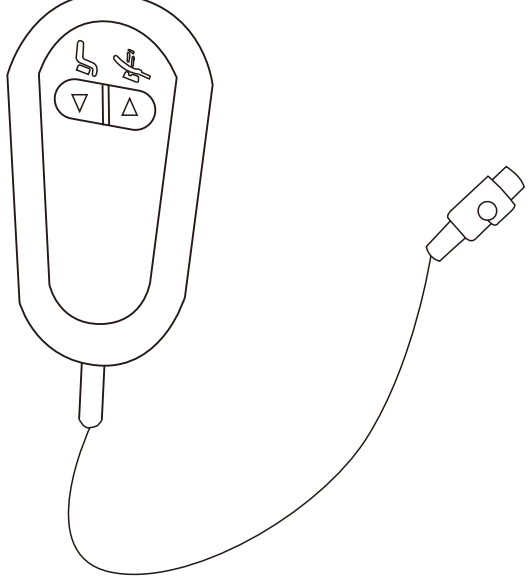
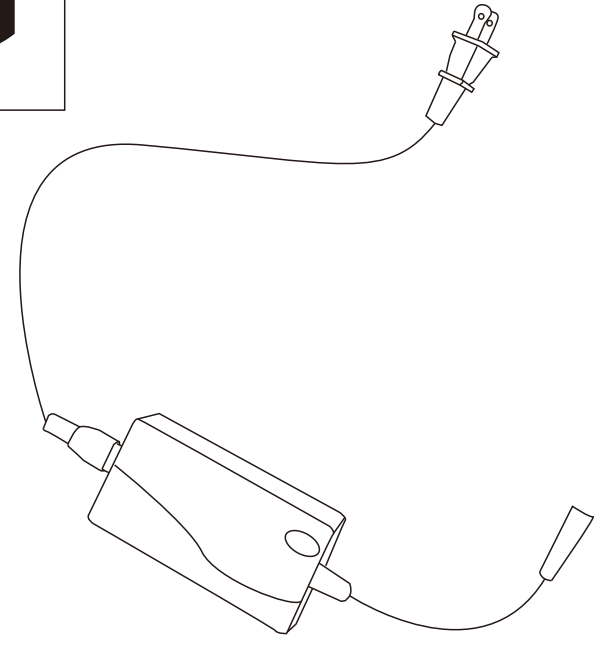


# Assembly Instruction


<b>A</b> 	<b>B</b> 	<b>C</b> 	<b>D</b> 	<b>E</b> 	<b>F</b> 	<b>G</b> 
Chair base	Backrest	Armrest	Remote control	Power converter box	Lift control	Power cord

## STEP 1

Connect the two armrests to the zippers of the armrests, making sure the zippers are in place.

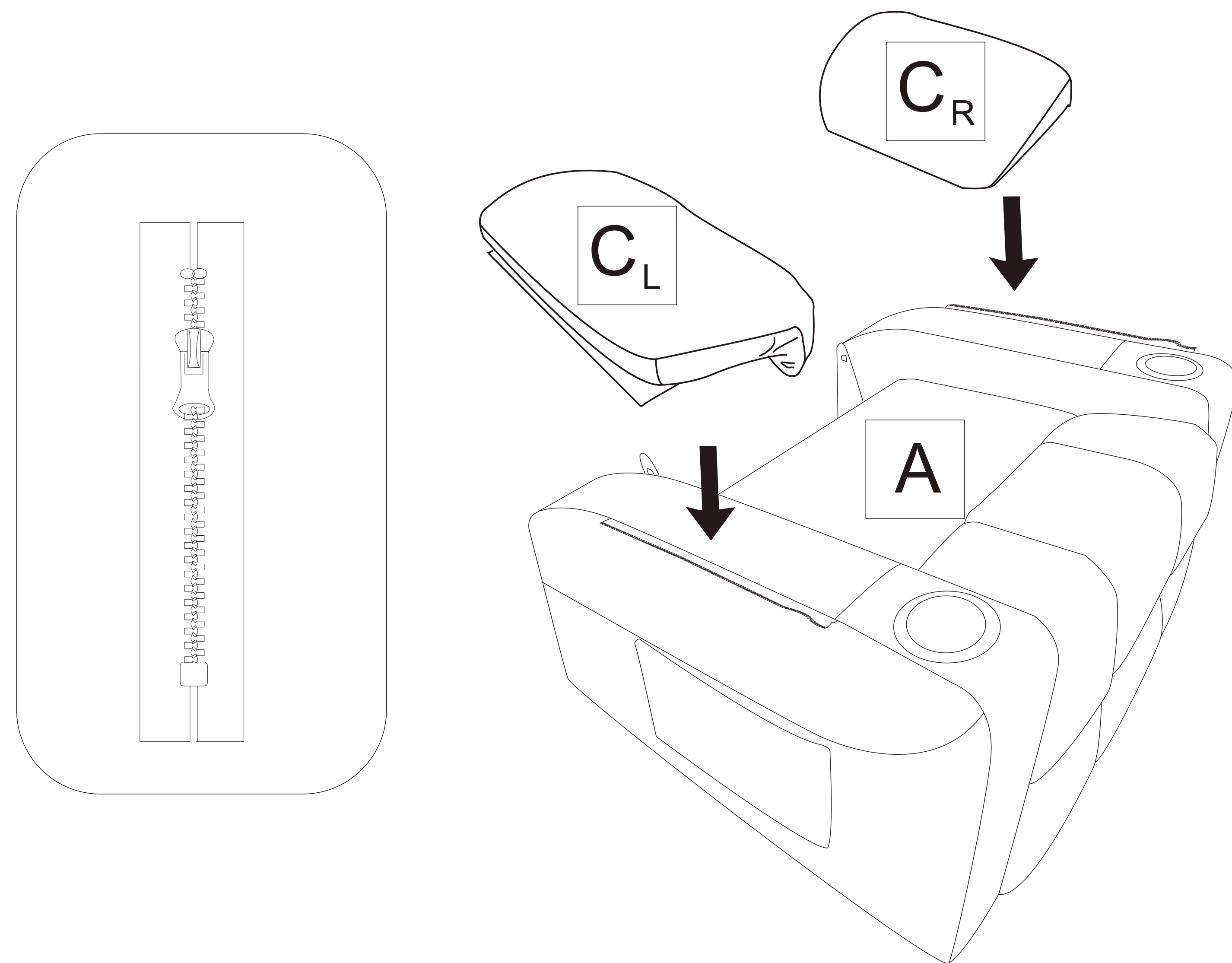

## STEP 2

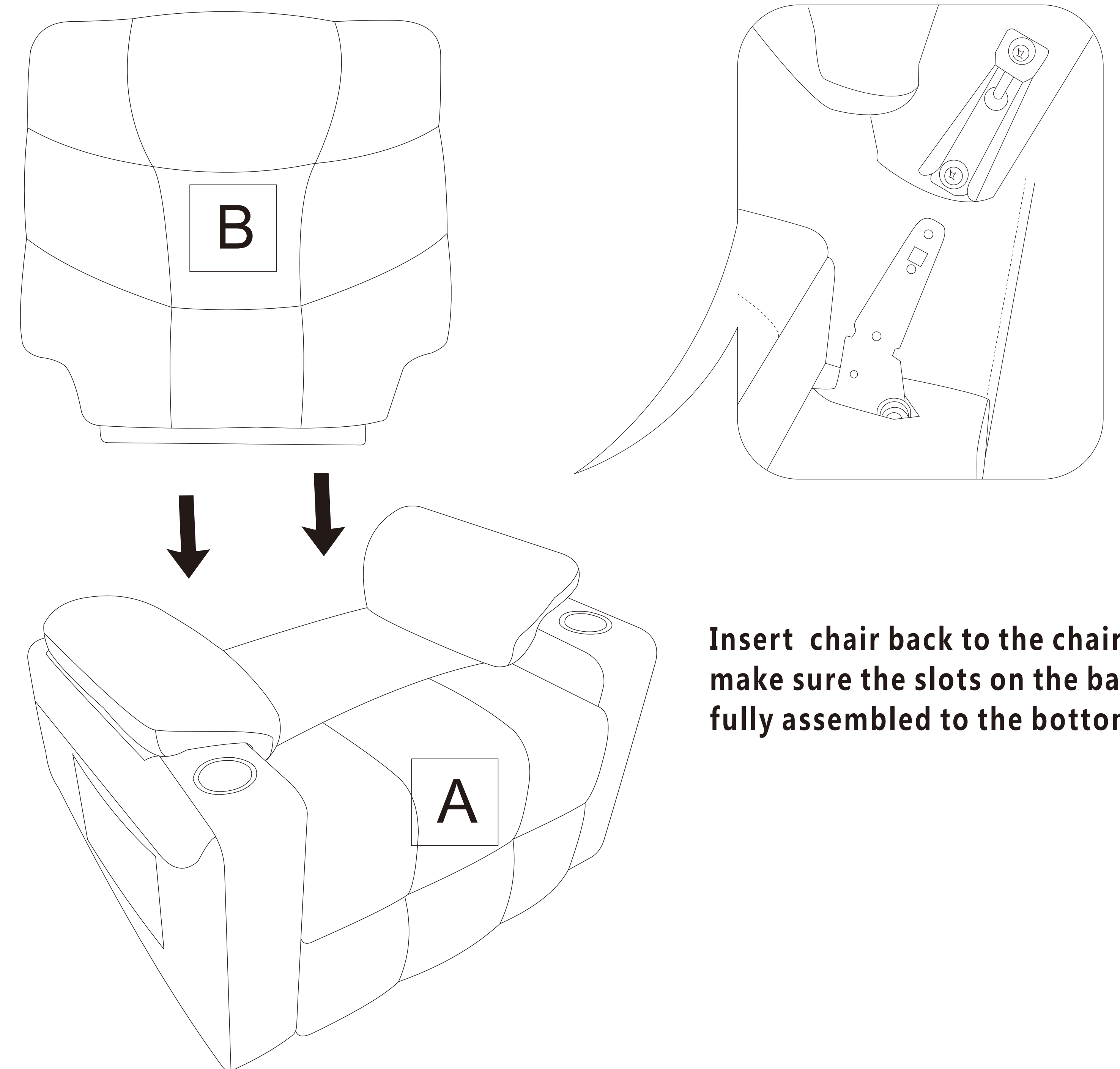

Insert chair back to the chair base, make sure the slots on the back fully assembled to the bottom part.


# Assembly Instruction

## STEP 3

- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power converter box(E)
- No.4 wire connects with backrest and remote control(D)
- No.5 wire connect the motor and Lift control ( F )
- No.6 wire connect the motor and power converter box (E).

### NOTE





- 1: Two same color wires connect.
- 2: Plug power cord and power source into outlet, then you can use all function.

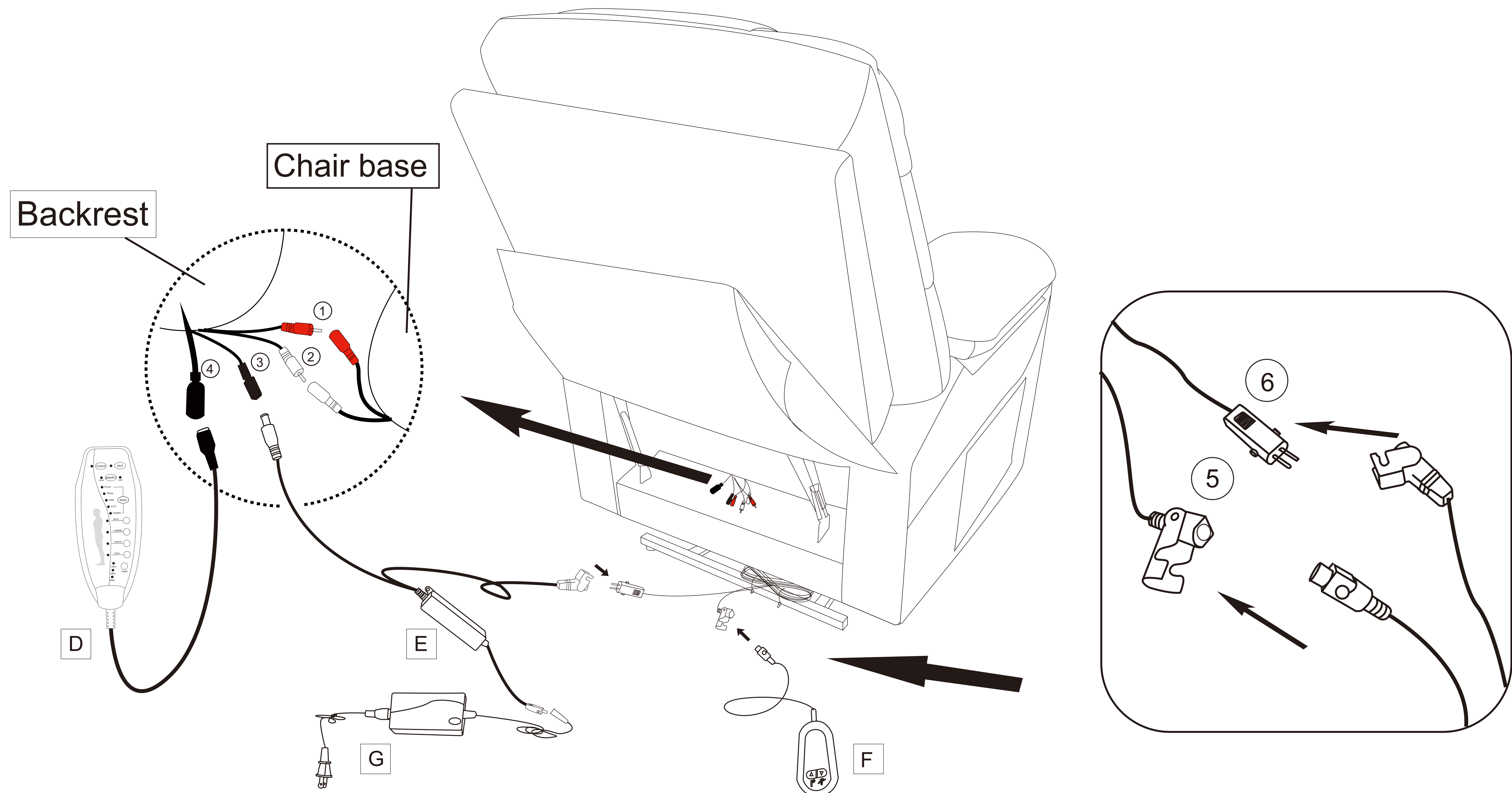

# Assembly Instruction







## How to recline?

Press

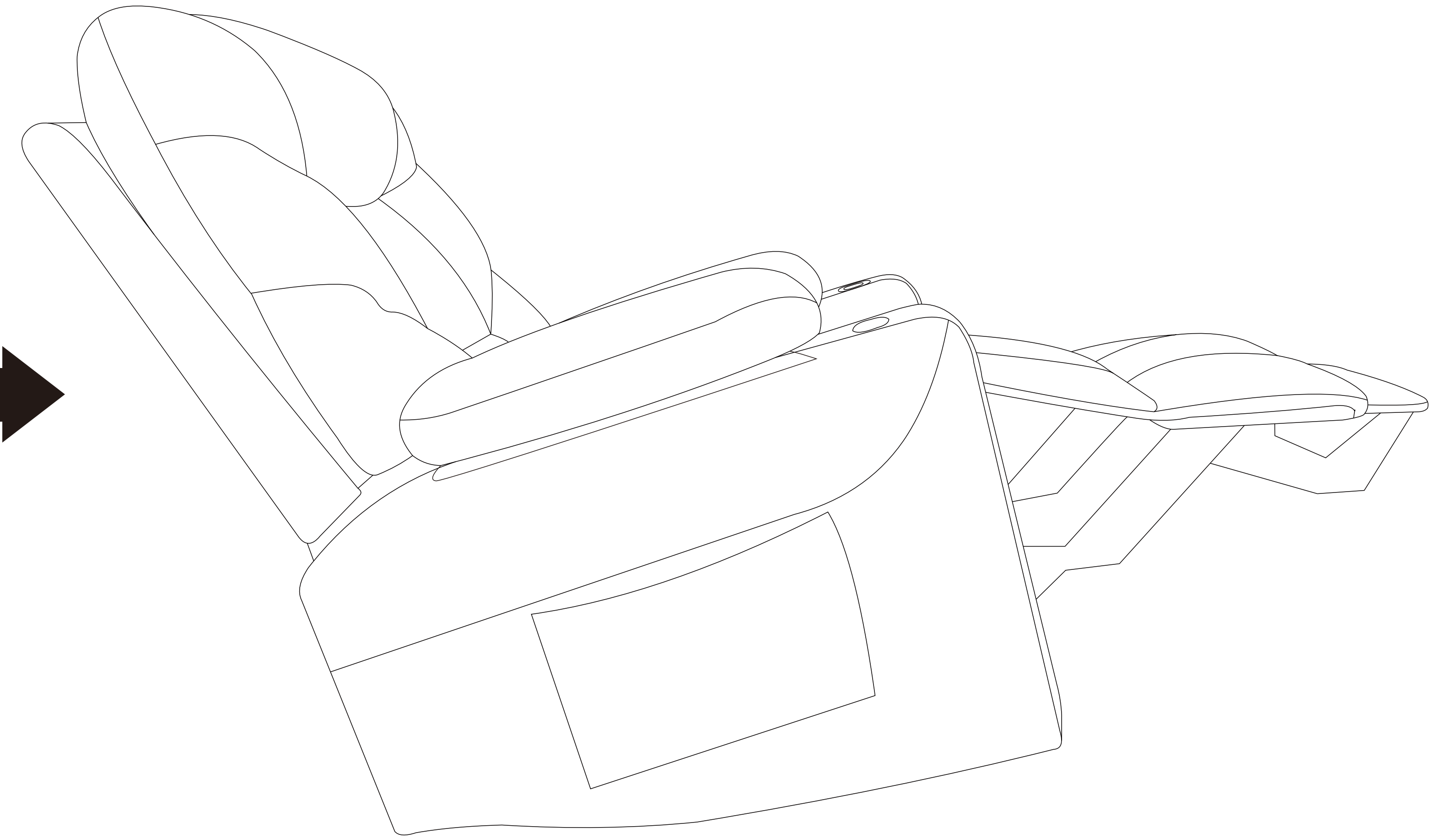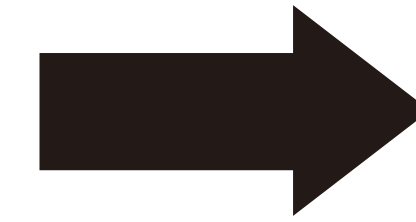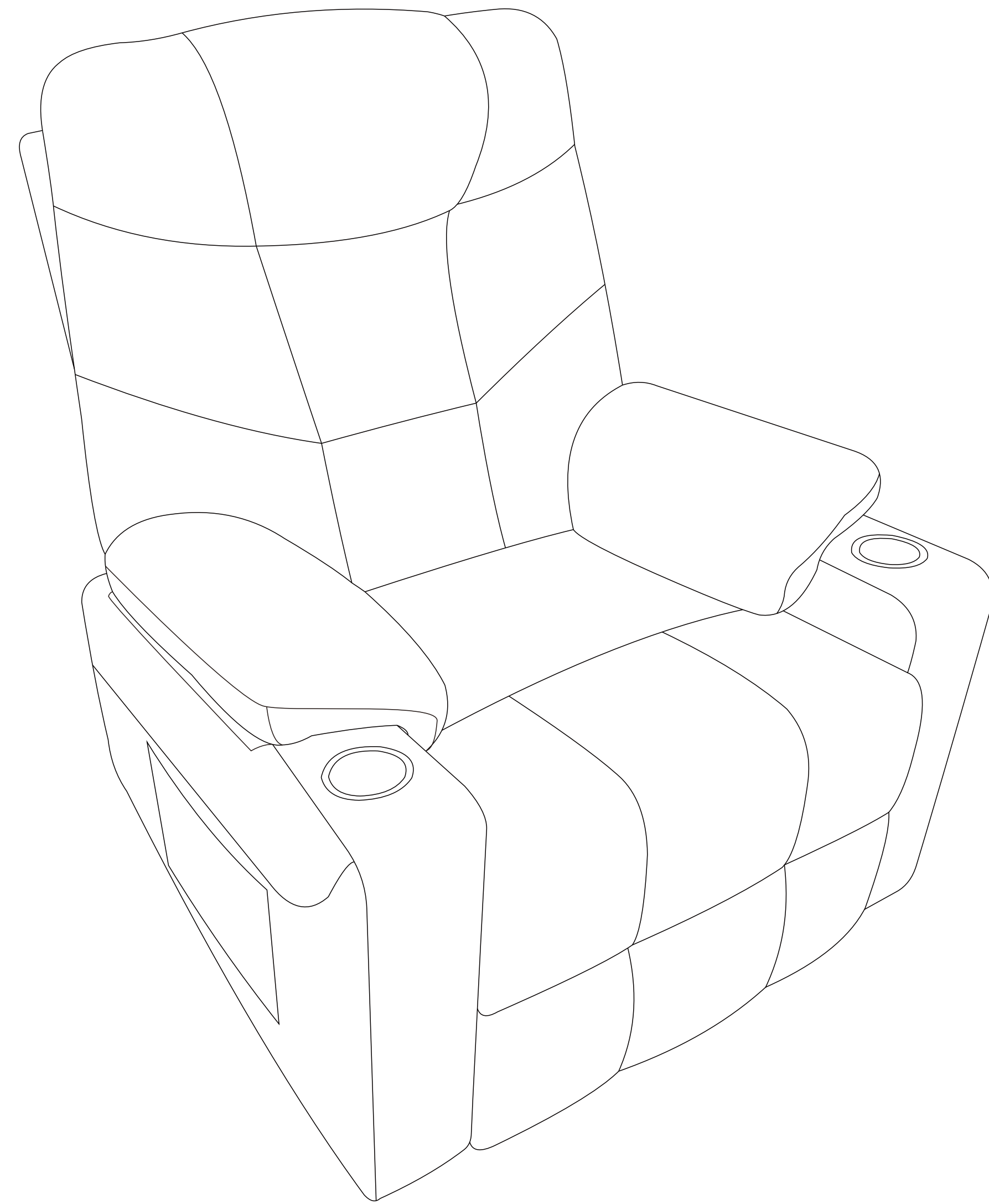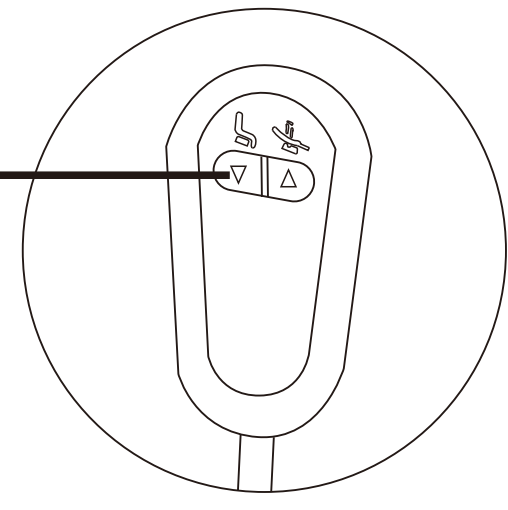

## How to get footrest latch down and stand?


Press

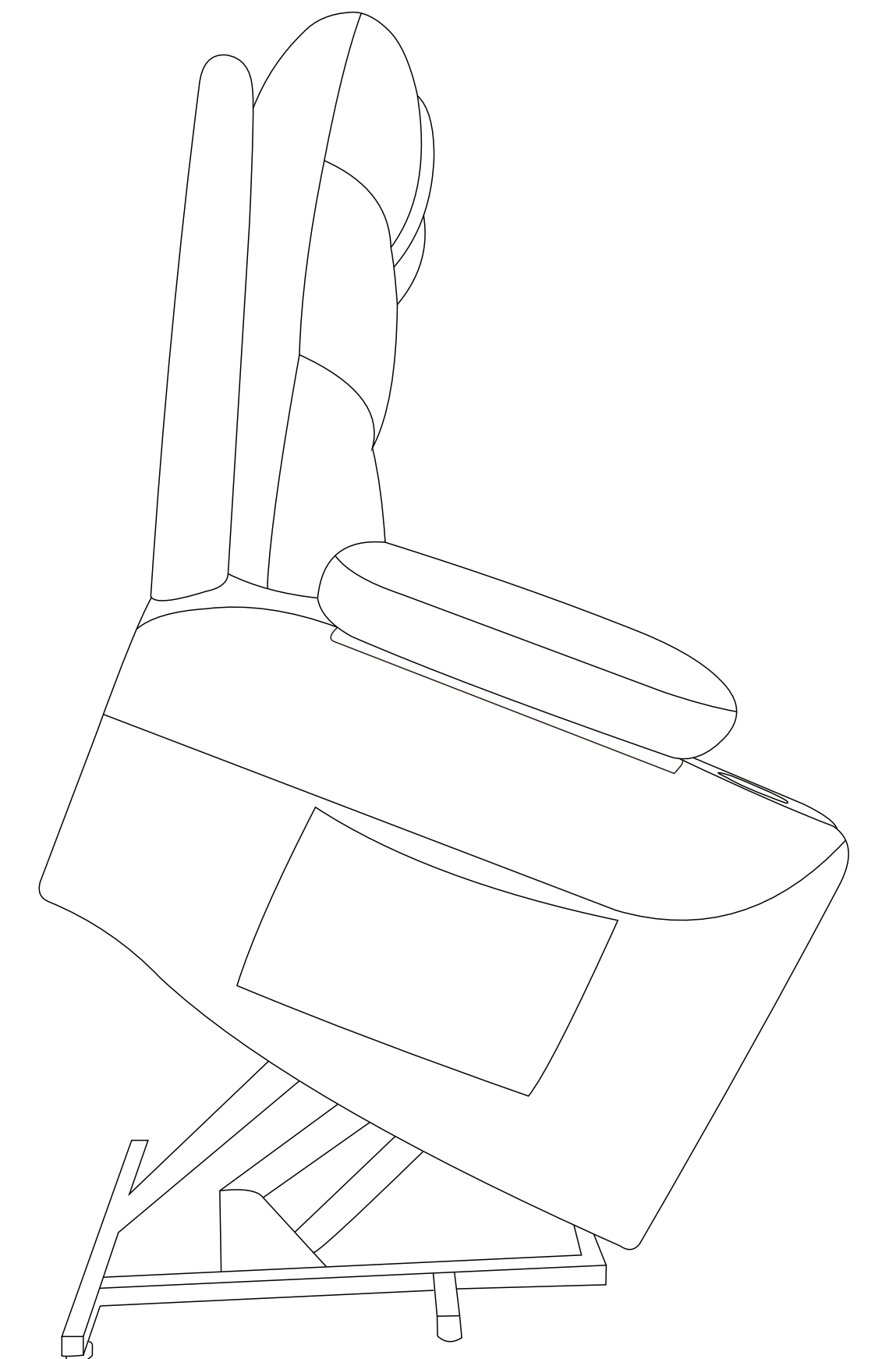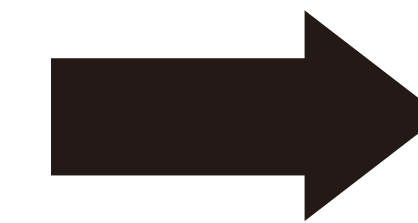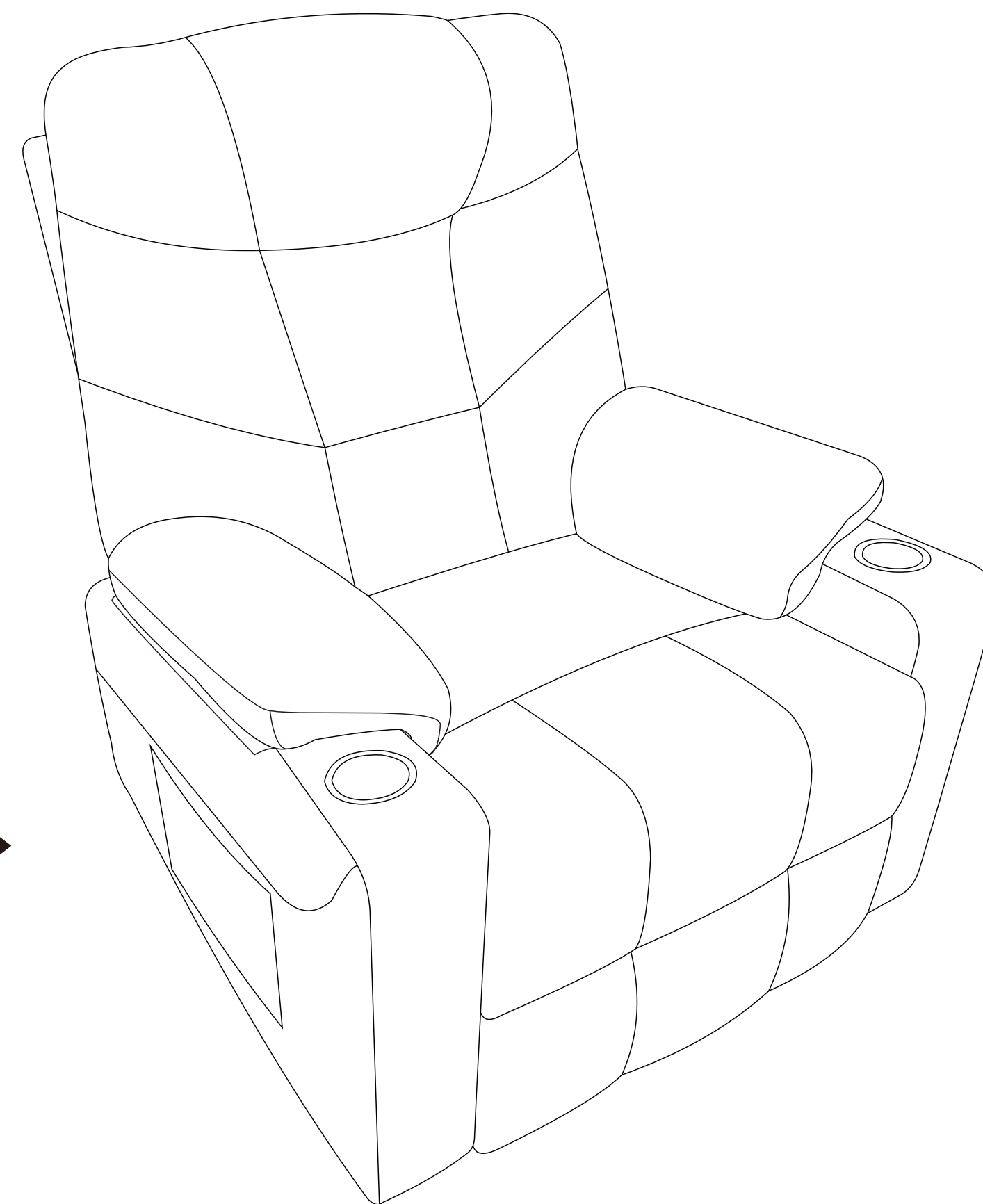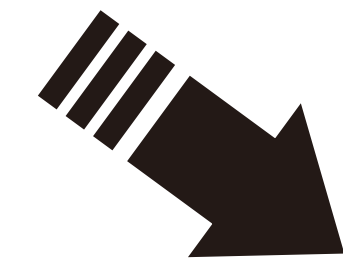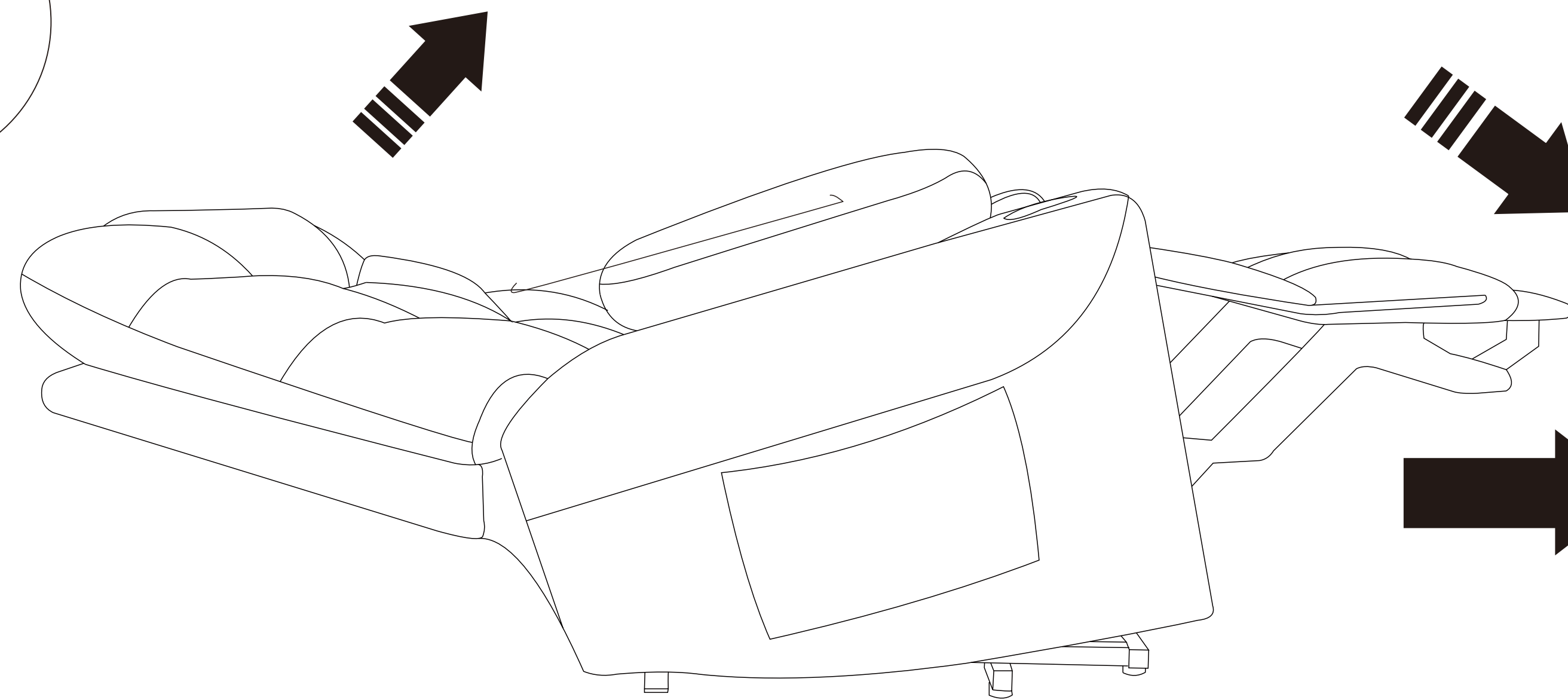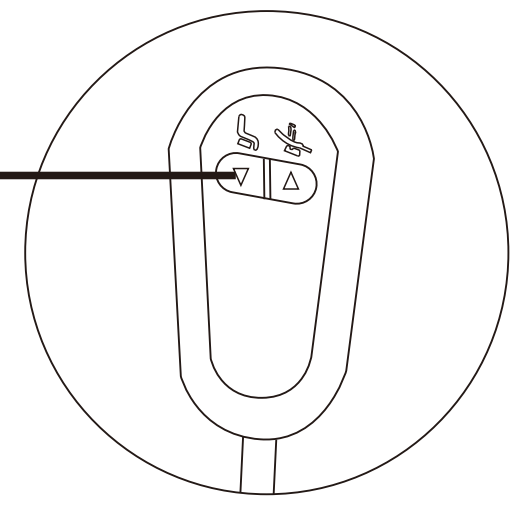

Maximum bearing capacity: 350lbs

# Assembly Instruction

## How to use remote control?

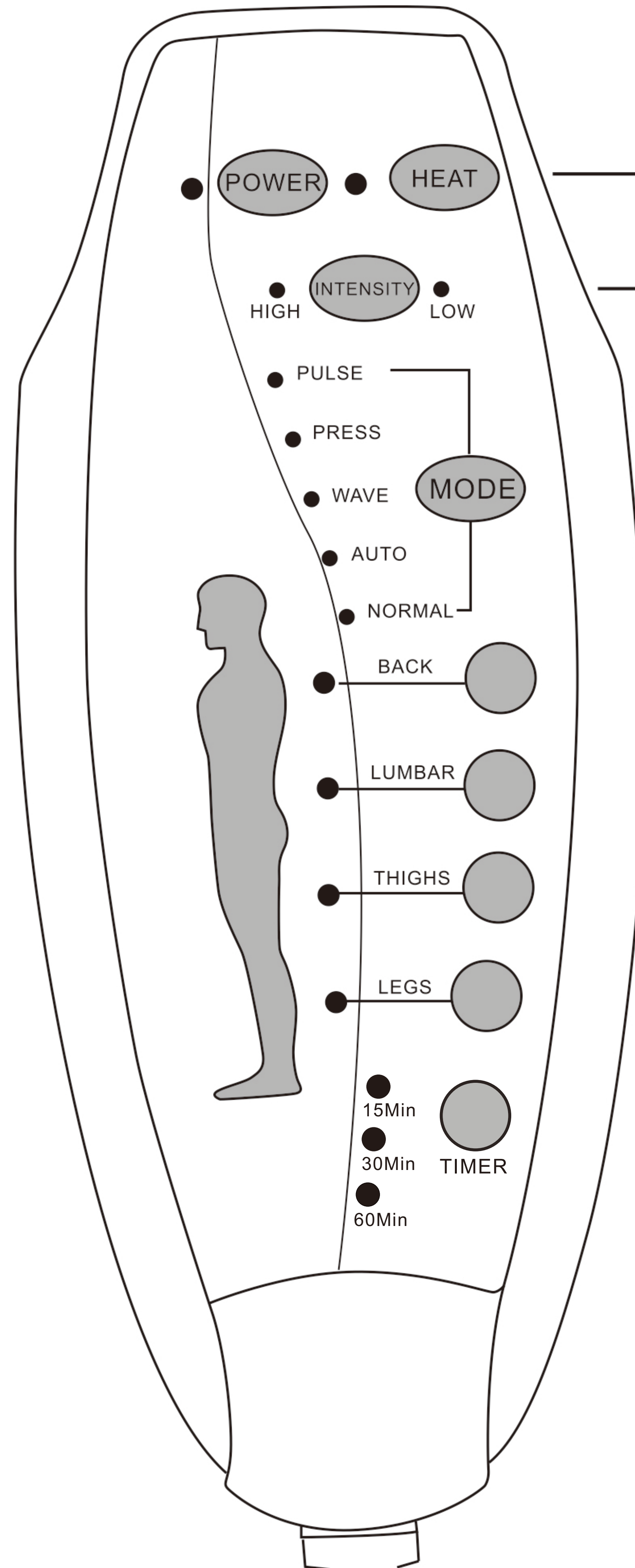

POWER : Turn on/off remote control

HEAT : Turn on/off heat function for lumbar part only.

ITENSIITY : Control massage function from low to high

MODE : Choose one massage mode you like(pulse, press,wave,auto, normal)

You can choose 1-4 massage parts(back, lumbar, thighs, legs)to enjoy.

TIMER 15mins/30mins/60mins

# Assembly Instruction

---

## FAQ

1.Q: Some parts damaged when arrived.

A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

2.Q: The massage function does not work.

A: Please check if all wires connect correctly, the power cord (G) may be defective if indicator does not light on, or if the lights doesn't turn on when you press "POWER" button on the remote, that means the remote is defective. Please kindly send messages with clear photos to describe the problem, then we will send necessary replacement to you.

3. Q: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.

A: Please kindly check if the power converter box (E) and control panel are connected well with lift motor(Wire #5 & #6). Sometimes the power converter box (E) and control panel may fall off from the motor. If they connect well, then check the lift motor if it has some anomaly, like some noise comes out when you press the lift and recline button. If you can make sure the motor is defective, please contact us for replacement.

4. Q: The Heat function does not work or can't feel.

A: Please check all wires connection the same as question 3 first. If the remote also works well, you will need wait for 5-10 mins after you press the "HEAT" button. The heating function only works at lumbar position and the heat temperature isn't very hot.