

ITEM # SGBSET12 / SGBSET16

Suction Grab Bar Set

Max Weight Capacity 66 lbs.



Use the grab bars for support when entering or exiting the bathing area.

READ ALL INSTRUCTIONS AND WARNINGS BEFORE USING THIS PRODUCT.

This manual provides important information on proper operation & maintenance. Every effort has been made to ensure the accuracy of this manual. These instructions are not meant to cover every possible condition and situation that may occur. We reserve the right to change this product at any time without prior notice.

**IF THERE IS ANY QUESTION ABOUT A CONDITION BEING SAFE OR UNSAFE,
DO NOT USE THIS PRODUCT!**

QUESTIONS? CONTACT CUSTOMER SERVICE.

A copy of the sales receipt is required.


Call Customer Service at 1-636-532-9888, Monday-Friday, 8 AM - 4 PM Central Time

KEEP MANUAL, SALES RECEIPT & WARRANTY FOR FUTURE REFERENCE.




To register your product warranty, please visit buffalotools.com or scan the QR code.


Installing the Grab Bar

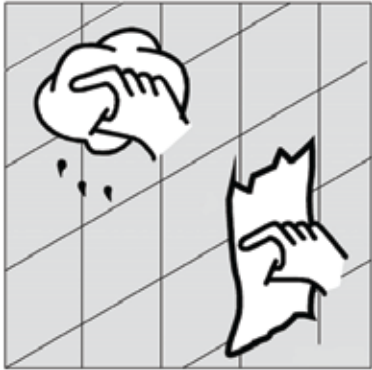

1. Clean and dry both the grab bar and the wall surface.

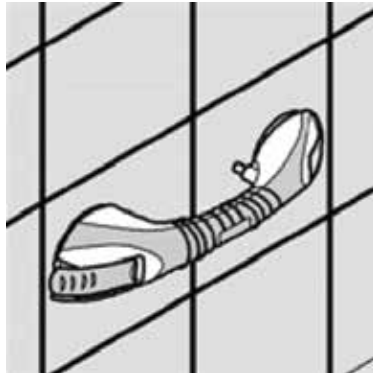

2. Place the grab bar so that it does not overlap any grout or gaps on the wall surface.

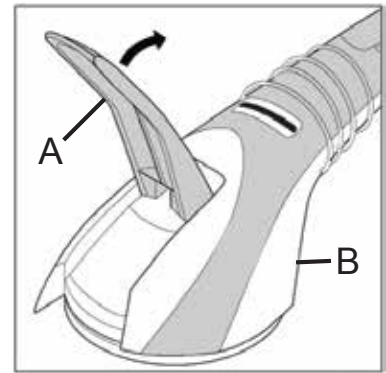

3. Lift the Lever A on each end of the grab bar (B). Position the grab bar on the wall and press Lever A down.

WARNING

The grab bar provides support and increases stability for an individual. The grab bar is not designed to support the full weight of an individual.

WARNING

Damaged grab bars may not adhere to the wall properly.

- DO NOT install grab bars if damage is found.

Grab bars may not get adequate suction to adhere to the wall.

- DO NOT install grab bars directly onto grout lines or gaps in the wall surface.

- DO NOT install grab bars onto dirty or wet walls.

WARNING

- DO NOT use as sports equipment

- DO NOT use as a climbing aid.

- ALWAYS check that the grab bar is secured to the wall before use.

- Use EXTREME caution on wet tub or floor surfaces.

- NEVER use the grab bar if any looseness is detected. Grab bars are only secure as the wall they adhere to.