





## INSTALLATION AND CARE GUIDE

### SUCTION ASSIST BAR

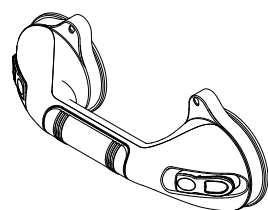

12 inch

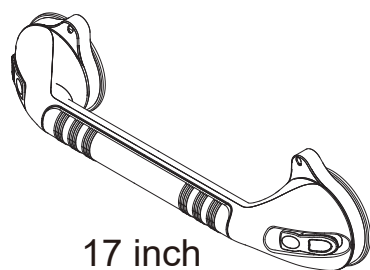

17 inch

REV.02 09/2021

We appreciate the trust and confidence you have placed in AmeriLuck through the purchase of this suction assist bar. We strive to continually create quality products designed to enhance your home.

Your Review Means A Lot!

LOVE it?

Please let the others know including a positive product review on WayFair.

#### Warranty

##### WHAT IS COVERED

This warrants the original purchaser that this product will be free of manufacturing defects under normal domestic use for one year of the product beginning from the original date of purchase.

Present warranty does not extend to anyone except the first purchaser at retail. Also, original sales receipt must be obtained in order for the warranty to be valid. To obtain warranty service, contact the seller from the website.

##### WHAT IS NOT COVERED

Present warranty excludes damages resulting in whole or part from installation error, product abuse, misuse, neglect, any alterations to products, improper maintenance, abrasives, corrosion and/or accidents, whether caused by a professional or anyone else.

Any and all accessories that are subject to any type of harsh abrasive, such as detergents or window cleaners, will not be covered under the warranty. The manufacturer is not responsible for any labor or shipping charges, or damages whatsoever incurred in whole or in part from installation, removal, re-installation, repair or replacement of any product or part, as well as any incidental or consequential damages, expensed, losses, direct or indirect, arising from any cause whatsoever.

#### Safety Information




**WARNING:** Install in accordance with these instructions. Do not modify or alter this product in any way. After installation, check your grab bar periodically to ensure all screws are secure.



**IMPOTANT:** Mount the grab bars to a secure surface. The manufacturer will not accept responsibility for any and all damage caused by lack of wall strength or improper installation.

#### Pre-installation

##### PACKAGE CONTENTS

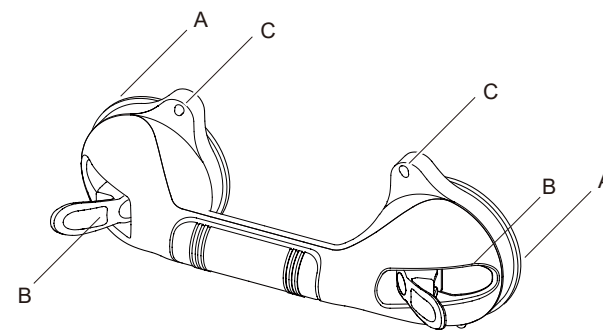

Part	Description	Quantity
A	Rubber disc	2
B	Release tab	2
C	Indicator	2

##### NOTE:

\*THE SUCTION GRAB BAR/BARS MUST BE INSTALLED ON SMOOTH, NON-POROUS SURFACES.

\*DON'T MOUNT OVER ANY GROUT LINES.

\*FOR BETTER SUCTION, PUSH THE GRAB BAR HARD AGAINST WALL TILL THE INDICATORS SHOW "GREEN" BEFORE PRESSING DOWN THE TABS (B).

##### HINTS:


FOR SAFETY PURPOSE, PLEASE REMOVE THE BAR FROM THE WALL AND REINSTALL IT TO MAKE SURE THE SUCTION WORK, EVERY MONTH, EVEN THE INDICATORS SHOW GREEN.

#### Installation

##### 1 Installing the suction assist bar

**NOTE:** The flip-up easy release tabs (B) should be loose at this point. Ensure that the rubber discs (A) are completely on the tile area or other smooth surface so that the discs do not cover any grout lines or other joints.


- Clean the rubber discs (A) and the surface it is to be attached to thoroughly. (Alcohol helps)
- Push the bar as hard as you could against the wall till the indicators turn green (80-90% at least). Press down on the flip-up easy release tabs (B) one by one, you will feel negative pressure. The greater the pressure, the better the suction.

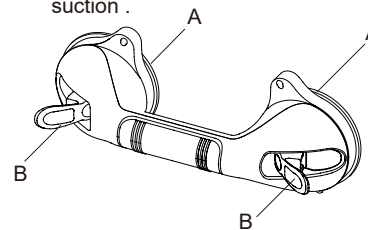

##### 2 Testing the suction assist bar before use

**NOTE:** The indicators on the side of the suction cup will change from red to green when a safe and secure hold is achieved.

- Before each use, test the firm hold of the suction device. Ensure that both indicators (C) are GREEN (80-90% at least).
- If the adhesion seems insufficient, release the suction assist bar and reattach again.
- If the required suction power has not been achieved, do not use the suction assist bar.

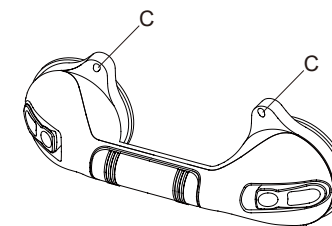

**Note:** The indicators are small color painted plaquettes driven by the springs underneath, they spin when certian pressure is achieved. But they are not 100% precise. Sometimes they are about 80-90% green, but when you test, the bar is firmly hold, it means the suction is working.