








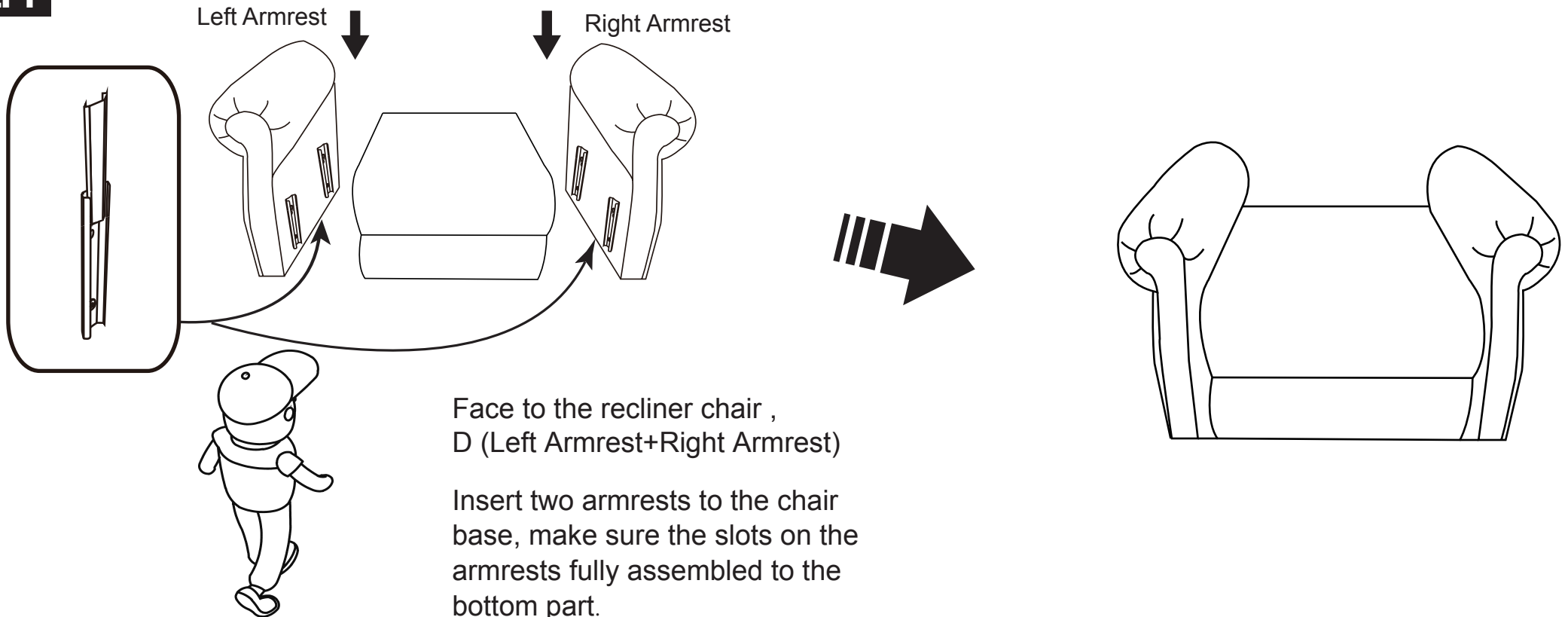


Massage Lift Chair Assembly Instruction-WF295180+WF295181

								
Chair base	Backrest	Left&Right Wings	Left&Right armrest	Power cord	Massage Remote	Power Source	Motor Remote	Wheel 2PCS


You will received totally 2 boxes, please make sure you get 1x (A) box and 1x (B) box before assembly.
Just feel free to chat us if you have any problem, we will try our best to help you solve the issue.

STEP 1

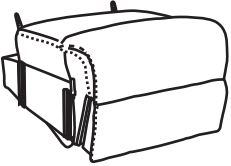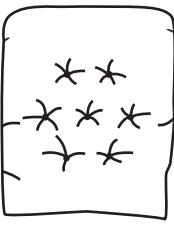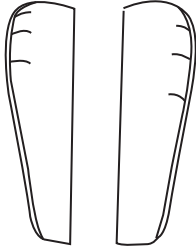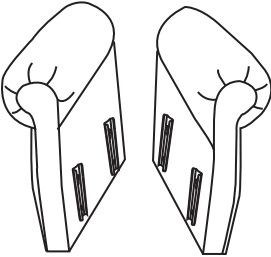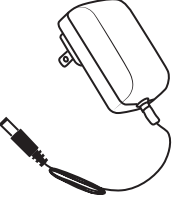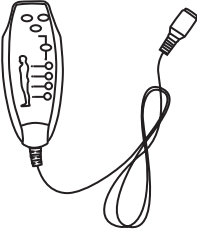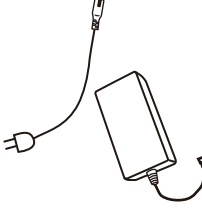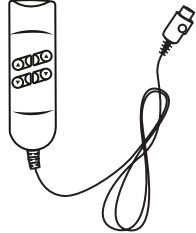

Massage Lift Chair Assembly Instruction

STEP 2

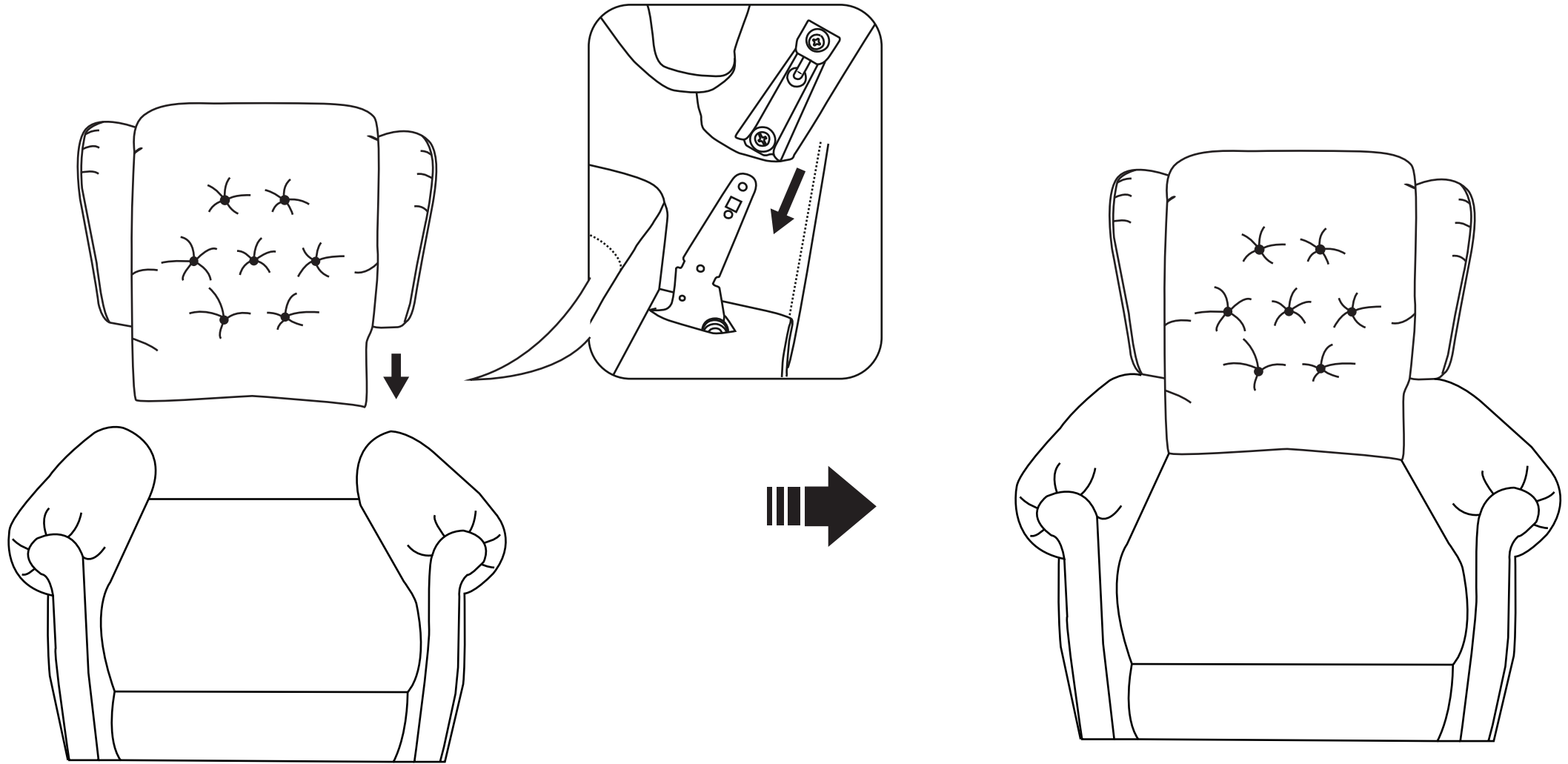


Insert chair back to the chair base, make sure the slots on the back fully assembled to the bottom part.

Massage Lift Chair Assembly Instruction

STEP 3


NOTE : Two same color wires connect.

No.1 and No.2 wire connect with the backrest and chair base.
No.3 wire connects with backrest and Power Cord(E).
No.4 wire connects with backrest and Massage Remote(F).

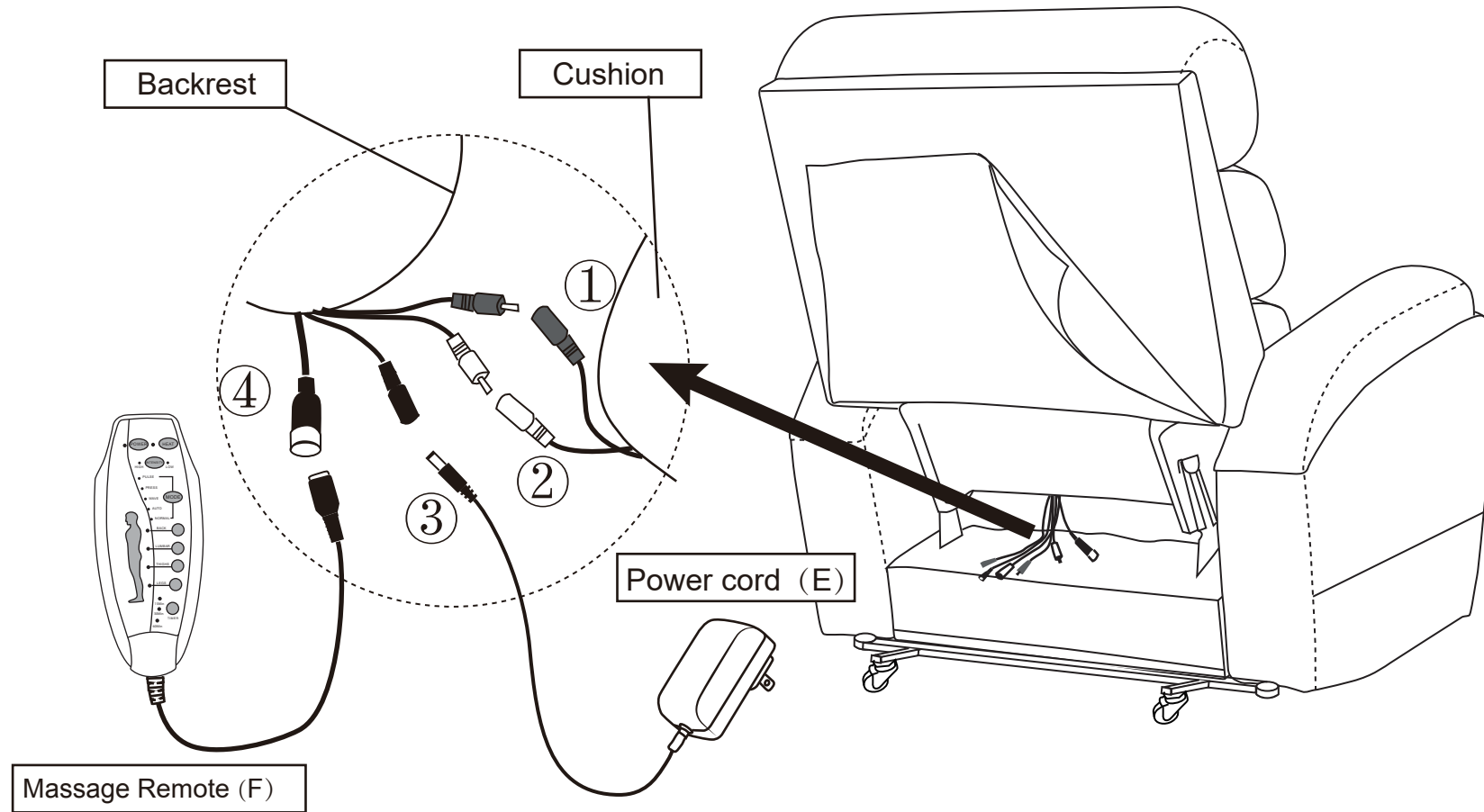

Massage Lift Chair Assembly Instruction

STEP 4

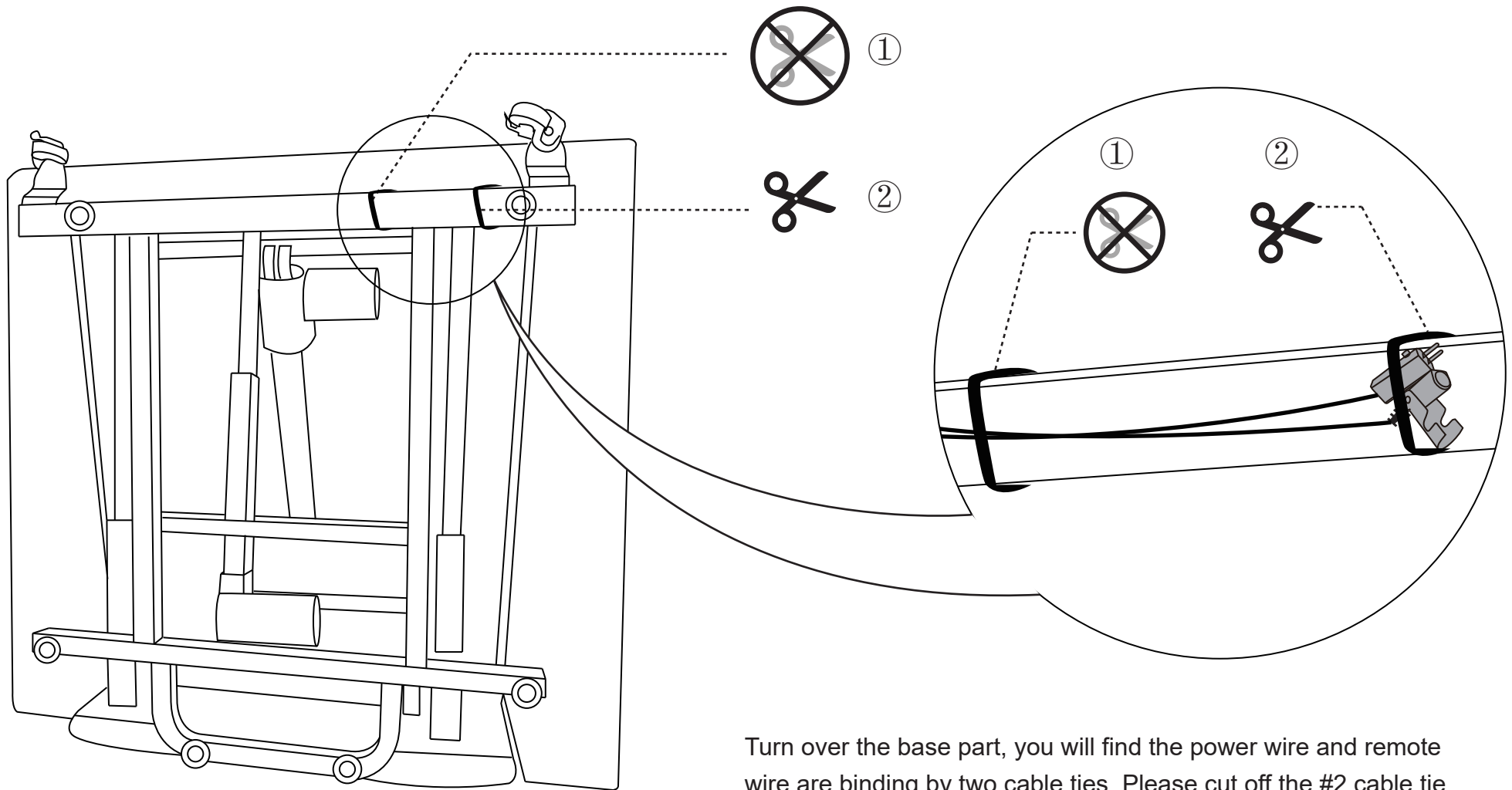


Turn over the base part, you will find the power wire and remote wire are binding by two cable ties. Please cut off the #2 cable tie and keep #1 to fix position.

Massage Lift Chair Assembly Instruction



STEP 5

NOTE : Plug power cord and power source into outlet, then you can use all function.

No.5 wire connect the motor and lift control remote.
No.6 wire connect the motor and power source (G).

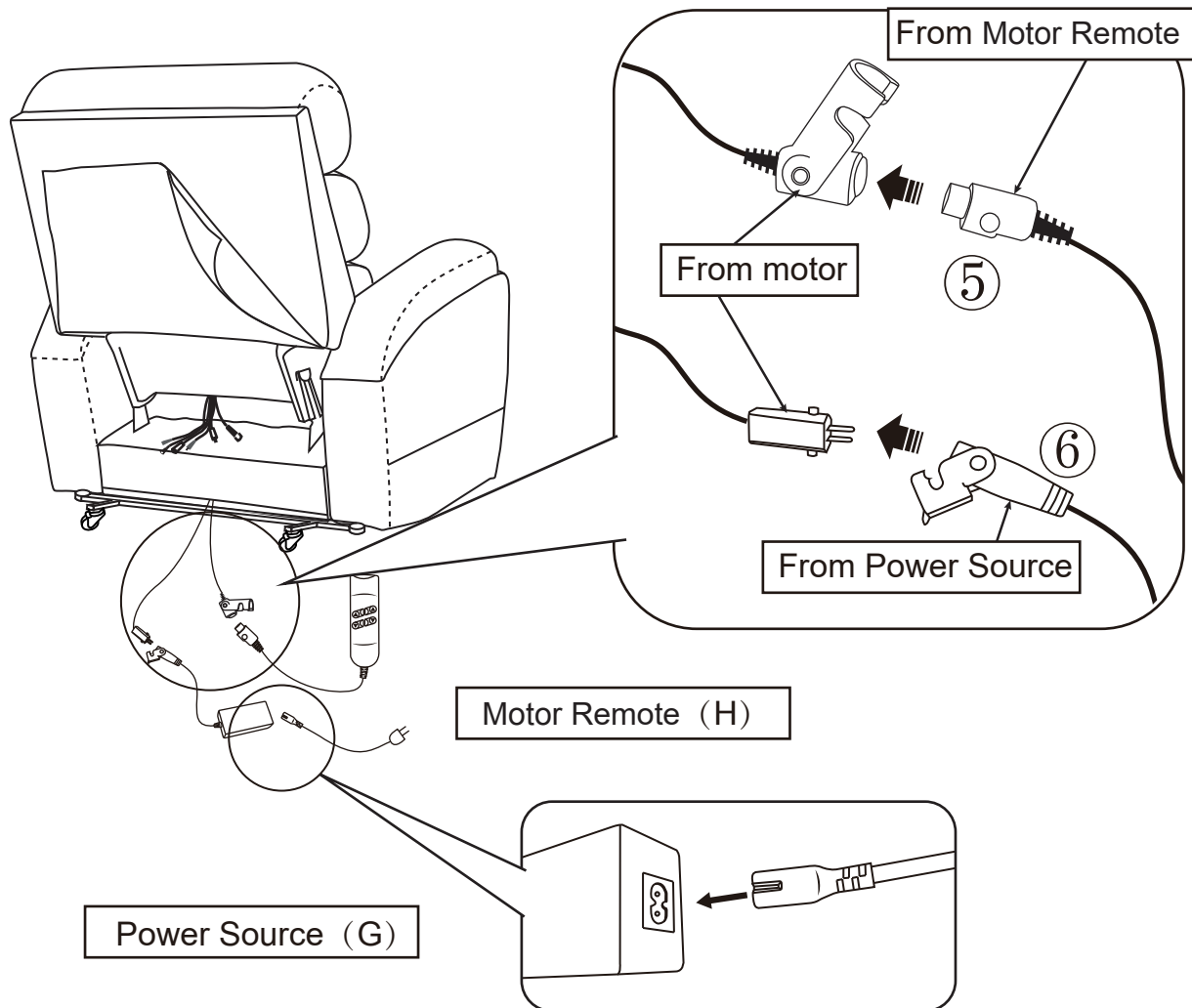

How to install the wheels?

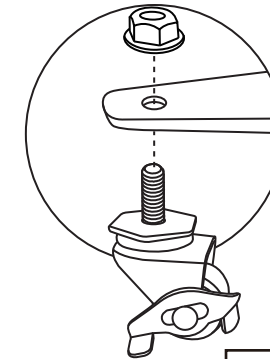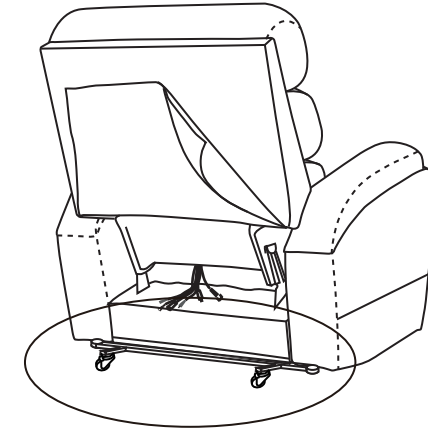


Wheel (I)

Must be assembled : Please connect the nut and the wheel to the chair base.

(Note: If any wheel is damaged, it can be removed and replaced.)



Massage Lift Chair Assembly Instruction

How to recline?

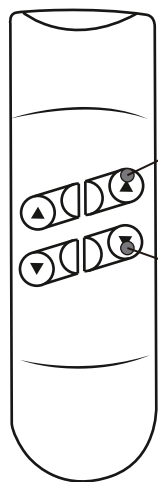

Press the button to lift the backrest.





Press the button to recline the backrest.

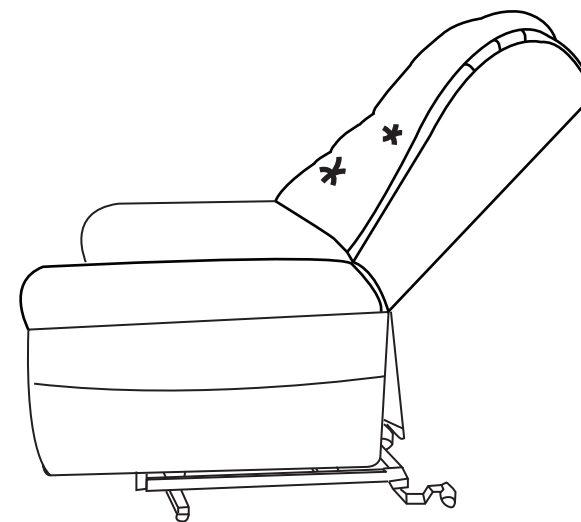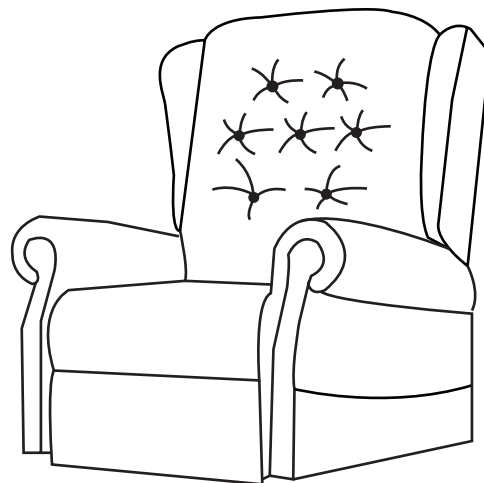

How to get footrest latch down and stand?

Press the button to rise the seat and retrieve footrest.

Press the button to descend the seat and unfold the footrest.

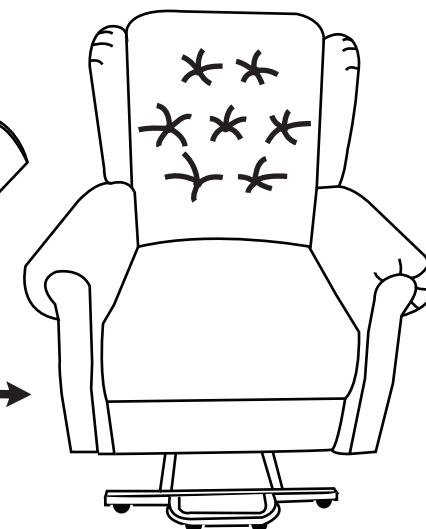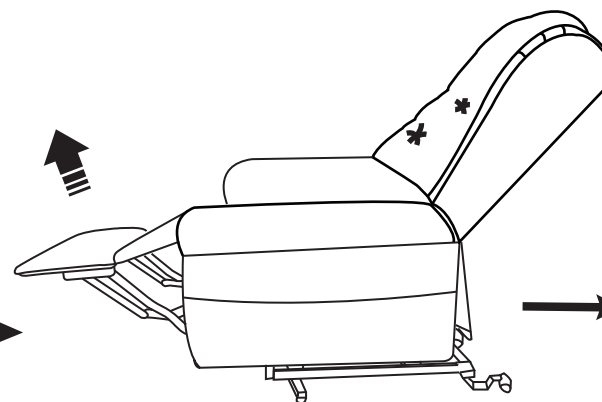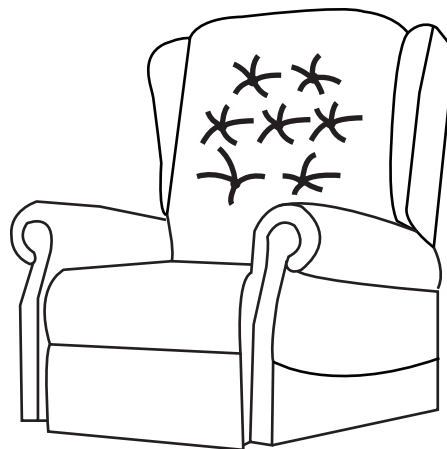
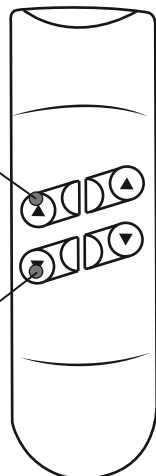


Massage Lift Chair Assembly Instruction

How to remove the chair ?



Please note that you need to take apart the backrest before removing the arms.

1. Force apart the buckles while take the backrest out from the arms.

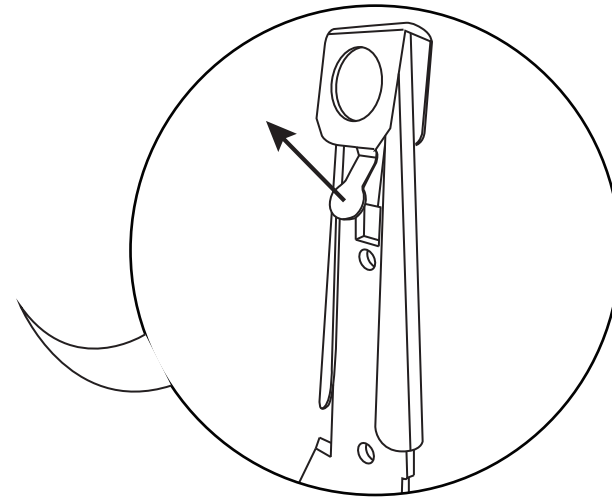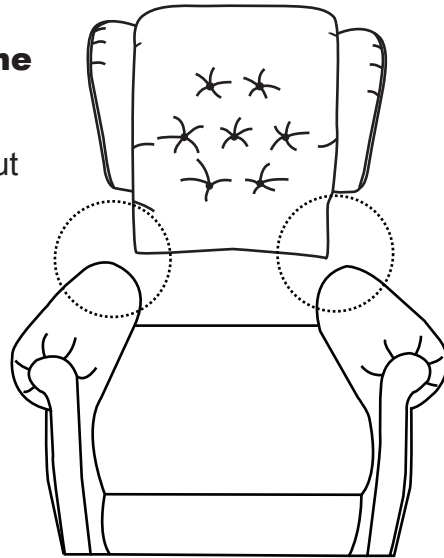

2. Then pull out the arms from the base.

Note: Please pull out the armrests from the middle of the armrests.

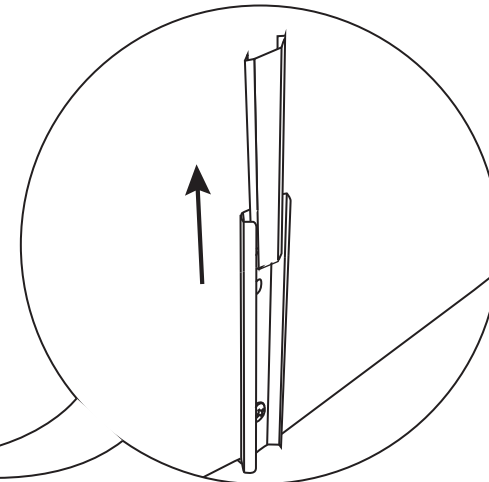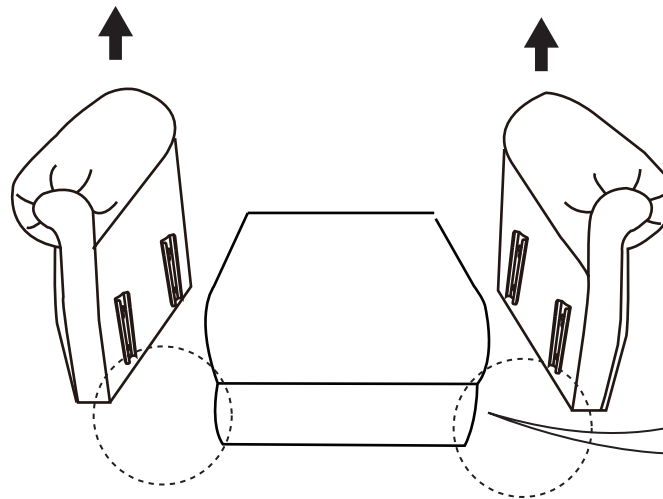

Massage Lift Chair Assembly Instruction

How to use remote control?

