INSTALLATION INSTRUCTIONS:

- 1. Lift up toilet seat cover and seat. Place plastic elevated toilet seat directly onto porcelain toilet seat bowl with the filler plug facing the toilet tank.
- 2. Bottom plastic inner rim of elevated toilet seat must fit inside of porcelain toilet bowl rim.

NOTE: Elevated toilet seat is designed for use with most common types of toilets including elongated (public washroom) toilets. This unit will not fit all toilet types.

CAUTION:

Before each use, check that unit is in proper position. Elevated toilet seat must sit evenly on porcelain toilet bowl rim. If unit rocks while on toilet rim, DO NOT USE. This product has a maximum user weight limitation of **250 pounds. NEVER MAKE SUBSTITUTONS OR MODIFY THIS PRODUCT IN ANY MANNER.** Do not jump or stand on seat. Liberty Hardware Manufacturing Corporation assumes no liability for any damage or injury caused by the improper installation or use of this product. It is recommended that you consult a health care professional for instructions specific to your needs. Physically challenged users should be supervised or assisted whenever using this product.

CLEANING INSTRUCTIONS:

Use non-abrasive mild detergent with warm water. Rinse with warm water and dry. **Do not use abrasive cleansers or bleach.**

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