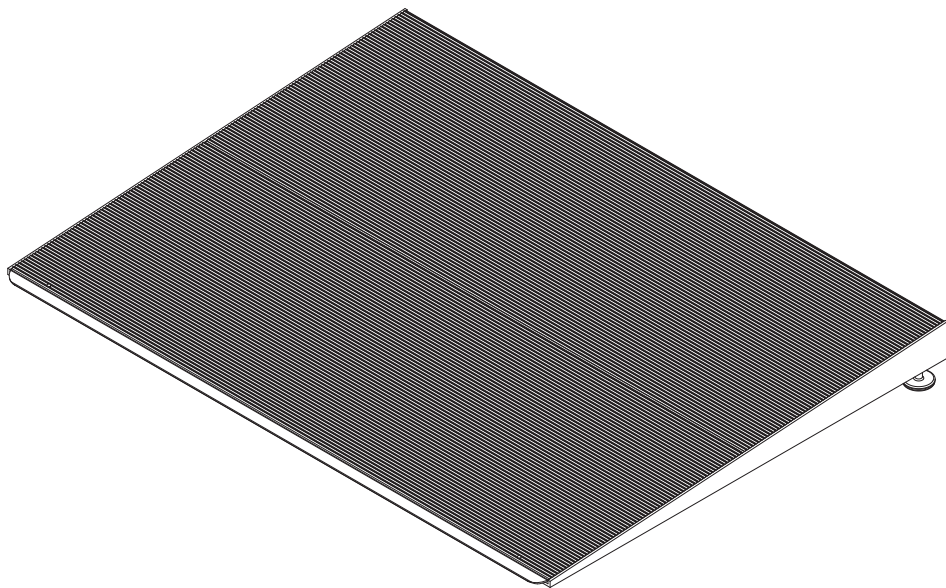
Prairie View Industries

ADJUSTABLE THRESHOLD RAMP

- MODEL • ATH1232
• ATH1236
• ATH2432
• ATH2436
• ATH3632
• ATH3636
• ATH4832
• ATH4836

Español p. 8


Serial Number _____ Purchase Date _____


Questions, problems, missing parts? Call our customer service department at
1-800-554-7267, 7 a.m. - 5 p.m., CST, Monday - Friday.

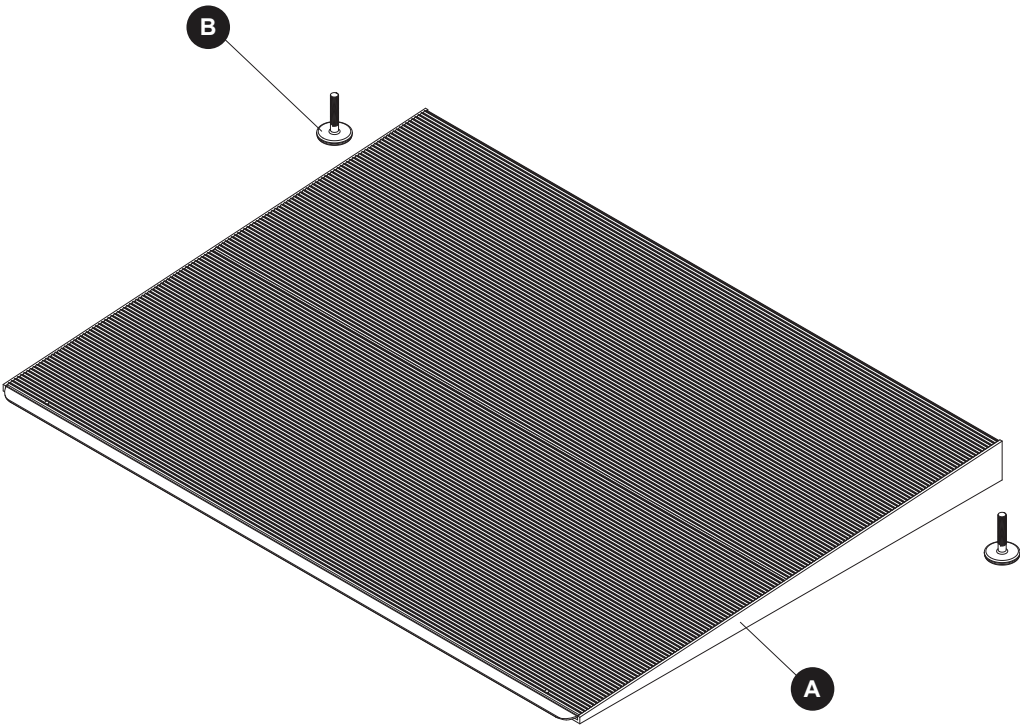PACKAGE CONTENTS

PART	DESCRIPTION	QUANTITY
A	Adjustable Threshold Ramp	1
B	Leveling Base	2

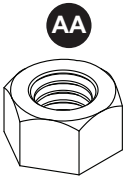HARDWARE CONTENTS (shown actual size)

Hex
Nut

Qty. 2

Flat Head
Pan Screw

Qty. 2

PREPARATION

Before beginning assembly of product, make sure all parts are present. Compare parts with package contents list and hardware contents. If any part is missing or damaged, do not attempt to assemble the product.

Estimated Installation Time: 2 minutes

Tools Required for Assembly (not included): None.

Helpful Tools (not included): Drill with 3/16 bit (wood, metal, or masonry depending on what type of surface you will be drilling pin holes into); Pen/Pencil.

SAFETY INFORMATION

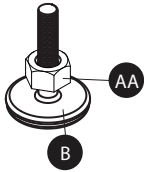
Please read and understand this entire manual before attempting to assemble, operate or install the product.

WARNING

- Ramps are designed for wheelchairs and electric scooters.
- Ramps are rated for a 600 pound capacity; rating is based on double axle, evenly distributed loads.
- For your safety, a caregiver or aid should be present at all times when using any ramp.
- Proper setup is essential for safe loading and unloading.
- Make sure that the hook (top end) is pressed firmly against the step or platform before using.
- When possible, use the provided flat head pan screws (BB) provided to lock ramp in place.
- Safe lifting practices should be used when lifting product.
- The Americans with Disabilities Act recommends a 1:12 slope; for 6-foot ramp a maximum height of step/platform not to exceed 6 inches.
- Failure to follow all warnings/instructions may result in ramp slipping off step while occupied.

ASSEMBLY INSTRUCTIONS

1

1. Screw hex nut (AA) onto leveling base (B) all the way down. Do not tighten.

Hardware Used

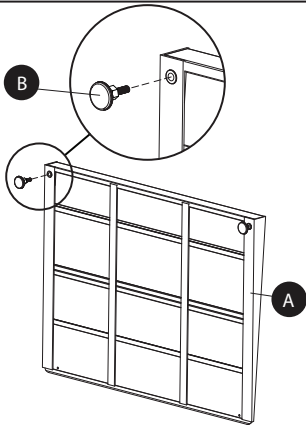
AA

Hex Nut


x 2

2


2. Attach leveling base (B) to bottom of adjustable threshold ramp (A) and adjust to required height.

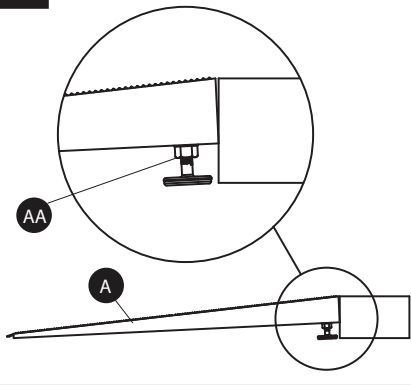3


3. Secure legs by tightening hex nut (AA) against bottom of adjustable ramp (A).

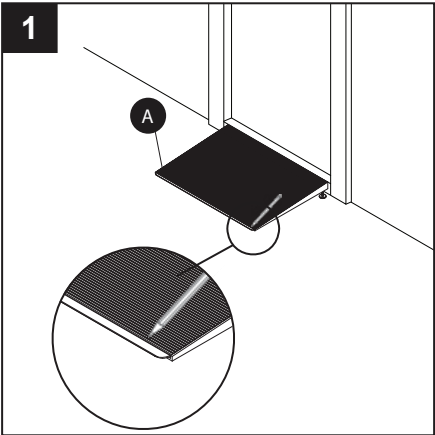
INSTALLATION INSTRUCTIONS

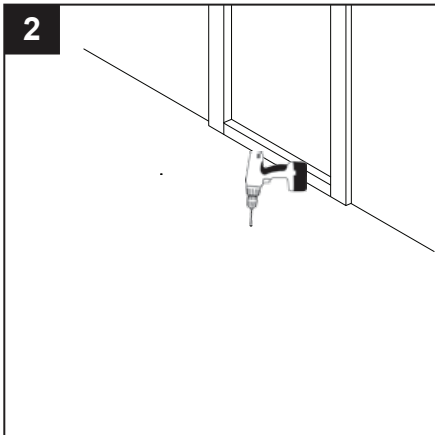1. Place adjustable threshold ramp (A) in desired location. Locate both holes at end of the adjustable threshold ramp (A). Mark the pin locations using a pencil.

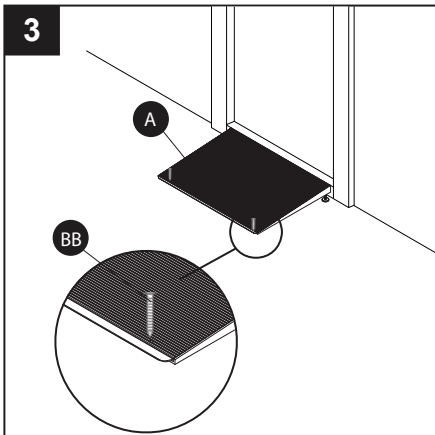2. Remove the adjustable threshold ramp (A) and then, using a drill and 3/16 bit, drill a hole approximately 2 inches deep. (Note: If anchoring into concrete, concrete screws need to be used.)


3. Replace the adjustable threshold ramp (A) and screw the flat head pan screws (BB) into the holes to secure it in place.

Hardware Used

BB

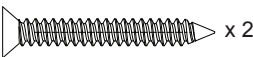Flat Head
Pan Screw

OPERATING INSTRUCTIONS

Measure height of threshold. Make sure leveling bases are adjusted to appropriate height. Place ramp as close to the threshold as possible and secure. Double check that both leveling bases come in contact with a surface.

When possible, use pan screws for additional security.

CARE AND MAINTENANCE


These ramps are maintenance free, but to keep product in excellent condition follow these tips.

- Store in a cool dry place when not in use.
- Keep ramp clean and free of debris.
- Do not leave ramp out during harsh weather conditions.
- If ice or snow accumulates on ramp, do NOT use metal objects to remove. Instead, use plastic shovel or non-corrosive ice melt (labeled as “pet friendly” or “pet safe” ice melt).

REPLACEMENT PARTS LIST

For replacement parts, call our customer service department at 1-800-554-7267, 7 a.m. - 5 p.m., CST, Monday - Friday.

PART	DESCRIPTION	PART #
B	Leveling Base	361653-B

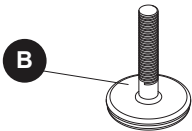WARRANTY

MANUFACTURER LIMITED WARRANTY

1. LIMITED LIFETIME WARRANTY — THE FOLLOWING APPLIES TO ALL END PURCHASERS OTHER THAN RESIDENTS OF THE STATE OF CALIFORNIA*:

The manufacturer warrants to the original end purchaser of the product that the product is free from defects in material and workmanship under normal use and service. This warranty does not apply to any product that has been subject to misuse, abuse, neglect, alteration, accident, usage not in accordance with product instructions, acts of God, or improper installation, or that has been used for rental purposes or repaired by someone other than the manufacturer. This warranty does not cover normal deterioration of the product due to wear and exposure.

*(FIVE YEAR WARRANTY — THE FOLLOWING APPLIES TO RESIDENTS OF THE STATE OF CALIFORNIA: For a period of five years after the purchase of the product by the original end purchaser of the product, the manufacturer warrants to the original end purchaser of the product that the product is free from defects in material and workmanship under normal use and service. This warranty does not apply to any product that has been subject to misuse, abuse, neglect, alteration, accident, usage not in accordance with product instructions, acts of God, or improper installation, or that has been used for rental purposes or repaired by someone other than the manufacturer. This warranty does not cover normal deterioration of the product due to wear and exposure.)

2. This warranty is limited to repairing or replacing, at the option of the manufacturer, any product that is returned to the manufacturer and is reasonably determined by the manufacturer to be defective. The repair or replacement of a defective product under this warranty will be made by the manufacturer without charge for parts or labor. This excludes shipping or delivery charges to and from the manufacturer's place of business. If the product has been discontinued, the manufacturer may replace the product with a new product of comparable value and function. The manufacturer also reserves the right to refund the purchase price as its exclusive warranty remedy.

3. Any claim alleging that the product fails to conform to this warranty may be made only by the original end purchaser and only while such purchaser owns the product. A defective product that is covered by this warranty must be returned, at the purchaser's expense, along with proof of date of original purchase (such as receipt, check or other document the manufacturer deems acceptable that shows the date of purchase and the identity of the product and purchaser), along with a description of the alleged defect, to the manufacturer.

4. EXCEPT AS SET FORTH HEREIN, THE MANUFACTURER MAKES NO WARRANTIES, EXPRESSED OR IMPLIED, AND THE MANUFACTURER DISCLAIMS AND NEGATES ALL OTHER WARRANTIES, INCLUDING, WITHOUT LIMITATION, IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, CONFORMITY TO MODELS OR SAMPLES, OR ANY WARRANTIES OR INDEMNITIES AGAINST INTELLECTUAL PROPERTY INFRINGEMENT.

SOME STATES DO NOT ALLOW LIMITATIONS ON IMPLIED WARRANTIES, SO THESE LIMITATIONS MAY NOT APPLY TO YOU. IN NO EVENT WILL THE MANUFACTURER BE LIABLE FOR ANY INDIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES OR FOR ANY LIABILITY IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT.

5. Before using the product, the purchaser must first determine the suitability of the product for its intended use, and the purchaser assumes all risk and liability whatsoever in connection therewith.

6. No person, agent, distributor, dealer or company is authorized to change, amend or modify the terms of this warranty.

7. This warranty gives the purchaser specific legal rights, and the purchaser may also have other rights that vary from state to state.

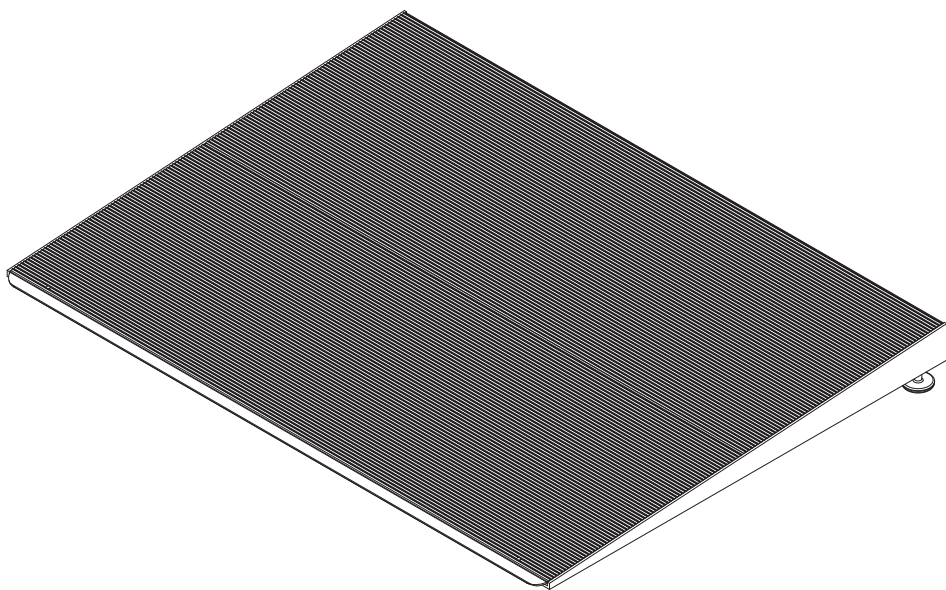
8. The purchaser may not assign the purchaser's rights or obligations under this warranty without the prior written consent of the manufacturer.

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Prairie View Industries, Inc. • Phone: 800-554-7267