


introduction

- Thank you for buying bath chair products.
- Before using the product, please read the instructions carefully and use and understand correctly.
- Please keep this manual properly after reading, so that you can consult and refer to it at any time.


Lifetime limited warranty

- If there are defects in the material and manufacturing process of the bath chair products purchased by you, we will A lifetime limited warranty is provided for the purchase. This service is built on strict standards of craftsmanship and On the basis of careful inspection before shipment, the lifetime limited warranty describes us on our products and products We have confidence in the craftsmanship and many years of reliable service to our customers.
- In the event of a defect covered by the warranty, we have the option to repair or replace the device under this warranty This warranty does not cover indurability due to customer misuse and negligence or normal wear and tear Components, such as rubber fittings, are normally worn and need to be replaced regularly.
- If you have any questions about our devices or quality assurance, please contact our authorized approved Contact the seller.

Matters needing attention

- Carefully check all parts before use, if you find low-end wearing parts abnormal, please replace in time.
 - Make sure the adjustment key is in place before use, that is Hear a 'click' to use.
- 
- Do not place the product in a high temperature or low temperature environment, which will cause aging and elasticity of rubber parts The phenomenon of inadequacy.
 - This product should be placed in a dry, ventilated, stable, non-corrosive substances indoor.
 - The bath chair is an auxiliary bath appliance that prohibits the whole body from being pressed on one side to prevent accidents.
 - Check the quality of the product regularly every week.
 - Footpads are easy to wear parts, wear after please contact the supplier paid replacement.
 - The size of the product marked in the manual measurement size, there is an error of 1-5cm, please note.

Multi-functional bath chair diagram

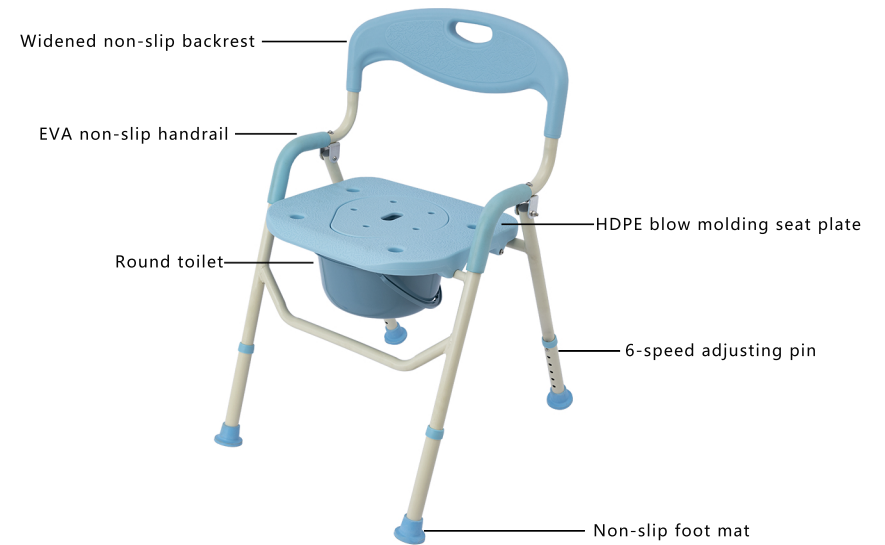

Product introduction: Bath chair is a new type of household supplies, improve the quality of life, suitable for people;The elderly, pregnant women, children, people with limited mobility, etc

Installation steps:

Unpack, unpack the bath chair, put it on the backboard, put it on the toilet. The four-pin pipe can be adjusted to the same height of the hole installation.