
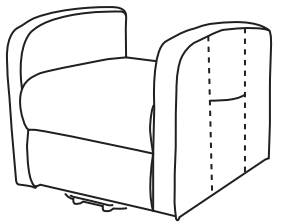
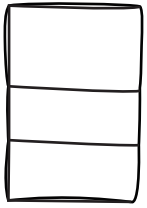
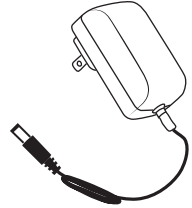
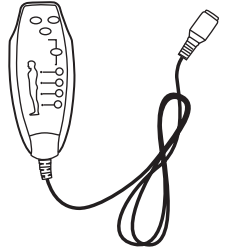
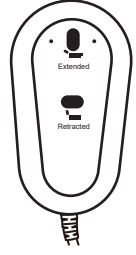
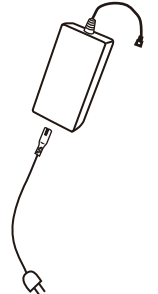


Electric Lift Chair Assembly Instruction

A 	B 	C 	D 	E 	F 
Chair base	Backrest	Power cord	Massage Remote	Chair Remote	Power Source

STEP 1

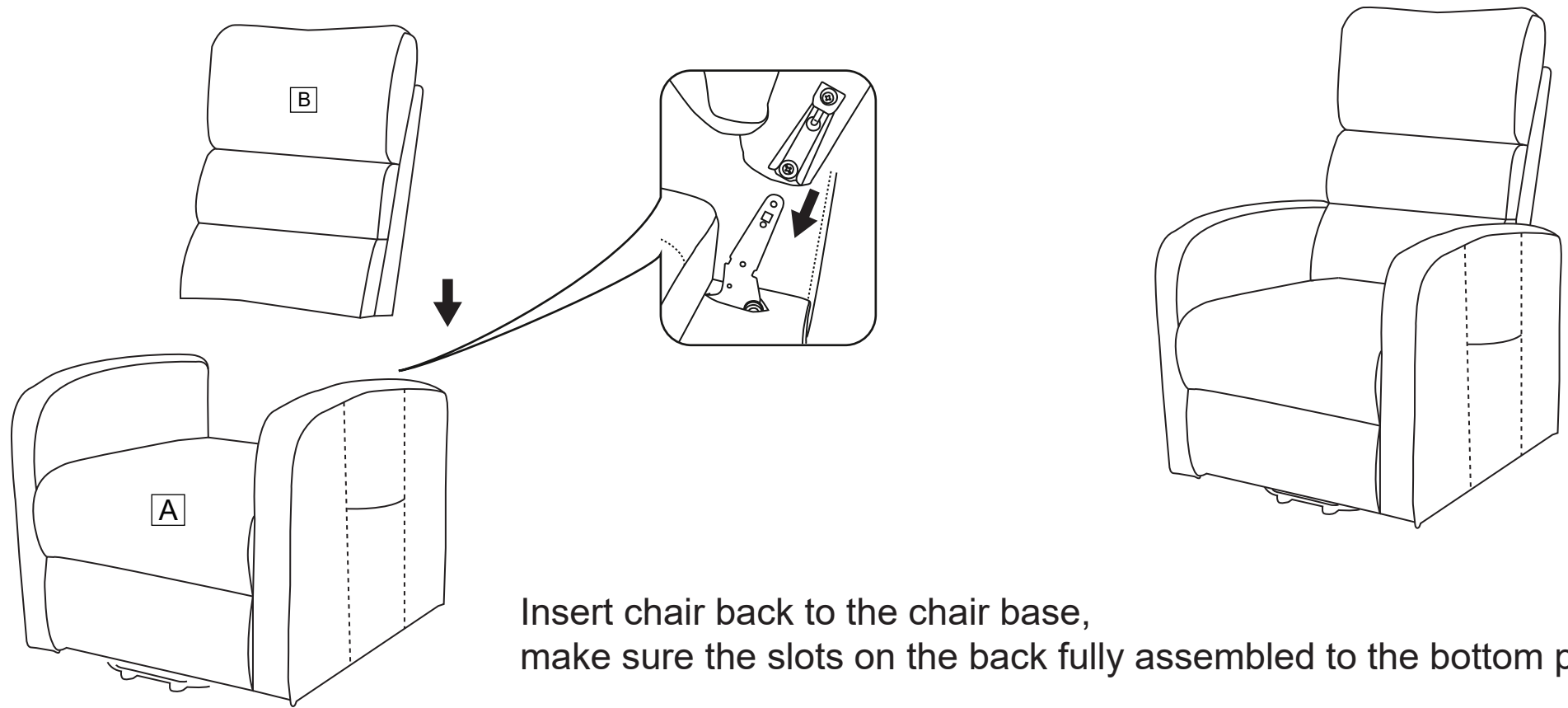

Insert chair back to the chair base, make sure the slots on the back fully assembled to the bottom part.


Electric Lift Chair Assembly Instruction

STEP 2

- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power cord(C).
- No.4 wire connects with backrest and remote control(D).
- No.5 wire connect the motor and the wire from Chair Remote(E).
- No.6 wire connect the motor and power source (F).




NOTE

- 1: Two same color wires connect.
- 2: Plug power cord and power source into outlet, then you can use all function.




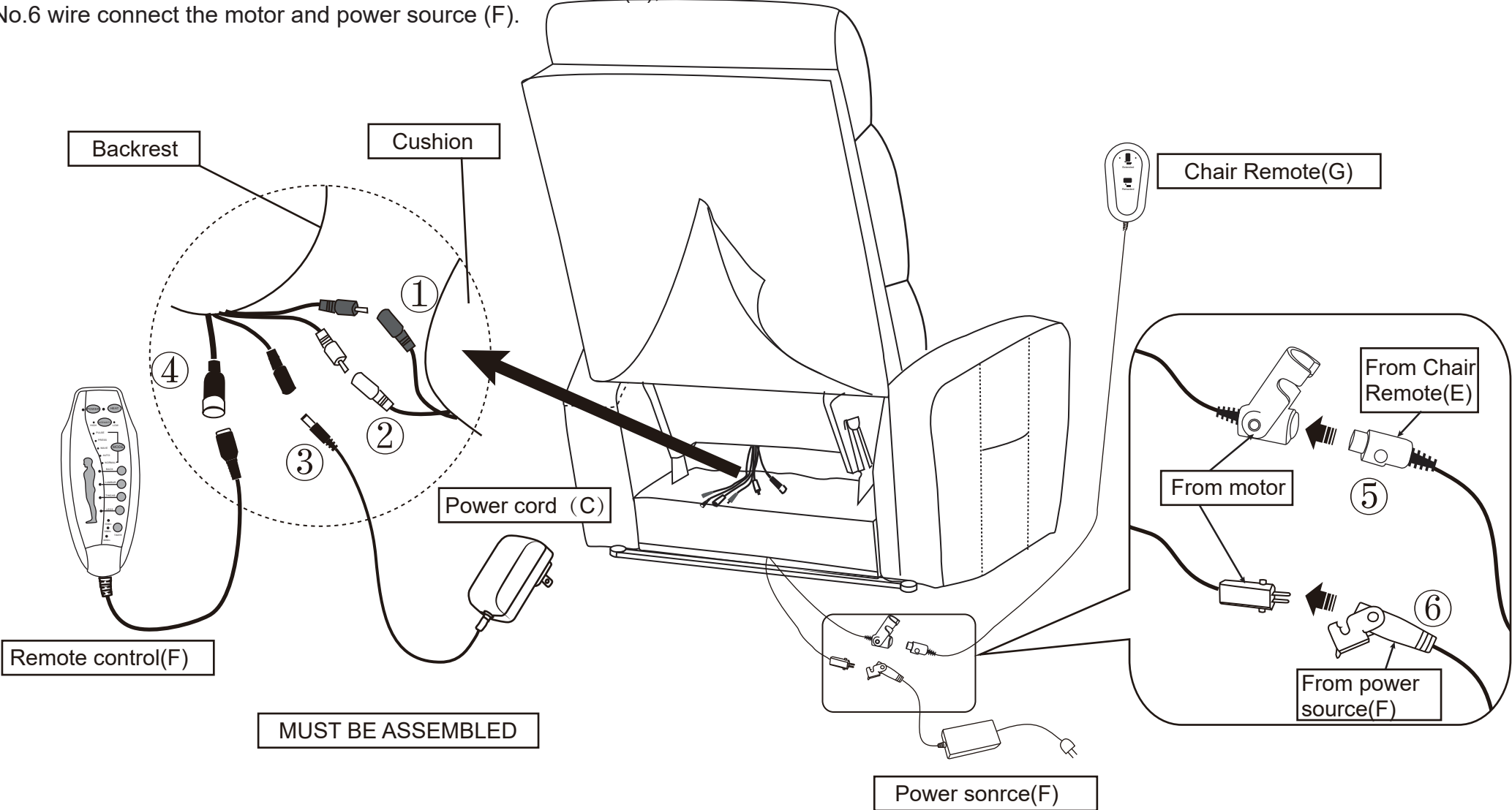


Electric Lift Chair Assembly Instruction

How to recline?

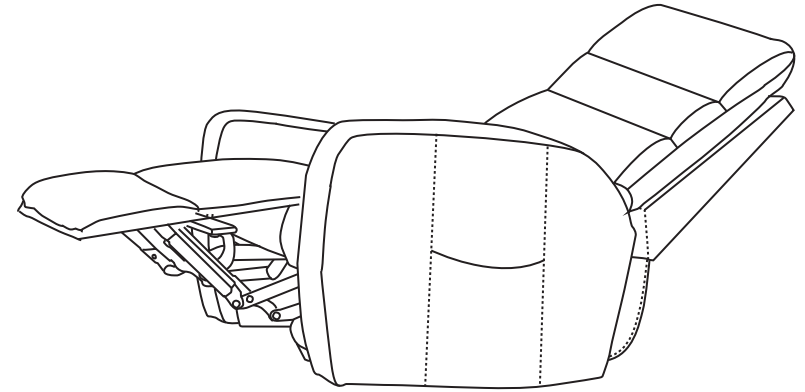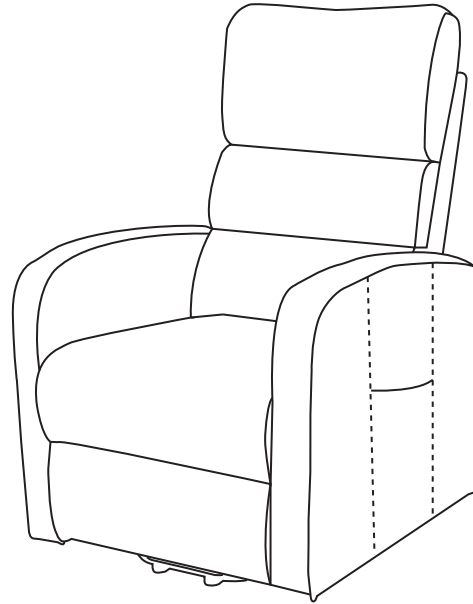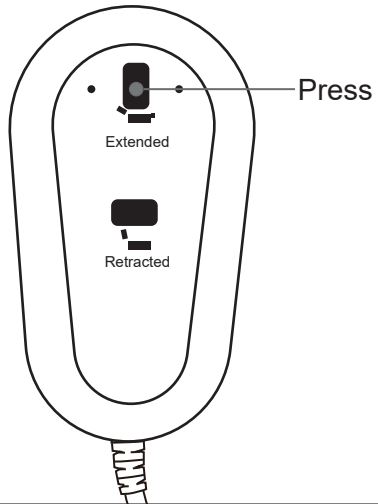


How to get footrest latch down and stand?

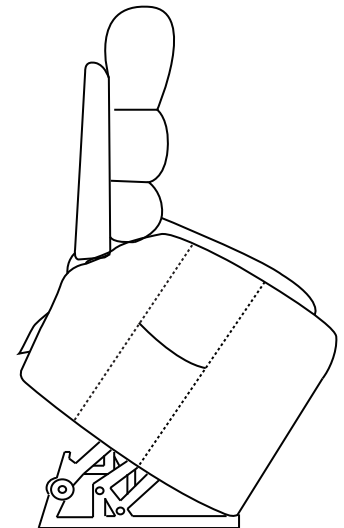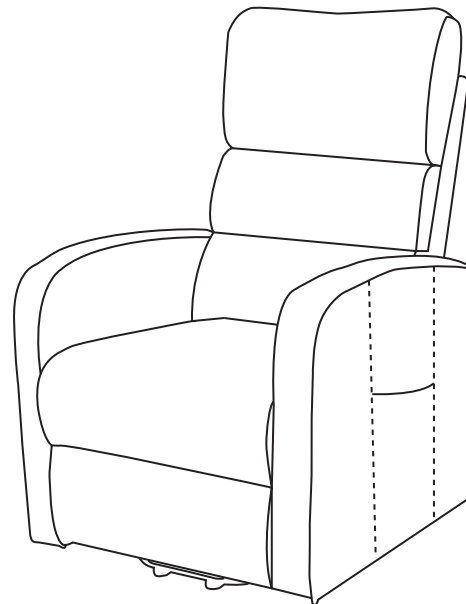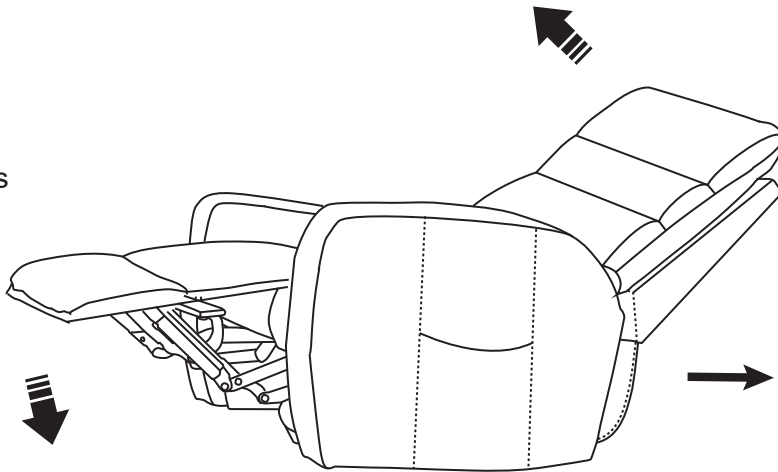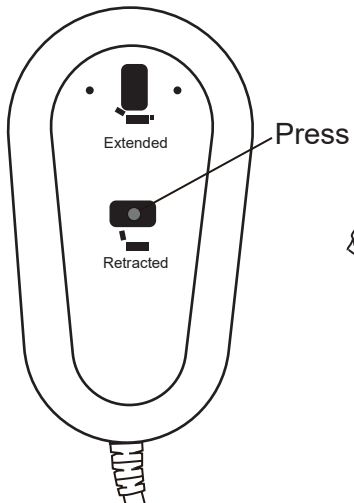

Electric Lift Chair Assembly Instruction

How to use remote control?

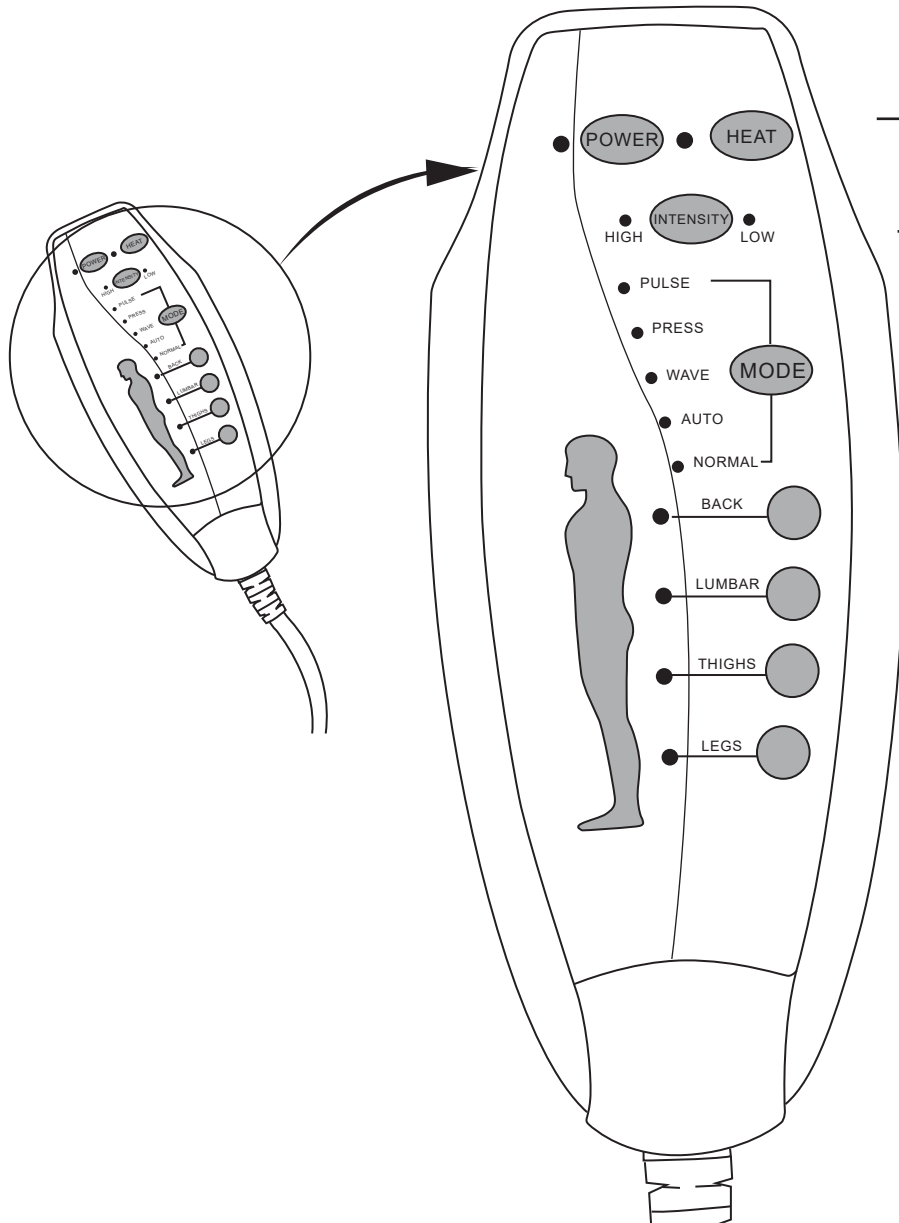

- POWER : Turn on/off remote control.
- HEAT : Turn on/off heat function for lumbar part only.
- INTENSITY : Control massage function from low to high.
- MODE : Choose one massage mode you like (pulse, press, wave, auto, normal)
- You can choose 1-4 massage parts (back, lumbar, thighs, legs) to enjoy.