

**Wave
Grab Bar**

WGB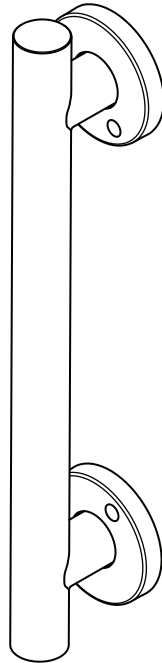

**Horizon
Grab Bar**



HGB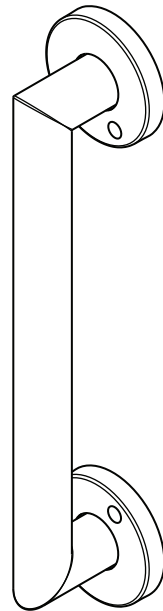







**Miter
Grab Bar**

MGB



A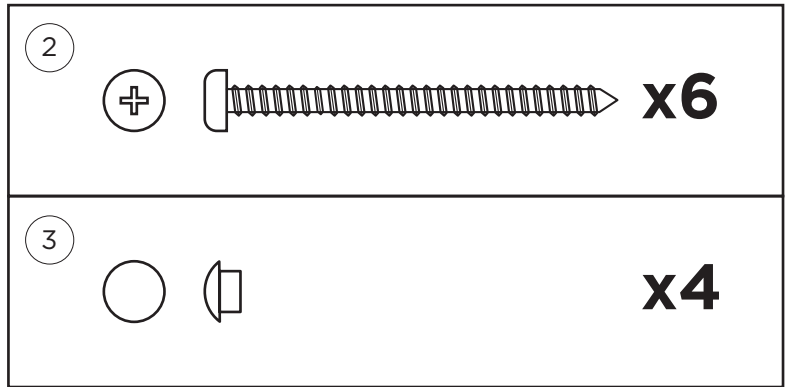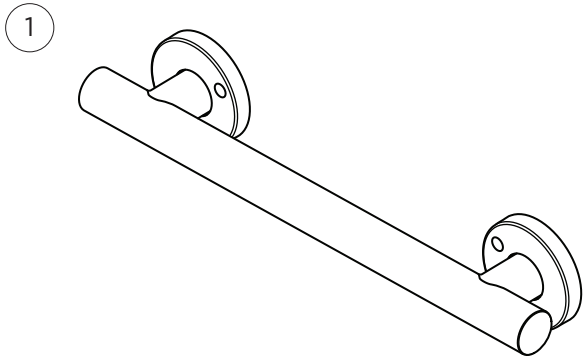



B

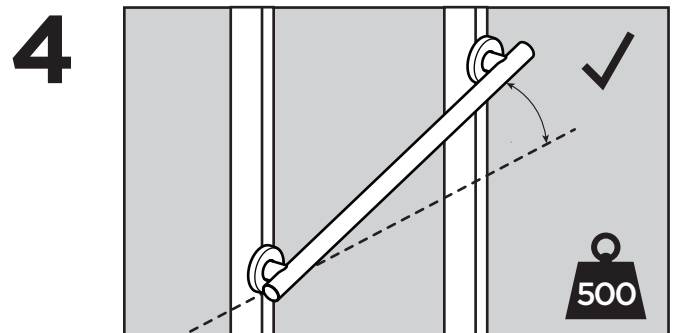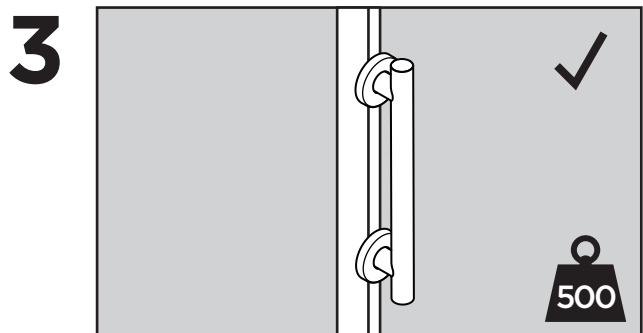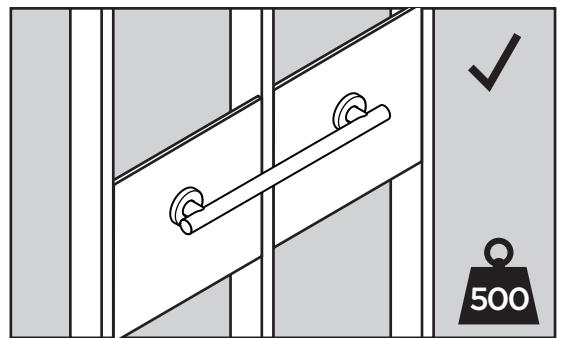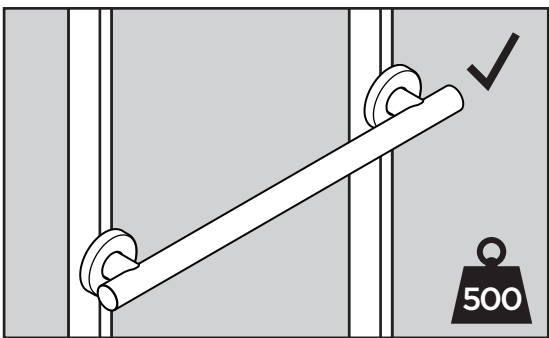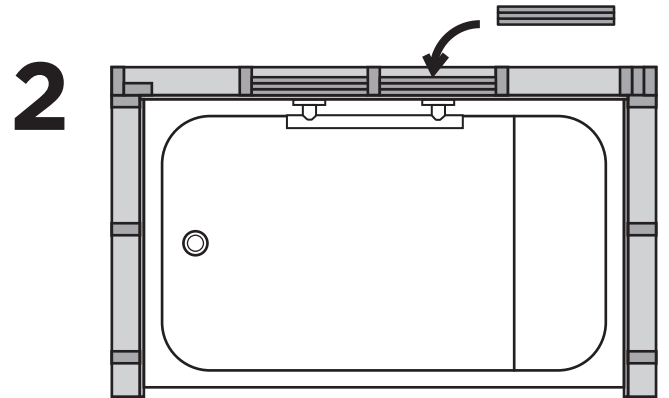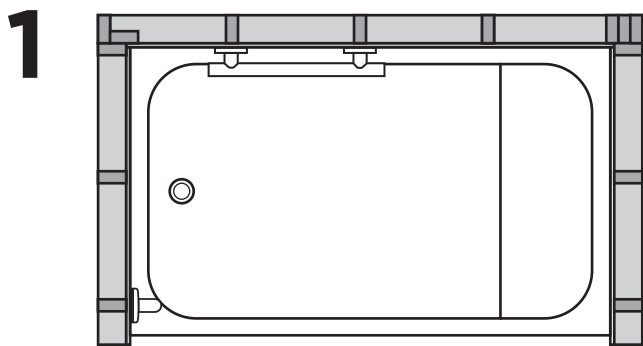
5

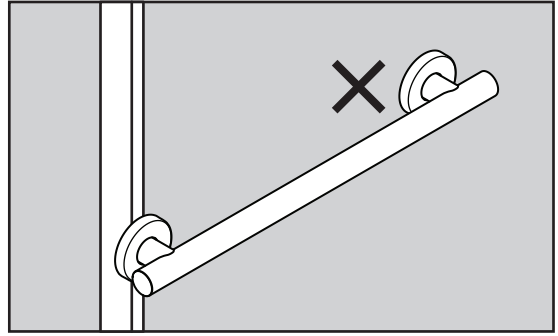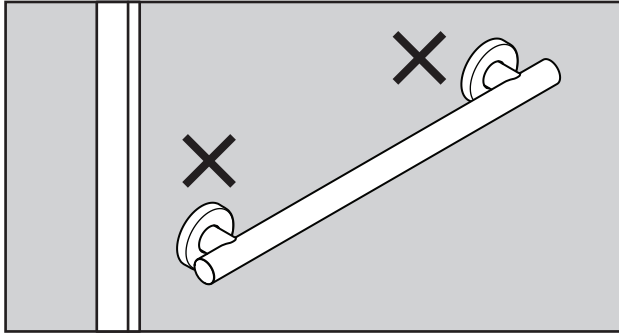
6

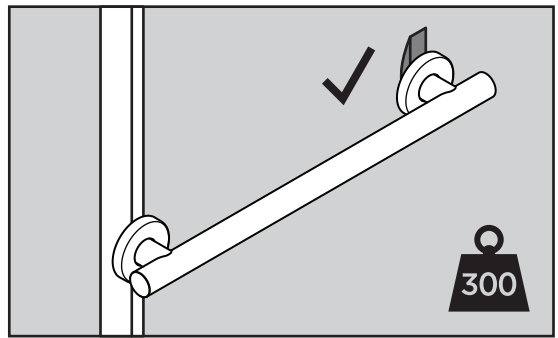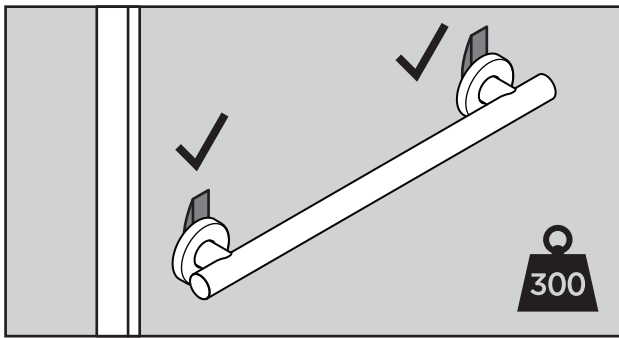
7

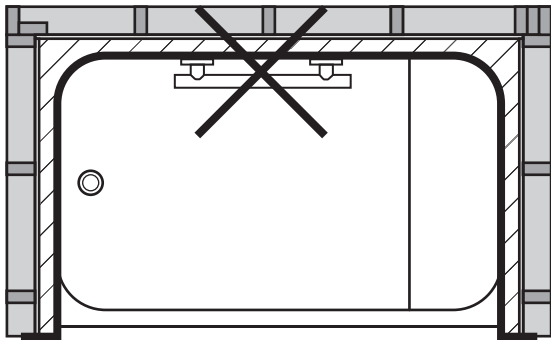8a

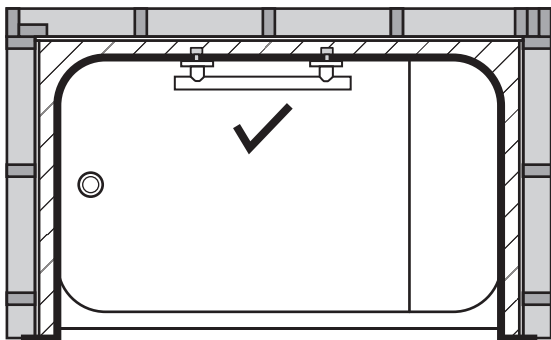
8b

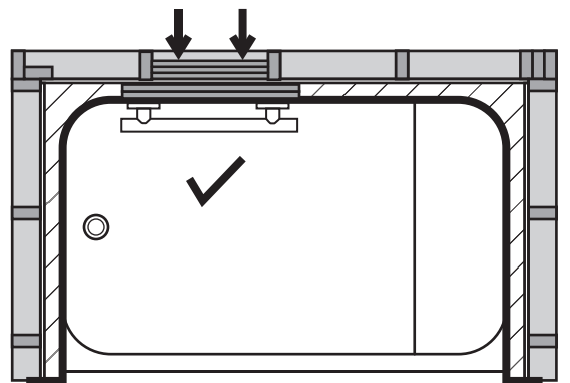
B



9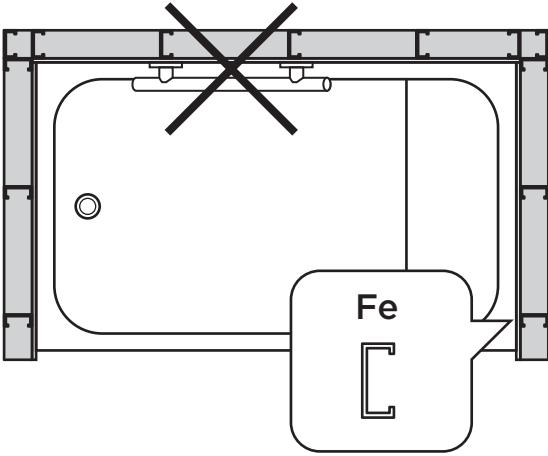


10

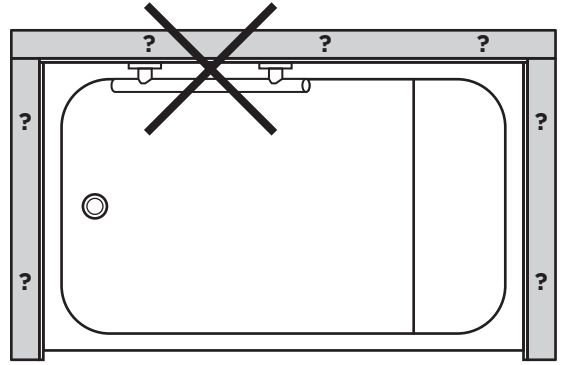
C

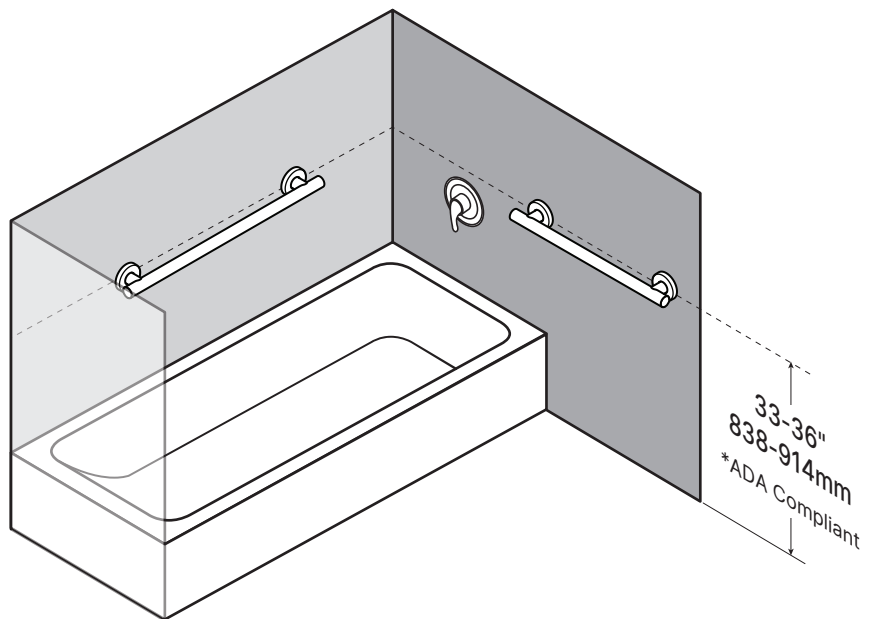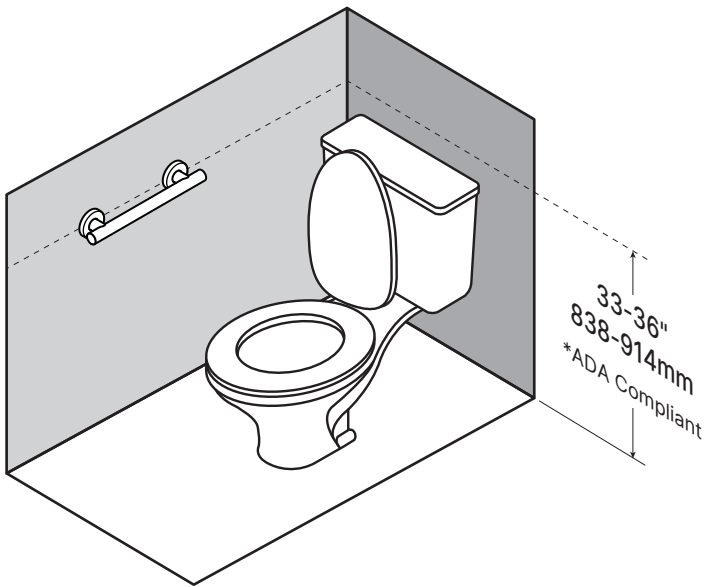
1

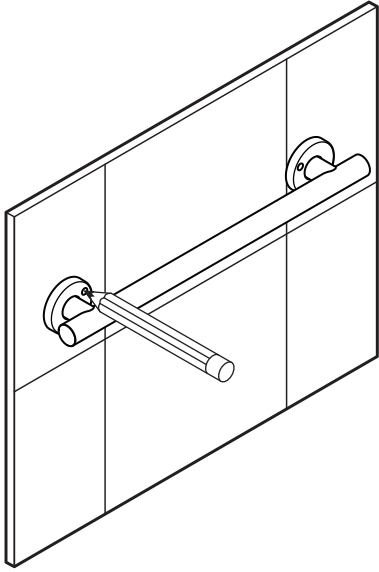
2

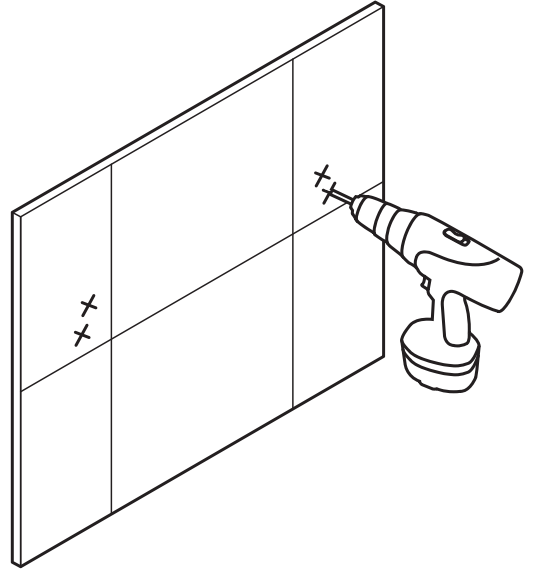
3



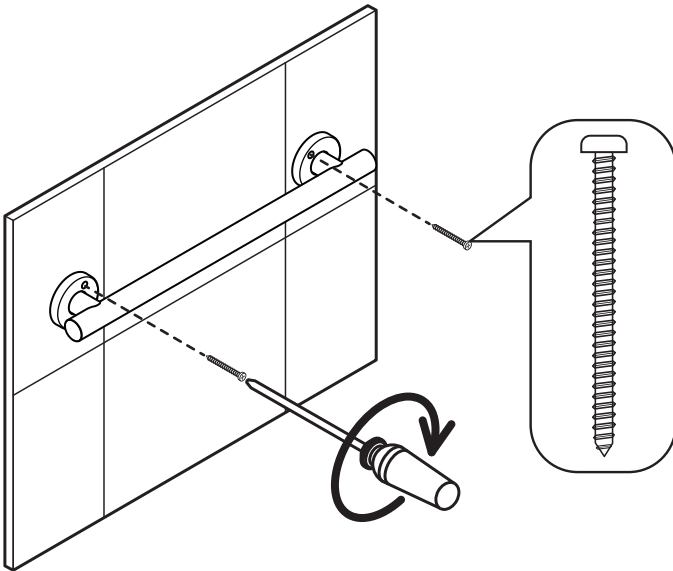
4

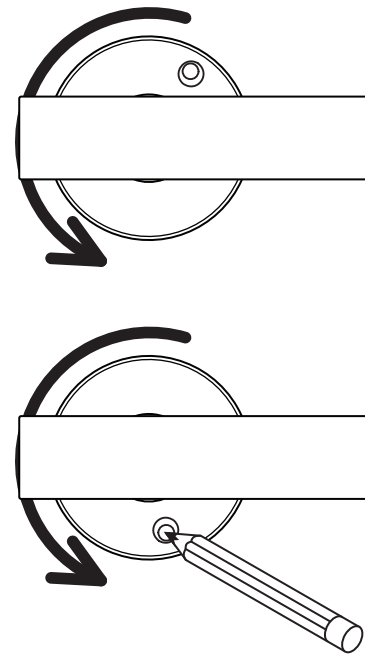5

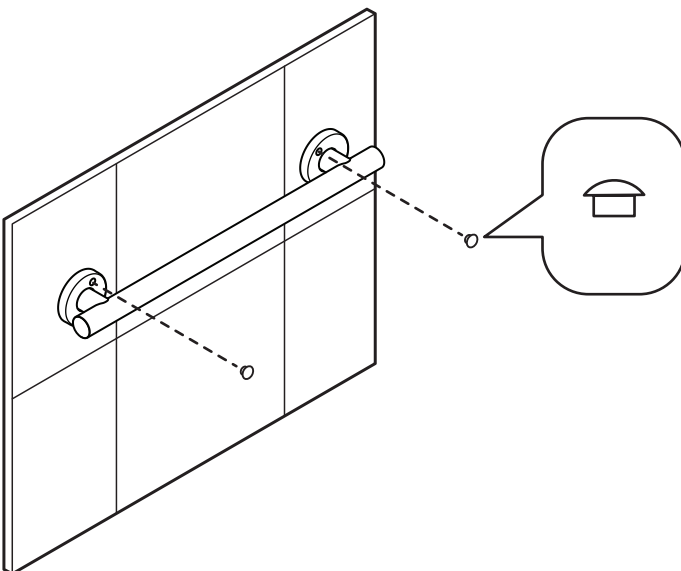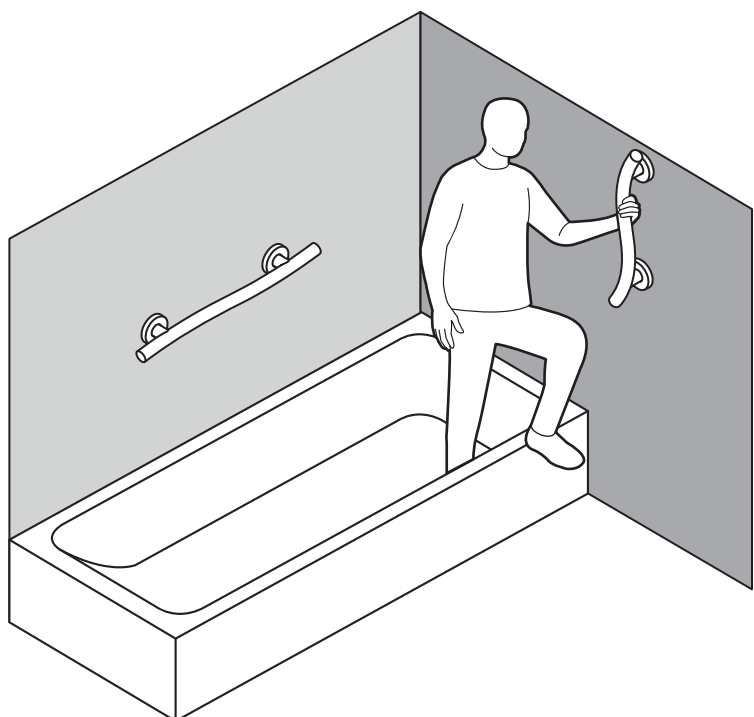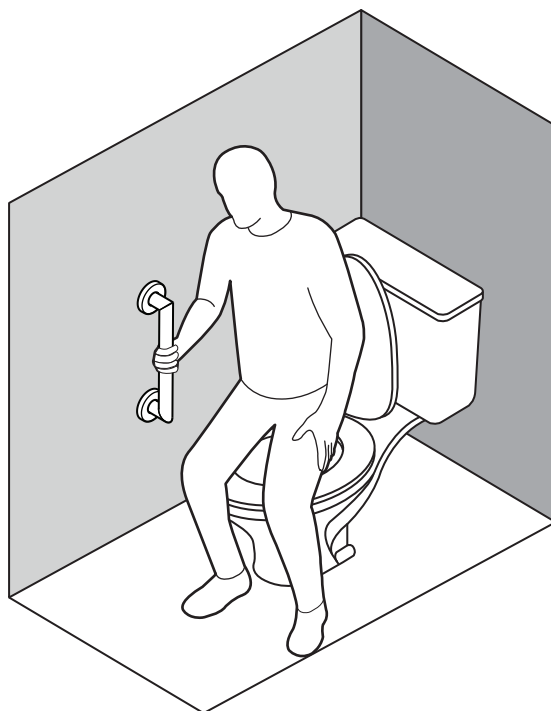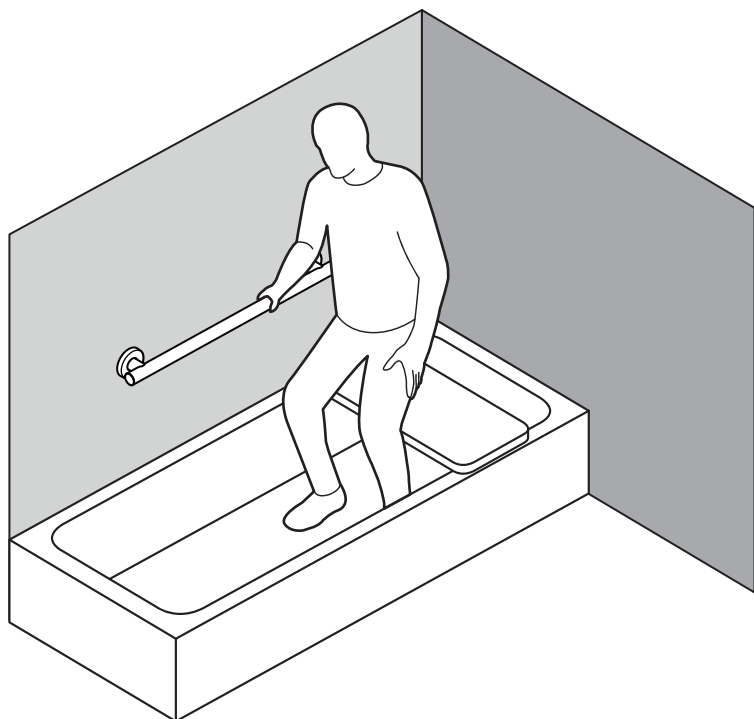
WGB

LENGTH (N)	LENGTH (L)	LENGTH (H)	PRODUCT WEIGHT
12"/305mm	19"/483mm	3.5"/89mm	2.71 lbs / 1.23kg
16"/406mm	24"/610mm	3"/76mm	3.23 lbs / 1.47kg


♻️ SS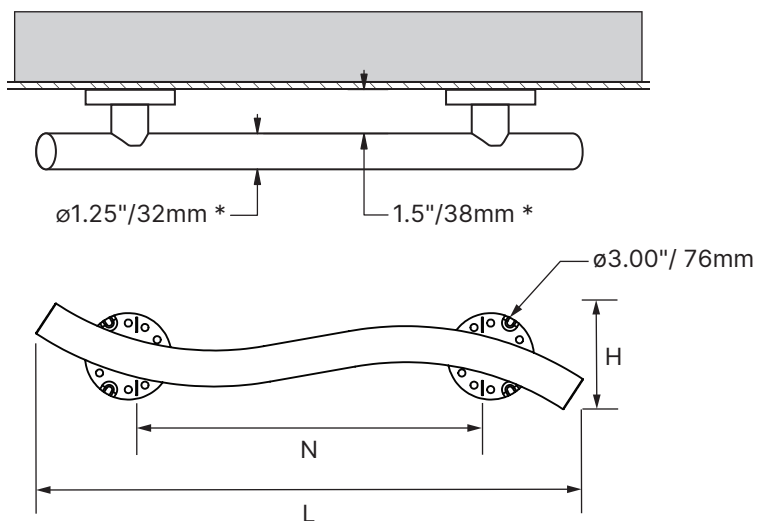

HGB

LENGTH (C)	LENGTH (L)	PRODUCT WEIGHT
12"/305mm	16"/405mm	1.49 lbs / 0.67kg
18"/457mm	22"/557mm	1.79 lbs / 0.81kg
24"/610mm	28"/710mm	2.10 lbs / 0.95kg


♻️ SS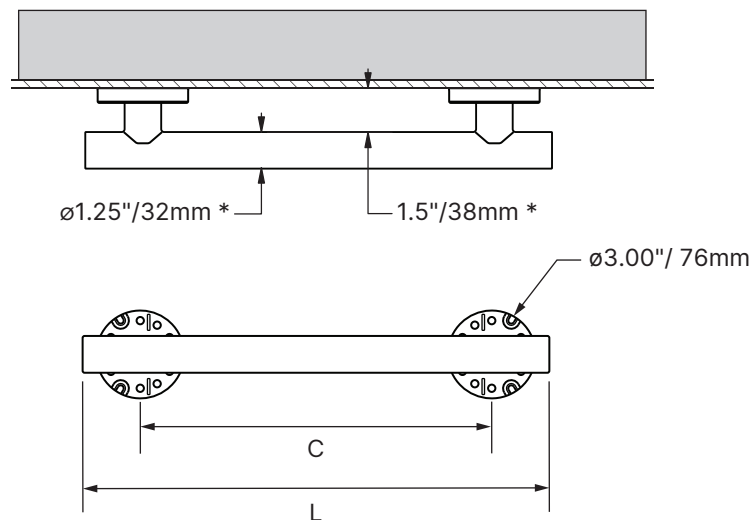

MGB

LENGTH (C)	LENGTH (L)	PRODUCT WEIGHT
12"/305mm	15"/381mm	1.30 lbs / 0.60kg
18"/457mm	21"/533mm	1.61 lbs / 0.73kg
24"/610mm	27"/686mm	1.92 lbs / 0.87kg

♻️ SS

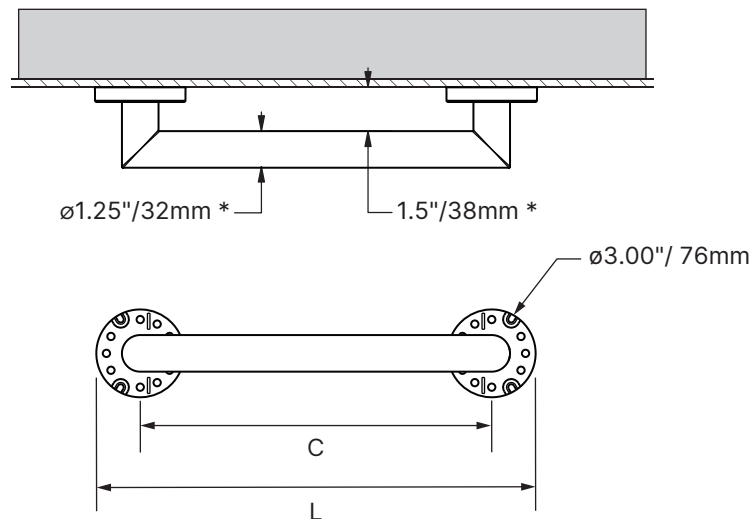MAX
500 lbs
227kg

*ADA Compliant

English

OVERVIEW

Thank you for choosing a HealthCraft Grab Bar (Wave, Horizon, or Miter). Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your grab bar is properly installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your HealthCraft Grab Bar installed by a qualified contractor.

The HealthCraft Grab Bars are intended for moderate vertical and horizontal load bearing to provide sitting and standing support for people with disability. The grab bar should be installed with appropriate fasteners to a structural wall. The maximum user weight for these products is 500lbs / 227kg. Note: Product is not intended to support full body weight. The grab bar is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

DELIVERY CONTENTS

FIGURE A.

1. Grab bar with flange covers
2. #10 x 2" wood screw, S/S (x6)
3. Plastic plugs (x4) - 2 colour options

INSTALLATION WARNINGS

FIGURE B.

1. Rail must be installed to walls that are structurally sound.
2. If desired installation location does not have adequate structure, add wood blocking.
3. Rail may be installed vertically to one stud.
4. Rail may be installed on an angle between 2 studs.
5. CAUTION - Do not install if there is no structure behind drywall. Do not install if there is no structure behind one side of the grab bar.
6. To reinforce hollow walls, we recommend Easy-Mount™ Anchors (each sold separately). Weight capacity 300lbs.
7. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength.
- 8a. To reinforce fiberglass, we recommend Easy-Mount™ Anchors OR
- 8b. Add wood blocking if required.
9. CAUTION - Do not install into metal studs.
10. CAUTION - Do not install if structure behind walls is unknown.

SUGGESTED PRODUCT PLACEMENT

FIGURE C.

To determine the optimal mounting location, we recommend holding the rail in place and walking through the motions of a transfer (sitting, standing, reaching, etc.). ADA recommended rail height of 33-36" / 838-914mm.

INSTALLATION

FIGURE D.

1. Hold the rail against the wall and adjust with level. Using the flanges as a locating guide, mark the screw hole locations through the hole in the flange cover.
2. Drill pilot holes for fasteners.
3. Fasten the rail to the wall using appropriate fasteners and waterproofing.
4. Rotate the flange cover to gain access to other mounting holes, and repeat steps 2-3.
5. Insert preferred plugs into the flange cover holes. Inspect hardware regularly, and tighten if required.

PRODUCT USAGE

FIGURE E.

CAUTION - Grab bar and bathtub are slippery when wet.

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