



HP-XJ-122

User's Guide

Content

Chapter 1. Advantage and specifications

- 1.our advantage.....1
- 2.Technical specifications.....1

Chapter 2. Operation of the machine

- 1.How to install the machine?
 - ① wheelbase model.....2
 - ② dental unit.....3
 - ③ spa model.....4
- 2.How to set the time?.....5
- 3.How to operate the whole teethwhitening treatment.....5

Chapter 3. Tips

- 1.Tips and suggestions

ADVANTAGE AND SPECIFICATIONS

■ our advantage

1. The high-intensity blue light of our advanceaffcti thent technology from Holland is 2-8 times more effectivfatnenother teeth whitening systems to ensure that you get perfectly white teeth in a short time.
2. The specific wavelength accelerates the oxidization proces and will quickly achieve white teeth.
3. The semiconductor light source will thoroughly prevent the disadvantage of halogen and harmful UV light arachieve truly luminescent whitening. The semiconductohas a guaranteed long life(over 50.000 hours).
4. The LED head unit has a unique design, the light source has a wide range and the light comes in at a perfect angleand radiates fully and uniformly.
5. The distance between the LED head unit and cheekretractor will make sure that the LED headunit keeps clean,prevents cross infection,and meet high hygiene standards.
6. The Gooseneck, which can mowe 360 degrees operates conveniently, and you can move the wheelbase anywherewhere you want.

■ Technical specifications

Power supply: InputAC 110V and 240V, 50/60Hz21 Osram Power LED light units

LED life expectancy: > 50,000hours

Magnifying lens: 3 x magnifying

Light intensity: 9000MCD

Blue wavelength: 465nm-470nmPower: 21W

Cooling: power centrifugal air cooling

OPERATION


1. How to install wheel base model?

A. Open the box; take the three parts and bolts out of the box.

B. Screw main unit ④ into pole ②, clockwise to tighten.

C. Attach pole ② into the wheelbase ①

D. Insert bolt from below and tighten with wrench. (Flat washer goes against wheel base and twisted washer in between bolt head and flat washer).

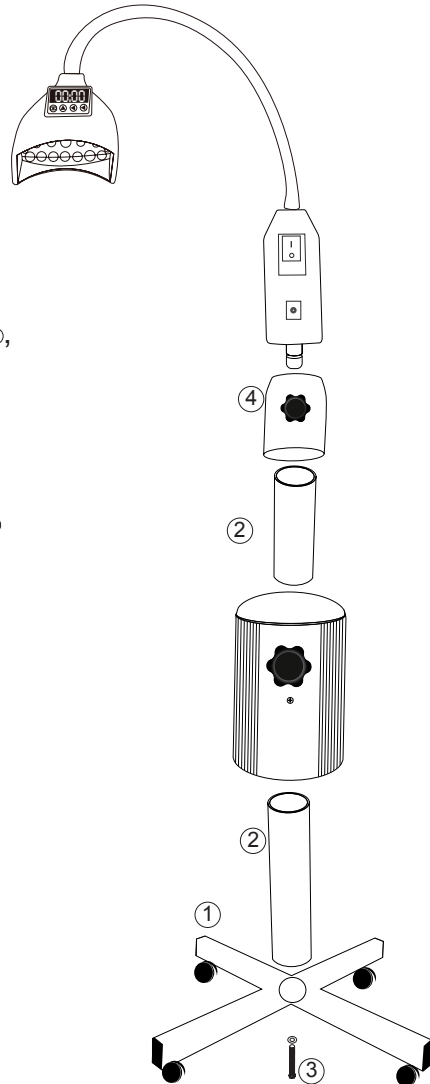


① Wheelbase ② Pole ③ Bolt and Washer ④ Main unit

Appendix:

Assemble the machine by following the steps shown in the picture

2. How to install dental unit?

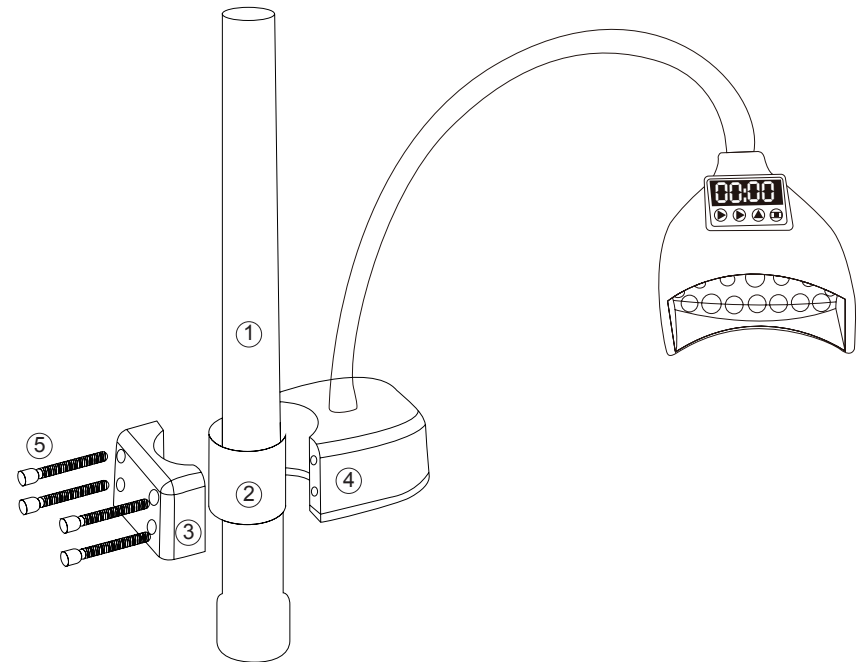

Appendix:

① Pole ② Rubber protective strip ③ Main unit
④ Part ⑤ Bolts

Assemble the machine by following the steps shown in the picture


A. Open the box; take the three parts and bolts out of the box

B. Start with the protective rubber strip ② wrap it around the pole ①

C. Put parts ① and ⑤ into its correct position according to the picture

D. Put the bolts through the holes in part ④ and tighten the bolts into the main unit part ③

1. How to install wheel base model?

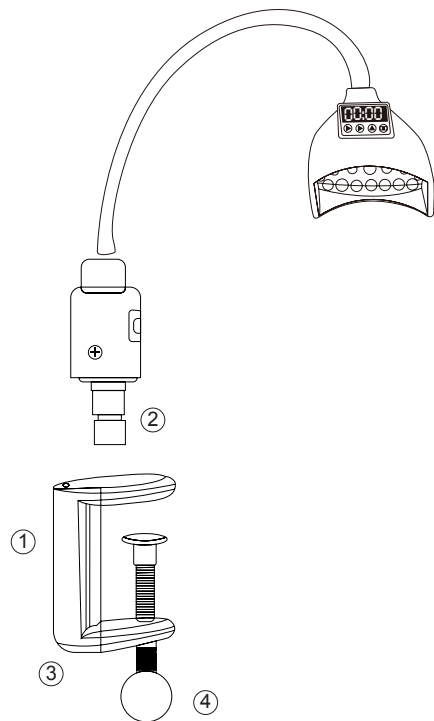


Appendix:

① Screw ② Light Connection ③ Clamp ④ Clamp Screw

Assemble the machine by following the steps shown in the picture

To secure the clamp to a table or specific area. Move the clamp ③ into position, tighten the clamp screw ④ clockwise to secure the clamp into place. Once in place insert the light connection ② in the top of the secured clamp. Tighten the Screw ① clockwise to secure the light connection ② into the clamp. ③

How to set the time?

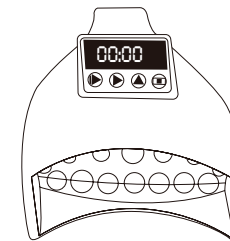

1. Turn on the power, and then activate the operation interface by pressing the power switch.
2. Press the ► button to set the timer, you will see the numbers on the display blink (there are three digits from left to right: hours, minutes, seconds).
3. Press the ► button to set the time by pushing it one or more times.
4. Press the ◻ button to lock the time.
5. After the time adjustment, press the ► button, the machine will enter into the countdown automatically.

How to operate the whole teeth whitening treatment?

1. Turn on the power and press the power switch, the machine will start to work. The system will cut off the power automatically after the time has expired.
2. After the first treatment (see 'how to set the timer' above) set new time for the second treatment or restart by pressing the ► button.
3. Repeat step 2 for the third treatment.
4. After the whitening process, press the power switch on the main unit and turn off the power.

In case of emergency:

If there is an emergency, press the @ switch on the head unit and turn off the power immediately by pressing the black switch on the main unit.

TIPS

Tips and instructions:

1. When adjusting the direction of the light head, you may rotate the gooseneck, but in order to maintain the stability of the machine, do not rotate the tube violently and, be aware that the tube can be stiff at the beginning.
2. The magnifying lens and the LED lights are precision optics in order to avoid dirt, it is recommended that you cover them or turn them downward when they are not in use.
3. The light source is a semiconductor that has a long life expectancy and an enjoyable stability; if something unusual happens while using our equipment, please contact your dealer.
4. We have carefully tested your equipment; please do not disassemble the light head or the control system without the authorization of a professional.

HP-XJ-122