



# Aluminum Telescoping Wheelchair Track Ramps Instructions for Part # TWR-WC-RAMPS-V2 Series



## General Guidelines ⚠

- It is the user's responsibility to read and follow all instructions.
- Keep these instructions with the product at all times and review before each use.
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product.
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects.
- Never modify this product in any way.
- All circumstances cannot be addressed in these instructions. Please use common sense and practice general safety measures when using this product.
- Parts and/or instructions are subject to change without notice.

## Safety ⚠

### **MAXIMUM \*OCCUPIED / \*\*UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL**

- 2' ramp: Occupied loading: 4" maximum, unoccupied loading height: 6"
- 3' ramp: Occupied loading: 6" maximum, unoccupied loading height: 9"
- 4' ramp: Occupied loading: 8" maximum, unoccupied loading height: 12"
- 5' ramp: Occupied loading: 10" maximum, unoccupied loading height: 15"
- 6' ramp: Occupied loading: 12" maximum, unoccupied loading height: 18"
- 7' ramp: Occupied loading: 14" maximum, unoccupied loading height: 21"
- 8' ramp: Occupied loading: 16" maximum, unoccupied loading height: 24"
- 9' ramp: Occupied loading: 18" maximum, unoccupied loading height: 27"
- 10' ramp: Occupied loading: 20" maximum, unoccupied loading height: 30"

\*Occupied is a person sitting in the wheelchair/scooter while loading (2:12 slope or 9.6 degree angle)

\*\*Unoccupied is loading with nobody sitting in the wheelchair/scooter (3:12 slope or 14.5 degree angle)

MAXIMUM WEIGHT CAPACITY IS 600 lbs. (Weight rating is based on evenly distributed load on 2 axles.)

## Warning

- NEVER exceed the rated capacity of your ramp.
- NEVER exceed the maximum height limit listed above.
- Use caution at all times when loading and unloading.
- ONLY load and unload onto a stable, level surface.
- NEVER stand in the path of equipment or vehicle. Keep bystanders and children away.
- DO NOT leave ramps unattended.
- Beware of pinch points. Keep all body parts clear of pinch points when extending or shortening the ramps.
- Ramp surfaces will have less traction when wet. Use caution.
- Always shorten and secure ramps with the latches before carrying.
- The ramp is not a toy. DO NOT use ramp as a jump, scaffold, bridge, ladder, or anything other than its designed purpose.
- ALWAYS make sure the top of the ramp attaching lip is properly secured onto the step, landing, or vehicle before using.
- ALWAYS follow manufacturer recommendations for chair or scooter.

## Operation Guidance

### **Loading wheelchair:**

- Position the wheelchair in the direction of the ramp with front wheel forward. Roll or ride the wheelchair slowly and steadily up the ramp, ensuring that the wheels follow the center of the ramp.
- Take extra care in wet conditions to ensure the wheels do not slide on the ramp.

### **Unloading wheelchair:**



- Position the ramp as detailed above directly behind the rear wheel of the wheelchair. Allow the wheelchair to roll backwards down the ramp, ensuring the wheels follow the center of the ramp.
- Always have a qualified assistant present when using the ramp.

## Maintenance

- It is the user's responsibility to check the loading ramp and components before each and every use. If the ramp is damaged, it should NOT be used.
- Maintain the ramp by regular cleaning.

# Extend or Shorten the Ramps

To extend or shorten the ramps, press in the button on the side of the ramp to release it. Slide the ramp to the desired position, taking care not to pinch your fingers. The ramp will lock into the short or long position when the button fits into the hole.

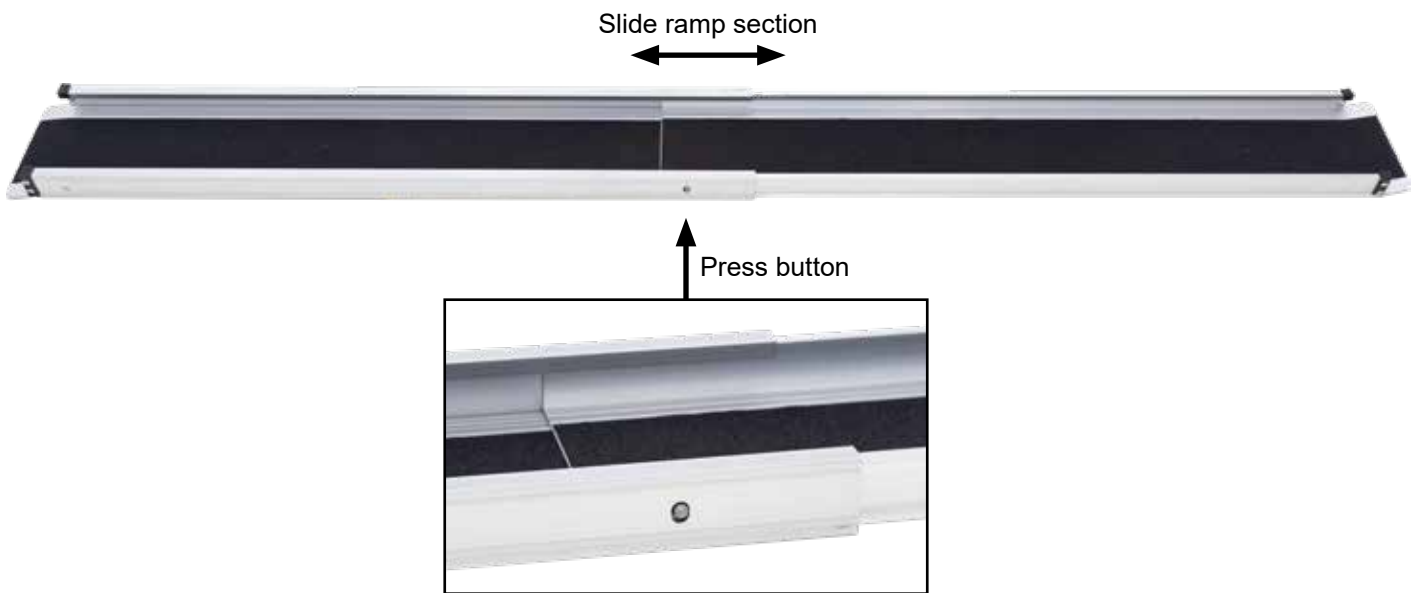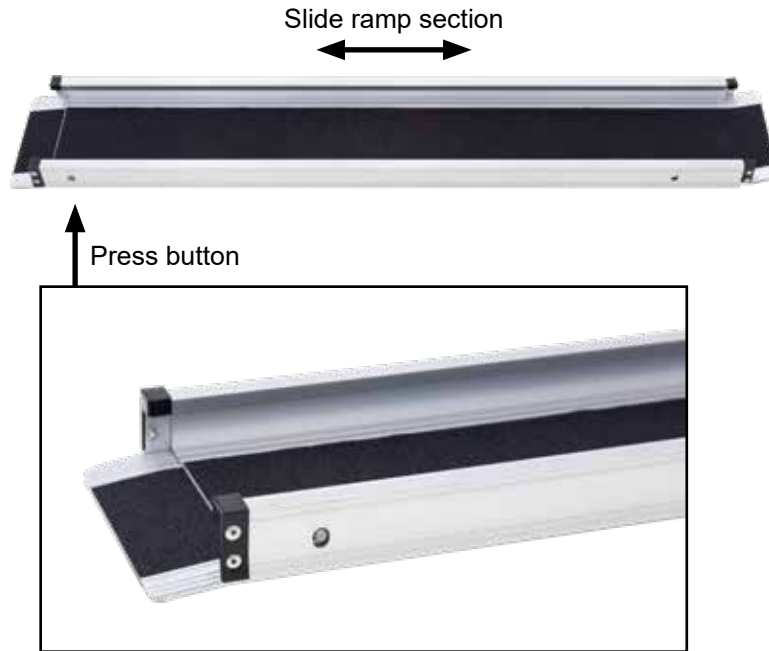

# Product Warranty and Liability

**GENERAL PRODUCT WARRANTY:** Products purchased from the Authorized Dealer (original place of purchase) or Merchant (“Dealer”) will be free of defects in material and workmanship at the time of receipt, and will meet the specifications stated at the place of purchase transaction or online at the Dealer’s website, under normal use and service when correctly installed, operated and maintained. This product warranty is effective for the period of time stated below, unless otherwise stated in the product instructions or depicted in the product advertisement. **All Authorized Dealer warranties are NON-TRANSFERABLE** and cover only the original end purchaser. This limited warranty does not cover products purchased through non-authorized dealers. Non-authorized dealer receipts are not accepted for warranty verification.

**\*\*\*Warranty claims must be made directly to the original place of purchase.\*\*\***

**WARRANTY PERIOD:** This warranty remains in force for one year from the date of the product’s accepted delivery. The Authorized Dealer offers a one year manufacturer’s warranty for most products unless otherwise specified on the product advertisement.

**WARRANTY SERVICE:** The Authorized Dealer will replace any defective or malfunctioning part at no charge, including payment of the shipping costs of parts or replacement product to and from the manufacturer. The purchaser is responsible for labor charges. If the product does not meet specifications as depicted in the advertisement, the Authorized Dealer will refund the full purchase price of the product.

Questions regarding the warranty on a specific product and warranty claims should be directed to the Authorized Dealer with whom the purchase transaction was made.

**WARRANTY LIMITATIONS:** The above warranty does not apply to products that are repaired, modified or altered by any party other than the Authorized Dealer; are subjected to unusual physical stress or conditions (such as overloaded ramps or corrosion), natural disaster, governmental actions, wars, strikes, labor disputes, riots, theft, vandalism, terrorism or any reason beyond reasonable control; are damaged due to improper installation, misuse, abuse, accident or negligence in use, storage, transportation or handling, or tampering; or to products that are considered consumable items or items requiring repair or replacement due to normal wear and tear.

Product should be inspected prior to signing for delivery. Product damage incurred during shipping, unless noted on the Bill of Lading at the time of delivery, renders this warranty void.

**LIMITED LIABILITY:** In no event shall THE AUTHORIZED DEALER be liable to the purchaser or any third party for any indirect, incidental, consequential, special, exemplary or punitive damages arising out of the use of the product, including, without limitation, property damage, loss of value of the product or any third party products that are used in or with the product, or loss of use of the product or any third party products that are used in or with the product.

