

Instruction Manual

Thanks for using our product,
Before using, please read the manual carefully.




Product size:

Seat: 510*310mm


Net weight: 4.4LBS

Seat height (seat to floor height): 410-560mm


Weight capacity: 330LBS

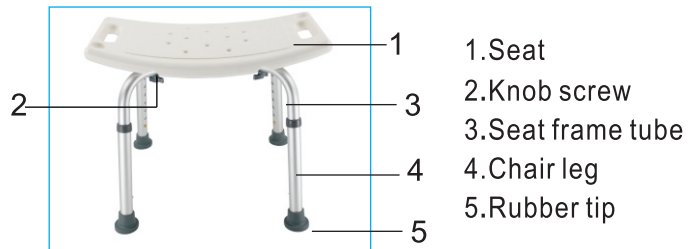


Installation Method:

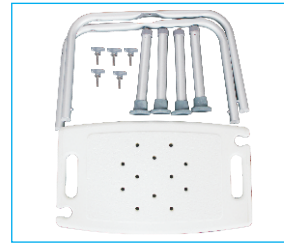

1. Take all the items out of package.

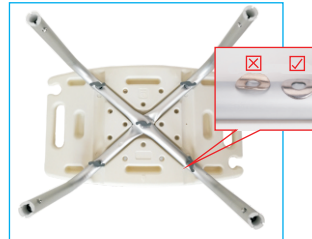

2. Put the seat frame tube on the seat board (the seat board and tubes are marked with 1 and 2, when you put it, 1 and 1 are in the same direction, 2 and 2 are in the same direction), lock the middle screw first, then lock the other four screws.
(Please note: Pay attention to the labeling of the tube, 1 to 1, 2 to 2 put spacer first, and then put screws.)

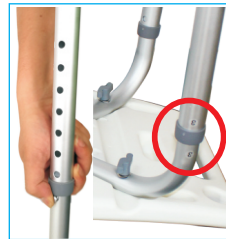

3. Install the foot tube. First press the marbles on the seat tube, then insert the foot tube into alignment, push to fit the hole position, the ball can jump out.
(Note: 4 pcs chair legs must be in the same level of hole)



4. Setup complete.

Scope of application and instructions

1. This shower chair is an assistant facility for disabled people to take a bath more conveniently.
2. Before using the shower chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole.
3. Check seat board, please sit in the middle position of the shower chair when using it.

Storage Method

Please put the shower chair in a dry and ventilated place. Do not exposure it under the sun for a long time.

Precautions

To prevent the injury caused by shower chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water for a long time and keep the board dry.