
 **HEALTHCRAFT**
Embrace life.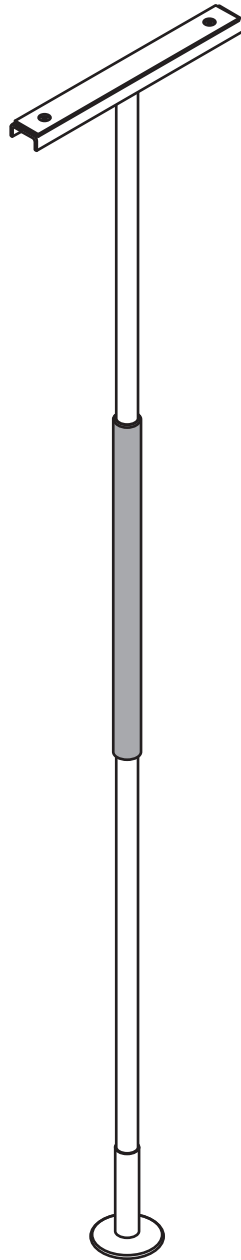

SuperPole™
SP-S
SP-HD
(DC200 - RevB)

healthcraftproducts.com

English

OVERVIEW

Thank you for choosing the SuperPole System from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your SuperPole System is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your SuperPole System installed by a qualified contractor.

The SuperPole System is intended for moderate vertical load bearing to provide sitting and standing support for people with disability. The floor to ceiling range and maximum user weight dependent on model (see Figure E for details and product / accessory compatibility chart). Product is not intended to support full body weight. The SuperPole System is not to be used in any other way than described above.

The SuperPole is a high quality, floor to ceiling safety pole. The SuperPole must be installed between a structural floor and ceiling by turning a jackscrew at the bottom of the pole. The pressure created by the jackscrew is strong enough for the SuperPole to support the user, yet it does not require drilling holes into the floor or ceiling. This is ideal for relocating or removing the pole. NOTE: The top plate has two holes that may be used to fix the plate directly to the ceiling joist if necessary. See Figure C for installation warnings.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. 1-888-619-9992. Patent Pending. © 2026 HealthCraft Products Inc.

DELIVERY CONTENTS

FIGURE A.

- | | |
|--------------------------------------|------------------|
| 1. Pole | 8. Top Plate |
| 2. Pole Grip, 20" length | 9. Base Plate |
| 3. Cover Sleeve (with warning label) | 10. Nylon Washer |
| 4. Threaded Collar | 11. Cotter Pin |
| 5. Jam Nut | 12. Clevis Pin |
| 6. Extension Screw | 13. C-Clip |
| 7. Plastic Washer | |

TECHNICAL DATA

FIGURE B.

INSTALLATION WARNINGS

FIGURE C.

1. Use caution while maneuvering pole during installation.
2. Standard pole can be installed directly under 1 ceiling joist.
3. Standard pole can be installed directly under 2 ceiling joists.
4. Heavy Duty (HD) pole must be secured to ceiling joist (s).
5. If subject to heavy usage, pole must be secured to ceiling joist (s).
6. CAUTION - Damage may occur if extension screw is overtightened.
7. CAUTION - Do not install if the structure above ceiling is unknown.
8. CAUTION - Do not install on loose floor coverings.
9. CAUTION - Do not install in rooms with radiant ceiling heat.
10. CAUTION - Do not install on angled ceilings.
11. CAUTION - Do not install pole where it is directly exposed to water.

SUGGESTED PLACEMENT

FIGURE D.

1. BEDROOM - Locate pole at least 12.5"/320mm from headboard or footboard, adjacent to waistline and either (a) against the mattress - no gaps, or (b) 12.5"/320mm away from mattress.
2. BATHTUB ONLY - Locate pole base against tub, approximately half way along its length.
3. SEATING (TOILET or CHAIR) - To provide clearance when standing, locate pole 3-10" / 76mm-254mm forward of the knees, and 2-6" / 51mm-154mm to the side of knees when sitting.

COMPATIBILITY CHART

FIGURE E.

INSTALLATION INSTRUCTIONS

FIGURE F.

1. Connect topplate to pole and align holes.
2. Insert clevis pin and nylon washer (on opposite side).
3. Install cotter pin into clevis pin to secure the connection.
4. Place base plate on floor at intended location, and tilt pole upright.
5. When upright, insert the pole into the base plate hole.
6. a) Lift cover sleeve and tape to pole, then
b) Insert shaft of screwdriver through hole of extension screw.
7. Hold pole and turn screwdriver clockwise to raise the pole.
8. Ensure top plate contacts the ceiling squarely.
9. Continue turning screwdriver approx. 2-5 turns until pole is secure.
10. Turn jam nut counter-clockwise until it's tight against bottom of pole.
11. Crank screwdriver and jam nut counter-clockwise at the same time until the jam nut is locked.
12. Cover exposed hole on the extension screw with c-clip.
13. Lower cover sleeve to conceal screw.
14. Inspect pole monthly - tighten extension screw if needed.

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