


Toilet Handrail Manual

Model: YC1215

Warm Reminder: For your safety, please read the instructions carefully before use!

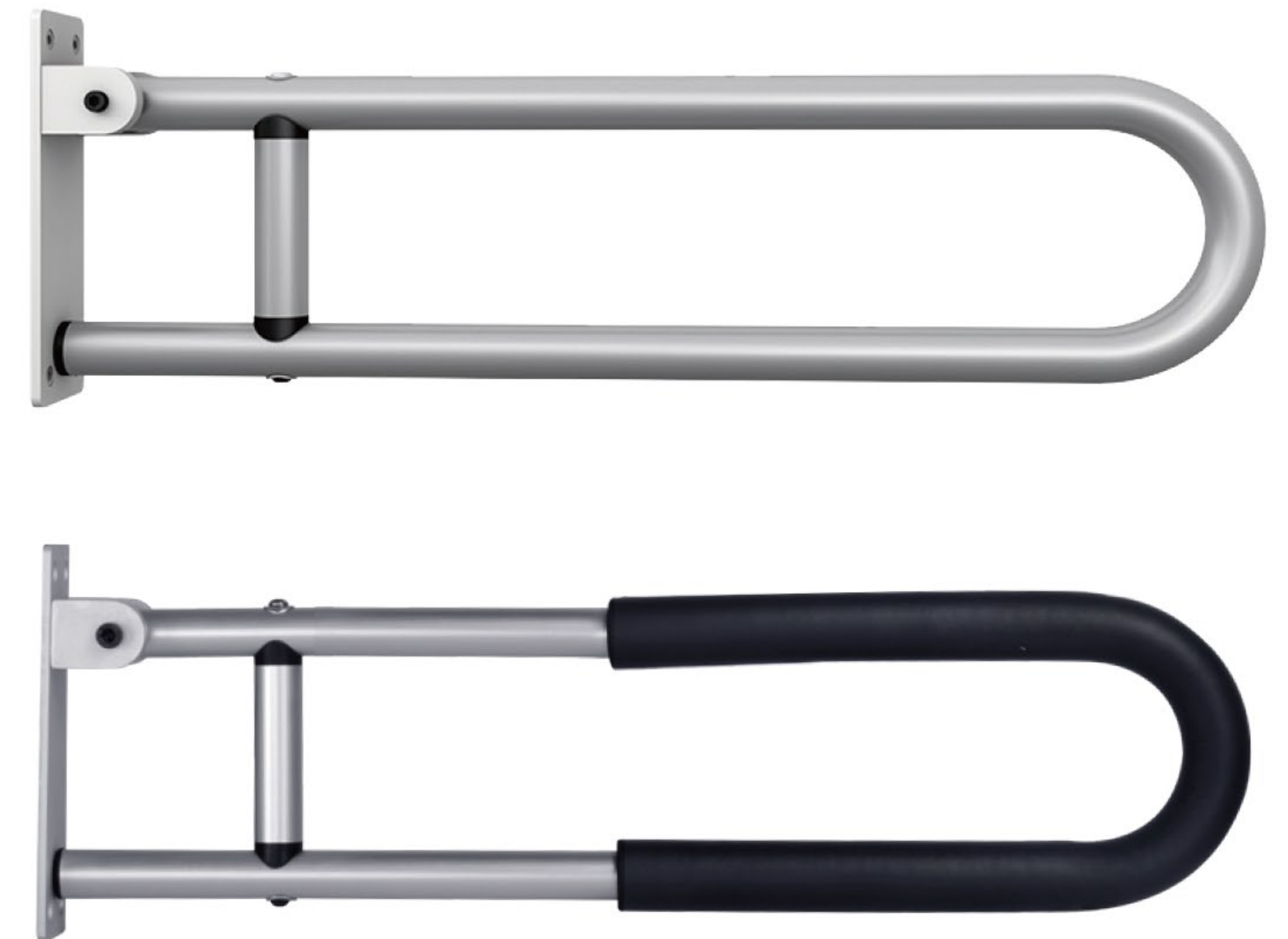

Dear Customer:


- Thank you for purchasing our YC1215 toilet handrail.
- Before using the handrail, please read the manual carefully for correct use.
- After reading, please keep this manual in a safe place so that you can check and refer to it at any time.

Product Description

YC1215 Product Description:

- YC1215 is a folding handrail, which is suitable for toilets, bathrooms, locker rooms, nursing homes, families, and other places. The product is made of aluminum alloy, which is waterproof, rust-free, easy to fold, space-saving, and simple and convenient to use.

Product Size

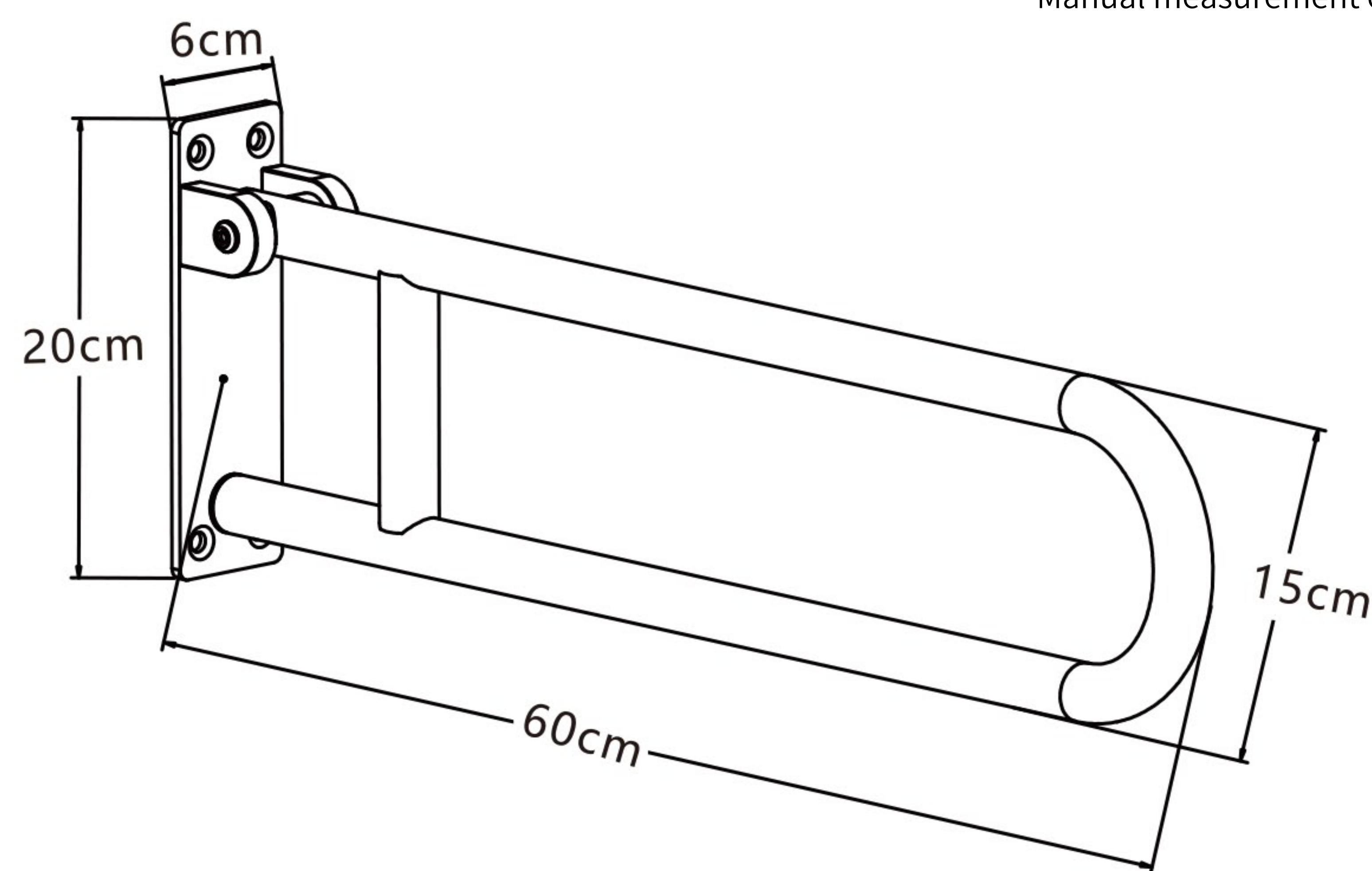

Instructions for Preparing Tools Before Installation



Impact Drill



8mm Impact Drill Bit
(Glass Drill Bit for Tile Walls)



Tape Measure




Marking Pen

Installation Notes:


Please make sure the mounting wall is solid before installation.

Suitable walls: concrete, solid brick, natural dense stone, reinforced walls, etc.


Not suitable walls: low load-bearing walls such as gypsum walls, gray sand bricks, thin hollow sand walls, wooden board walls, etc.

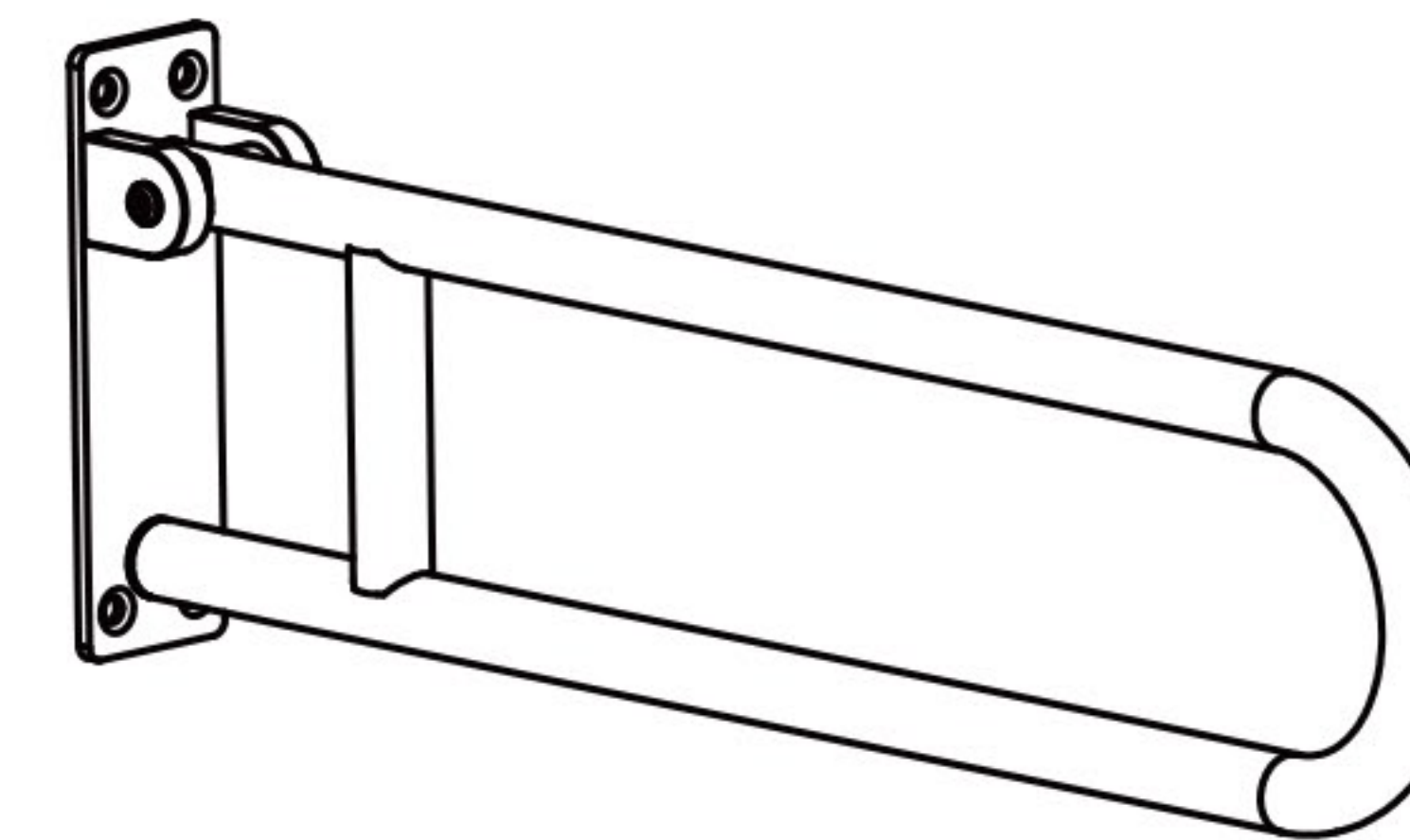


1. Open the case and take out all accessories.

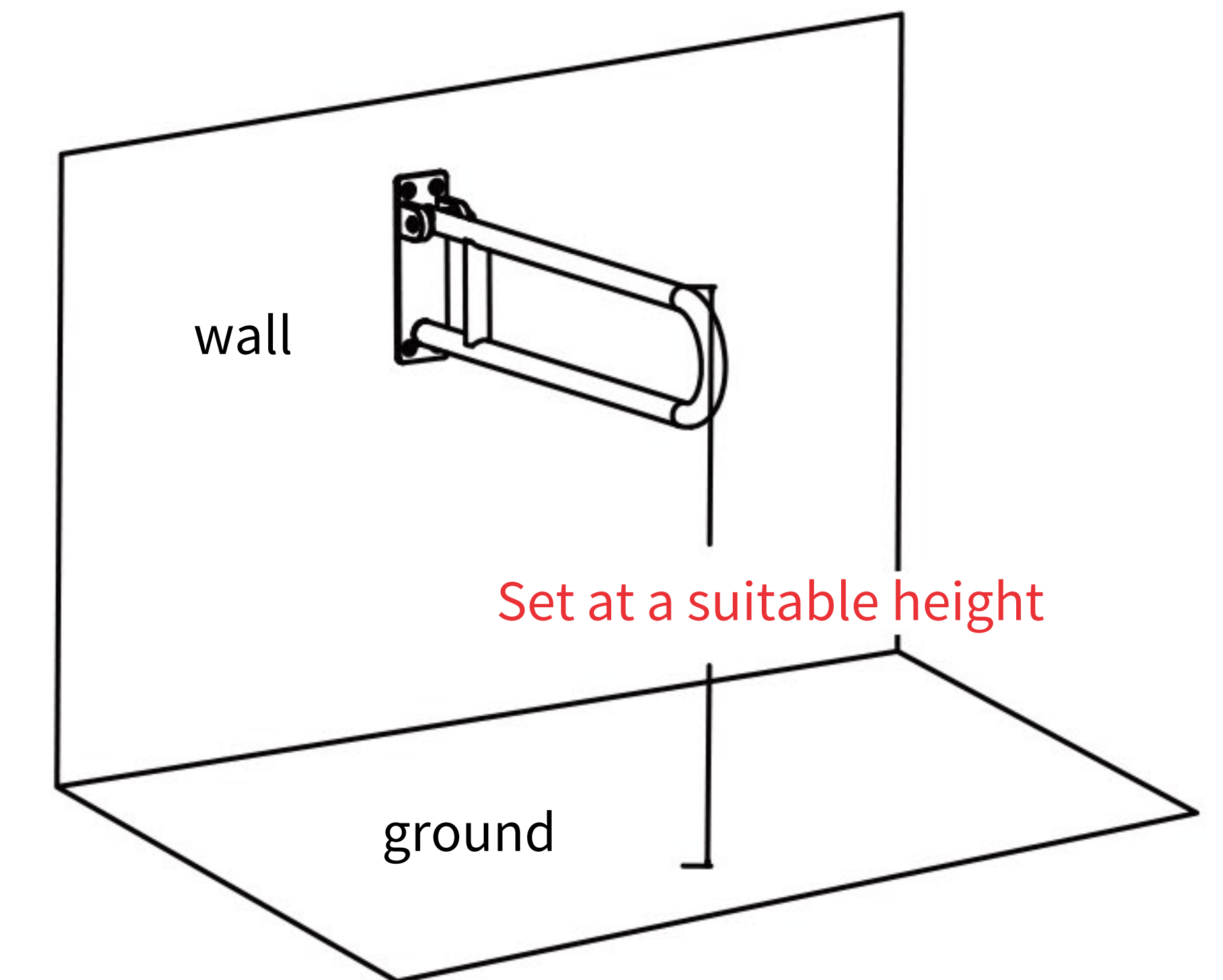


2. Put the handrail against the wall and adjust it to the appropriate installation position according to personal height.

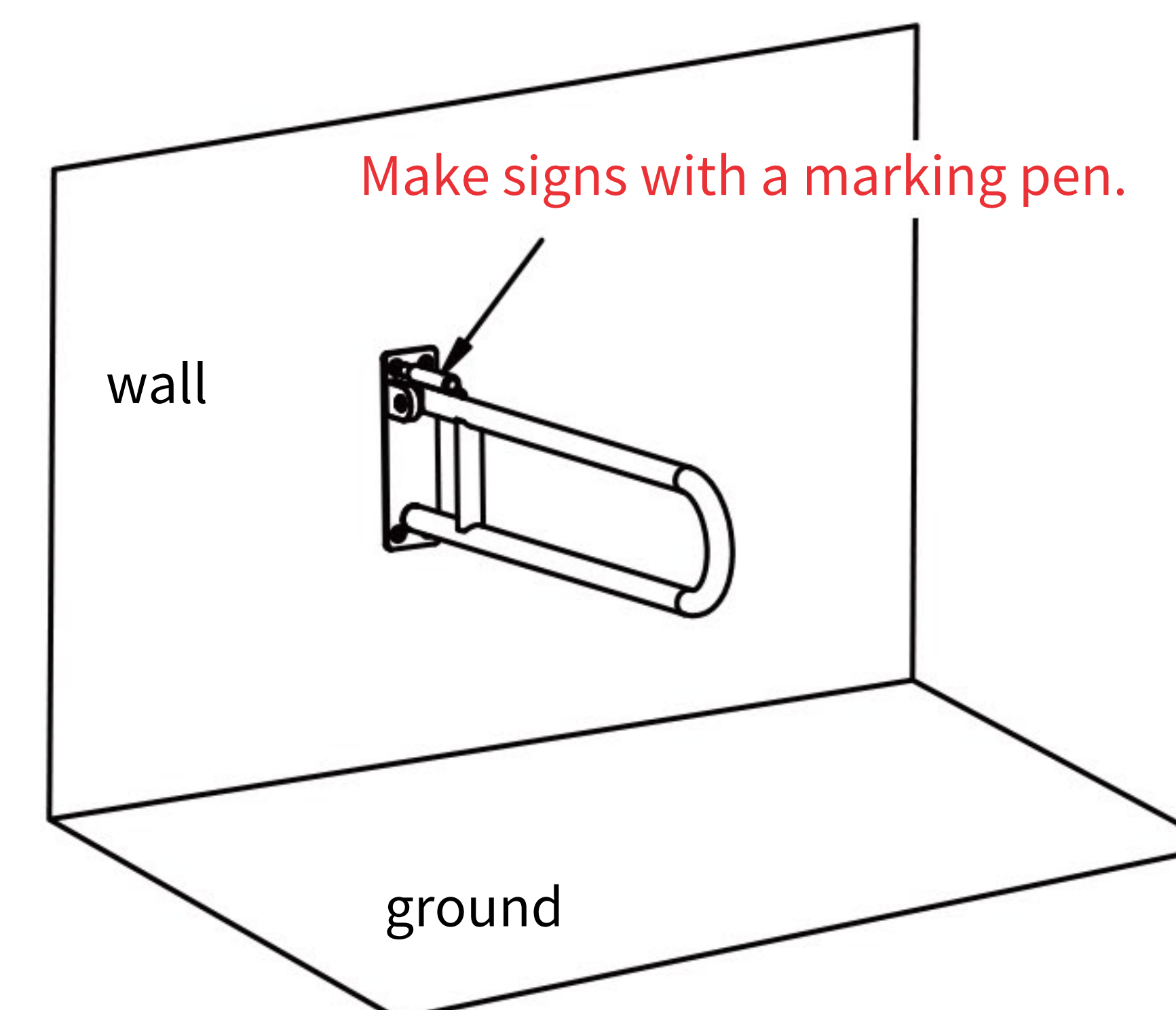


3. Mark all the holes with a marking pen.

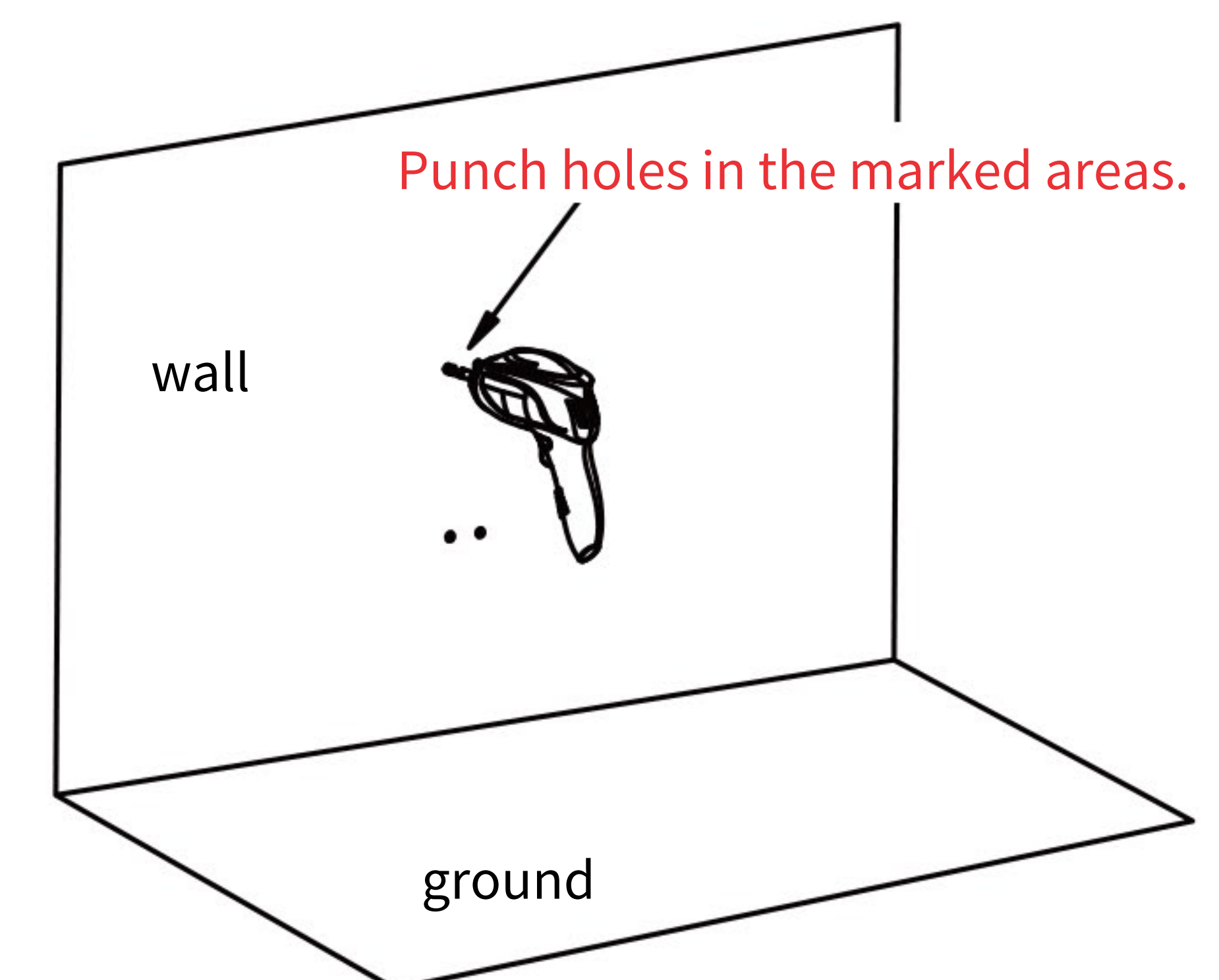

4. Drill holes (drill bit diameter 8mm) and the depth of the holes should not be less than 50mm.

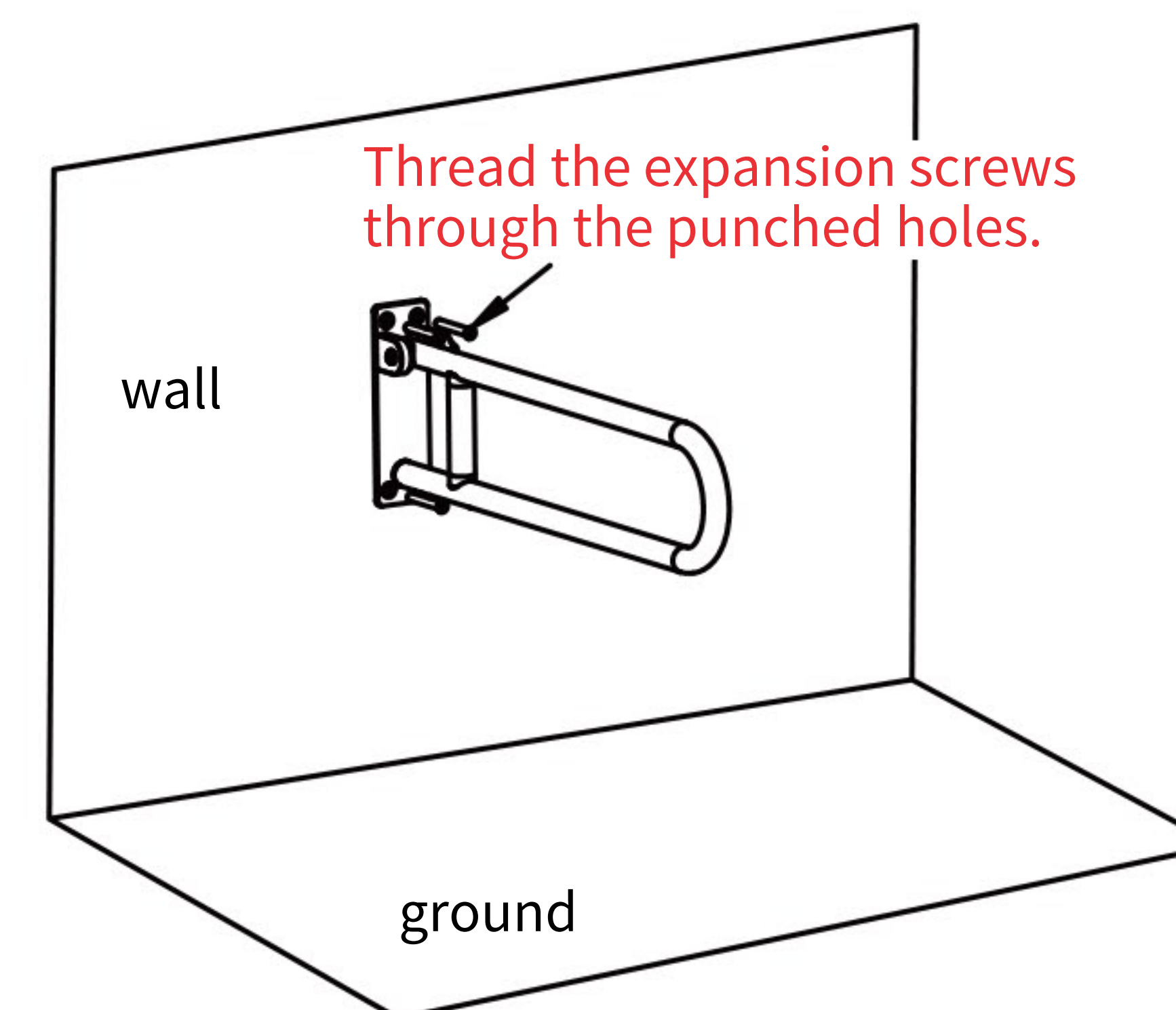

5. Thread the expansion screws into the holes in the fixing plate and align the holes punched in the wall.

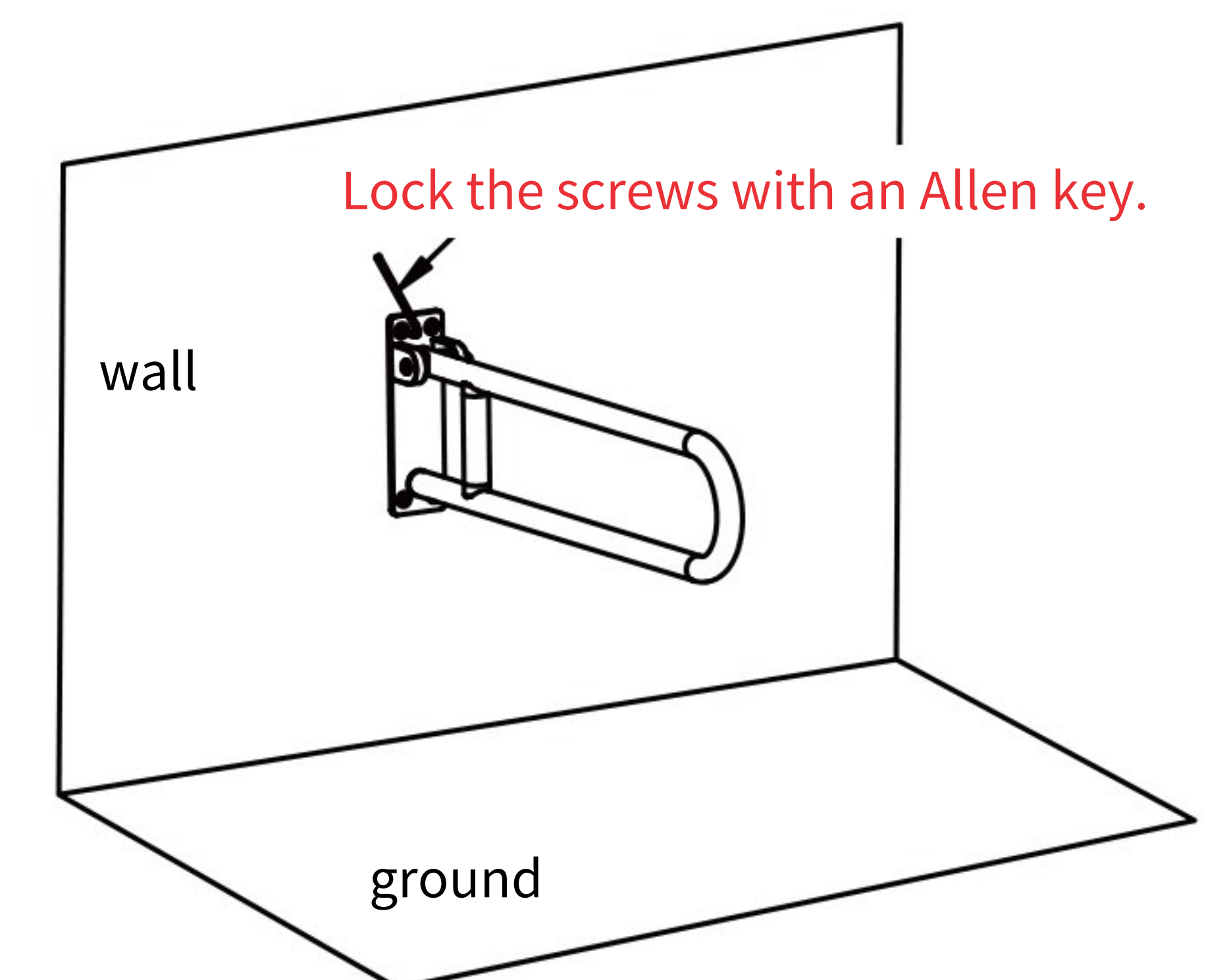

6. Lock the screws with a 4mm Allen key to complete the installation.