

OPERATION METHOD


1. Plug the matching power source we provide.
2. Put your instrument to be sterilized on the tray and close the cover.
3. Adjust the time dial according to your requirement, activating the device. The red indicator will light on which means the device is under working mode. Next, adjust the temperature dial according to your requirement, and by now the green indicator will light on. Both of the lights will go out when the device has finished its work. Please note that neither of the indicator will light on if you only adjust the temperature dial. Generally speaking, all instrument and goods can be totally sterilized and disinfected in 15 minutes at 125 Celsius degrees.
4. Adjust the temperature dial by dialing the left hand knob.



5. Unplug the power supply after each use.

SAFETY WARNING

- Keep the device far away from children's reach
- After sterilization, hook the ring onto the tray and fetch it out the device.
- DO NOT TOUCH THE HOT TRAY WITH YOUR HANDS!

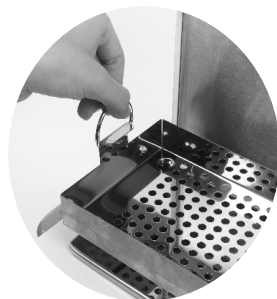

SANITIZING BOX

ITEM NO. CH-360T

INSTRUCTIONS



Rating Voltage: 220 V/50HZ

Power: 300W

It is well-known that high temperature can kill bacteria and virus attached to article. The maximum temperature of our CH-360T can reach 220 Celsius degree. This device is widely used in laboratories, catering services and beauty salons.

OPERATION METHOD


1. Plug the matching power source we provide.
2. Put your instrument to be sterilized on the tray and close the cover.
3. Adjust the time dial according to your requirement, activating the device. The red indicator will light on which means the device is under working mode. Next, adjust the temperature dial according to your requirement, and by now the green indicator will light on. Both of the lights will go out when the device has finished its work. Please note that neither of the indicator will light on if you only adjust the temperature dial. Generally speaking, all instrument and goods can be totally sterilized and disinfected in 15 minutes at 125 Celsius degrees.
4. Adjust the temperature dial by dialing the left hand knob.



5. Unplug the power supply after each use.

SAFETY WARNING

- Keep the device far away from children's reach
- After sterilization, hook the ring onto the tray and fetch it out the device.
- DO NOT TOUCH THE HOT TRAY WITH YOUR HANDS!

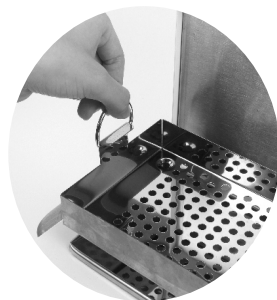

SANITIZING BOX

ITEM NO. CH-360T

INSTRUCTIONS



Rating Voltage: 110V/60HZ

Power: 300W

It is well-known that high temperature can kill bacteria and virus attached to article. The maximum temperature of our CH-360T can reach 220 Celsius degree. This device is widely used in laboratories, catering services and beauty salons.