



713-034



STEP 1:

Please take out two main frame, there have left and right side, and you need to distinguish which part is front also.


Please check that front side has a plastic part for the foot rest.



Step 2:

Please find out those two piece “U” shape Tube, and do the assembling like above Photos.

And please don’ t forget to put in the Commode Pot steel support Frame in the holes before tightening the screws.

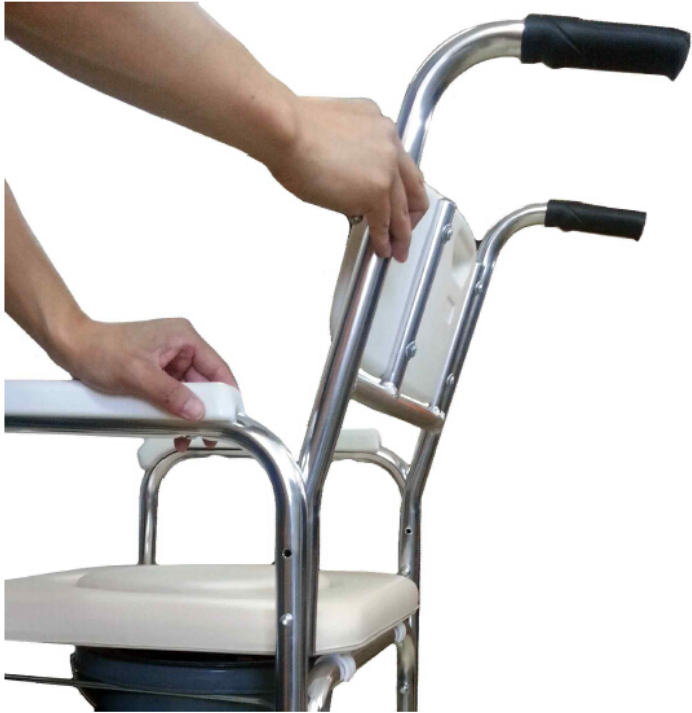

Step 3:

When you finished “U” Tube, and Commode Support Frame’ s assembling, please don’ t tightening them so hard, as you also need to assembling the backrest also, like above photos, But still keep on the screws in a little loose situation, as you need to leave some adjustable for the footrest tube.



Step 4:

Take out the Footrest and those two pieces “Round” Metal part.

Please put into those two pieces “Round” Metal Parts into the footrest tube hole for screws steady.

Then insert to the main frame Plastic Part. And tightening the screws again.

When you finished this step, and please tightening all of the loose screws of this product, and make sure they are in hard tightening now.



STEP 5:

Final, Please find out the Soft PVC seat and pot, just put to the “U” tube with the snap joint in match.

Now, it is finished, enjoy the using.