

Remove Demo Watermark from <http://www.wancaiinfo.com/>



Oversized Drop Arm Commode CST-2082

CAUTION
UP TO 550LBS



Do NOT install commode without first reading and understanding this assembly and instruction manual. Failure to follow instructions may result in serious injuries. If you cannot understand these instructions, then contact a dealer or technical professional before attempting to install; Otherwise, injury or damage may occur. In particular, we are not to be held liable for any indirect or direct damage or consequential damage, loss or expenses in connection with the use or inability to use the device for a specific purpose.

WARNINGS

⚠ Inspect all parts for shipping damage before assembly. If there is shipping damage – DO NOT USE. Contact dealer / carrier for further instructions.

⚠ Ensure ALL screws, nuts and / or bolts are tightened. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.

⚠ Users with limited physical strength should be supervised or assisted while using commode

⚠ Do not exceed the maximum static pressure bearing or serious injury could result, maximum static pressure bearing: 550lbs(250kg)

⚠ DO NOT place commode chair on uneven surfaces.

Storage Method

Please put the Commode chair in a dry and ventilated place. Do not exposure it under the sun for a long time.

Before removing / installing seat and lid, allow seat / ring to reach room temperature. This will help prevent the seat clamps from breaking when being removed / installed onto commode back frame rail.

⚠ Prior to each use check to assure that all screws, joints and fittings on the commode chair are tight.

⚠ Toilet seat must be in down position before use. Failure to do so may result in breakage of unit and possible injury to user.

⚠ Before using the commode chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole, all four leg extensions with rubber tips MUST touch the floor simultaneously at all times.

⚠ Check seat board, the backrest are securely installed, please sit in the middle position of the Commode chair when using it.

⚠ Keep away from babies and children in order to avoid the risk suffocation or choking.

⚠ To prevent the injury caused by commode chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water and keep the commode chair dry.

Oversized Drop Arm Commode CST-2082

Thanks for using our product.

Before using, please read the manual carefully.



Seat Size:23.2"*18.9"/59*48cm

Seat Height:20"-24"/51-61cm
(Seat to floor height)


Product Height:30.7"-34.6"/78-88cm

Product Width:29.5"/75cm

Net Weight:20.7lbs/9.4kg

Static pressure bearing force:550lbs/250kg

Elements of the chair and manual for installation:

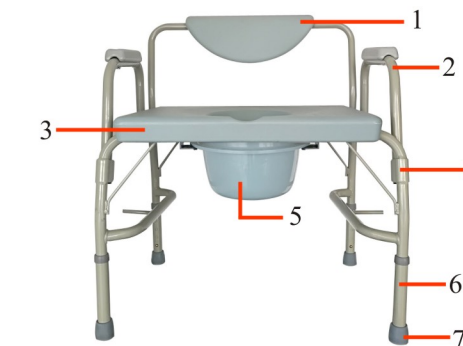


1. Backrest combination
2. Armrest
3. Seat pan
4. Main frame
5. Bucket
6. Leg
7. Rubber tip



1. Take all the items out of package.



2. Pull both sides of the armrest up in turn and pull down to the armrest tube to lock completely.

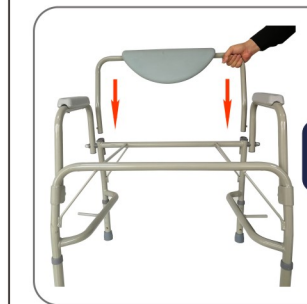

3. Insert the backrest into the main frame, press down until the marble pop up.



4. Align the seat plate position to the mounting position of the seat, then press down hard to lock the plate.



5. Put on the bucket and complete the installation.

Note: When adjusting the height, press and hold the marbles to move up or down. All four foot tubes should be adjusted to the same height before being used.