
INVISIA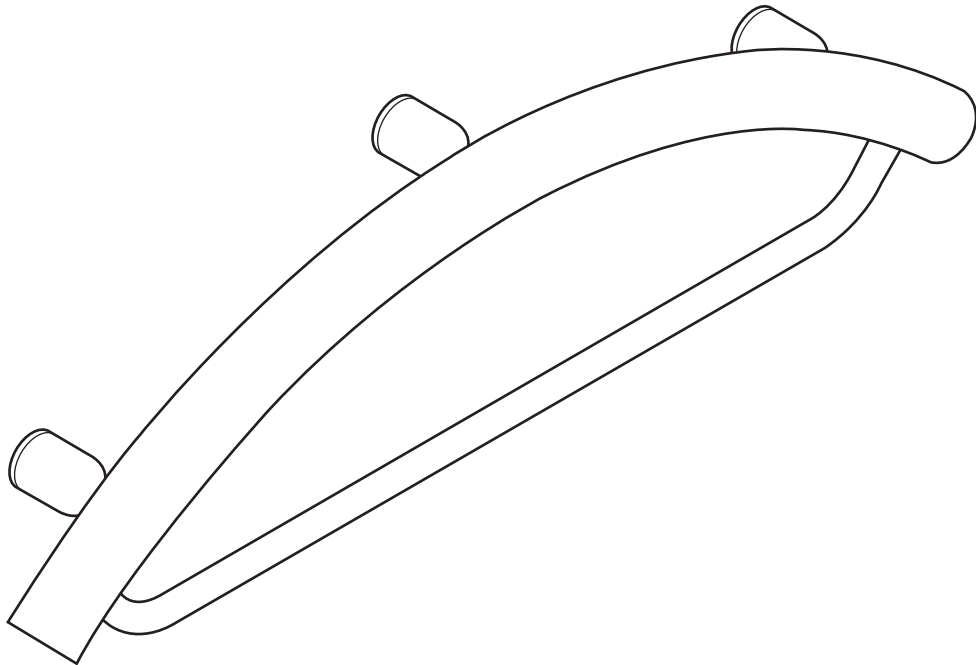

Towel Bar

INV-TB
(DC154 - RevF)

healthcraftproducts.com

English

OVERVIEW

Thank you for choosing the INVISIA Towel Bar. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your INVISIA Towel Bar is properly installed and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your INVISIA Towel Bar installed by a qualified contractor.

The INVISIA Towel Bar is a combination grab bar and towel bar designed to disguise safety in a bathroom accessory. It is intended to be used as a grab bar and its secondary function is to hang a towel in bathrooms near bathtubs/toilets. It is intended for moderate vertical and horizontal load-bearing, providing sitting and standing support for people with disabilities. It should be installed with appropriate fasteners to a structural wall. The towel rung (which holds a towel) is NOT intended to be a support product. The maximum user weight for this product is 500lbs / 227kgs.

NOTE: Product is not intended to support full body weight. The INVISIA Towel Bar is not to be used in any other way than described above.

LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. 1-888-619-9992. Patent Pending. © 2026 HealthCraft Products Inc.

DELIVERY CONTENTS

- | | |
|----------------------------|----------------------------------|
| 1. Grab Bar - Towel Bar | 5. 3/32" Hex Key |
| 2. Wall Stem Assembly (x3) | 6. Lock Washer, S/S (x3) |
| 3. #10-32 Set Screw (x3) | 7. #10 x 3" Wood Screw, S/S (x3) |
| 4. Center Punch | |

FIGURE A.

INSTALLATION WARNINGS

FIGURE B.

1. Bar must be installed to walls that are structurally sound.
- 2a. CAUTION - Do not install if there is no structure behind one or more wall stems.
- 2b. If desired installation location does not have adequate structure, add wood blocking.
- 3a. CAUTION - Do not install in fiberglass / acrylic shower enclosures with insufficient stiffness and strength. 3b. Add wood blocking if required.
4. CAUTION - Do not install into metal studs.
5. CAUTION - Do not install if structure behind walls is unknown.

SUGGESTED PRODUCT LOCATION

FIGURE C.

To determine the optimal mounting location, we recommend holding the bar in place and walking through the motions of a transfer (sitting, standing, reaching, etc.). ADA recommended bar height of 33-36" / 838-914mm.

INSTALLATION

FIGURE D.

1. Back off set screws from bar sockets.
2. Allow wall stems to freely move in and out of bar sockets.
3. Peel off label on back of wall stems to expose sticker.
4. With wall stems inside bar sockets, push the bar against the wall. Ensure level.
5. Firmly press the bar so stickers on back of wall stems contact wall.
6. Remove bar carefully so wall stems stay on the wall.
7. Using wall stems as locating guides, mark the tile with center punch.
8. Drill pilot holes for fasteners.
9. Fasten the wall stems to the wall using appropriate fasteners and waterproofing.
10. Place bar back onto the fastened wall stems.
11. Secure bar to wall stems by tightening set screws. Inspect hardware regularly, and tighten if required.

PRODUCT USAGE

FIGURE E.

CAUTION - Grab bar and bathtub are slippery when wet.

TECHNICAL DATA

FIGURE F.

- *Complies with ADA 609.2 Cross Section - Grab Bars.
- *Complies with ADA 609.3 Spacing - Grab Bars.
- *Complies with ADA 609.8 Structural Strength - Grab Bars.

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