

GENERAL INSTALLATION INSTRUCTIONS

Tools Needed

•Toggle Bolts • Pencil • Level • Tape Measure • Drill

IMPORTANT: Be sure to re-inforce wall before installation. We recommend a 2" x 12" securely fastened to wall studs at the desired height for your installation.



1. If wall studs are not easily accessible, you will need to install wood blocks behind the finished wall. For a safe installation please seek professional help. **Wall should be re-inforced before continuing.**
2. Mark the location for your installation at desired height on wall ensuring that position is over studs. Install your shower seat approximately 17"- 21" above the shower floor, or 17"-19" for ADA compliance. **If your seat has legs be sure to put legs down when choosing mounting height to ensure legs touch floor after installation.**
3. Using the template provided mark the locations for the mounting holes. Use shower seat if template was not provided to determine the mounting holes locations.
4. Use appropriate drill bit for your wall construction to drill mounting holes, and use appropriate mounting hardware to secure the mounting brackets.
5. Place bracket covers over the mounting brackets and use mounting clips to secure seat assembly in place. If included, insert the seat assembly into the bracket cover slots.
6. Complete your installation by snapping the mounting pin caps into the mounting bracket covers.

WARNING:

Wall construction and materials may vary from home to home. This seat CANNOT be mounted to acrylic, drywall, fiberglass, plaster, sheetrock, or any other similar wall surfaces without the required wood backing support material. Seek professional help to ensure a safe installation.