

Power Lift Chair (4/21/2019)

A Catnapper Power Lift Chair will arrive packaged in a box/carton similar to the provided picture.

Upon arrival the solid line on the box should run parallel to the floor indicating that the chair is properly oriented.

The solid line is indicated by the red arrow on the provided picture.



Rotate the fully packaged chair end-over-end until the solid line on the box is "up" as indicated by the red arrow in the provided picture.



Exercise caution opening the bottom of the box, the flaps are secured using clench staples. Failure to do so can result in serious injury or product damage.



0:55

Fold the box flaps down and rotate the entire box back to its original position in the first step

The solid line on the box will once again be parallel to the ground.



1:05

When the solid line on the box is parallel to the ground, carefully lift the box/carton off of the furniture.

Use caution when performing this step, the staples used to secure the box flaps can cause injury or product damage.



1:10

Remove and set aside the box/carton.



1:25

Remove the protective plastic bag covering the chair.
Dispose of packaging materials in the proper manner depending upon local guidelines.



1:25

Remove the protective plastic bag containing the back chair frame.
Dispose of packaging materials in the proper manner depending upon local guidelines.



1:52

Position the back frame as pictured and align the metal KD sleeves on the back frame with the metal KD posts.



2:00

Picture shows a properly aligned KD bracket.

The metal KD sleeve on the back frame is indicated by the red arrow while the metal KD post is indicated by the blue arrow.



2:05

When both KD brackets are aligned, press firmly down on the top of the back frame until the locking tabs “click” into place.

Before operating the chair, lift up on the back frame to ensure that it is properly locked into place. The back should not come off unless the tabs are physically unlocked by the user.



2:05

The provided picture shows individuals firmly pressing down on the back frame to ensure the KD brackets are properly locked in place.




Rotate chair forward. Place protective plastic on the floor to avoid damage to the furniture surface,
Secure the outside back cover along the bottom edge using the existing hook and loop fastener.




Remove the transformer from the bottom of the chair. The transformer will be located in a red bag under the chair.

Make sure that the wires are routed away from any moving parts to ensure that they will not become pinched or damaged during operation.

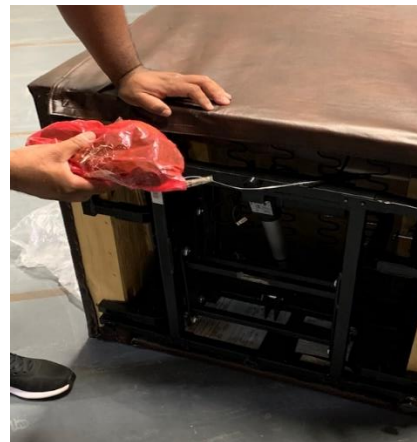

The transformer for a lift chair has two cables, one that goes from the wall plug to the transformer, and one that goes from the transformer to the chair.


The first cable is indicated by the red arrow and the second cable is indicated by the blue arrow.

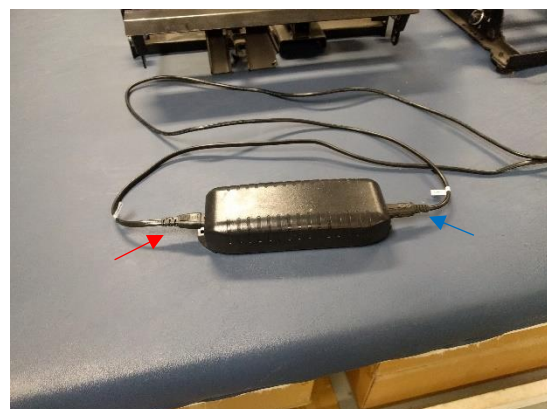


The transformer on the bottom has a slot for (2) 9 volt batteries. These batteries will allow the chair to be returned to the “at rest” position in the event of a power outage.

Normally a brand new fresh set of batteries is capable of 1 use in an emergency, after this one use the batteries must be replaced.

Check the batteries periodically to ensure that they are in working condition.

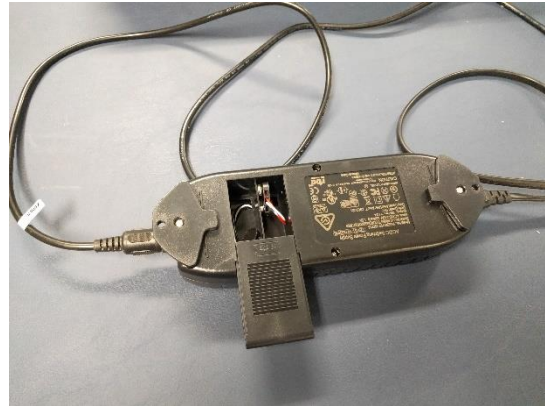

Plug the transformer directly into a wall receptacle/outlet. Do not plug transformer into an extension cord, surge protector, or port replicator.

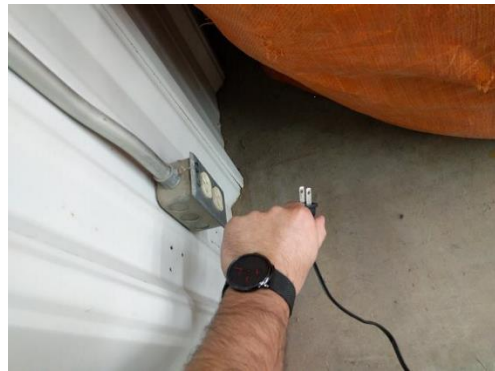

Before operation, make sure that all leg levelers are completely screwed in.



Tighten the leg levelers completely so that they are all the way in.

If adjustment is needed after the chair is in operation these leg levelers can be used for small adjustments to ensure the chair is properly stabilized even on slightly uneven surfaces.



3:37

This picture shows the chair in the “at rest” or seated position.

Always return to the chair to this position when not in use.

NEVER leave the chair in the raised/lifted position when unoccupied.



4:30

To lift the chair, hold the “up” button on the hand wand control. Releasing the button at any time will stop the chair from moving.



This picture shows the chair in the fully lifted position.



To return the chair to the seated or reclined position, simply hold the “down” button. Stop the chair in the desired position by releasing the “down” button.

Picture shows the chair in a partially reclined position.



This picture shows the chair in the “full recline” position.

Recline angle will vary depending on product style/model.



Before plugging any device into the provided USB port, consult the device owner's manual to ensure compatibility. Failure to do so can result in damage to the chair and/or device.

Always use (OEM) factory approved charging cables. Aftermarket/substitute cables may not have the same safety features/standards as OEM cables and the use of these non-approved cables could result in damage to the chair and/or device.

