


JKGQMH-000400 可调节成人移动马桶（扶手不可调） 英文说明书


Adjustable Adult Toilet

Instruction Manual

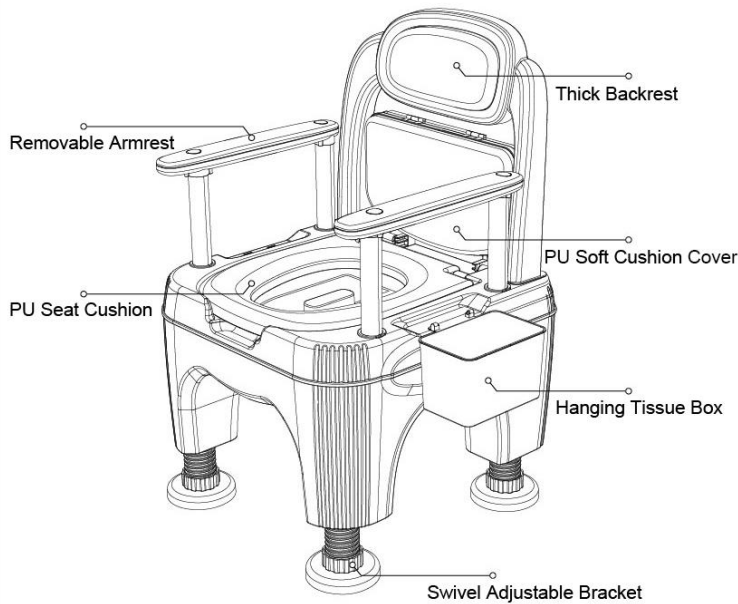

- ✓ Anti-rollover
- ✓ Thoughtful Design
- ✓ Strong Load Bearing
- ✓ Easy to Clean

• Product Components

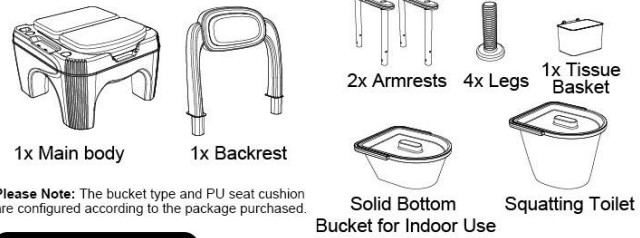

Please Note: The bucket type and PU seat cushion are configured according to the package purchased.

• Installation Steps

-
-
-
-
- 1 Put the toilet body upside down, rotate the legs to the right position by aligning them with the holes in turn, check and make sure that the legs are firmly installed and parallel;
 - 2 Align the backrest with the holes and press down firmly, check and make sure the backrest is firmly installed;
 - 3 Align the armrests with the holes and press down, install the other armrest in the same way, check and ensure that both armrests are firmly installed;
 - 4 Remove the lid and base, put in the inner bucket and fix the paper towel basket.