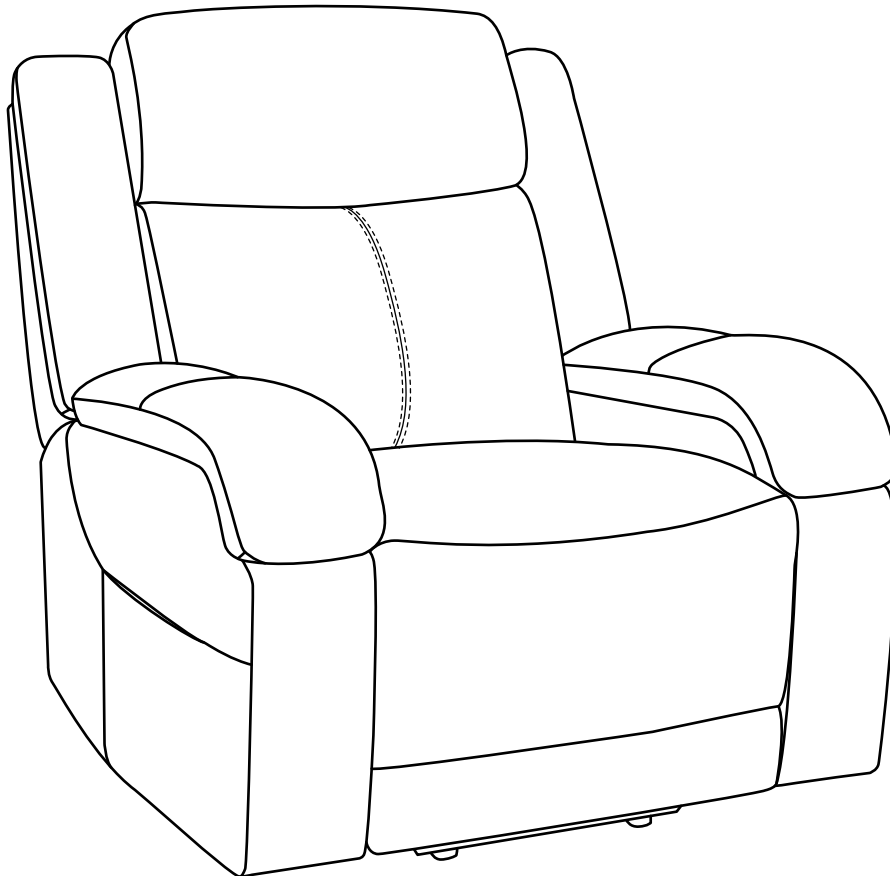


KJW.A5029

Assembly Manual

Dual Motor Power Lift Chair with Massage and Heat





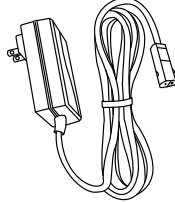
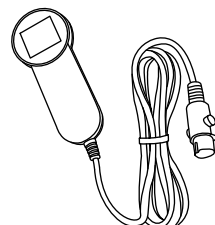


▲ CAUTION

THIS ASSEMBLY MANUAL CONTAINS IMPORTANT SAFETY INFORMATION.


PLEASE READ AND KEEP FOR FUTURE REFERENCE.

Count parts before assembling.
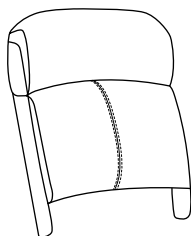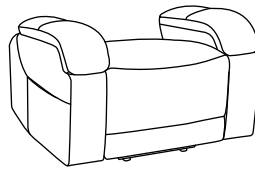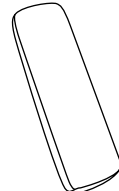
<p>A 1pc</p> 	<p>B 1pc</p> 	<p>C 1pc</p> 
<p>Backrest</p>	<p>Seat</p>	<p>Left Wing</p>
<p>D 1pc</p> 	<p>E 1pc</p> 	<p>F 1pc</p> 
<p>Right Wing</p>	<p>Power Supply</p>	<p>Hand Controller</p>


Step 1

Cut the plastic ties to remove the hardware pack (Includes: Power Supply (E) and Hand Controller (F)). Locate the Left Wing (C) and Right Wing (D) at the bottom of the Seat (B), then take them out.

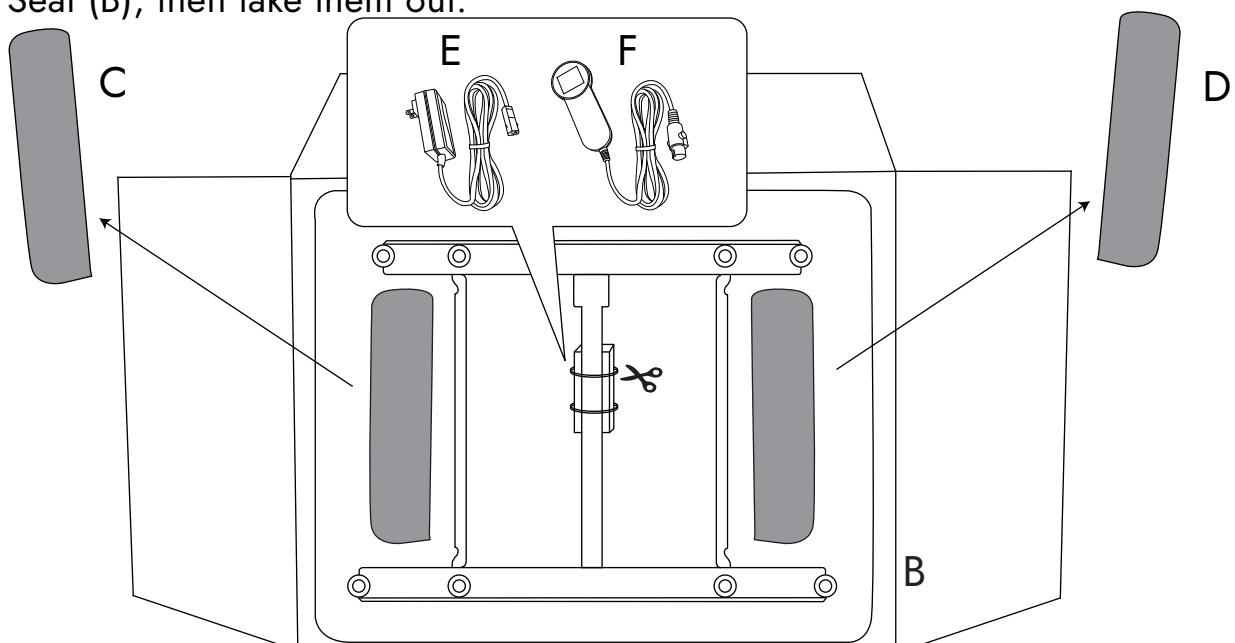


Step 2

Zip up the Backrest (A) and the Seat (B) together.

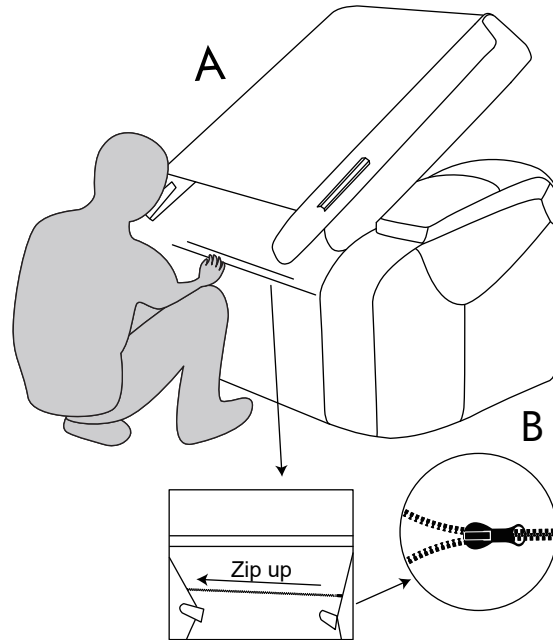


Step 3

Slide the Backrest (A) into the metallic quick connectors of the Seat (B). Push down the Backrest (A) firmly until each side is locked into place and an audible "click" is heard.

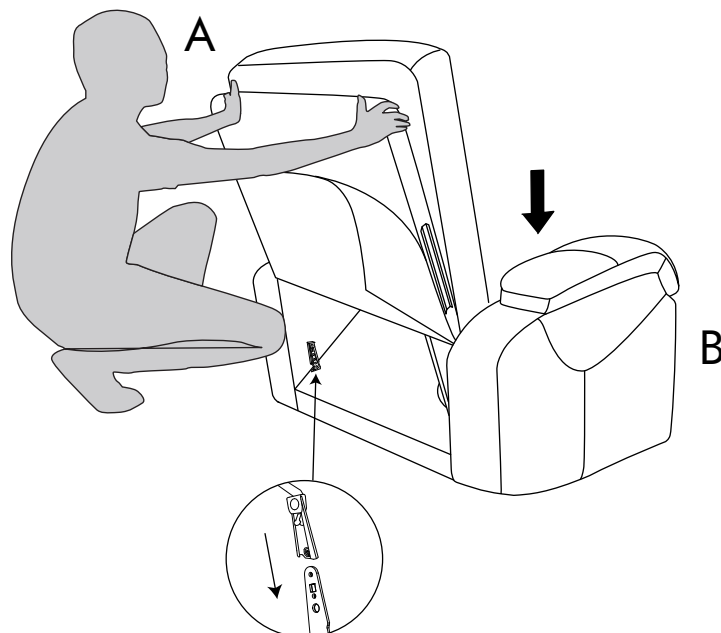


Step 4

Slide the Left Wing (C) and Right Wing (D) into the Backrest (A).

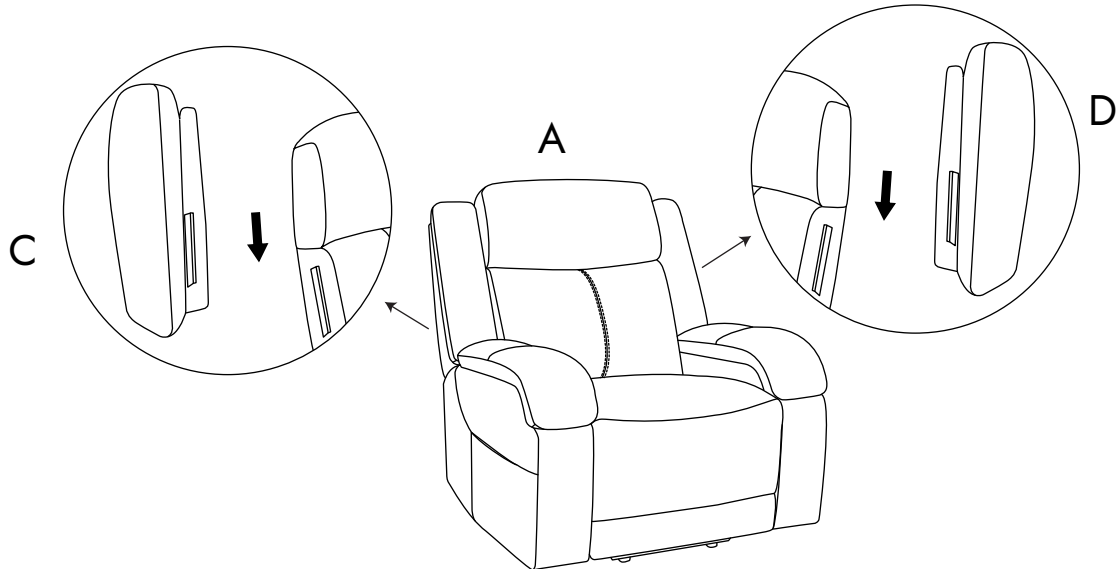

Step 5


Cables for hand controller are connected as shown in below table. If you want to place the hand controller (F) on the right side of the armrest, please fasten the hand controller cable to G. If you want to place the hand controller (F) on the left side of the armrest, please fasten the hand controller cable to H.

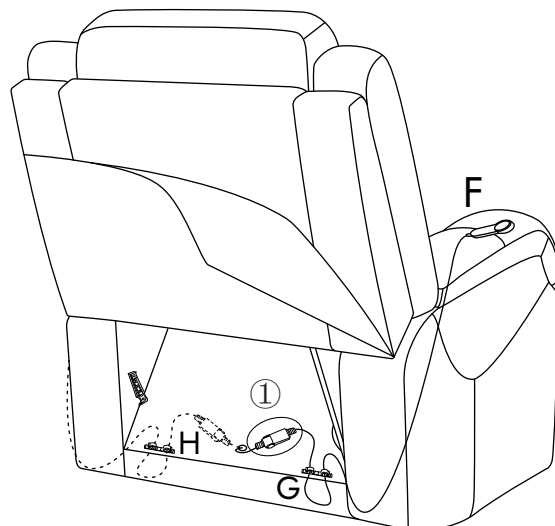

Number	Connection method	Description
①		Connect a with b and fasten the buckle.

Step 6


The remaining cables on the Backrest (A) shall be connected as shown in below table.

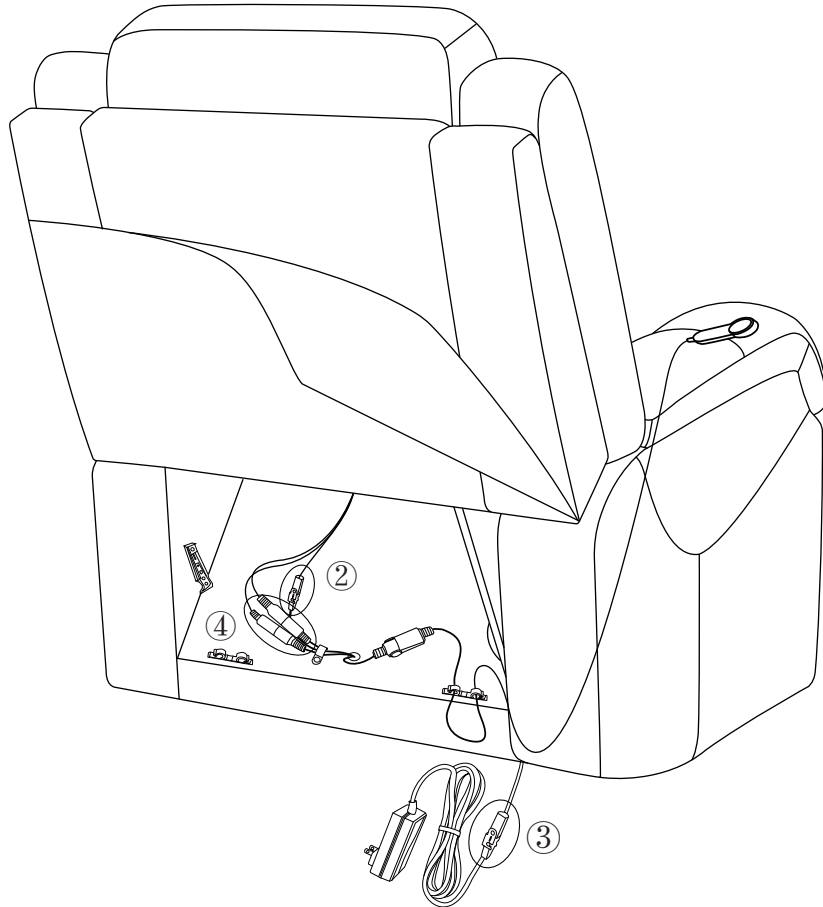

Number	Connection method	Description
②③		Connect c with d and fasten the buckle.
④		Find the pattern on the connector . Align it with the arrow and connect e and f. (Connect all the wires by matching the colors (red to red and white to white).)


Step 7

Attach the back cover to the bottom of the seat with velcro.

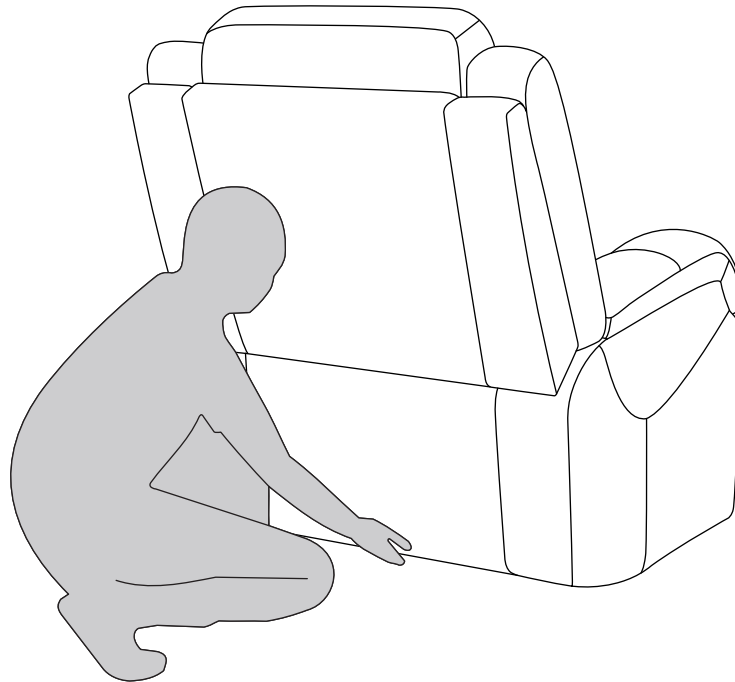

Step 8

Plug the Power Supply (E) into the power outlet. After the chair is powered on, you can enjoy by adjusting it with the Hand Controller (F).

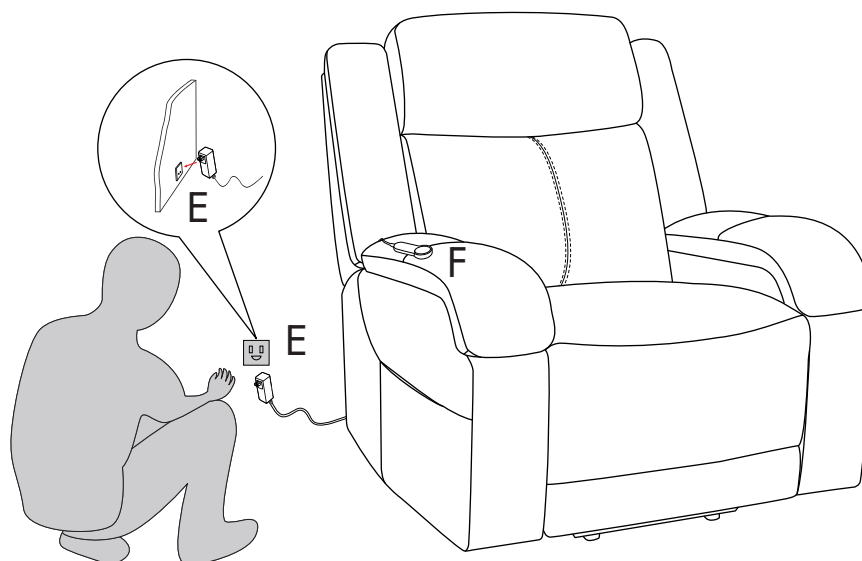

Hand Controller User Manual

