

## Scope of application and instructions

1. This shower chair is an assistant facility for disabled people to take a bath more conveniently.
2. Before using the shower chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole.
3. Check seat board, the backrest are securely installed, please sit in the middle position of the Shower chair when using it.

## Storage Method

Please put the shower chair in a dry and ventilated place. Do not exposure it under the sun for a long time. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

## Precautions

To prevent the injury caused by shower chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water for a long time and keep the board dry.

## WARNINGS:

- ⚠️ Prior to each use check to assure that all screws, joints and fittings on the bath bench are tight. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times. Be sure that all gold locking buttons fully protrude thru holes of each Adjustable Leg.
- ⚠️ DO NOT place bath bench on uneven surfaces.
- ⚠️ DO NOT use if wobbly, unstable or not level.
- ⚠️ DO NOT exceed user weight capacity of 205kg(450LBS).
- ⚠️ DO NOT stand on the bath bench.
- ⚠️ DO NOT use the armrest to fully support body weight when using the unit.
- ⚠️ Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions
- ⚠️ This shower chair cannot be used in the bathtub.
- ✅ Excessive twisting or turning while seated on the bench may cause a loss of balance and damage to the product.
- ✅ Check rubber tips for rips, cracks or wear. If these conditions exist, DO NOT use the bath bench and replace the tips immediately.
- ✅ Check the bench for cracks or ruptures in the plastic.
- ✅ Check that all push buttons are fully protruded from the mainframe through the leg extensions and are properly engaged.
- ✅ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use.



## SHOWER CHAIR CST-3011

**CAUTION**  
UP TO 450LBS





To ensure the safe installation, adjustment, and operation of the shower chair, these instructions MUST be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER AND/OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions, and instructions, contact a professional or dealer before attempting to use this equipment. In particular, we are not to be held liable for any indirect or direct damage or consequential damage, loss or expenses in connection with the use or inability to use the device for a specific purpose.

## SHOWER CHAIR CST-3011


Thanks for using our product.  
Before using, please read the manual carefully.



Seat: 510\*310mm / 20×12.2 "   
Net weight: 2kg / 4.4LBS   
Seat height (seat to floor height): 410-560mm / 16.1-22 "   
Static pressure bearing force: 205kg / 450LBS

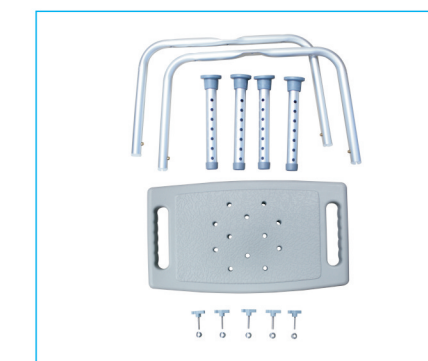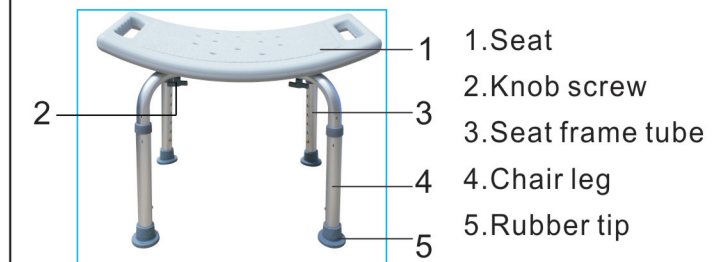

1. Take all the items out of package.

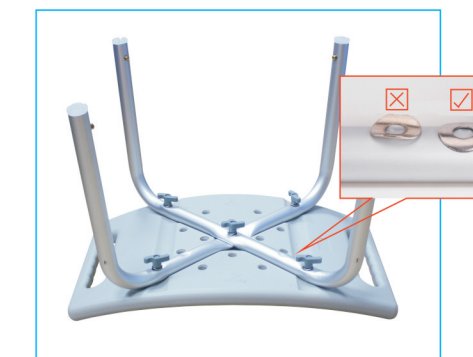

2. Put the seat frame tube on the seat board (the seat board and tubes are marked with 1 and 2, when you put it, 1 and 1 are in the same direction, 2 and 2 are in the same direction), lock the middle screw first, then lock the other four screws. (Please note: Pay attention to the labeling of the tube, 1 to 1, 2 to 2 put spacer first, and then put screws.)



3. Install the foot tube. First press the marbles on the seat tube, then insert the foot tube into alignment, push to fit the hole position, the ball can jump out. (Note: 4 pcs chair legs must be in the same level of hole)



4. Setup complete.