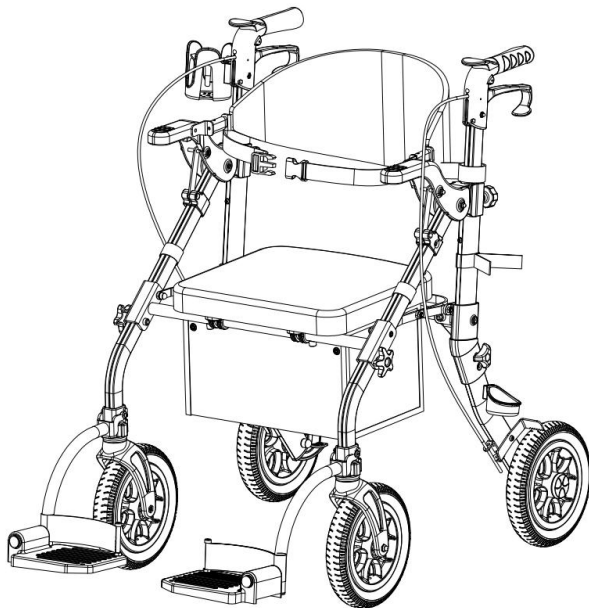
Upgrade · The Home Creator Way

Manual Rolling Walker

MODEL: HFK-9269KD-1

MODEL: HFK-9269KD-1

Note: The product picture is for reference, the actual details shall prevail

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

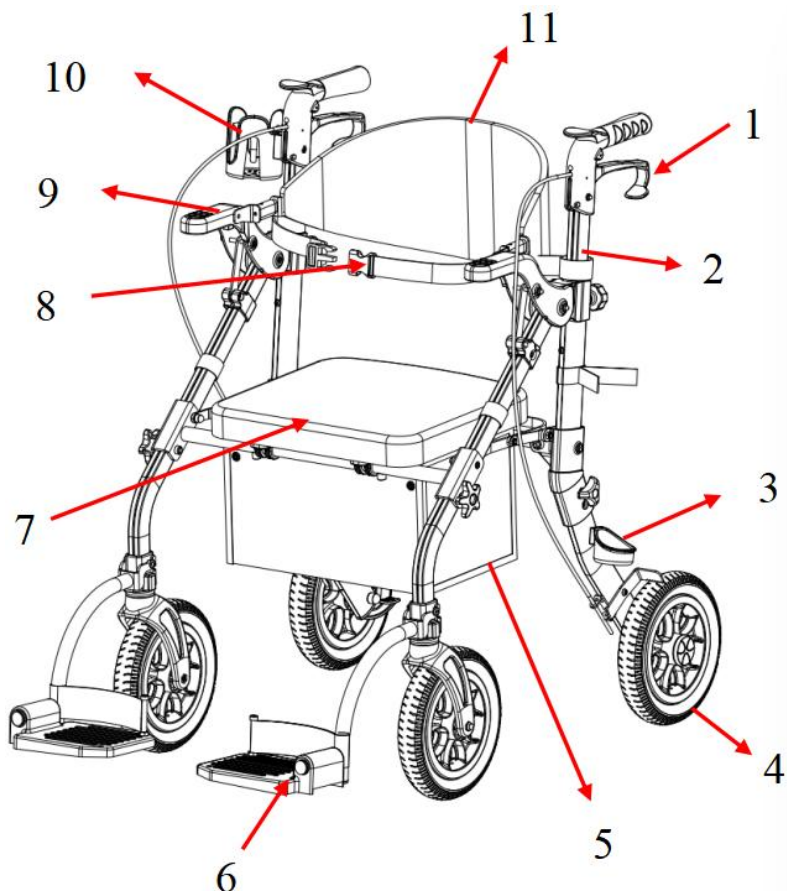
Please retain instructions for future reference

Protect yourself and others by observing all safety information, warnings and cautions. Failure to comply with instructions could result in personal injury and/or damage to product property.

WEEE symbol. Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your Local Authority or local store for recycling advice.

PRODUCT STRUCTURE DIAGRAM

- 1.Brake lever
- 4.10-inch PU wheel
- 7.Seat cushion
- 10.Cup holder

- 2.Adjustable armrest
- 5.Shopping bag
- 8.Safety belt
- 11.Backrest

- 3.Umbrella holder
- 6.Removable footrest
- 9.Press-down armrest

APPLICATION SCOPE

- 1.Indoor use: This four-wheeled vehicle is an essential item for individuals with limited mobility to move freely at home.
- 2.Outdoor use: Effectively meets the needs of individuals with limited mobility to breathe fresh air outdoors or travel short distances.
- 3.Meets the user's need for short breaks.

MAIN PARAMETER

Model	HFK-9269KD-1
Wheel	10-inch PU wheel
Load Capacity	300 lbs
Width	24.2inch

ASSEMBLY

Pre-Assembly Preparation


After unpacking the carton, first use scissors to remove the packaging. Please take care to protect all the parts inside the carton. If assembling on the floor, lay down kraft paper or a carpet first to prevent scratches. Then remove the inner carton and take out the parts. The carton contains loose components, as shown in Figure 1.

Item	Part Name	QTY	Picture
1	Rear wheel & Adjustable handle	2	
2	Main frame	1	
3	Front wheel	2	


3	Footplate	2	
4	Back rest	1	
5	Safe belt	1	
6	Cup holder	1	
7	Cane holder	1	
8	Star knob screw	2	
9	Triangle knob screw	2	
10	Round knob screw	2	

Figure 1

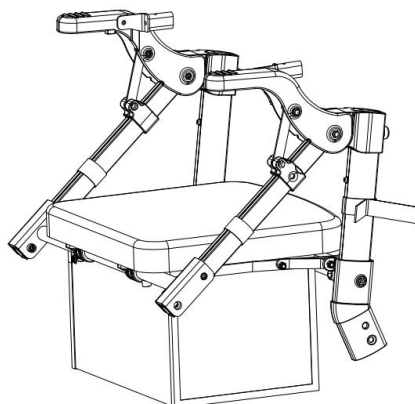
Press the crossbar under the seat plate by hand to open the main frame and lock the seat plate in place. Then place the main frame of the four-wheeled vehicle on a solid, level surface, press down the armrest and pull it tightly into position toward the hole end.

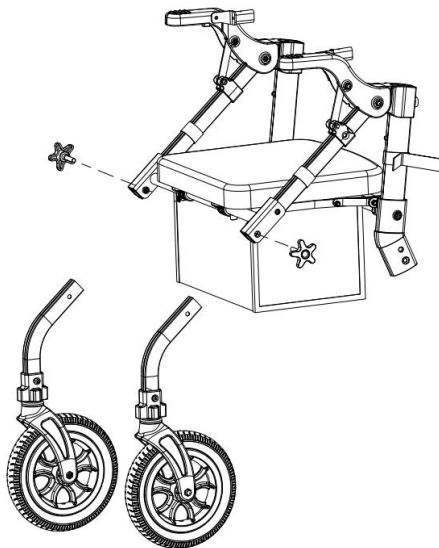Attach the two front wheel tubes to the main frame, then secure and tighten them with the star knobs, as shown in the figure below: (Repeat the same steps for the other side)

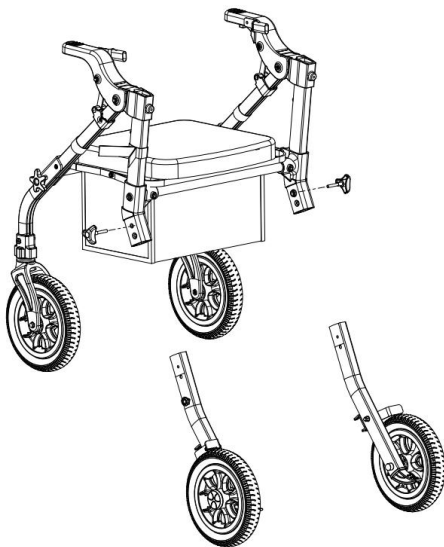
Attach the two rear wheel tubes to the main frame, then secure and tighten them with the three-point knobs, as shown in the figure below: (Repeat the same steps for the other side)


First, thread the safety belt onto the armrest tube, then insert the armrest tube (with the brake cable attached) into the hole on the main frame. Select your desired height position and tighten it with the round knob. (Repeat the same steps for the armrest tube on the other side)

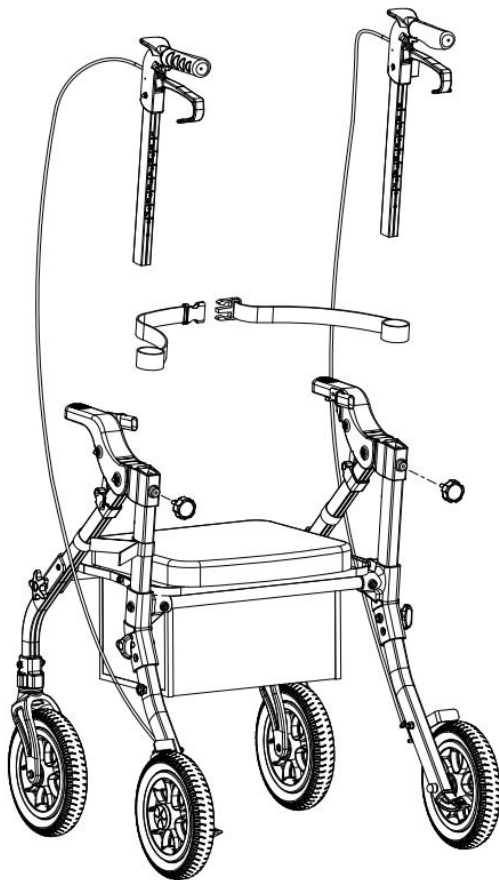
Note: Ensure the brake cable is positioned on the outside of the frame.

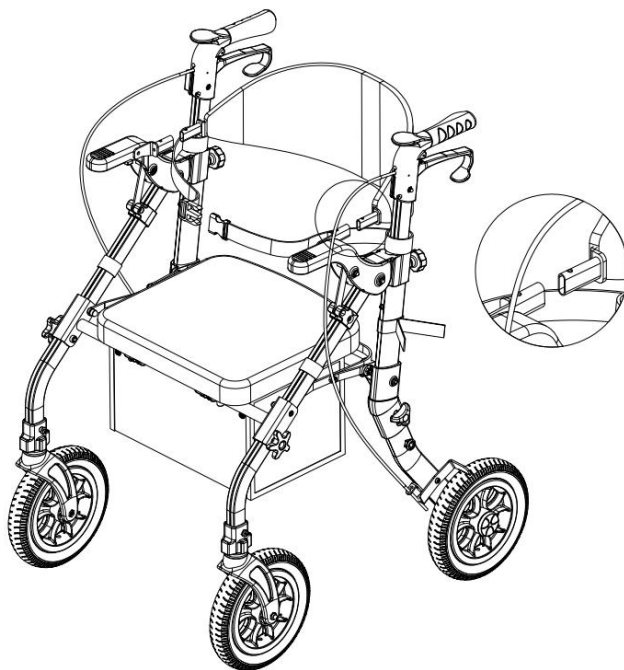
Install the backrest: Attach the backrest to the main frame by aligning it with the corresponding holes.

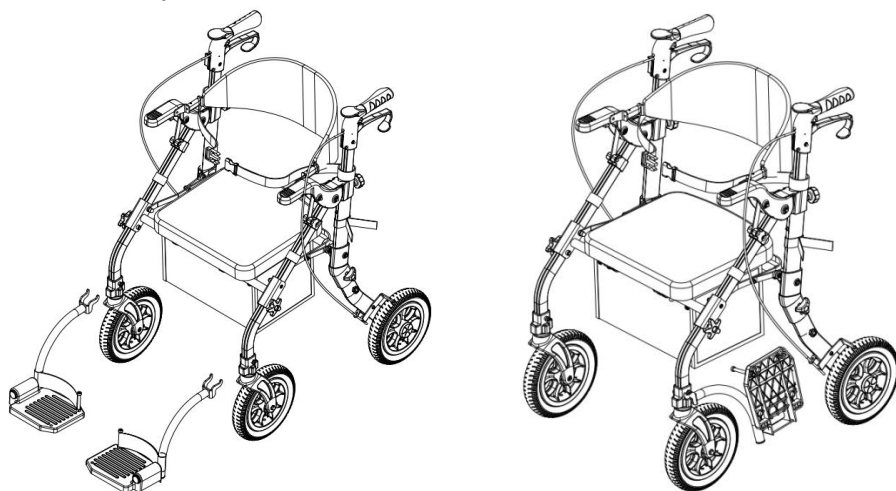Install the footrest: Hook the footrest onto the screw; when you want to use the footrest, place it in front of the main frame. When you don't need to use the footrest, you can place it behind the main frame.

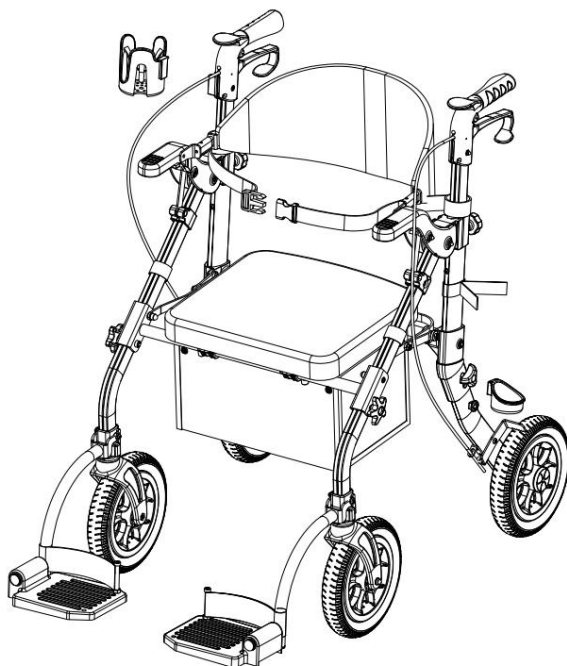Finally, install the umbrella holder and cup holder.

Post-Installation Inspection:

1. Verify that the four-wheeled vehicle is assembled correctly by comparing it with the diagrams in the manual.
2. Confirm that all components of the four-wheeled vehicle are fully assembled, and check for any leftover parts.
3. Check if any part of the four-wheeled vehicle is loose; if so, tighten the screws and nuts.
4. Check if all external components of the four-wheeled vehicle are misaligned; make appropriate adjustments if needed.
5. Check if the brake functions properly, the folding operation works smoothly, and the steering moves freely.

INSTRUCTIONS FOR USE

1. When getting out of bed: Place the walker next to the patient's affected leg, shift your body toward the edge of the bed, move the affected leg off the bed, then swing the unaffected leg off the bed. Straighten your body and stand up while holding onto the walker.
2. When sitting down: Use a chair with a backrest and armrests (you can add a seat cushion), and sit down slowly; be mindful of your position when moving backward.
3. When standing up from a chair: First, shift your body to the edge of the chair. Place the affected limb forward, let the unaffected limb bear most of your body weight, stabilize the walker, grip the armrests firmly, straighten your torso, and stand steadily. Bend your elbows at approximately 30°, and align the top of the walker with the inner junction of your wrists to ensure the patient's arms are comfortable.
4. When walking: Before use, adjust the width and height of the walker. Sit on the edge of the bed or a chair, grip the walker handles, support yourself with the walker, stand steadily, keep your head up and chest out, look forward, and lean your center of gravity slightly forward. Stand in the middle of the walker, move the walker forward 20cm, step forward with the affected limb, then move the unaffected limb to the middle of the walker.

SAFETY PRECAUTIONS

1. Do not let children play with the packaging unsupervised, as it poses a suffocation risk to children.
2. Keep this product away from children. It is not a toy—children may tip it over and injure themselves.
3. Position the four-wheeled vehicle as close to your body as possible, and keep your body as straight and forward-facing as you can.
4. The four-wheeled vehicle may only be used for its intended functions.
5. The maximum weight capacity of the four-wheeled vehicle is 158kg.
6. The four-wheeled vehicle may only be used when it is in good condition.
7. Before use, ensure the four-wheeled vehicle functions properly and all screws

are tightened.

8. The seat of the four-wheeled vehicle is only for short-term rest and cannot be used as a replacement for a chair.
9. Do not stand on the four-wheeled vehicle, and do not use it to carry other people.
10. Exercise extreme caution when using it on road shoulders, near obstacles, on sloped, uneven, or slippery surfaces, as this may lead to falls or injuries.
11. The four-wheeled vehicle may only be used after a thorough inspection, and when it is fully deployed and locked in place.
12. Be careful not to get your hands caught when folding or deploying the vehicle.

PRODUCT MAINTENANCE AND CARE METHODS

1. Do not scratch or scuff the product surface with hard objects to avoid damaging the material.
2. For daily cleaning and maintenance, use a neutral synthetic detergent: wipe with a damp cloth first, then dry with a dry cloth.
3. Do not let the product come into contact with strong acids or alkalis to prevent corrosion.

STORAGE CONDITIONS AND METHODS

After packaging, the four-wheeled vehicle should be stored indoors in a clean, well-ventilated area with no corrosive gases, where the temperature stays within -10°C to 40°C and the relative humidity does not exceed 80%. During transportation, protect it from moisture, avoid exposure to light, and keep it away from heat sources.

CORRECT DISPOSAL

This product is subject to the provision of European Directive 2012/19/EU. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this

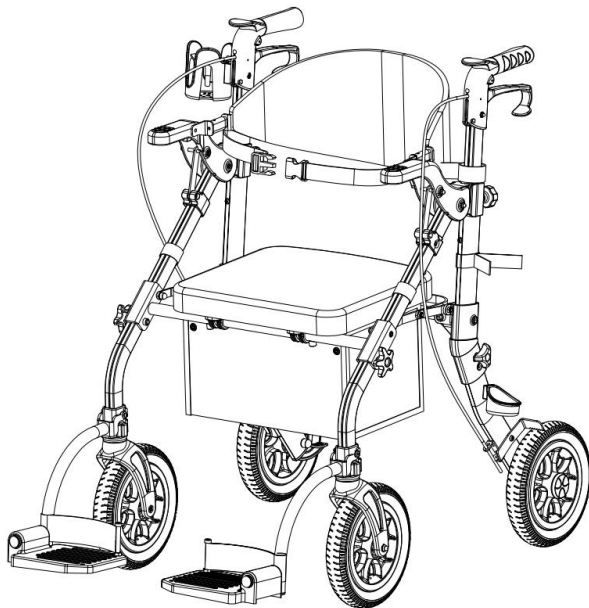
symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.

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