




ErgoAngleBar

with Optional Toilet Paper Holder


Model #4007-SSB, 4007-SSP, 4007-ORB
PW-94007-76BSS, PW-94007-76PSS, PW-94007-76BRZSS

Owner's Manual

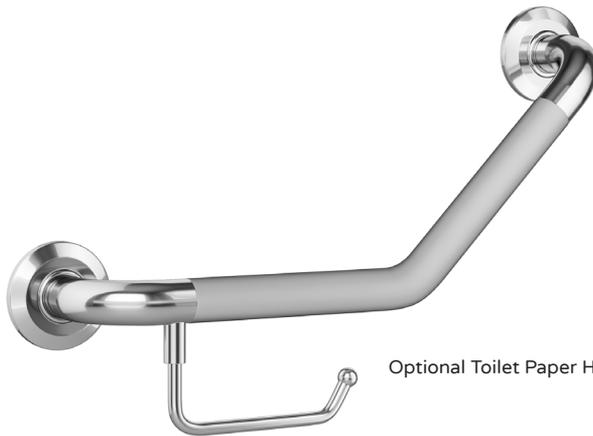

Optional Toilet Paper Holder Included

TOOLS REQUIRED FOR INSTALLATION

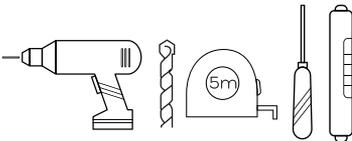

- Electric Drill
- 1/8" wood or metal drill bit
- 5/16" masonry drill bit
- Tape measure
- Screwdriver
- Level

IMPORTANT

Check local code for proper placement of ErgoAngleBar to meet requirement such as ADA or ANSI 117.1 Consumer Safety Requirements.

ERGOANGLEBAR CAN BE MOUNTED THROUGH SHOWER WALL AND SCREWED DIRECTLY TO WOOD OR METAL STUD WHEN APPLICABLE.


INSTALLATION QUESTIONS

Before returning your ErgoAngleBar to the store or have any question about installation or missing parts please visit the CUSTOMER CARE area on our website www.PulseShowerSpas.com.

Installation Instructions

- 1 Position mounting flanges and mark hole locations. For wood or solid backing applications drill 1/8" pilot holes through shower wall into wood or metal stud. NOTE: If wall is tiled, drill a 1/8" pilot hole through tile then change to the a 5/16" diameter masonry tile drill bit.
- 2 Install screws through mounting flange and directly into wood/metal stud or bracing. Use masonry anchors if necessary.
- 3 Once the ErgoAngleBar is securely fastened to wall, align cover plate locking tabs with slots in flange. Push cover against mounting until it locks.

DETAIL MOUNTING INSTALLATION

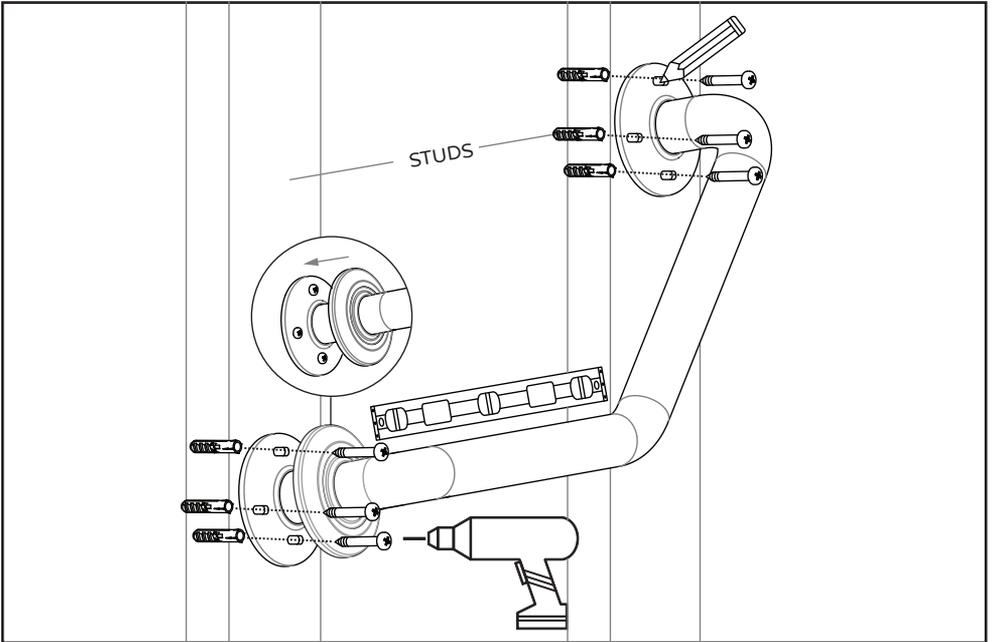

INSTALLATION FOR OPTIONAL TOILET PAPER HOLDER

