



ErgoSafetyBar

with Optional Towel Bar


Model #4006-SSB, 4006-SSP, 4006-ORB

PW-94006-75BSS, PW-94006-75PSS, PW-94006-75BRZSS

Owner's Manual



TOOLS REQUIRED FOR INSTALLATION

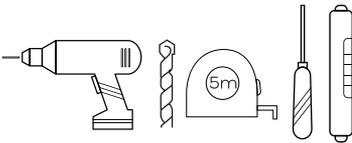

- Electric Drill
- 1/8" wood or metal drill bit
- 5/16" masonry drill bit
- Tape measure
- Screwdriver
- Level

IMPORTANT

Check local code for proper placement of ErgoSafetyBar to meet requirement such as ADA or ANSI 117.1 Consumer Safety Requirements.

ERGOSAFETYBAR CAN BE MOUNTED THROUGH SHOWER WALL AND SCREWED DIRECTLY TO WOOD OR METAL STUD WHEN APPLICABLE.


INSTALLATION QUESTIONS

Before returning your ErgoSafetyBar to the store or have any question about installation or missing parts please visit the CUSTOMER CARE area on our website www.PulseShowerSpas.com.

Installation Instructions

- 1** Position mounting flanges and mark hole locations.
For wood or solid backing applications drill 1/8" pilot holes through shower wall into wood or metal stud.
NOTE: If wall is tiled, drill a 1/8" pilot hole through tile then change to the a 5/16" diameter masonry tile drill bit.
- 2** Install screws through mounting flange and directly into wood/metal stud or bracing.
Use masonry anchors if necessary.
- 3** Once the ErgoSafetyBar is securely fastened to wall, align cover plate locking tabs with slots in flange. Push cover against mounting until it locks.

DETAIL MOUNTING INSTALLATION

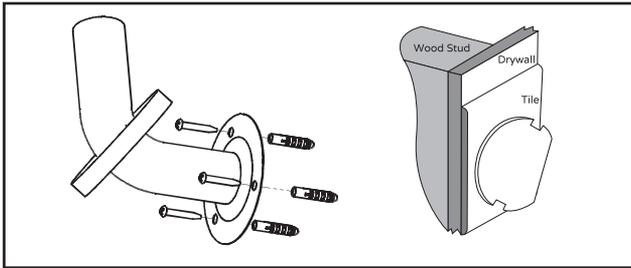

INSTALLATION FOR OPTIONAL TOWEL BAR



Towel Bar sold separately at www.PulseShowerSpas.com

