



Aluminum Bath/Toilet Chair CST-2182

CAUTION
UP TO 300LBS



Do NOT install commode without first reading and understanding this assembly and instruction manual. Failure to follow instructions may result in serious injuries. If you cannot understand these instructions, then contact a dealer or technical professional before attempting to install; Otherwise, injury or damage may occur. In particular, we are not to be held liable for any indirect or direct damage or consequential damage, loss or expenses in connection with the use or inability to use the device for a specific purpose.

WARNINGS

- ⚠️ Prior to each use check to assure that all screws, joints and fittings on the commode chair are tight.
- ⚠️ Toilet seat must be in down position before use. Failure to do so may result in breakage of unit and possible injury to user.
- ⚠️ Before using the commode chair, please adjust it to the suitable seating height. Adjustment method: Press the pinball with your finger up and down to adjust these four legs into appropriate sitting height, please note that these four legs must be adjusted to the same height of the hole, all four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
- ⚠️ Check seat board, the backrest are securely installed, please sit in the middle position of the Commode chair when using it.
- ⚠️ Keep away from babies and children in order to avoid the risk suffocation or choking.
- ⚠️ To prevent the injury caused by commode chairs overturned, please do not take only one side support when stand up or sit down. Be carefully when using it on the uneven ground. Please clean and make it dry promptly after using. Do not take it under in water and keep the commode chair dry.

⚠️ Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer / carrier for further instructions.

⚠️ Ensure ALL screws, nuts and / or bolts are tightened.

⚠️ Users with limited physical strength should be supervised or assisted while using commode

⚠️ Do not exceed the maximum weight capacity or serious injury could result: maximum weight capacity: 300lbs(136kg).

⚠️ DO NOT place commode chair on uneven surfaces.

⚠️ Don't push it outside for use.

⚠️ Step on the brake while holding it in place and check that the brake is working properly.

Storage Method

Please put the Commode chair in a dry and ventilated place. Do not exposure it under the sun for a long time.

Before removing / installing seat and lid, allow seat / ring to reach room temperature. This will help prevent the seat clamps from breaking when being removed / installed onto commode back frame rail.

Aluminum Bath/Toilet Chair CST-2182

Thanks for using our product.


Before using, please read the manual carefully.



Material: aluminum\HDPE/foam
Static pressure bearing force:136kg / 300LBS
The wheel diameter: 100mm/ 3.9 "
overall size: L480*W560*H960mm/
L18.9*W22*H37.8 "
The seat dimension: L450*W420*H550mm /
L17.7*W16.5*H21.7 "

Elements of the chair and manual for installation:


Product size:

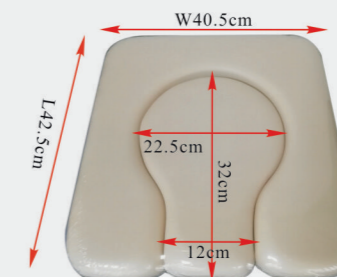


Parameter size is manual measurement, there may be some errors. For reference only, the final data shall prevail.

Installation process:


Installation of arc washer
According to the left schema method

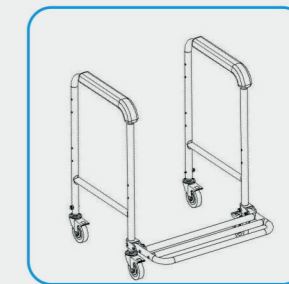


1. Two side armrest components and foot pedal combined installation.

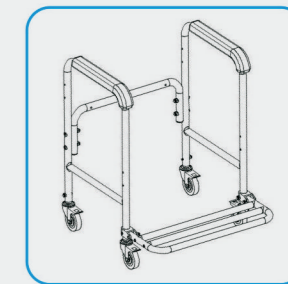

2. Install the rear seat tube and tighten the four screws with the tool.

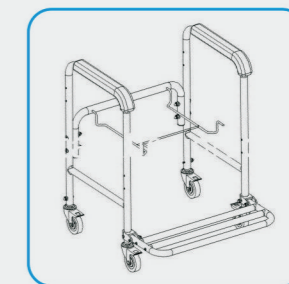

3. Install the bucket rack, and pay attention to the opening direction, as picture.

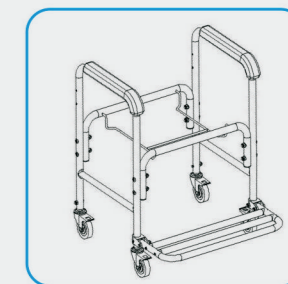

4. Install the front seat tube and fasten the four screws with the tool.



5. Install the back tube and fix the four screws with the tool.



6. The bucket is placed on the rack, then take the seat clip and align it to the rear seat tube. Press down hard.