

MANUAL

Extra Wide Adjustable Headrest Power Lift Recliner



1. ASSEMBLE GUIDE

Thank you very much for choosing our product!

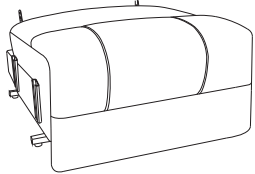
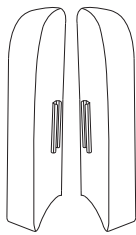
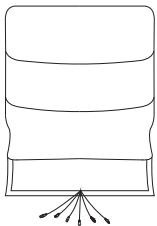
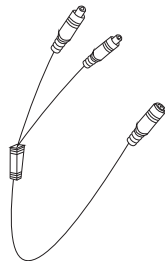
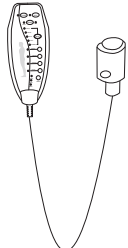
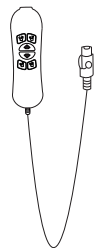
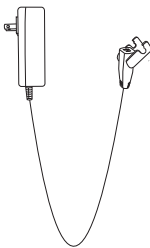
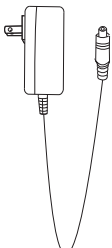
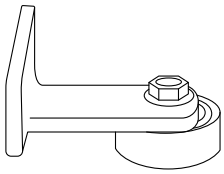
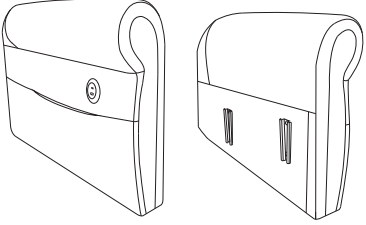
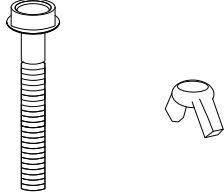
Please take time to read these instructions thoroughly and follow each step carefully for safe and easy operation. Keep this manual for future reference, especially for the safety information.

Also, it is better for you to assemble the chair in cooperation with 2-person, having a second person to help you makes it even easier.


Please read and understand this manual before any assembly. Before beginning assembly of product, make sure all parts are present. Compare parts with packaging contents list.

If any part is missing, damaged and any questions, please feel free to contact us.


2. PARTS LIST

BOX 1					
Part	Description				Qty
A					1PC
BOX 2					
Part	B	C		D	
Description					
Qty	2PCS		1PC		1PC
Part	E	F	G	H	I
Description					
Qty	1PC	1PC	1PC	1PC	2PCS
Part	J				K
Description					
Qty	2PCS				4PCS

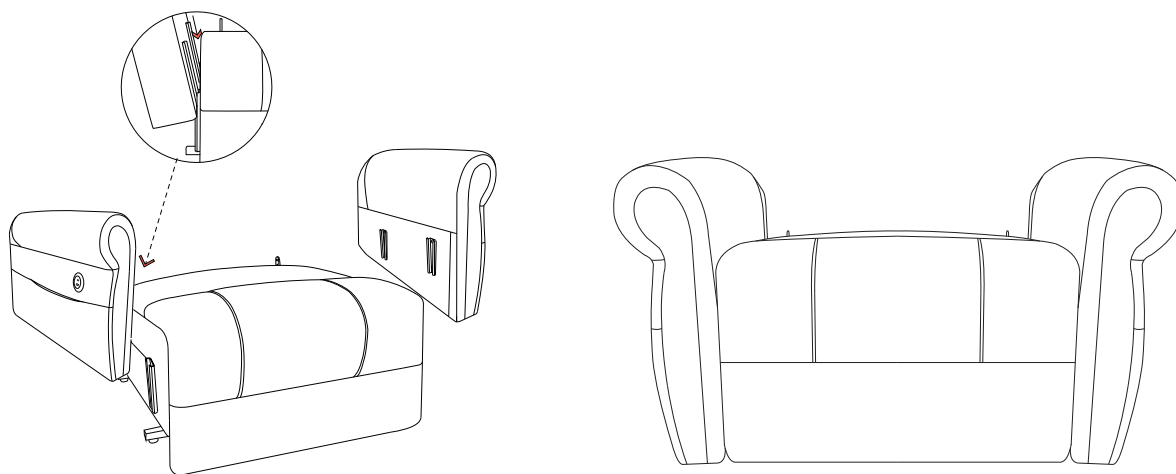
3. ASSEMBLY STEPS



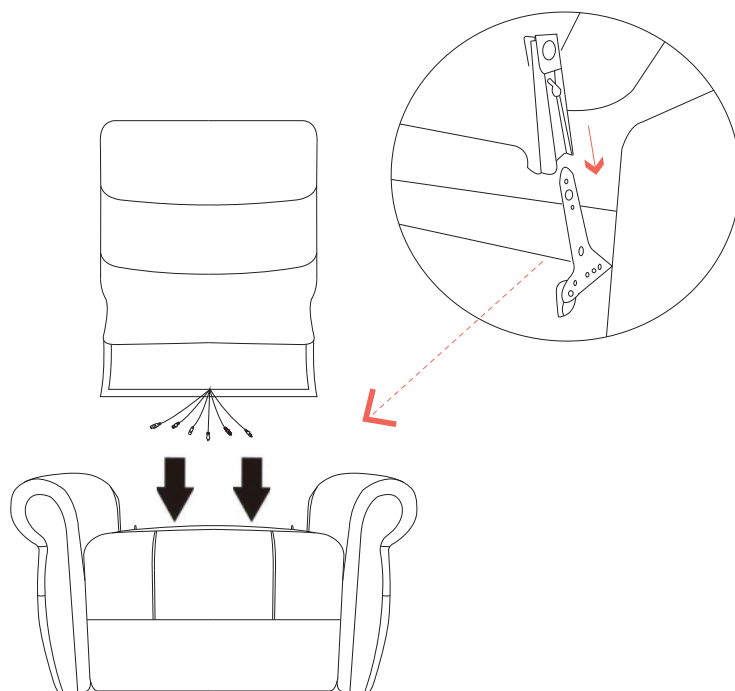
Step1: Align the two side slots and install the Armrest.



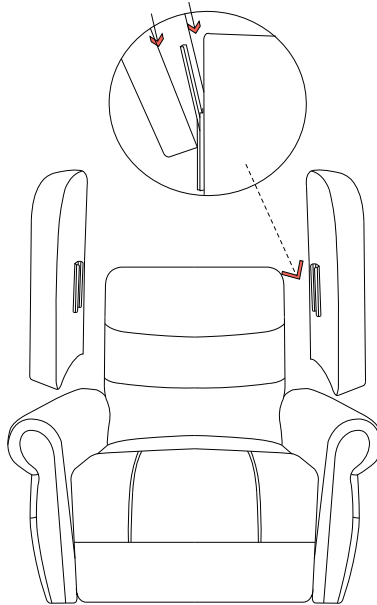Step2: Align the back to the seat base through the slots then to press down backrest, until hear a “click” , and then ensure that the backrest is stable and will not be pulled out.



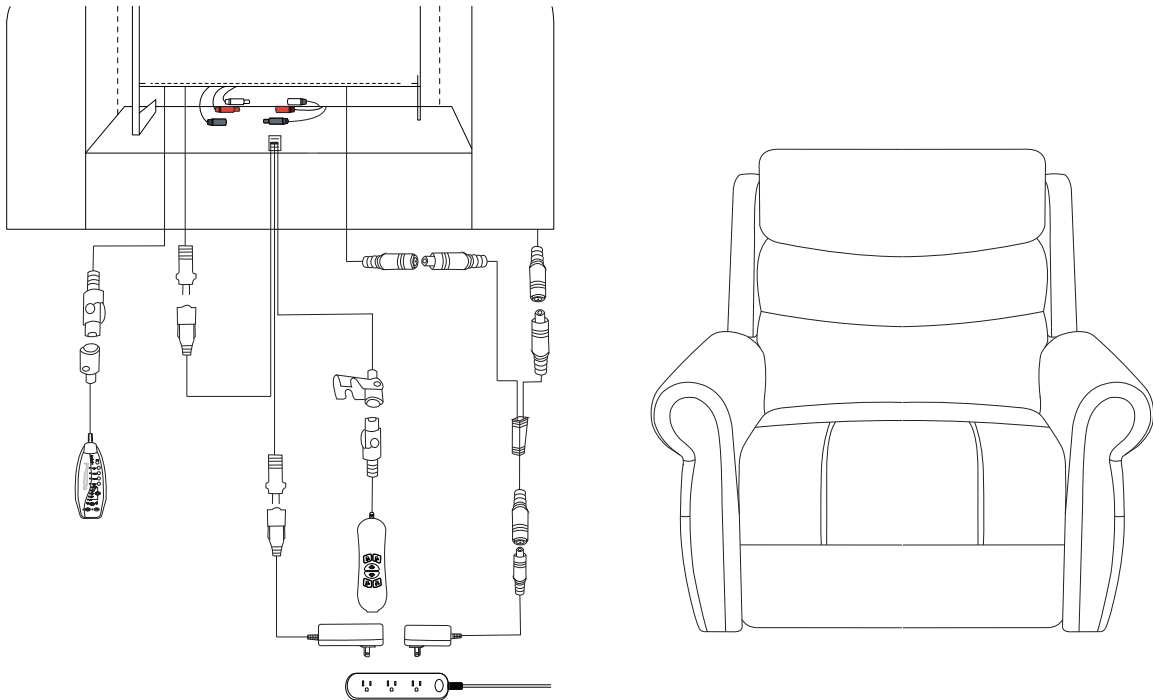Step3: Align the two side slots on both sides of the backrest.




Step4: Connect the remaining power cables one by one as shown in the diagram.

Please note:

The USB socket in the bottom of the recliner armrest.

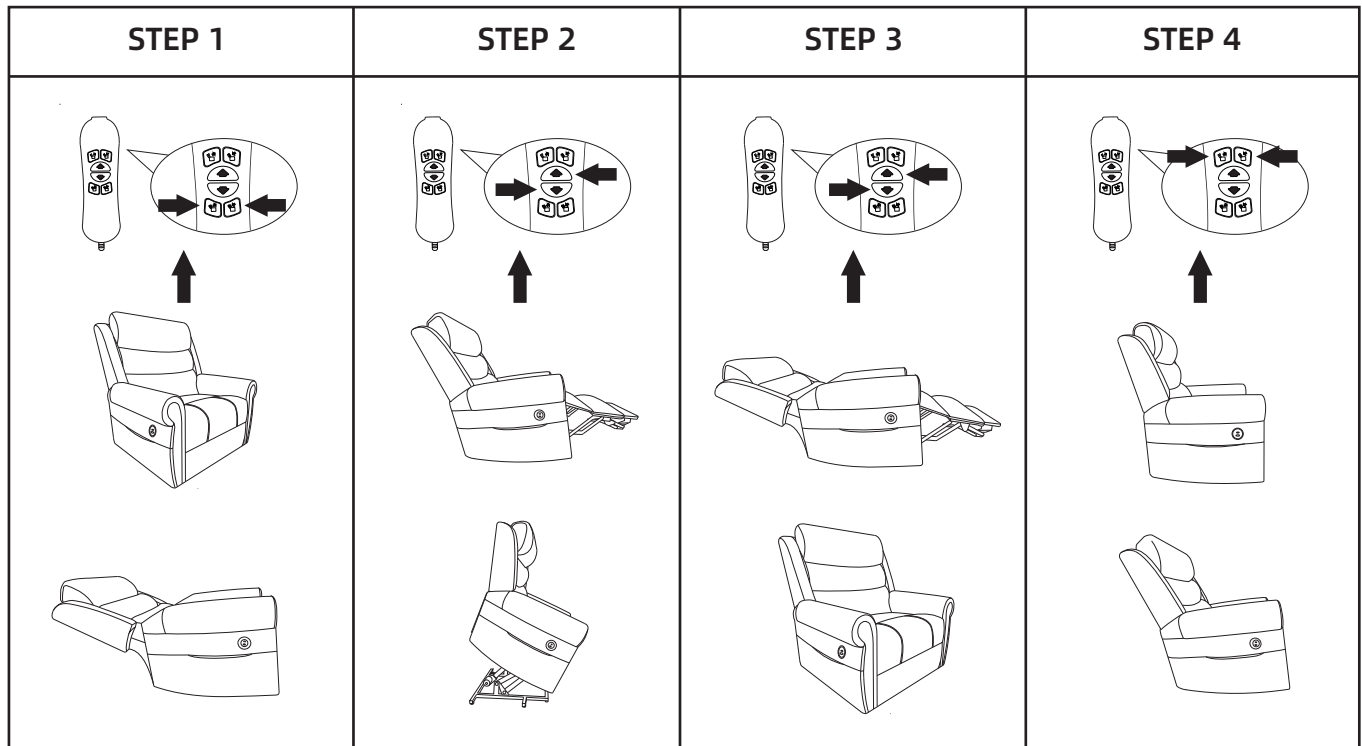4. OPERATING INSTRUCTIONS



How to use the remote control for massage?

