


Lift Chair


Assembly Instructions

Your recliner comes packaged in 3 boxes, and may not arrive at the same time. Kindly wait and you will get soon.

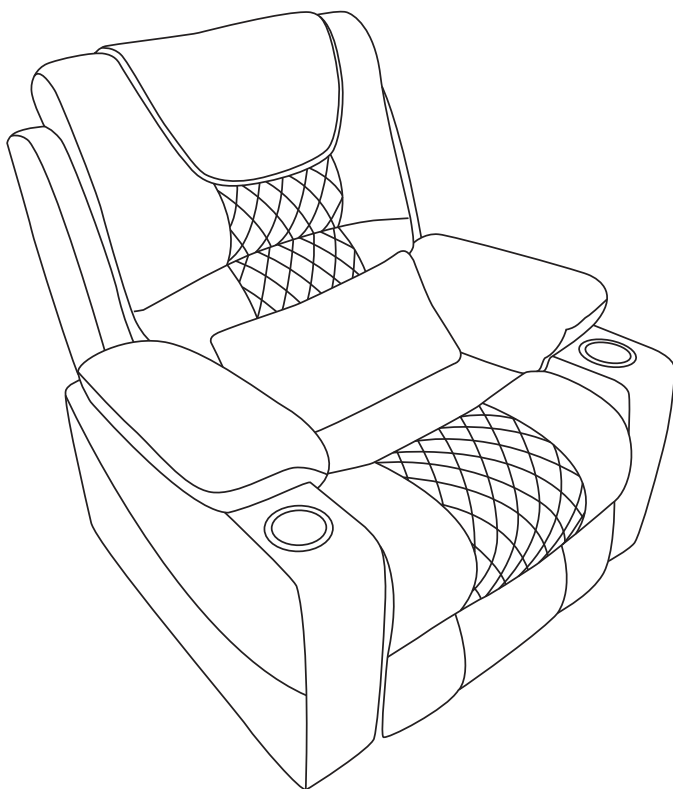

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION.
PLEASE READ AND KEEP FOR FUTURE REFERENCE.







GENERAL ASSEMBLY INFORMATION GUIDE

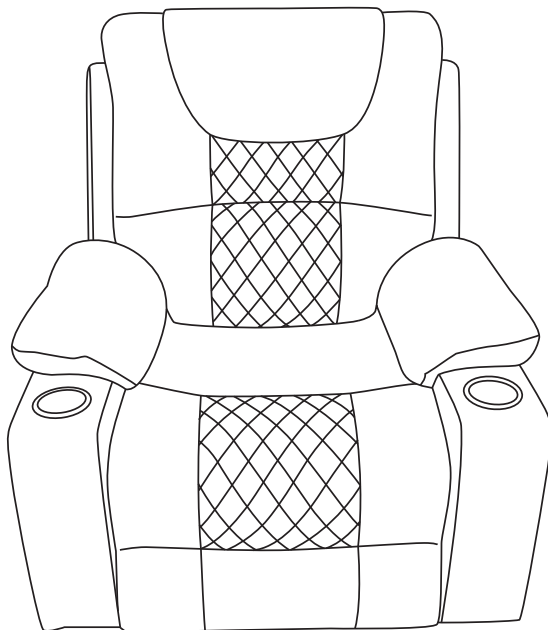

Please read the following instructions prior to assembling this product:


1. To avoid damage, assemble this product on a non-abrasive surface such as carpet.
2. Make sure all bolts are tightly fastened before use.
3. For home use or light commercial use only.
4. Please make sure you have all parts before assembly.

Package Contents

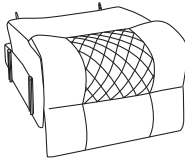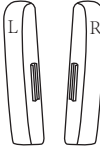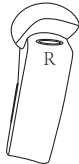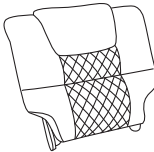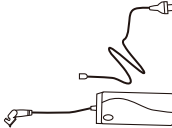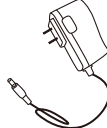				
A 1* Chair Seat	B 1*Ear	C 1*Left Armrest	D 1*Right Armrest	
Box 1		Box 2		
				
E 1*Backrest	F 1*Power Supply	G 1*Power Plug	H 1*Massage Remote	I 1*Remote
Box 3				

Please find the spare parts at the bottom of the seat(Box1).


Step 1 Remove this hardboard before assembly.

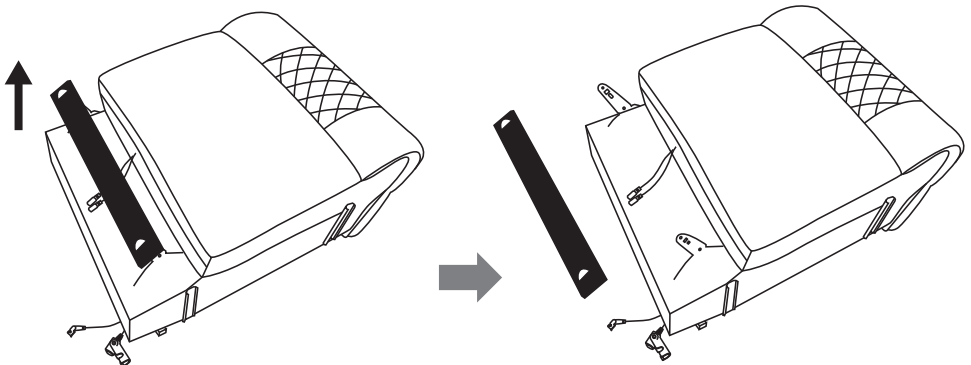

Step 2 Assemble the armrests.

Connect the Armrests(C&D) to the chair seat(A) by aligning the metal grooves and sliding down to the bottom.


PS: Pls ask another adult to pull the top cushion of the seat aside so that you can see clearly the metal grooves, it will be easier to make it, pls align the front metal groove first instead of the back one.

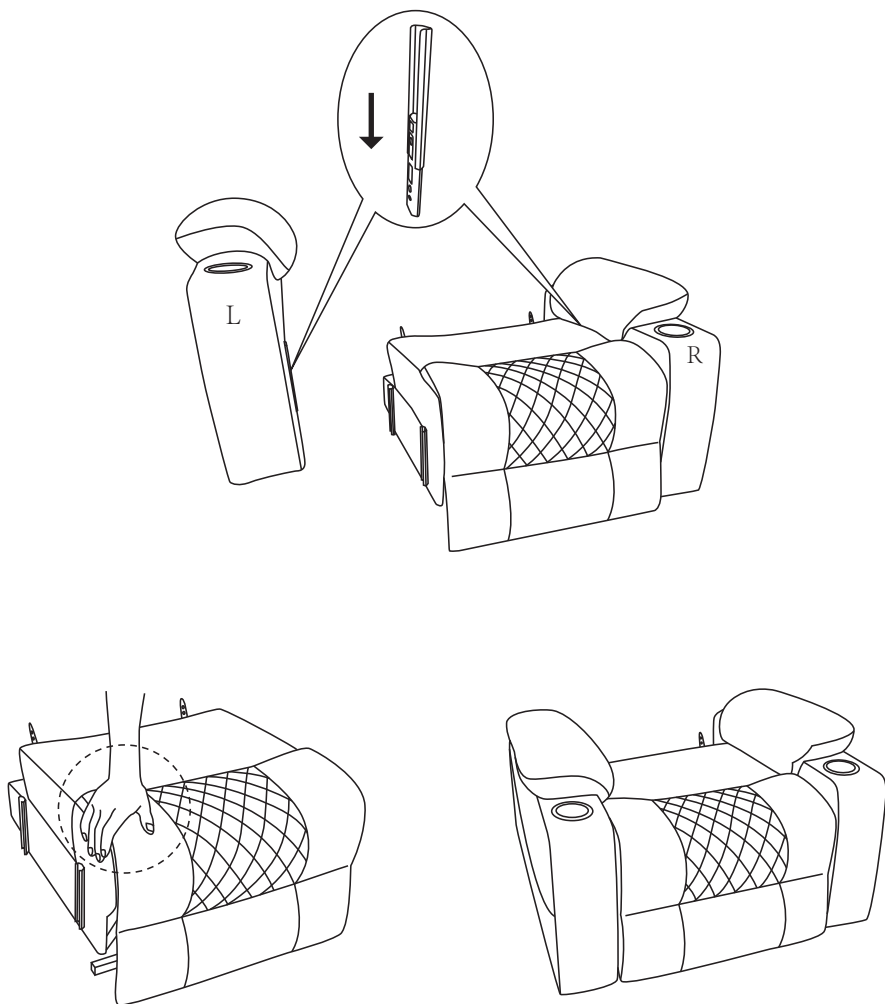

Step 3


Connect the chair backrest(E) to the chair seat(A) by aligning the metal grooves(M&N) and sliding down to the bottom.

PS: If the metal groove M slides down to the bottom of metal groove N, the backrest will not be taken off easily.

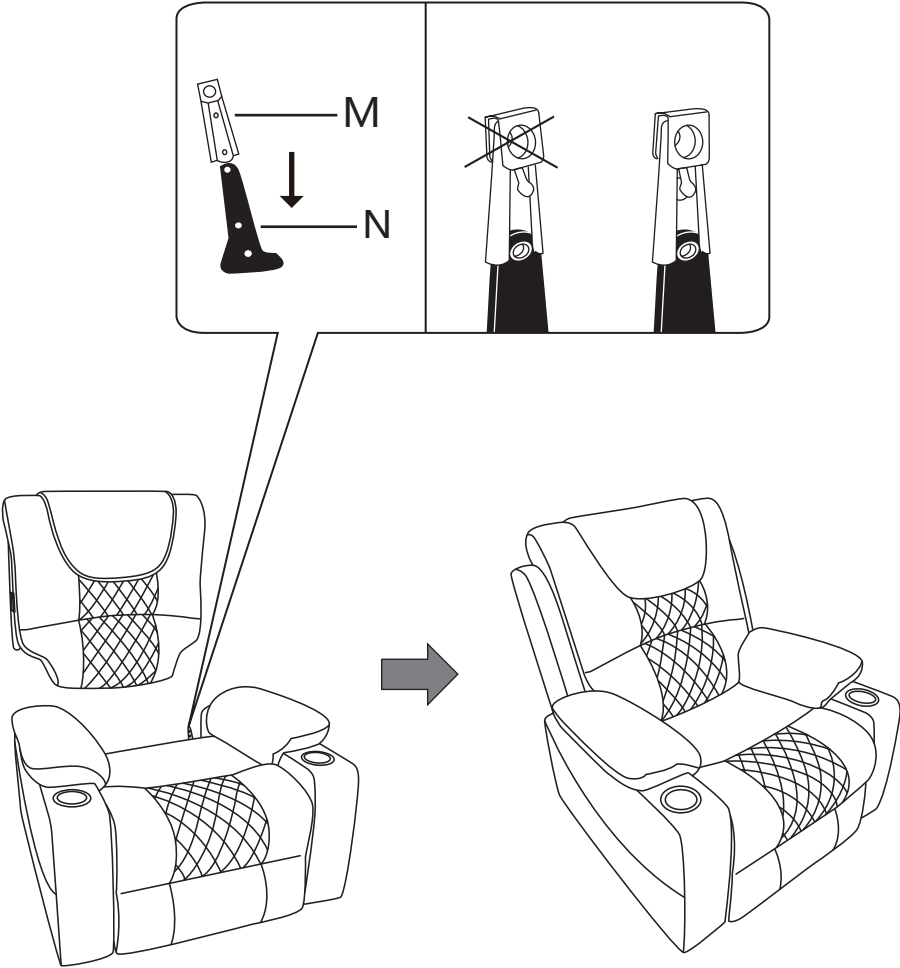


Step 4

Connect each Ear (B) to the chair backrest(E) by aligning the metal grooves and sliding down to the bottom.

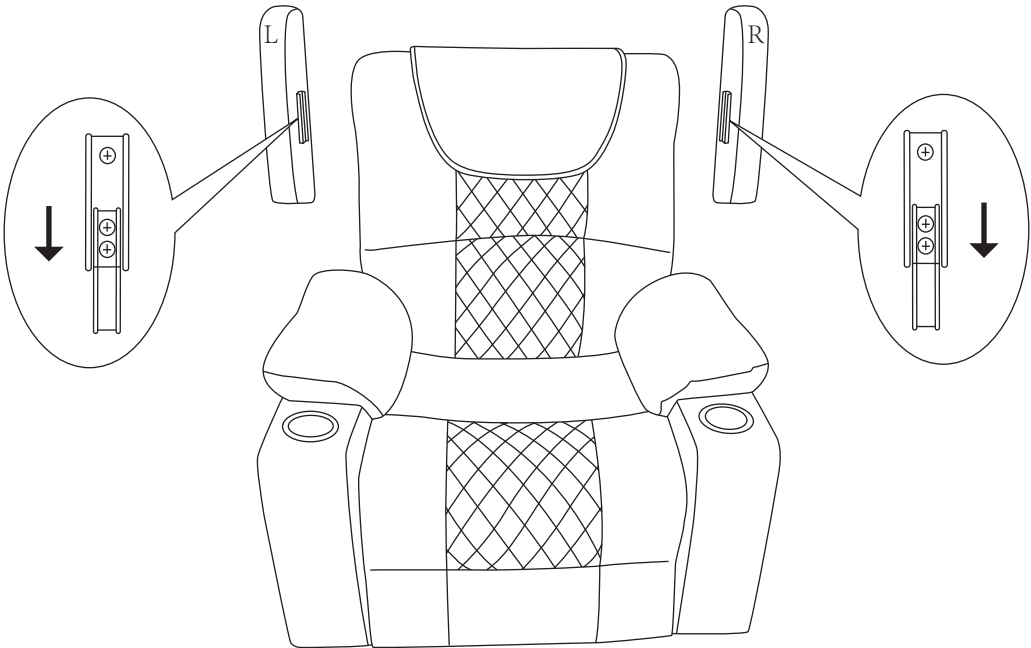


Step 5

Please adjust the supporting feet to stabilize the recliner.

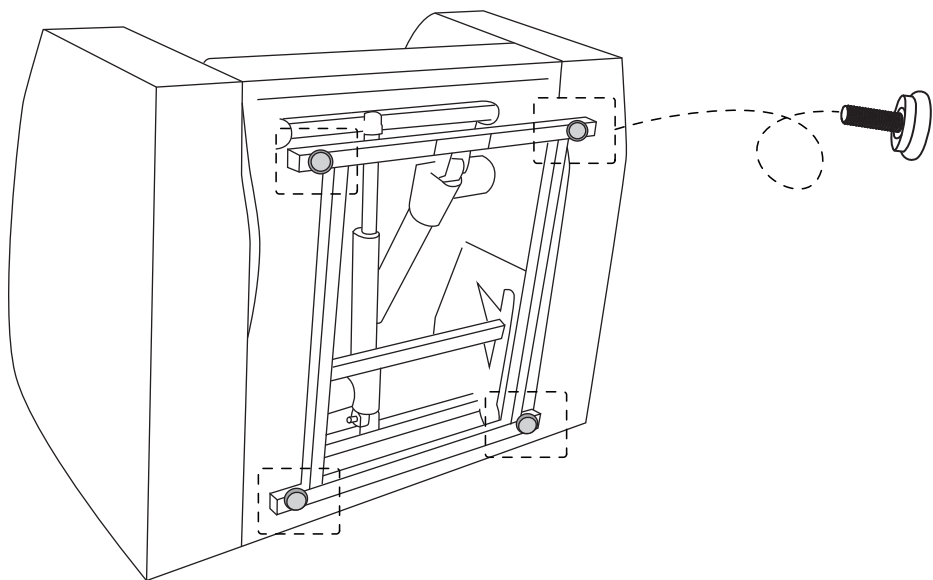

Step 6

Because the backrest is filled with doll cotton, 3cm difference is allowed, and you can slap the higher part to align the backrest.

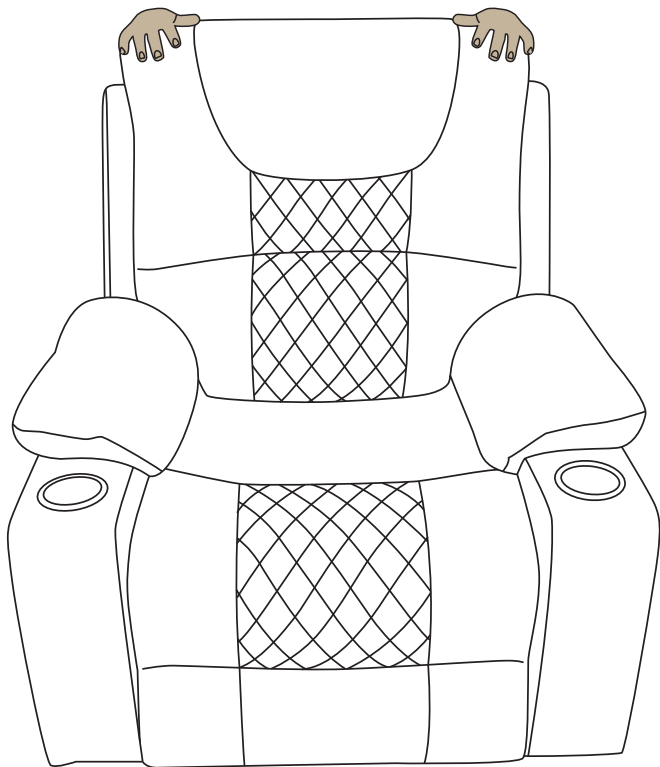

Instructions for Connecting Power Cords

Pls be sure the wires with the same color are connected.

Tip: Lines of the same color must be connected. If the wires with different colors are connected, it can not work.


No.1 and No.2 wires connect the backrest to the chair seat.

No.3 wire connects the backrest to the power plug(G).

No.4 wire connects to the backrest and massage remote(H).

No.5 wire connects the motor with the wire tied in iron frame, which has a control panel.



No.6 wire connects the motor to the power supply(F).

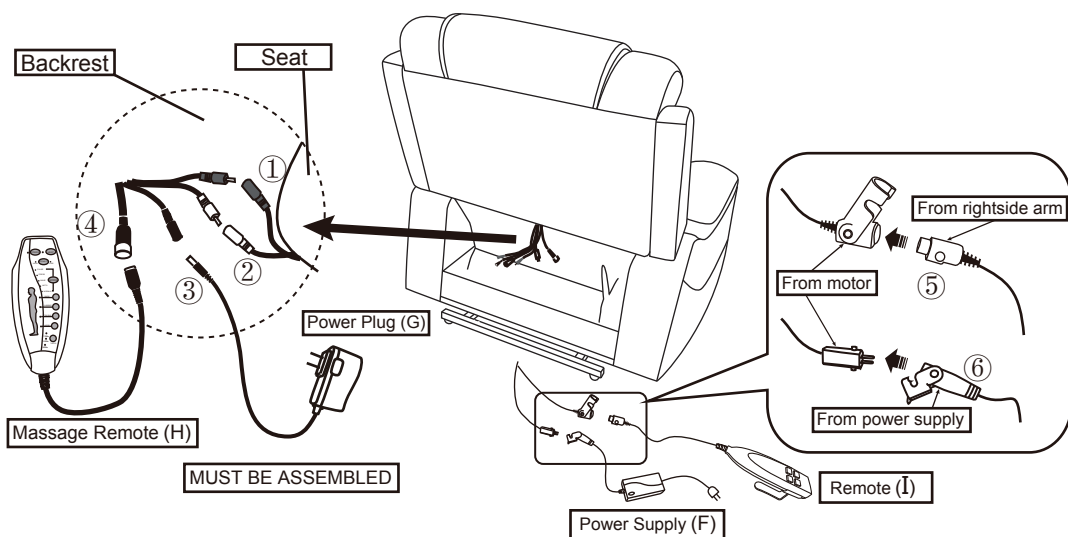

Note:



1. The connection plugs have different colors, black to black, white to white, and red to red.
2. Plug the Power Plug(G) and Power Supply(F) into the outlet to get the recliner working.

Instruction for Remote control

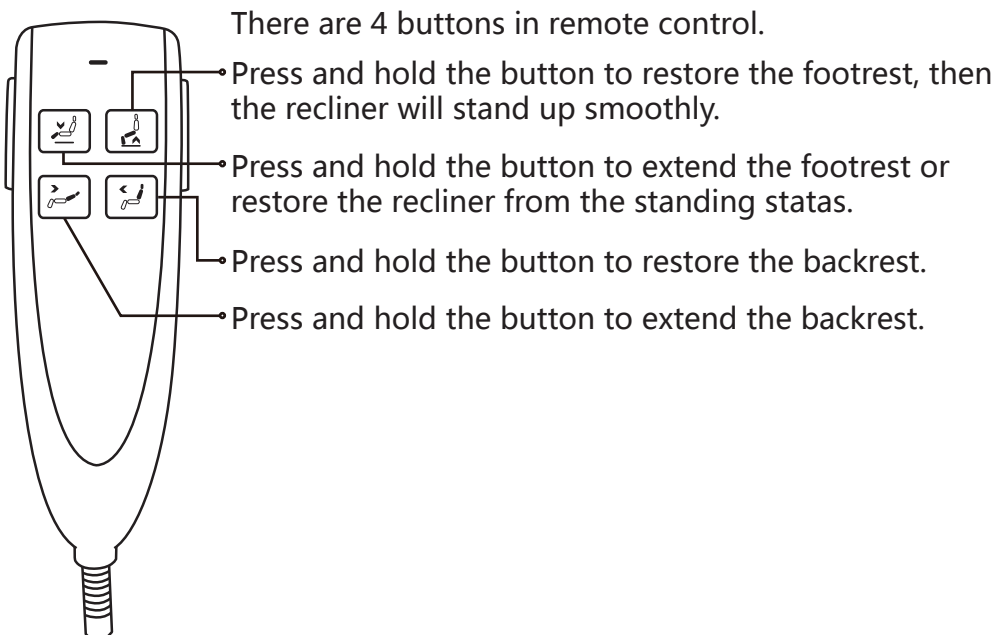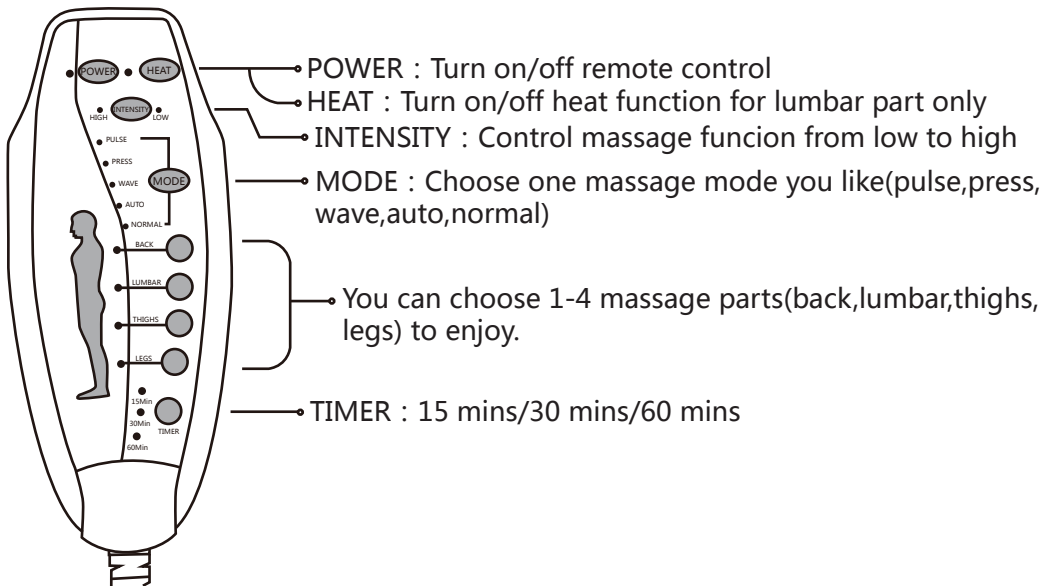

Warnings

1. Please check if the chair's components are all included and can be used generally as the 6th tip.
2. Follow instructions and assemble all components correctly before connecting the chair with the power.
3. Please DO NOT sit on the armrests or footrest to eliminate the risk of tipping over.
4. Please be careful when lifting to avoid sliding from the chair.
5. If the chair you received is damaged or doesn't with the website, please send clear pictures of the product, and take a picture of one side of every outer box. (This side is printed with a camera pattern). Then send us these photos or videos, and we'll help you to solve all problems.
6. If the lift recliner doesn't work, you can try the following measures for self-checking:
 - ① Check if all the cables are connected;
 - ② Check if the pilot lamp of remote control is on or off;
 - ③ Check if the pilot lamp of the power supply is on or off
(1. on is normal. 2. off, check the wire connection of power supply, if connection is correct but the light is not on, please contact us to change a new power supply for you.)
 - ④ If the motor has a weak buzz on the condition that the two steps above are normal. If the chair still doesn't work, please leave the message to customer service in time. We will help solve the problem for you as soon as possible.
7. If you have any questions about the product, please get in touch with us.

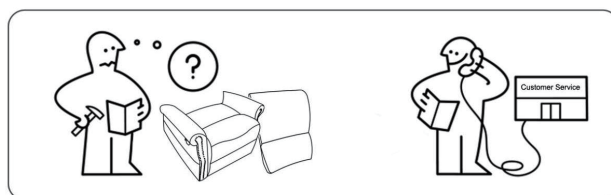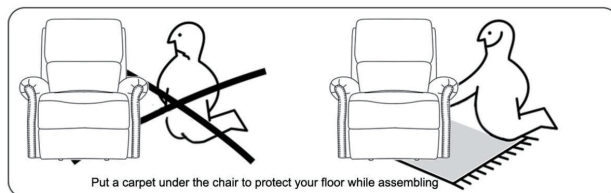

HEALTH CAUTION

- ☒ If you have any concerns regarding to your health, consult your doctor before using this product.
- ☒ Individuals with pacemakers should not use this product.
- ☒ If you have pain in a muscle or joint after using for a prolonged period of time, discontinue using and consult your doctor. Persist pain could be a symptom of a more serious condition.

MAINTAINANCE INSTRUCTIONS

TO STORE:

Place unit in its box or safety dry and cool place. Avoid putting chair in contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap attached power cord around the unit. DO NOT pull the unit by power cord.

TO CLEAN:

- ☒ Ensure the unit is unplugged from the power source, and allow the unit to cool down before cleaning.
- ☒ DO not use any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat with clean, dry cloth.
- ☒ Do not use any abrasive cleaners, brushes, gasoline, kerosene, glass furniture polish, paint thinner or water hotter than 120° F to clean the unit.